

Spare Parts Catalogue

Version: 2019/11/15a EN





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information

| | |
|---|---|
| Abbreviations | 7 |
| Frame colours & CPS colours | 8 |
| Forks and Front wheels Adjustment | 9 |

Chassis

| | |
|---|----|
| Chassis | 10 |
| Swing away frame | 11 |
| Fixed front end 80 degree angle frame | 13 |
| Folding system | 15 |
| Swing Away Chassis | 16 |
| Fixed front end Chassis | 26 |

Seat

| | |
|---|----|
| Seat | 36 |
| Seat upholstery, depth adjustable | 37 |
| Seat upholstery, basic | 39 |
| Seat upholstery, tension & depth adjustable | 41 |

Backrest

Backrest type

| | |
|--|----|
| Backrest type | 43 |
| Backrest attachment type | 44 |
| Fixed backrest | 45 |
| Backrest height adjustable 7°, without & rearsset pushhandle | 46 |
| Backrest height adjustable, Integrated pushhandle | 47 |

Backrest Upholsteries

| | |
|---|----|
| Backrest Upholsteries | 48 |
| Backrest upholstery fixed, for fixed & integrated pushhandle | 49 |
| Backrest upholstery adjustable, for fixed & integrated pushhandle | 51 |
| Backrest upholstery adjustable, for without & rearsset pushhandle | 54 |
| Backrest Upholsteries adjustable télescopique | 56 |
| Backrest upholstery MatrX | 58 |
| Mounting Hardware MatrX | 59 |

Legrests

| | |
|--|----|
| Legrests | 60 |
| Hanger 70° | 61 |
| Hanger 80° | 62 |
| Hanger 90° | 63 |
| Elevating Legrest Standard 2019.04.15 => | 64 |
| Elevating legrest =>2019.04.14 | 65 |
| Calf Pad Medium 2019.04.15 => | 66 |
| Calf Pad =>2019.04.14 | 67 |
| Amputee Legrest | 68 |



Yes, you can.®

TABLE OF CONTENTS

Footrests

| | |
|--|----|
| Footrests | 69 |
| Footrest fixed, swing away frame | 70 |
| Footrest fixed, fixed frame | 72 |
| Footrest fixed, high mounted tube, swing away frame | 74 |
| Footrest angle & depth adjustable, swing away frame | 76 |
| Footrest angle & depth adjustable, fixed frame | 79 |
| Footrest angle & depth adjustable, high mounted tube, swing away frame | 82 |
| Angle and depth adjustable footrest AA-Legrest(Medium) 2019.04.15 => | 84 |
| Footplate cover Medium 2019.04.15 => | 85 |
| One piece footplate for fixed frame | 86 |

Siderests

| | |
|---|----|
| Siderests | 88 |
| Side guard 2017.02.16=> | 89 |
| Mudguard 2018.05.18=> | 90 |
| Armrest n°1, Removable, height adjustable T-arm, desk armpad 280mm 2017.02.16=> | 91 |
| Armrest n°1, Removable, height adjustable T-arm, full lenght armpad 380mm 2017.02.16=> | 92 |
| Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm) | 93 |
| Armrest n°3, Swing away detachable & height adjustable, full length armpad (380mm) | 95 |
| Tubular Armrest | 97 |
| Hemi Dartex armpad multiposition standard 2017.02.16=> | 98 |

Front wheels

| | |
|------------------------------------|-----|
| Front wheels | 99 |
| Forks for swing away frame | 100 |
| Forks for fixed frame | 102 |
| Forks support for swing away frame | 104 |
| Forks support for fixed frame | 105 |
| Front wheels | 106 |
| Forks and Front wheels Adjustement | 9 |

Rear Wheels

| | |
|---|-----|
| Rear Wheels | 107 |
| 22"/24" HD Cross Spoked wheels | 108 |
| 22"/24"/25" Lightweight | 110 |
| 22"/24" Composite wheels | 112 |
| 24"/25" Spinergy Wheels | 113 |
| 22"/24" Hub brake Wheels (New) 2019.10.27=> | 114 |
| 22"/24" Hub Brake Wheels =>2019.10.26 | 116 |
| Standard & hub brake wheels accessories | 117 |
| Lightweight wheels accessories | 119 |
| Composite wheels accessories | 121 |
| Spinergy wheels accessories | 122 |
| Spoke protector | 123 |
| Rear wheel support | 124 |
| Pump | 125 |



Yes, you can.®

TABLE OF CONTENTS

| | |
|---|-----|
| Bracket set for rear wheel support Twion & E-motion | 126 |
|---|-----|

Brakes

| | |
|---------------------------|-----|
| Brakes | 127 |
| Push Brakes | 128 |
| Pull Brakes | 129 |
| Attendant Hub Brake | 130 |

Safety Options

| | |
|-------------------------------------|-----|
| Safety Options | 131 |
| Buckled belt | 132 |
| Buckled belt [UK] | 133 |
| Anti-tipper, Angle adjustable | 134 |
| Reflectors | 135 |
| Kit anti barrier | 136 |


Seating Options

| | |
|---|-----|
| Seating Options | 137 |
| Cushion foam Nylon & anti-slip cover | 138 |
| Profil Air Cushions [FR] | 139 |
| Rigidizer [FR] | 140 |
| Classic Visco Cushions [FR] | 141 |
| Matrx Flo-Tech CONTOUR VISCO cushions | 143 |
| Matrx Flo-Tech IMAGE cushions | 144 |
| Matrx Flo-Tech LITE cushions | 145 |
| Matrx Flo-Tech XTRA cushions | 146 |
| Matrx VI Cushions | 147 |
| Matrx LIBRA Cushions | 148 |
| Clip to keep the chair folded | 149 |
| Heels strap | 150 |
| Calf strap | 151 |
| Folding backbrace | 152 |
| Backbrace for rigid chassis | 153 |

Other Options & Accessories

| | |
|-----------------------------------|-----|
| Other Options & Accessories | 154 |
| Cane holder | 155 |
| Tipping aid | 156 |
| Tool Kit | 157 |
| Case holder | 158 |


Abbreviations

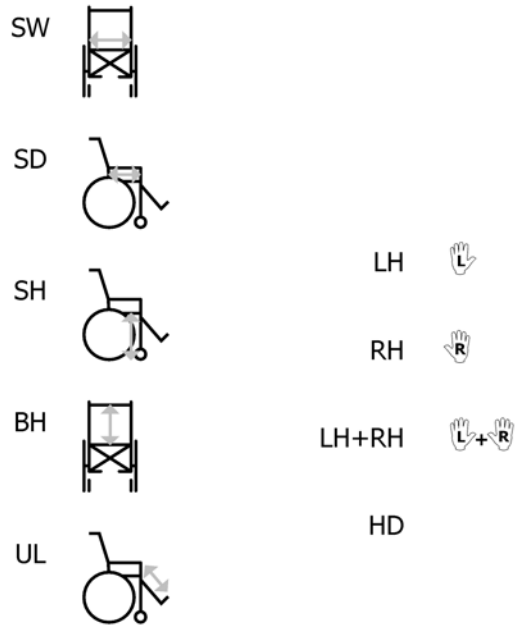

15P5133721

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| - | SW | Seat width (2 digits in cm) (3 digits in mm) | | 1 |
| - | SD | Seat depth (2 digits in cm) (3 digits in mm) | | 1 |
| - | SH | Seat height (2 digits in cm) (3 digits in mm) | | 1 |
| - | BH | Backrest height (2 digits in cm) (3 digits in mm) | | 1 |
| - | UL | Knee-to-heel height (2 digits in cm) (3 digits in mm) | | 1 |
| - | LH | Left side (sitting in the wheelchair) | | 1 |
| - | RH | Right side (sitting in the wheelchair) | | 1 |
| - | LH+RH | An assembly of a left and a right part, referred to as "pair" | | 1 |
| - | HD | Heavy duty version | | 1 |

Frame colours & CPS colours


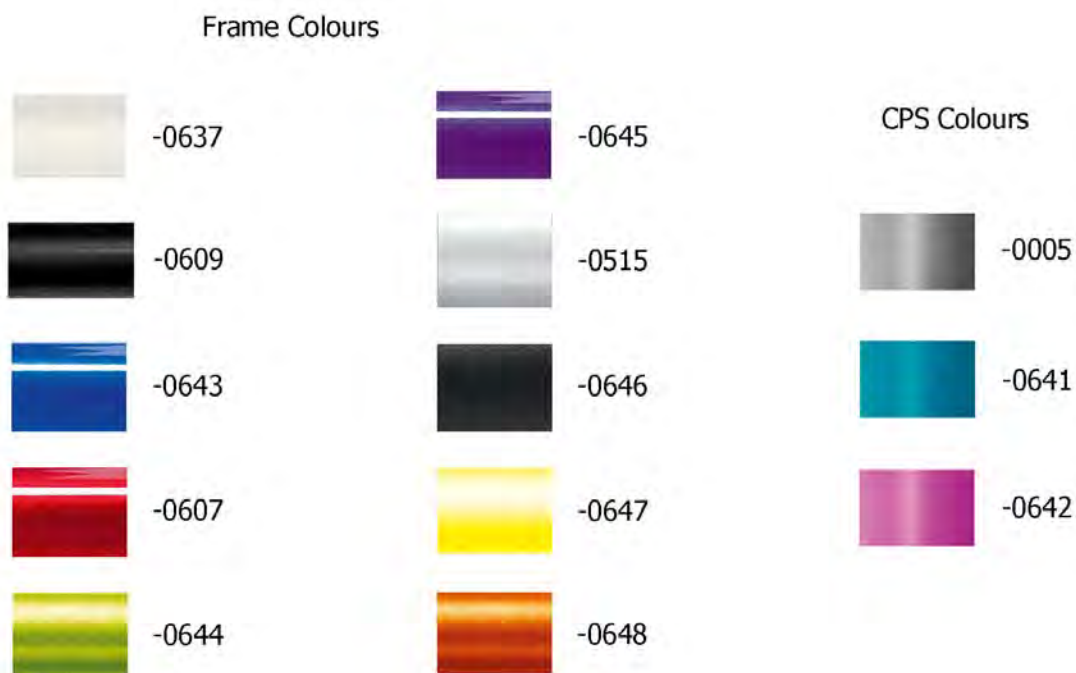


185201602017

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| - | -0637 | Colour code for frame, Pearl White | | 1 |
| - | -0609 | Colour code for frame, Anthracite Black | | 1 |
| - | -0643 | Colour code for frame, Pop Blue | | 1 |
| - | -0607 | Colour code for frame, Dark Red | | 1 |
| - | -0644 | Colour code for frame, Electric Green | | 1 |
| - | -0645 | Colour code for frame, Hypnotic Purple | | 1 |
| - | -0515 | Colour code for frame, Pearl Grey | | 1 |
| - | -0646 | Colour code for frame, Matt Black | | 1 |
| - | -0647 | Colour code for frame, Intense Yellow | | 1 |
| - | -0648 | Colour code for frame, Vibrating Orange | | 1 |
| - | -0005 | Colour code for CPS, without accent | | 1 |
| - | -0641 | Colour code for CPS, Blue | | 1 |
| - | -0642 | Colour code for CPS, Purple | | 1 |


Forks and Front wheels Adjustment

| | | Seat height in front (FSTF) in mm | | | | | | | |
|------------------|---|-----------------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|
| | | SH360 / 14" [SJ_1200] | SH385 / 15" [SJ_1210] | SH410 / 16" [SJ_1220] | SH435 / 17" [SJ_1230] | SH460 / 18" [SJ_1240] | SH485 / 19" [SJ_1250] | SH510 / 20" [SJ_1260] | SH510 / 21" [SJ_1270] |
| Front wheel size | 4" (100 mm) [SJ_7000] [SJ_7020] [SJ_7030] [SJ_7040] | UL1 Ultra Low | | S2 | S4 | L3 | L2 | | |
| | 5" (125 mm) [SJ_7055] [SJ_7060] [SJ_7070] | | UL2 Ultra Low | S1 | S3 | L2 | L1 | L3 | |
| | 6" (150 mm) [SJ_7080] [SJ_7090] [SJ_7100] | | | | S2 | S4 | L3 | L2 | |
| | 7" (180 mm) [SJ_7110] | | | | | S3 | L2 | L1 | L3 |

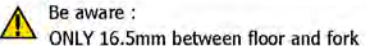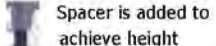
| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| - | UL1 | Fork Ultra Low, Front wheel axle Position 1 | | 1 |
| - | UL2 | Fork Ultra Low, Front wheel axle Position 2 | | 1 |
| - | S1 | Fork SMALL, Rear wheel axle position 1 | | 1 |
| - | S2 | Fork SMALL, Rear wheel axle position 2 | | 1 |
| - | S3 | Fork Small, Rear wheel axle position 3 | | 1 |
| - | S4 | Fork SMALL, Rear wheel axle position 4 | | 1 |
| - | L1 | Fork LARGE, Front wheel axle position 1 | | 1 |
| - | L2 | Fork LARGE, Front wheel axle position 2 | | 1 |
| - | L3 | Fork Large, Rear wheel axle position 3 | | 1 |

Chassis

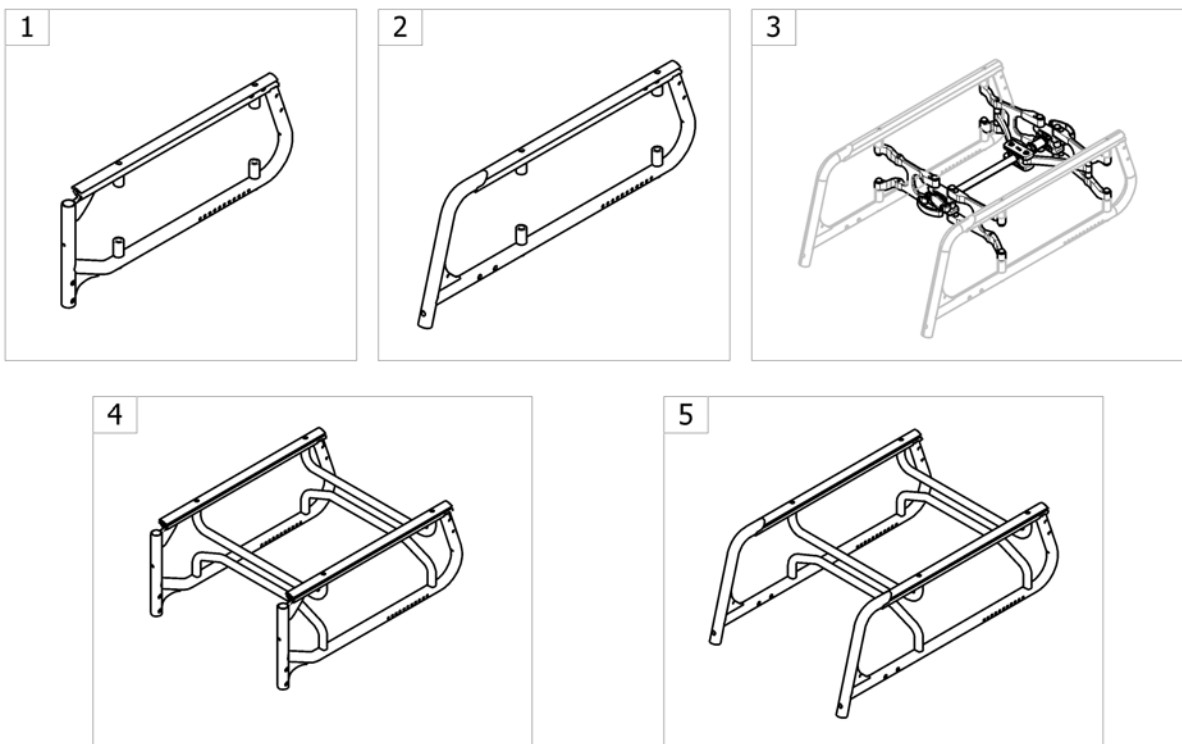

-<SPCACTIONS10

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Swing away frame [SJ_0200] | | 1 |
| 2 | - | Fixed front end 80 degree angle frame [SJ_0210] | | 1 |
| 3 | - | Folding system | | 1 |
| 4 | - | Swing Away Chassis [SJ_0200] | | 1 |
| 5 | - | Fixed front end Chassis [SJ_0210] | | 1 |

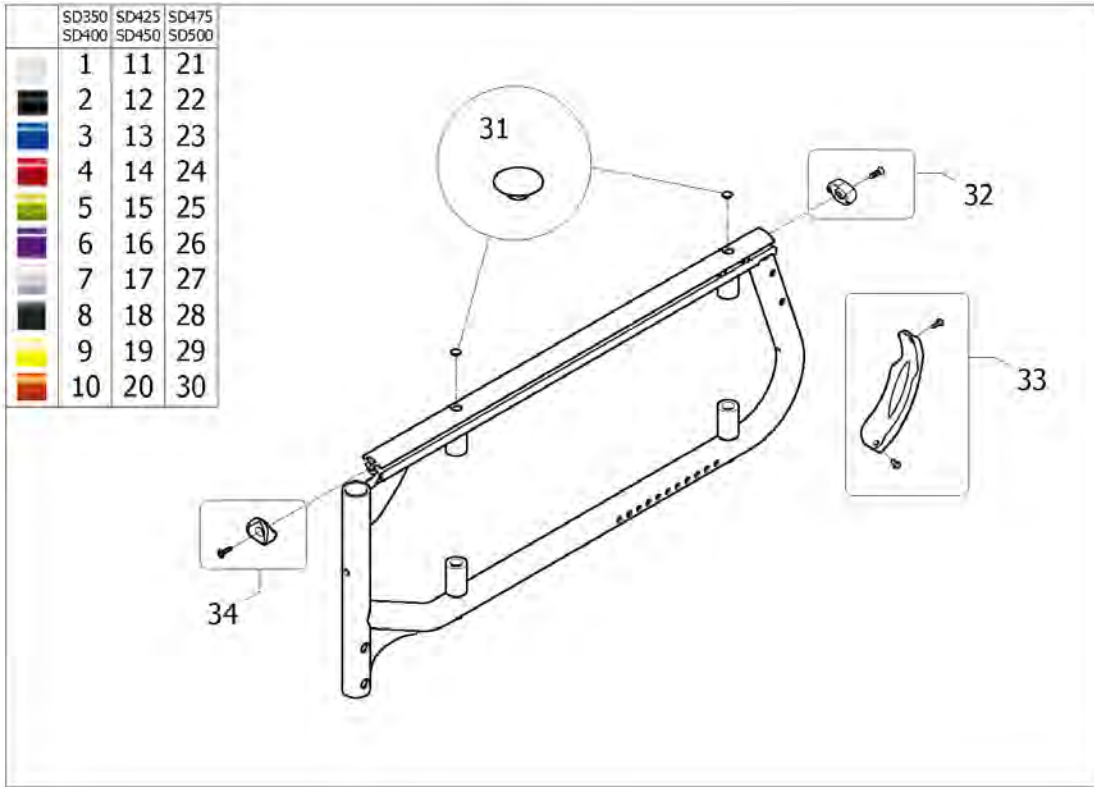


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1582901-0637 | Kit swing away frames, SD350/375/400mm, Pearl White (pair) [SJ_0200+SJ_6000] | | 1 |
| 2 | SP1582901-0609 | Kit swing away frames, SD350/375/400mm, Anthracite (pair) [SJ_0200+SJ_6010] | | 1 |
| 3 | SP1582901-0643 | Kit swing away frames, SD350/375/400mm, Pop Blue (pair) [SJ_0200+SJ_6020] | | 1 |
| 4 | SP1582901-0607 | Kit swing away frames, SD350/375/400mm, Dark Red (pair) [SJ_0200+SJ_6030] | | 1 |
| 5 | SP1582901-0644 | Kit swing away frames, SD350/375/400mm, Electric Green (pair) [SJ_0200+SJ_6040] | | 1 |
| 6 | SP1582901-0645 | Kit swing away frames, SD350/375/400mm, Hypnotic Purple (pair) [SJ_0200+SJ_6050] | | 1 |
| 7 | SP1582901-0515 | Kit swing away frames, SD350/375/400mm, Pearl Grey (pair) [SJ_0200+SJ_6060] | | 1 |
| 8 | SP1582901-0646 | Kit swing away frames, SD350/375/400mm, Matt Black (pair) [SJ_0200+SJ_6070] | | 1 |
| 9 | SP1582901-0647 | Kit swing away frames, SD350/375/400mm, Intense Yellow (pair) [SJ_0200+SJ_6080] | | 1 |
| 10 | SP1582901-0648 | Kit swing away frames, SD350/375/400mm, Vibrating Orange (pair) [SJ_0200+SJ_6090] | | 1 |
| 11 | SP1582902-0637 | Kit swing away frames, SD425/450mm, Pearl White (pair) [SJ_0200+SJ_6000] | | 1 |
| 12 | SP1582902-0609 | Kit swing away frames, SD425/450mm, Anthracite (pair) [SJ_0200+SJ_6010] | | 1 |
| 13 | SP1582902-0643 | Kit swing away frames, SD425/450mm, Pop Blue (pair) [SJ_0200+SJ_6020] | | 1 |
| 14 | SP1582902-0607 | Kit swing away frames, SD425/450mm, Dark Red (pair) [SJ_0200+SJ_6030] | | 1 |
| 15 | SP1582902-0644 | Kit swing away frames, SD425/450mm, Electric Green (pair) [SJ_0200+SJ_6040] | | 1 |
| 16 | SP1582902-0645 | Kit swing away frames, SD425/450mm, Hypnotic Purple (pair) [SJ_0200+SJ_6050] | | 1 |
| 17 | SP1582902-0515 | Kit swing away frames, SD425/450mm, Pearl Grey (pair) [SJ_0200+SJ_6060] | | 1 |
| 18 | SP1582902-0646 | Kit swing away frames, SD425/450mm, Matt Black (pair) [SJ_0200+SJ_6070] | | 1 |

Swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 19 | SP1582902-0647 | Kit swing away frames, SD425/450mm, Intense Yellow (pair) <i>[SJ_0200+SJ_6080]</i> | | 1 |
| 20 | SP1582902-0648 | Kit swing away frames, SD425/450mm, Vibrating Orange (pair) <i>[SJ_0200+SJ_6090]</i> | | 1 |
| 21 | SP1582903-0637 | Kit swing away frames, SD475/500mm, Pearl White (pair) <i>[SJ_0200+SJ_6000]</i> | | 1 |
| 22 | SP1582903-0609 | Kit swing away frames, SD475/500mm, Anthracite (pair) <i>[SJ_0200+SJ_6010]</i> | | 1 |
| 23 | SP1582903-0643 | Kit swing away frames, SD475/500mm, Pop Blue (pair) <i>[SJ_0200+SJ_6020]</i> | | 1 |
| 24 | SP1582903-0607 | Kit swing away frames, SD475/500mm, Dark Red (pair) <i>[SJ_0200+SJ_6030]</i> | | 1 |
| 25 | SP1582903-0644 | Kit swing away frames, SD475/500mm, Electric Green (pair) <i>[SJ_0200+SJ_6040]</i> | | 1 |
| 26 | SP1582903-0645 | Kit swing away frames, SD475/500mm, Hypnotic Purple (pair) <i>[SJ_0200+SJ_6050]</i> | | 1 |
| 27 | SP1582903-0515 | Kit swing away frames, SD475/500mm, Pearl Grey (pair) <i>[SJ_0200+SJ_6060]</i> | | 1 |
| 28 | SP1582903-0646 | Kit swing away frames, SD475/500mm, Matt Black (pair) <i>[SJ_0200+SJ_6070]</i> | | 1 |
| 29 | SP1582903-0647 | Kit swing away frames, SD475/500mm, Intense Yellow (pair) <i>[SJ_0200+SJ_6080]</i> | | 1 |
| 30 | SP1582903-0648 | Kit swing away frames, SD475/500mm, Vibrating Orange (pair) <i>[SJ_0200+SJ_6090]</i> | | 1 |
| 31 | SP1590794 | Kit end cap (pair) | | 2 |
| 32 | SP1589683 | Kit rear cap reflector for frame | | 2 |
| 33 | SP1589684 | Kit frame plastic protector | | 2 |
| 34 | SP1589692 | Kit front cap for frame | | 2 |
| 35 | SP1584051 | Sticker Action5 | | 2 |

Fixed front end 80 degree angle frame

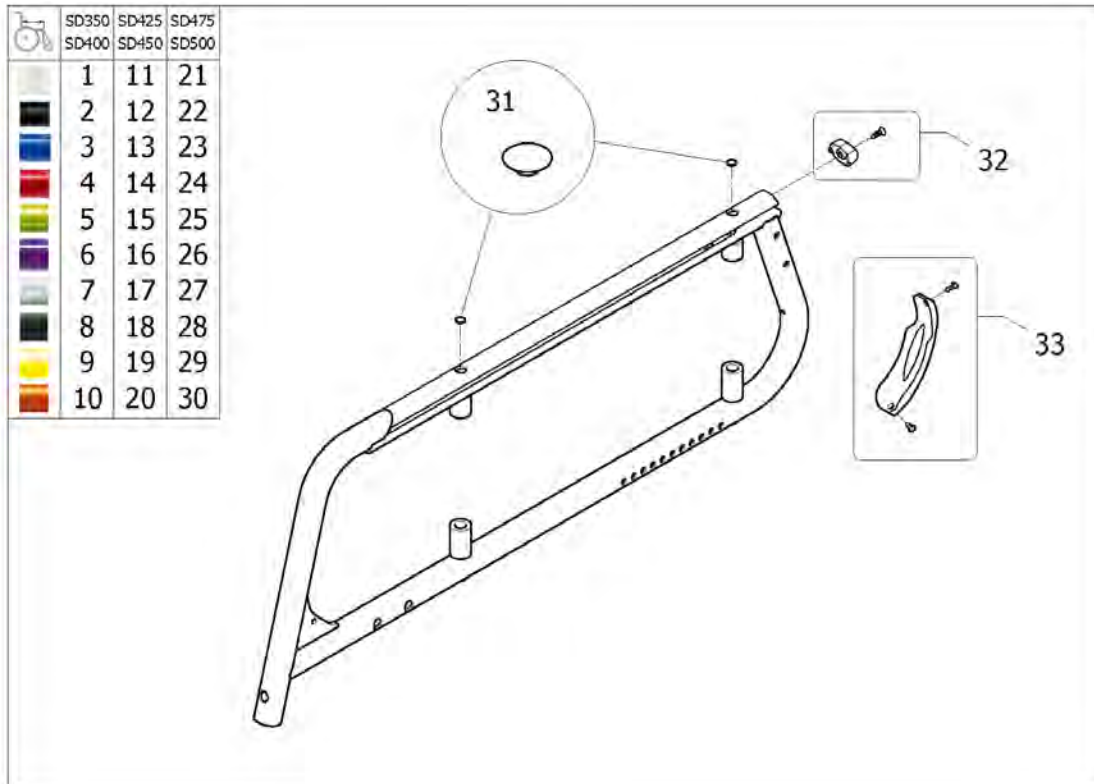


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 1 | SP1582904-0637 | Kit fixed frames, SD350/375/400, Pearl White (pair) [SJ_0210+SJ_6000] | | 1 |
| 2 | SP1582904-0609 | Kit fixed frames, SD350/375/400, Anthracite (pair) [SJ_0210+SJ_6010] | | 1 |
| 3 | SP1582904-0643 | Kit fixed frames, SD350/375/400, Pop Blue (pair) [SJ_0210+SJ_6020] | | 1 |
| 4 | SP1582904-0607 | Kit fixed frames, SD350/375/400, Dark Red (pair) [SJ_0210+SJ_6030] | | 1 |
| 5 | SP1582904-0644 | Kit fixed frames, SD350/375/400, Electric Green (pair) [SJ_0210+SJ_6040] | | 1 |
| 6 | SP1582904-0645 | Kit fixed frames, SD350/375/400, Hypnotic Purple (pair) [SJ_0210+SJ_6050] | | 1 |
| 7 | SP1582904-0515 | Kit fixed frames, SD350/375/400, Pearl Grey (pair) [SJ_0210+SJ_6060] | | 1 |
| 8 | SP1582904-0646 | Kit fixed frames, SD350/375/400, Matt Black (pair) [SJ_0210+SJ_6070] | | 1 |
| 9 | SP1582904-0647 | Kit fixed frames, SD350/375/400, Intense Yellow (pair) [SJ_0210+SJ_6080] | | 1 |
| 10 | SP1582904-0648 | Kit fixed frames, SD350/375/400, Vibrating Orange (pair) [SJ_0210+SJ_6090] | | 1 |
| 11 | SP1582905-0637 | Kit fixed frames, SD425/450, Pearl White (pair) [SJ_0210+SJ_6000] | | 1 |
| 12 | SP1582905-0609 | Kit fixed frames, SD425/450, Anthracite (pair) [SJ_0210+SJ_6010] | | 1 |
| 13 | SP1582905-0643 | Kit fixed frames, SD425/450, Pop Blue (pair) [SJ_0210+SJ_6020] | | 1 |
| 14 | SP1582905-0607 | Kit fixed frames, SD425/450, Dark Red (pair) [SJ_0210+SJ_6030] | | 1 |
| 15 | SP1582905-0644 | Kit fixed frames, SD425/450, Electric Green (pair) [SJ_0210+SJ_6040] | | 1 |
| 16 | SP1582905-0645 | Kit fixed frames, SD425/450, Hypnotic Purple (pair) [SJ_0210+SJ_6050] | | 1 |
| 17 | SP1582905-0515 | Kit fixed frames, SD425/450, Pearl Grey (pair) [SJ_0210+SJ_6060] | | 1 |
| 18 | SP1582905-0646 | Kit fixed frames, SD425/450, Matt Black (pair) [SJ_0210+SJ_6070] | | 1 |

Fixed front end 80 degree angle frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 19 | SP1582905-0647 | Kit fixed frames, SD425/450, Intense Yellow (pair) <i>[SJ_0210+SJ_6080]</i> | | 1 |
| 20 | SP1582905-0648 | Kit fixed frames, SD425/450, Vibrating Orange (pair) <i>[SJ_0210+SJ_6090]</i> | | 1 |
| 21 | SP1582906-0637 | Kit fixed frames, SD475/500, Pearl White (pair) <i>[SJ_0210+SJ_6000]</i> | | 1 |
| 22 | SP1582906-0609 | Kit fixed frames, SD475/500, Anthracite (pair) <i>[SJ_0210+SJ_6010]</i> | | 1 |
| 23 | SP1582906-0643 | Kit fixed frames, SD475/500, Pop Blue (pair) <i>[SJ_0210+SJ_6020]</i> | | 1 |
| 24 | SP1582906-0607 | Kit fixed frames, SD475/500, Dark Red (pair) <i>[SJ_0210+SJ_6030]</i> | | 1 |
| 25 | SP1582906-0644 | Kit fixed frames, SD475/500, Electric Green (pair) <i>[SJ_0210+SJ_6040]</i> | | 1 |
| 26 | SP1582906-0645 | Kit fixed frames, SD475/500, Hypnotic Purple (pair) <i>[SJ_0210+SJ_6050]</i> | | 1 |
| 27 | SP1582906-0515 | Kit fixed frames, SD475/500, Pearl Grey (pair) <i>[SJ_0210+SJ_6060]</i> | | 1 |
| 28 | SP1582906-0646 | Kit fixed frames, SD475/500, Matt Black (pair) <i>[SJ_0210+SJ_6070]</i> | | 1 |
| 29 | SP1582906-0647 | Kit fixed frames, SD475/500, Intense Yellow (pair) <i>[SJ_0210+SJ_6080]</i> | | 1 |
| 30 | SP1582906-0648 | Kit fixed frames, SD475/500, Vibrating Orange (pair) <i>[SJ_0210+SJ_6090]</i> | | 1 |
| 31 | SP1590794 | Kit end cap (pair) | | 2 |
| 32 | SP1589683 | Kit rear cap reflector for frame | | 2 |
| 33 | SP1589684 | Kit frame plastic protector | | 2 |
| 34 | SP1584051 | Sticker Action5 | | 2 |


Folding system

| | SD350 SD400 | SD425 SD450 | SD475 SD500 |
|-------|----------------|----------------|----------------|
| SW305 | 1 | 11 | - |
| SW330 | 2 | 12 | - |
| SW355 | 3 | 13 | 21 |
| SW380 | 4 | 14 | 22 |
| SW405 | 5 | 15 | 23 |
| SW430 | 6 | 16 | 24 |
| SW455 | 7 | 17 | 25 |
| SW480 | 8 | 18 | 26 |
| SW505 | 9 | 19 | 27 |
| SW530 | 10 | 20 | 28 |

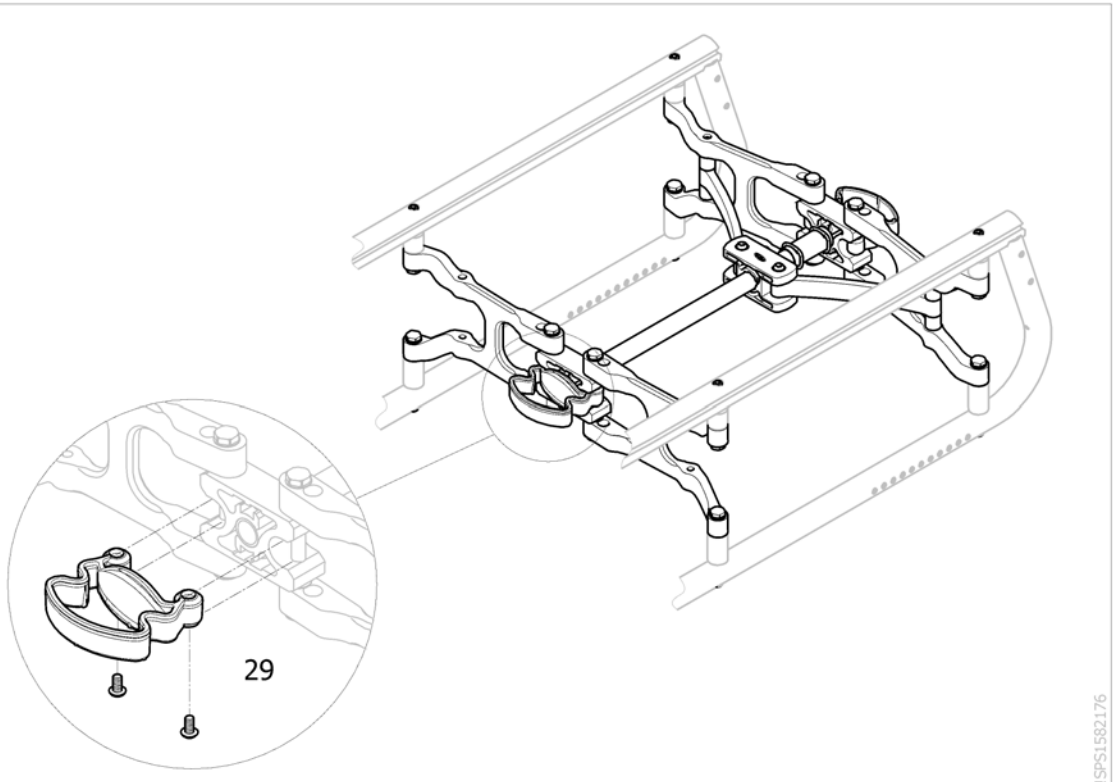

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1583191 | Folding system, SW305 mm x SD350/375/400 mm | | 1 |
| 2 | SP1583193 | Folding system, SW330 mm x SD350/375/400 mm | | 1 |
| 3 | SP1583195 | Folding system, SW355 mm x SD350/375/400 mm | | 1 |
| 4 | SP1583198 | Folding system, SW380 mm x SD350/375/400 mm | | 1 |
| 5 | SP1583253 | Folding system, SW405 mm x SD350/375/400 mm | | 1 |
| 6 | SP1583262 | Folding system, SW430 mm x SD350/375/400 mm | | 1 |
| 7 | SP1583265 | Folding system, SW455 mm x SD350/375/400 mm | | 1 |
| 8 | SP1583268 | Folding system, SW480 mm x SD350/375/400 mm | | 1 |
| 9 | SP1589975 | Folding system, SW505 mm x SD400 mm | | 1 |
| 10 | SP1589976 | Folding system, SW530 mm x SD400 mm | | 1 |
| 11 | SP1583192 | Folding system, SW305 mm x SD425/450 mm | | 1 |
| 12 | SP1583194 | Folding system, SW330 mm x SD425/450 mm | | 1 |
| 13 | SP1583196 | Folding system, SW355 mm x SD425/450 mm | | 1 |
| 14 | SP1583199 | Folding system, SW380 mm x SD425/450 mm | | 1 |
| 15 | SP1583260 | Folding System, SW405 mm x SD425/450 mm | | 1 |
| 16 | SP1583263 | Folding system, SW430 mm x SD425/450 mm | | 1 |
| 17 | SP1583266 | Folding system, SW455 mm x SD425/450 mm | | 1 |
| 18 | SP1583269 | Folding system, SW425 mm x SD425/450 mm | | 1 |
| 19 | SP1583271 | Folding system, SW505 mm x SD425/450 mm | | 1 |
| 20 | SP1583313 | Folding system, SW530 mm x SD425/450 mm | | 1 |
| 21 | SP1583197 | Folding system, SW355 mm x SD475/500 mm | | 1 |
| 22 | SP1583252 | Folding system, SW380 mm x SD475/500 mm | | 1 |
| 23 | SP1583261 | Folding system, SW475 mm x SD475/500 mm | | 1 |
| 24 | SP1583264 | Folding system, SW430 mm x SD475/500 mm | | 1 |
| 25 | SP1583267 | Folding system, SW455 mm x SD475/500 mm | | 1 |
| 26 | SP1583270 | Folding system, SW480 mm x SD475/500 mm | | 1 |
| 27 | SP1583272 | Folding system, SW505 mm x SD475/500 mm | | 1 |
| 28 | SP1583314 | Folding system, SW530 mm x SD475/500 mm | | 1 |
| 29 | SP1589803 | Kit handle + screw for folding system | | 1 |

Swing Away Chassis

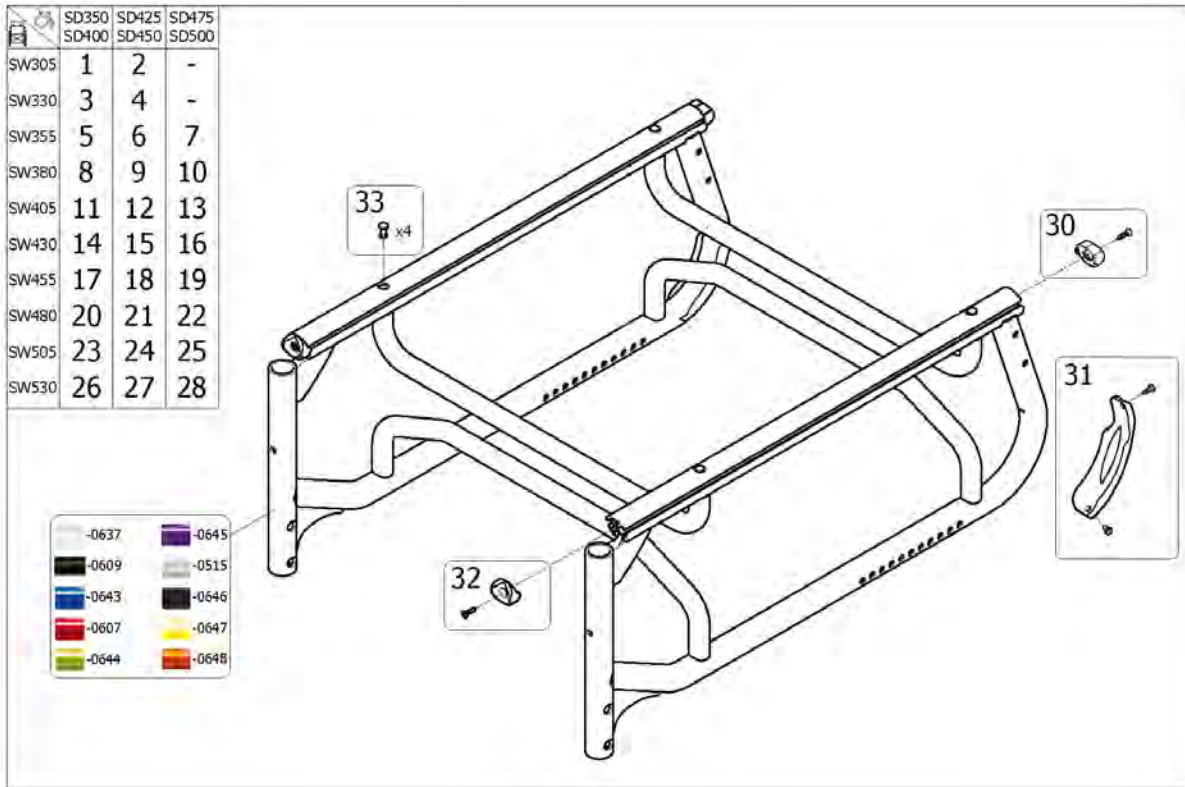

29
Action 5

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 1 | SP1651617-0515 | Swing away chassis SW305 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 1 | SP1651617-0607 | Swing away chassis SW305 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 1 | SP1651617-0609 | Swing away chassis SW305 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 1 | SP1651617-0637 | Swing away chassis SW305 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 1 | SP1651617-0643 | Swing away chassis SW305 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 1 | SP1651617-0644 | Swing away chassis SW305 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 1 | SP1651617-0645 | Swing away chassis SW305 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 1 | SP1651617-0646 | Swing away chassis SW305 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 1 | SP1651617-0647 | Swing away chassis SW305 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 1 | SP1651617-0648 | Swing away chassis SW305 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 2 | SP1651618-0515 | Swing away chassis SW305 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 2 | SP1651618-0607 | Swing away chassis SW305 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 2 | SP1651618-0609 | Swing away chassis SW305 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 2 | SP1651618-0637 | Swing away chassis SW305 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 2 | SP1651618-0643 | Swing away chassis SW305 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 2 | SP1651618-0644 | Swing away chassis SW305 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 2 | SP1651618-0645 | Swing away chassis SW305 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 2 | SP1651618-0646 | Swing away chassis SW305 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 2 | SP1651618-0647 | Swing away chassis SW305 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 2 | SP1651618-0648 | Swing away chassis SW305 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 3 | SP1651619-0515 | Swing away chassis SW330 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 3 | SP1651619-0607 | Swing away chassis SW330 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 3 | SP1651619-0609 | Swing away chassis SW330 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 3 | SP1651619-0637 | Swing away chassis SW330 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 3 | SP1651619-0643 | Swing away chassis SW330 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 3 | SP1651619-0644 | Swing away chassis SW330 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 3 | SP1651619-0645 | Swing away chassis SW330 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 3 | SP1651619-0646 | Swing away chassis SW330 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 3 | SP1651619-0647 | Swing away chassis SW330 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 3 | SP1651619-0648 | Swing away chassis SW330 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 4 | SP1651620-0515 | Swing away chassis SW330 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 4 | SP1651620-0607 | Swing away chassis SW330 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 4 | SP1651620-0609 | Swing away chassis SW330 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 4 | SP1651620-0637 | Swing away chassis SW330 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 4 | SP1651620-0643 | Swing away chassis SW330 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 4 | SP1651620-0644 | Swing away chassis SW330 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 4 | SP1651620-0645 | Swing away chassis SW330 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 4 | SP1651620-0646 | Swing away chassis SW330 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 4 | SP1651620-0647 | Swing away chassis SW330 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 4 | SP1651620-0648 | Swing away chassis SW330 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 5 | SP1651621-0515 | Swing away chassis SW355 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 5 | SP1651621-0607 | Swing away chassis SW355 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 5 | SP1651621-0609 | Swing away chassis SW355 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 5 | SP1651621-0637 | Swing away chassis SW355 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 5 | SP1651621-0643 | Swing away chassis SW355 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 5 | SP1651621-0644 | Swing away chassis SW355 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 5 | SP1651621-0645 | Swing away chassis SW355 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 5 | SP1651621-0646 | Swing away chassis SW355 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 5 | SP1651621-0647 | Swing away chassis SW355 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 5 | SP1651621-0648 | Swing away chassis SW355 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |

Swing Away Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 6 | SP1651622-0515 | Swing away chassis SW355 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 6 | SP1651622-0607 | Swing away chassis SW355 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 6 | SP1651622-0609 | Swing away chassis SW355 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 6 | SP1651622-0637 | Swing away chassis SW355 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 6 | SP1651622-0643 | Swing away chassis SW355 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 6 | SP1651622-0644 | Swing away chassis SW355 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 6 | SP1651622-0645 | Swing away chassis SW355 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 6 | SP1651622-0646 | Swing away chassis SW355 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 6 | SP1651622-0647 | Swing away chassis SW355 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 6 | SP1651622-0648 | Swing away chassis SW355 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 7 | SP1651623-0515 | Swing away chassis SW355 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 7 | SP1651623-0607 | Swing away chassis SW355 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 7 | SP1651623-0609 | Swing away chassis SW355 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 7 | SP1651623-0637 | Swing away chassis SW355 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 7 | SP1651623-0643 | Swing away chassis SW355 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 7 | SP1651623-0644 | Swing away chassis SW355 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 7 | SP1651623-0645 | Swing away chassis SW355 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 7 | SP1651623-0646 | Swing away chassis SW355 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 7 | SP1651623-0647 | Swing away chassis SW355 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 7 | SP1651623-0648 | Swing away chassis SW355 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 8 | SP1651624-0515 | Swing away chassis SW380 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 8 | SP1651624-0607 | Swing away chassis SW380 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 8 | SP1651624-0609 | Swing away chassis SW380 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 8 | SP1651624-0637 | Swing away chassis SW380 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 8 | SP1651624-0643 | Swing away chassis SW380 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 8 | SP1651624-0644 | Swing away chassis SW380 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 8 | SP1651624-0645 | Swing away chassis SW380 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 8 | SP1651624-0646 | Swing away chassis SW380 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 8 | SP1651624-0647 | Swing away chassis SW380 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 8 | SP1651624-0648 | Swing away chassis SW380 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 9 | SP1651625-0515 | Swing away chassis SW380 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 9 | SP1651625-0607 | Swing away chassis SW380 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 9 | SP1651625-0609 | Swing away chassis SW380 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 9 | SP1651625-0637 | Swing away chassis SW380 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 9 | SP1651625-0643 | Swing away chassis SW380 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 9 | SP1651625-0644 | Swing away chassis SW380 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 9 | SP1651625-0645 | Swing away chassis SW380 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 9 | SP1651625-0646 | Swing away chassis SW380 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 9 | SP1651625-0647 | Swing away chassis SW380 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 9 | SP1651625-0648 | Swing away chassis SW380 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 10 | SP1651626-0515 | Swing away chassis SW380 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 10 | SP1651626-0607 | Swing away chassis SW380 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 10 | SP1651626-0609 | Swing away chassis SW380 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 10 | SP1651626-0637 | Swing away chassis SW380 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 10 | SP1651626-0643 | Swing away chassis SW380 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 10 | SP1651626-0644 | Swing away chassis SW380 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 10 | SP1651626-0645 | Swing away chassis SW380 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 10 | SP1651626-0646 | Swing away chassis SW380 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 10 | SP1651626-0647 | Swing away chassis SW380 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 10 | SP1651626-0648 | Swing away chassis SW380 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 11 | SP1651627-0515 | Swing away chassis SW405 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 11 | SP1651627-0607 | Swing away chassis SW405 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 11 | SP1651627-0609 | Swing away chassis SW405 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 11 | SP1651627-0637 | Swing away chassis SW405 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 11 | SP1651627-0643 | Swing away chassis SW405 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 11 | SP1651627-0644 | Swing away chassis SW405 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 11 | SP1651627-0645 | Swing away chassis SW405 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 11 | SP1651627-0646 | Swing away chassis SW405 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 11 | SP1651627-0647 | Swing away chassis SW405 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 11 | SP1651627-0648 | Swing away chassis SW405 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 12 | SP1651628-0515 | Swing away chassis SW405 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 12 | SP1651628-0607 | Swing away chassis SW405 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 12 | SP1651628-0609 | Swing away chassis SW405 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 12 | SP1651628-0637 | Swing away chassis SW405 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |

Swing Away Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 12 | SP1651628-0643 | Swing away chassis SW405 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 12 | SP1651628-0644 | Swing away chassis SW405 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 12 | SP1651628-0645 | Swing away chassis SW405 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 12 | SP1651628-0646 | Swing away chassis SW405 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 12 | SP1651628-0647 | Swing away chassis SW405 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 12 | SP1651628-0648 | Swing away chassis SW405 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 13 | SP1651629-0515 | Swing away chassis SW405 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 13 | SP1651629-0607 | Swing away chassis SW405 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 13 | SP1651629-0609 | Swing away chassis SW405 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 13 | SP1651629-0637 | Swing away chassis SW405 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 13 | SP1651629-0643 | Swing away chassis SW405 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 13 | SP1651629-0644 | Swing away chassis SW405 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 13 | SP1651629-0645 | Swing away chassis SW405 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 13 | SP1651629-0646 | Swing away chassis SW405 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 13 | SP1651629-0647 | Swing away chassis SW405 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 13 | SP1651629-0648 | Swing away chassis SW405 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 14 | SP1651630-0515 | Swing away chassis SW430 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 14 | SP1651630-0607 | Swing away chassis SW430 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 14 | SP1651630-0609 | Swing away chassis SW430 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 14 | SP1651630-0637 | Swing away chassis SW430 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 14 | SP1651630-0643 | Swing away chassis SW430 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 14 | SP1651630-0644 | Swing away chassis SW430 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 14 | SP1651630-0645 | Swing away chassis SW430 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 14 | SP1651630-0646 | Swing away chassis SW430 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 14 | SP1651630-0647 | Swing away chassis SW430 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 14 | SP1651630-0648 | Swing away chassis SW430 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 15 | SP1651631-0515 | Swing away chassis SW430 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 15 | SP1651631-0607 | Swing away chassis SW430 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 15 | SP1651631-0609 | Swing away chassis SW430 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 15 | SP1651631-0637 | Swing away chassis SW430 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 15 | SP1651631-0643 | Swing away chassis SW430 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 15 | SP1651631-0644 | Swing away chassis SW430 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 15 | SP1651631-0645 | Swing away chassis SW430 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 15 | SP1651631-0646 | Swing away chassis SW430 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 15 | SP1651631-0647 | Swing away chassis SW430 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 15 | SP1651631-0648 | Swing away chassis SW430 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 16 | SP1651632-0515 | Swing away chassis SW430 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 16 | SP1651632-0607 | Swing away chassis SW430 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 16 | SP1651632-0609 | Swing away chassis SW430 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 16 | SP1651632-0637 | Swing away chassis SW430 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 16 | SP1651632-0643 | Swing away chassis SW430 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 16 | SP1651632-0644 | Swing away chassis SW430 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 16 | SP1651632-0645 | Swing away chassis SW430 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 16 | SP1651632-0646 | Swing away chassis SW430 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 16 | SP1651632-0647 | Swing away chassis SW430 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 16 | SP1651632-0648 | Swing away chassis SW430 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 17 | SP1651633-0515 | Swing away chassis SW455 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 17 | SP1651633-0607 | Swing away chassis SW455 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 17 | SP1651633-0609 | Swing away chassis SW455 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 17 | SP1651633-0637 | Swing away chassis SW455 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 17 | SP1651633-0643 | Swing away chassis SW455 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 17 | SP1651633-0644 | Swing away chassis SW455 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 17 | SP1651633-0645 | Swing away chassis SW455 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 17 | SP1651633-0646 | Swing away chassis SW455 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 17 | SP1651633-0647 | Swing away chassis SW455 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 17 | SP1651633-0648 | Swing away chassis SW455 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 18 | SP1651634-0515 | Swing away chassis SW455 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 18 | SP1651634-0607 | Swing away chassis SW455 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 18 | SP1651634-0609 | Swing away chassis SW455 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 18 | SP1651634-0637 | Swing away chassis SW455 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 18 | SP1651634-0643 | Swing away chassis SW455 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 18 | SP1651634-0644 | Swing away chassis SW455 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 18 | SP1651634-0645 | Swing away chassis SW455 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 18 | SP1651634-0646 | Swing away chassis SW455 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |

Swing Away Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 18 | SP1651634-0647 | Swing away chassis SW455 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 18 | SP1651634-0648 | Swing away chassis SW455 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 19 | SP1651635-0515 | Swing away chassis SW455 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 19 | SP1651635-0607 | Swing away chassis SW455 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 19 | SP1651635-0609 | Swing away chassis SW455 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 19 | SP1651635-0637 | Swing away chassis SW455 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 19 | SP1651635-0643 | Swing away chassis SW455 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 19 | SP1651635-0644 | Swing away chassis SW455 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 19 | SP1651635-0645 | Swing away chassis SW455 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 19 | SP1651635-0646 | Swing away chassis SW455 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 19 | SP1651635-0647 | Swing away chassis SW455 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 19 | SP1651635-0648 | Swing away chassis SW455 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 20 | SP1651636-0515 | Swing away chassis SW480 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 20 | SP1651636-0607 | Swing away chassis SW480 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 20 | SP1651636-0609 | Swing away chassis SW480 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 20 | SP1651636-0637 | Swing away chassis SW480 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 20 | SP1651636-0643 | Swing away chassis SW480 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 20 | SP1651636-0644 | Swing away chassis SW480 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 20 | SP1651636-0645 | Swing away chassis SW480 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 20 | SP1651636-0646 | Swing away chassis SW480 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 20 | SP1651636-0647 | Swing away chassis SW480 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 20 | SP1651636-0648 | Swing away chassis SW480 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 21 | SP1651637-0515 | Swing away chassis SW480 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 21 | SP1651637-0607 | Swing away chassis SW480 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 21 | SP1651637-0609 | Swing away chassis SW480 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 21 | SP1651637-0637 | Swing away chassis SW480 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 21 | SP1651637-0643 | Swing away chassis SW480 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 21 | SP1651637-0644 | Swing away chassis SW480 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 21 | SP1651637-0645 | Swing away chassis SW480 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 21 | SP1651637-0646 | Swing away chassis SW480 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 21 | SP1651637-0647 | Swing away chassis SW480 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 21 | SP1651637-0648 | Swing away chassis SW480 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 22 | SP1651638-0515 | Swing away chassis SW480 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 22 | SP1651638-0607 | Swing away chassis SW480 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 22 | SP1651638-0609 | Swing away chassis SW480 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 22 | SP1651638-0637 | Swing away chassis SW480 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 22 | SP1651638-0643 | Swing away chassis SW480 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 22 | SP1651638-0644 | Swing away chassis SW480 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 22 | SP1651638-0645 | Swing away chassis SW480 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 22 | SP1651638-0646 | Swing away chassis SW480 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 22 | SP1651638-0647 | Swing away chassis SW480 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 22 | SP1651638-0648 | Swing away chassis SW480 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 23 | SP1651639-0515 | Swing away chassis SW505 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 23 | SP1651639-0607 | Swing away chassis SW505 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 23 | SP1651639-0609 | Swing away chassis SW505 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 23 | SP1651639-0637 | Swing away chassis SW505 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 23 | SP1651639-0643 | Swing away chassis SW505 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 23 | SP1651639-0644 | Swing away chassis SW505 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 23 | SP1651639-0645 | Swing away chassis SW505 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 23 | SP1651639-0646 | Swing away chassis SW505 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 23 | SP1651639-0647 | Swing away chassis SW505 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 23 | SP1651639-0648 | Swing away chassis SW505 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 24 | SP1651640-0515 | Swing away chassis SW505 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 24 | SP1651640-0607 | Swing away chassis SW505 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 24 | SP1651640-0609 | Swing away chassis SW505 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 24 | SP1651640-0637 | Swing away chassis SW505 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 24 | SP1651640-0643 | Swing away chassis SW505 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 24 | SP1651640-0644 | Swing away chassis SW505 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 24 | SP1651640-0645 | Swing away chassis SW505 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 24 | SP1651640-0646 | Swing away chassis SW505 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 24 | SP1651640-0647 | Swing away chassis SW505 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 24 | SP1651640-0648 | Swing away chassis SW505 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 25 | SP1651641-0515 | Swing away chassis SW505 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 25 | SP1651641-0607 | Swing away chassis SW505 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |


Swing Away Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 25 | SP1651641-0609 | Swing away chassis SW505 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 25 | SP1651641-0637 | Swing away chassis SW505 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 25 | SP1651641-0643 | Swing away chassis SW505 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 25 | SP1651641-0644 | Swing away chassis SW505 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 25 | SP1651641-0645 | Swing away chassis SW505 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 25 | SP1651641-0646 | Swing away chassis SW505 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 25 | SP1651641-0647 | Swing away chassis SW505 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 25 | SP1651641-0648 | Swing away chassis SW505 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 26 | SP1651642-0515 | Swing away chassis SW530 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 26 | SP1651642-0607 | Swing away chassis SW530 SD400 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 26 | SP1651642-0609 | Swing away chassis SW530 SD400 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 26 | SP1651642-0637 | Swing away chassis SW530 SD400 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 26 | SP1651642-0643 | Swing away chassis SW530 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 26 | SP1651642-0644 | Swing away chassis SW530 SD400 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 26 | SP1651642-0645 | Swing away chassis SW530 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 26 | SP1651642-0646 | Swing away chassis SW530 SD400 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 26 | SP1651642-0647 | Swing away chassis SW530 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 26 | SP1651642-0648 | Swing away chassis SW530 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 27 | SP1651675-0515 | Swing away chassis SW530 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 27 | SP1651675-0607 | Swing away chassis SW530 SD450 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 27 | SP1651675-0609 | Swing away chassis SW530 SD450 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 27 | SP1651675-0637 | Swing away chassis SW530 SD450 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |
| 27 | SP1651675-0643 | Swing away chassis SW530 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 27 | SP1651675-0644 | Swing away chassis SW530 SD450 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 27 | SP1651675-0645 | Swing away chassis SW530 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 27 | SP1651675-0646 | Swing away chassis SW530 SD450 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 27 | SP1651675-0647 | Swing away chassis SW530 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 27 | SP1651675-0648 | Swing away chassis SW530 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 28 | SP1651676-0515 | Swing away chassis SW530 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0200+SJA6060]</i> | | 1 |
| 28 | SP1651676-0607 | Swing away chassis SW530 SD500 mm Dark red <i>Delay + 12 days, [SJA0200+SJA6030]</i> | | 1 |
| 28 | SP1651676-0609 | Swing away chassis SW530 SD500 mm Anthracite <i>Delay + 12 days, [SJA0200+SJA6010]</i> | | 1 |
| 28 | SP1651676-0637 | Swing away chassis SW530 SD500 mm Pearl White <i>Delay + 12 days, [SJA0200+SJA6000]</i> | | 1 |

Invacare® Action 5 | Chassis
Swing Away Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 28 | SP1651676-0643 | Swing away chassis SW530 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0200+SJA6020]</i> | | 1 |
| 28 | SP1651676-0644 | Swing away chassis SW530 SD500 mm Electric Green <i>Delay + 12 days, [SJA0200+SJA6040]</i> | | 1 |
| 28 | SP1651676-0645 | Swing away chassis SW530 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0200+SJA6050]</i> | | 1 |
| 28 | SP1651676-0646 | Swing away chassis SW530 SD500 mm Matt Black <i>Delay + 12 days, [SJA0200+SJA6070]</i> | | 1 |
| 28 | SP1651676-0647 | Swing away chassis SW530 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0200+SJA6080]</i> | | 1 |
| 28 | SP1651676-0648 | Swing away chassis SW530 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0200+SJA6090]</i> | | 1 |
| 29 | SP1648788 | Sticker Action 5 Rigid | | 1 |
| 30 | SP1589683 | Kit rear cap reflector for frame | | 2 |
| 31 | SP1589684 | Kit frame plastic protector | | 2 |
| 32 | SP1589692 | Kit front cap for frame | | 2 |
| 33 | SP1652169 | Kit rivet clip | | 1 |

Fixed front end Chassis

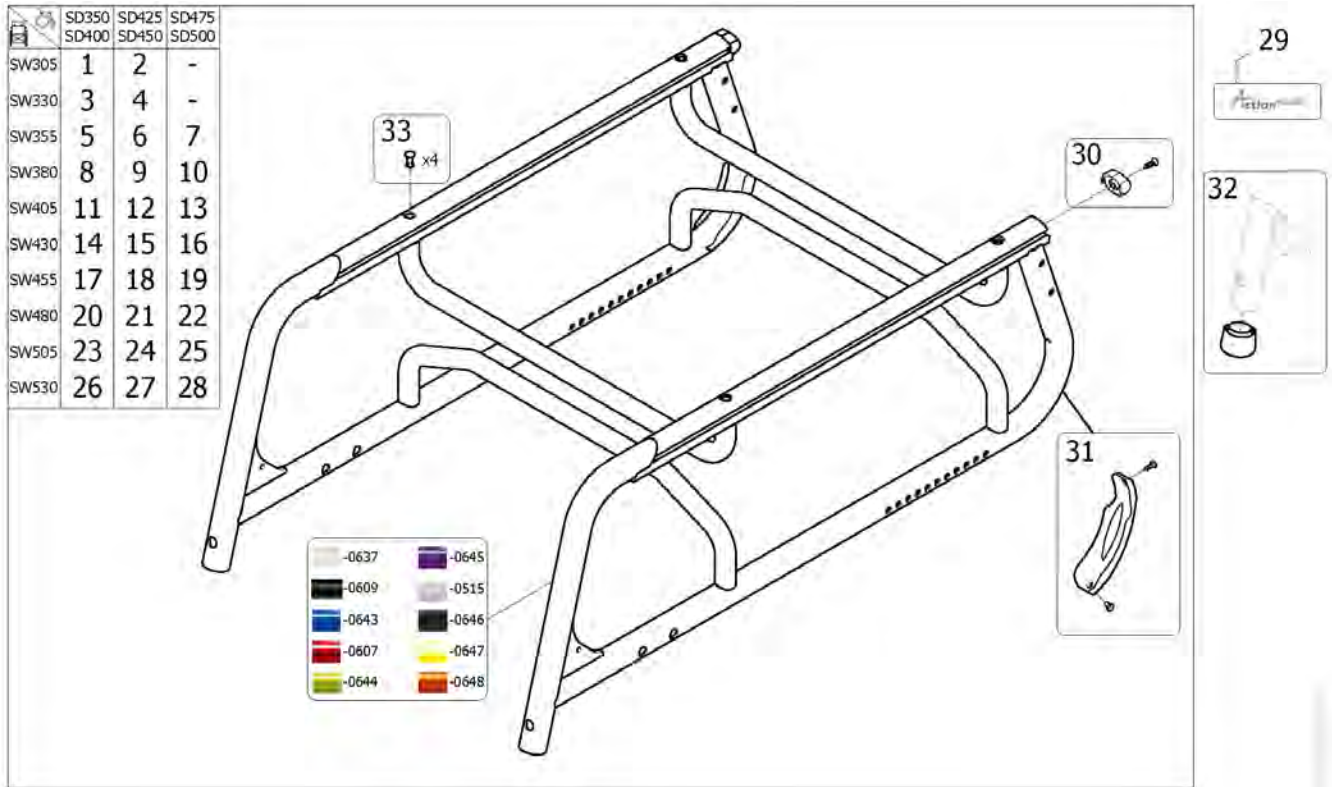

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1651677-0515 | Fixed chassis SW305 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 1 | SP1651677-0607 | Fixed chassis SW305 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 1 | SP1651677-0609 | Fixed chassis SW305 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 1 | SP1651677-0637 | Fixed chassis SW305 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 1 | SP1651677-0643 | Fixed chassis SW305 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 1 | SP1651677-0644 | Fixed chassis SW305 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 1 | SP1651677-0645 | Fixed chassis SW305 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 1 | SP1651677-0646 | Fixed chassis SW305 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 1 | SP1651677-0647 | Fixed chassis SW305 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 1 | SP1651677-0648 | Fixed chassis SW305 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 2 | SP1651678-0515 | Fixed chassis SW305 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 2 | SP1651678-0607 | Fixed chassis SW305 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 2 | SP1651678-0609 | Fixed chassis SW305 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 2 | SP1651678-0637 | Fixed chassis SW305 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 2 | SP1651678-0643 | Fixed chassis SW305 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 2 | SP1651678-0644 | Fixed chassis SW305 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 2 | SP1651678-0645 | Fixed chassis SW305 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 2 | SP1651678-0646 | Fixed chassis SW305 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 2 | SP1651678-0647 | Fixed chassis SW305 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 2 | SP1651678-0648 | Fixed chassis SW305 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 3 | SP1651679-0515 | Fixed chassis SW330 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 3 | SP1651679-0607 | Fixed chassis SW330 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 3 | SP1651679-0609 | Fixed chassis SW330 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 3 | SP1651679-0637 | Fixed chassis SW330 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 3 | SP1651679-0643 | Fixed chassis SW330 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 3 | SP1651679-0644 | Fixed chassis SW330 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 3 | SP1651679-0645 | Fixed chassis SW330 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 3 | SP1651679-0646 | Fixed chassis SW330 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 3 | SP1651679-0647 | Fixed chassis SW330 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 3 | SP1651679-0648 | Fixed chassis SW330 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 4 | SP1651680-0515 | Fixed chassis SW330 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 4 | SP1651680-0607 | Fixed chassis SW330 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 4 | SP1651680-0609 | Fixed chassis SW330 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 4 | SP1651680-0637 | Fixed chassis SW330 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 4 | SP1651680-0643 | Fixed chassis SW330 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 4 | SP1651680-0644 | Fixed chassis SW330 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 4 | SP1651680-0645 | Fixed chassis SW330 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 4 | SP1651680-0646 | Fixed chassis SW330 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 4 | SP1651680-0647 | Fixed chassis SW330 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 4 | SP1651680-0648 | Fixed chassis SW330 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 5 | SP1651681-0515 | Fixed chassis SW355 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 5 | SP1651681-0607 | Fixed chassis SW355 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 5 | SP1651681-0609 | Fixed chassis SW355 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 5 | SP1651681-0637 | Fixed chassis SW355 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 5 | SP1651681-0643 | Fixed chassis SW355 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 5 | SP1651681-0644 | Fixed chassis SW355 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 5 | SP1651681-0645 | Fixed chassis SW355 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 5 | SP1651681-0646 | Fixed chassis SW355 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 5 | SP1651681-0647 | Fixed chassis SW355 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 5 | SP1651681-0648 | Fixed chassis SW355 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |

Fixed front end Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 6 | SP1651682-0515 | Fixed chassis SW355 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 6 | SP1651682-0607 | Fixed chassis SW355 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 6 | SP1651682-0609 | Fixed chassis SW355 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 6 | SP1651682-0637 | Fixed chassis SW355 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 6 | SP1651682-0643 | Fixed chassis SW355 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 6 | SP1651682-0644 | Fixed chassis SW355 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 6 | SP1651682-0645 | Fixed chassis SW355 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 6 | SP1651682-0646 | Fixed chassis SW355 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 6 | SP1651682-0647 | Fixed chassis SW355 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 6 | SP1651682-0648 | Fixed chassis SW355 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 7 | SP1651683-0515 | Fixed chassis SW355 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 7 | SP1651683-0607 | Fixed chassis SW355 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 7 | SP1651683-0609 | Fixed chassis SW355 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 7 | SP1651683-0637 | Fixed chassis SW355 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 7 | SP1651683-0643 | Fixed chassis SW355 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 7 | SP1651683-0644 | Fixed chassis SW355 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 7 | SP1651683-0645 | Fixed chassis SW355 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 7 | SP1651683-0646 | Fixed chassis SW355 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 7 | SP1651683-0647 | Fixed chassis SW355 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 7 | SP1651683-0648 | Fixed chassis SW355 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 8 | SP1651684-0515 | Fixed chassis SW380 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 8 | SP1651684-0607 | Fixed chassis SW380 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 8 | SP1651684-0609 | Fixed chassis SW380 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 8 | SP1651684-0637 | Fixed chassis SW380 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 8 | SP1651684-0643 | Fixed chassis SW380 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 8 | SP1651684-0644 | Fixed chassis SW380 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 8 | SP1651684-0645 | Fixed chassis SW380 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 8 | SP1651684-0646 | Fixed chassis SW380 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 8 | SP1651684-0647 | Fixed chassis SW380 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 8 | SP1651684-0648 | Fixed chassis SW380 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 9 | SP1651685-0515 | Fixed chassis SW380 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 9 | SP1651685-0607 | Fixed chassis SW380 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 9 | SP1651685-0609 | Fixed chassis SW380 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 9 | SP1651685-0637 | Fixed chassis SW380 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 9 | SP1651685-0643 | Fixed chassis SW380 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 9 | SP1651685-0644 | Fixed chassis SW380 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 9 | SP1651685-0645 | Fixed chassis SW380 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 9 | SP1651685-0646 | Fixed chassis SW380 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 9 | SP1651685-0647 | Fixed chassis SW380 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 9 | SP1651685-0648 | Fixed chassis SW380 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 10 | SP1651686-0515 | Fixed chassis SW380 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 10 | SP1651686-0607 | Fixed chassis SW380 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 10 | SP1651686-0609 | Fixed chassis SW380 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 10 | SP1651686-0637 | Fixed chassis SW380 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 10 | SP1651686-0643 | Fixed chassis SW380 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 10 | SP1651686-0644 | Fixed chassis SW380 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 10 | SP1651686-0645 | Fixed chassis SW380 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 10 | SP1651686-0646 | Fixed chassis SW380 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 10 | SP1651686-0647 | Fixed chassis SW380 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 10 | SP1651686-0648 | Fixed chassis SW380 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 11 | SP1651687-0515 | Fixed chassis SW405 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 11 | SP1651687-0607 | Fixed chassis SW405 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 11 | SP1651687-0609 | Fixed chassis SW405 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 11 | SP1651687-0637 | Fixed chassis SW405 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 11 | SP1651687-0643 | Fixed chassis SW405 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 11 | SP1651687-0644 | Fixed chassis SW405 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 11 | SP1651687-0645 | Fixed chassis SW405 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 11 | SP1651687-0646 | Fixed chassis SW405 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 11 | SP1651687-0647 | Fixed chassis SW405 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 11 | SP1651687-0648 | Fixed chassis SW405 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 12 | SP1651688-0515 | Fixed chassis SW405 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 12 | SP1651688-0607 | Fixed chassis SW405 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 12 | SP1651688-0609 | Fixed chassis SW405 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 12 | SP1651688-0637 | Fixed chassis SW405 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |

Fixed front end Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 12 | SP1651688-0643 | Fixed chassis SW405 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 12 | SP1651688-0644 | Fixed chassis SW405 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 12 | SP1651688-0645 | Fixed chassis SW405 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 12 | SP1651688-0646 | Fixed chassis SW405 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 12 | SP1651688-0647 | Fixed chassis SW405 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 12 | SP1651688-0648 | Fixed chassis SW405 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 13 | SP1651689-0515 | Fixed chassis SW405 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 13 | SP1651689-0607 | Fixed chassis SW405 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 13 | SP1651689-0609 | Fixed chassis SW405 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 13 | SP1651689-0637 | Fixed chassis SW405 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 13 | SP1651689-0643 | Fixed chassis SW405 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 13 | SP1651689-0644 | Fixed chassis SW405 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 13 | SP1651689-0645 | Fixed chassis SW405 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 13 | SP1651689-0646 | Fixed chassis SW405 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 13 | SP1651689-0647 | Fixed chassis SW405 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 13 | SP1651689-0648 | Fixed chassis SW405 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 14 | SP1651700-0515 | Fixed chassis SW430 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 14 | SP1651700-0607 | Fixed chassis SW430 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 14 | SP1651700-0609 | Fixed chassis SW430 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 14 | SP1651700-0637 | Fixed chassis SW430 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 14 | SP1651700-0643 | Fixed chassis SW430 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 14 | SP1651700-0644 | Fixed chassis SW430 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 14 | SP1651700-0645 | Fixed chassis SW430 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 14 | SP1651700-0646 | Fixed chassis SW430 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 14 | SP1651700-0647 | Fixed chassis SW430 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 14 | SP1651700-0648 | Fixed chassis SW430 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 15 | SP1651701-0515 | Fixed chassis SW430 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 15 | SP1651701-0607 | Fixed chassis SW430 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 15 | SP1651701-0609 | Fixed chassis SW430 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 15 | SP1651701-0637 | Fixed chassis SW430 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 15 | SP1651701-0643 | Fixed chassis SW430 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 15 | SP1651701-0644 | Fixed chassis SW430 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 15 | SP1651701-0645 | Fixed chassis SW430 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 15 | SP1651701-0646 | Fixed chassis SW430 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 15 | SP1651701-0647 | Fixed chassis SW430 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 15 | SP1651701-0648 | Fixed chassis SW430 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 16 | SP1651750-0515 | Fixed chassis SW430 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 16 | SP1651750-0607 | Fixed chassis SW430 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 16 | SP1651750-0609 | Fixed chassis SW430 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 16 | SP1651750-0637 | Fixed chassis SW430 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 16 | SP1651750-0643 | Fixed chassis SW430 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 16 | SP1651750-0644 | Fixed chassis SW430 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 16 | SP1651750-0645 | Fixed chassis SW430 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 16 | SP1651750-0646 | Fixed chassis SW430 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 16 | SP1651750-0647 | Fixed chassis SW430 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 16 | SP1651750-0648 | Fixed chassis SW430 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 17 | SP1651751-0515 | Fixed chassis SW455 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 17 | SP1651751-0607 | Fixed chassis SW455 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 17 | SP1651751-0609 | Fixed chassis SW455 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 17 | SP1651751-0637 | Fixed chassis SW455 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 17 | SP1651751-0643 | Fixed chassis SW455 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 17 | SP1651751-0644 | Fixed chassis SW455 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 17 | SP1651751-0645 | Fixed chassis SW455 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 17 | SP1651751-0646 | Fixed chassis SW455 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 17 | SP1651751-0647 | Fixed chassis SW455 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 17 | SP1651751-0648 | Fixed chassis SW455 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 18 | SP1651752-0515 | Fixed chassis SW455 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 18 | SP1651752-0607 | Fixed chassis SW455 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 18 | SP1651752-0609 | Fixed chassis SW455 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 18 | SP1651752-0637 | Fixed chassis SW455 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 18 | SP1651752-0643 | Fixed chassis SW455 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 18 | SP1651752-0644 | Fixed chassis SW455 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 18 | SP1651752-0645 | Fixed chassis SW455 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 18 | SP1651752-0646 | Fixed chassis SW455 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |

Fixed front end Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 18 | SP1651752-0647 | Fixed chassis SW455 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 18 | SP1651752-0648 | Fixed chassis SW455 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 19 | SP1651753-0515 | Fixed chassis SW455 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 19 | SP1651753-0607 | Fixed chassis SW455 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 19 | SP1651753-0609 | Fixed chassis SW455 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 19 | SP1651753-0637 | Fixed chassis SW455 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 19 | SP1651753-0643 | Fixed chassis SW455 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 19 | SP1651753-0644 | Fixed chassis SW455 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 19 | SP1651753-0645 | Fixed chassis SW455 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 19 | SP1651753-0646 | Fixed chassis SW455 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 19 | SP1651753-0647 | Fixed chassis SW455 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 19 | SP1651753-0648 | Fixed chassis SW455 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 20 | SP1651764-0515 | Fixed chassis SW480 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 20 | SP1651764-0607 | Fixed chassis SW480 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 20 | SP1651764-0609 | Fixed chassis SW480 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 20 | SP1651764-0637 | Fixed chassis SW480 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 20 | SP1651764-0643 | Fixed chassis SW480 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 20 | SP1651764-0644 | Fixed chassis SW480 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 20 | SP1651764-0645 | Fixed chassis SW480 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 20 | SP1651764-0646 | Fixed chassis SW480 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 20 | SP1651764-0647 | Fixed chassis SW480 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 20 | SP1651764-0648 | Fixed chassis SW480 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 21 | SP1651765-0515 | Fixed chassis SW480 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 21 | SP1651765-0607 | Fixed chassis SW480 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 21 | SP1651765-0609 | Fixed chassis SW480 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 21 | SP1651765-0637 | Fixed chassis SW480 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 21 | SP1651765-0643 | Fixed chassis SW480 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 21 | SP1651765-0644 | Fixed chassis SW480 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 21 | SP1651765-0645 | Fixed chassis SW480 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 21 | SP1651765-0646 | Fixed chassis SW480 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 21 | SP1651765-0647 | Fixed chassis SW480 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 21 | SP1651765-0648 | Fixed chassis SW480 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 22 | SP1651766-0515 | Fixed chassis SW480 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 22 | SP1651766-0607 | Fixed chassis SW480 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 22 | SP1651766-0609 | Fixed chassis SW480 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 22 | SP1651766-0637 | Fixed chassis SW480 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 22 | SP1651766-0643 | Fixed chassis SW480 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 22 | SP1651766-0644 | Fixed chassis SW480 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 22 | SP1651766-0645 | Fixed chassis SW480 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 22 | SP1651766-0646 | Fixed chassis SW480 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 22 | SP1651766-0647 | Fixed chassis SW480 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 22 | SP1651766-0648 | Fixed chassis SW480 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 23 | SP1651767-0515 | Fixed chassis SW505 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 23 | SP1651767-0607 | Fixed chassis SW505 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 23 | SP1651767-0609 | Fixed chassis SW505 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 23 | SP1651767-0637 | Fixed chassis SW505 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 23 | SP1651767-0643 | Fixed chassis SW505 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 23 | SP1651767-0644 | Fixed chassis SW505 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 23 | SP1651767-0645 | Fixed chassis SW505 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 23 | SP1651767-0646 | Fixed chassis SW505 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 23 | SP1651767-0647 | Fixed chassis SW505 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 23 | SP1651767-0648 | Fixed chassis SW505 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 24 | SP1651768-0515 | Fixed chassis SW505 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 24 | SP1651768-0607 | Fixed chassis SW505 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 24 | SP1651768-0609 | Fixed chassis SW505 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 24 | SP1651768-0637 | Fixed chassis SW505 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 24 | SP1651768-0643 | Fixed chassis SW505 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 24 | SP1651768-0644 | Fixed chassis SW505 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 24 | SP1651768-0645 | Fixed chassis SW505 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 24 | SP1651768-0646 | Fixed chassis SW505 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 24 | SP1651768-0647 | Fixed chassis SW505 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 24 | SP1651768-0648 | Fixed chassis SW505 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 25 | SP1651769-0515 | Fixed chassis SW505 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 25 | SP1651769-0607 | Fixed chassis SW505 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |




Fixed front end Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 25 | SP1651769-0609 | Fixed chassis SW505 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 25 | SP1651769-0637 | Fixed chassis SW505 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 25 | SP1651769-0643 | Fixed chassis SW505 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 25 | SP1651769-0644 | Fixed chassis SW505 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 25 | SP1651769-0645 | Fixed chassis SW505 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 25 | SP1651769-0646 | Fixed chassis SW505 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 25 | SP1651769-0647 | Fixed chassis SW505 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 25 | SP1651769-0648 | Fixed chassis SW505 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 26 | SP1651770-0515 | Fixed chassis SW530 SD400 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 26 | SP1651770-0607 | Fixed chassis SW530 SD400 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 26 | SP1651770-0609 | Fixed chassis SW530 SD400 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 26 | SP1651770-0637 | Fixed chassis SW530 SD400 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 26 | SP1651770-0643 | Fixed chassis SW530 SD400 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 26 | SP1651770-0644 | Fixed chassis SW530 SD400 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 26 | SP1651770-0645 | Fixed chassis SW530 SD400 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 26 | SP1651770-0646 | Fixed chassis SW530 SD400 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 26 | SP1651770-0647 | Fixed chassis SW530 SD400 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 26 | SP1651770-0648 | Fixed chassis SW530 SD400 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 27 | SP1651814-0515 | Fixed chassis SW530 SD450 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 27 | SP1651814-0607 | Fixed chassis SW530 SD450 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 27 | SP1651814-0609 | Fixed chassis SW530 SD450 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 27 | SP1651814-0637 | Fixed chassis SW530 SD450 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |
| 27 | SP1651814-0643 | Fixed chassis SW530 SD450 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 27 | SP1651814-0644 | Fixed chassis SW530 SD450 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 27 | SP1651814-0645 | Fixed chassis SW530 SD450 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 27 | SP1651814-0646 | Fixed chassis SW530 SD450 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 27 | SP1651814-0647 | Fixed chassis SW530 SD450 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 27 | SP1651814-0648 | Fixed chassis SW530 SD450 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 28 | SP1651828-0515 | Fixed chassis SW530 SD500 mm Pearl Grey <i>Delay + 12 days, [SJA0210+SJA6060]</i> | | 1 |
| 28 | SP1651828-0607 | Fixed chassis SW530 SD500 mm Dark red <i>Delay + 12 days, [SJA0210+SJA6030]</i> | | 1 |
| 28 | SP1651828-0609 | Fixed chassis SW530 SD500 mm Anthracite <i>Delay + 12 days, [SJA0210+SJA6010]</i> | | 1 |
| 28 | SP1651828-0637 | Fixed chassis SW530 SD500 mm Pearl White <i>Delay + 12 days, [SJA0210+SJA6000]</i> | | 1 |

Invacare® Action 5 | Chassis
Fixed front end Chassis

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 28 | SP1651828-0643 | Fixed chassis SW530 SD500 mm Pop Blue <i>Delay + 12 days, [SJA0210+SJA6020]</i> | | 1 |
| 28 | SP1651828-0644 | Fixed chassis SW530 SD500 mm Electric Green <i>Delay + 12 days, [SJA0210+SJA6040]</i> | | 1 |
| 28 | SP1651828-0645 | Fixed chassis SW530 SD500 mm Hypnotic Purple <i>Delay + 12 days, [SJA0210+SJA6050]</i> | | 1 |
| 28 | SP1651828-0646 | Fixed chassis SW530 SD500 mm Matt Black <i>Delay + 12 days, [SJA0210+SJA6070]</i> | | 1 |
| 28 | SP1651828-0647 | Fixed chassis SW530 SD500 mm Intense Yellow <i>Delay + 12 days, [SJA0210+SJA6080]</i> | | 1 |
| 28 | SP1651828-0648 | Fixed chassis SW530 SD500 mm Vibrating Orange <i>Delay + 12 days, [SJA0210+SJA6090]</i> | | 1 |
| 29 | SP1648788 | Sticker Action 5 Rigid | | 1 |
| 30 | SP1589683 | Kit rear cap reflector for frame | | 2 |
| 31 | SP1589684 | Kit frame plastic protector | | 2 |
| 32 | 1026124 | Foot Rest Support Tip | | 2 |
| 33 | SP1652169 | Kit rivet clip | | 1 |

Seat

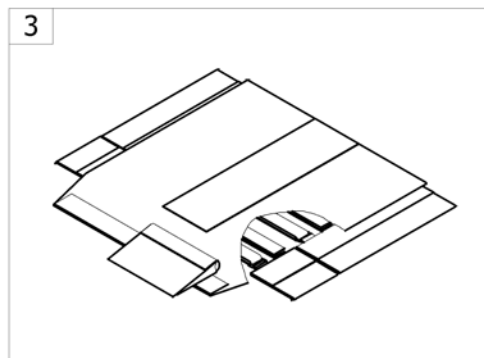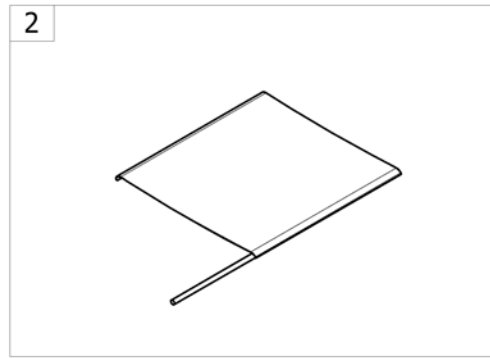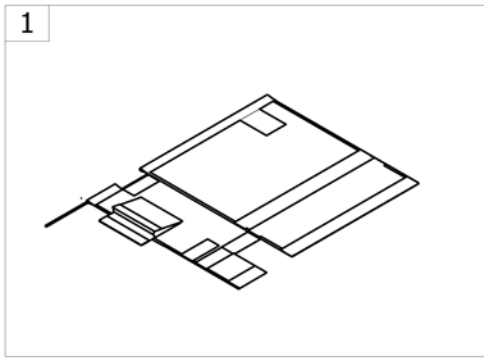


-<SPCACTIONS20

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | - | Seat upholstery, depth adjustable <i>[SJ_1500]</i> | | 1 |
| 2 | - | Seat upholstery, basic <i>[SJ_1510]</i> | | 1 |
| 3 | - | Seat upholstery, tension & depth adjustable <i>[SJ_1520]</i> | | 1 |

Seat upholstery, depth adjustable

| | SD350 SD400 | SD400 SD450 | SD450 SD500 |
|-------|----------------|----------------|----------------|
| SW305 | 1 | 2 | - |
| SW330 | 1 | 2 | - |
| SW355 | 3 | 4 | 5 |
| SW380 | 3 | 4 | 5 |
| SW405 | 6 | 7 | 8 |
| SW430 | 6 | 7 | 8 |
| SW455 | 9 | 10 | 11 |
| SW480 | 9 | 10 | 11 |
| SW505 | - | 12 | 13 |
| SW530 | - | 12 | 13 |

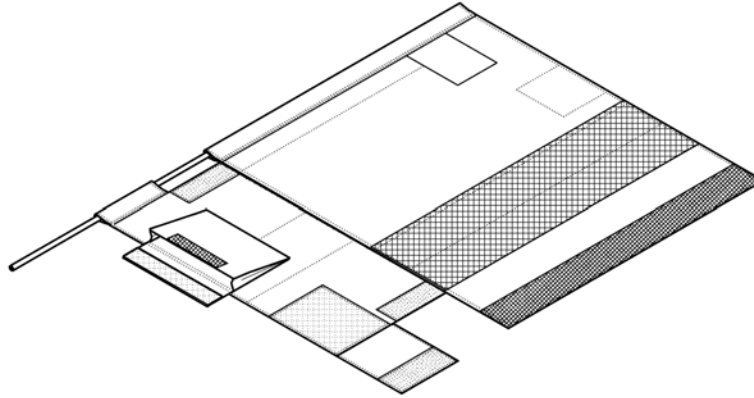

SPS1582178 ~ sps1577822


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1582406 | Seat upholstery depth adjustable, complete, SW305/330mm SD350/400mm <i>[SJ_1500]</i> | | 1 |
| 2 | SP1582407 | Seat upholstery depth adjustable, complete, SW305/330mm SD400/450mm <i>[SJ_1500]</i> | | 1 |
| 3 | SP1582408 | Seat upholstery depth adjustable, complete, SW355/380mm SD350/400mm <i>[SJ_1500]</i> | | 1 |
| 4 | SP1582433 | Seat upholstery depth adjustable, complete, SW355/380mm SD400/450mm <i>[SJ_1500]</i> | | 1 |
| 5 | SP1582434 | Seat upholstery depth adjustable, complete, SW355/380mm SD450/500mm <i>[SJ_1500]</i> | | 1 |
| 6 | SP1582435 | Seat upholstery depth adjustable, complete, SW405/430mm SD350/400mm <i>[SJ_1500]</i> | | 1 |
| 7 | SP1582589 | Seat upholstery depth adjustable, complete, SW405/430mm SD400/450mm <i>[SJ_1500]</i> | | 1 |
| 8 | SP1582590 | Seat upholstery depth adjustable, complete, SW405/430mm SD450/500mm <i>[SJ_1500]</i> | | 1 |
| 9 | SP1582591 | Seat upholstery depth adjustable, complete, SW455/480mm SD350/400mm <i>[SJ_1500]</i> | | 1 |
| 10 | SP1582592 | Seat upholstery depth adjustable, complete, SW455/480mm SD400/450mm <i>[SJ_1500]</i> | | 1 |
| 11 | SP1582593 | Seat upholstery depth adjustable, complete, SW455/480mm SD450/500mm <i>[SJ_1500]</i> | | 1 |
| 12 | SP1582594 | Seat upholstery depth adjustable, complete, SW505/530mm SD400/450mm <i>[SJ_1500]</i> | | 1 |

Seat upholstery, depth adjustable

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 13 | SP1582595 | Seat upholstery depth adjustable, complete, SW505/530mm SD450/500mm <i>[SJ_1500]</i> | | 1 |

Seat upholstery, basic

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | 1 | 2 | 3 | 4 | 5 | - | - |
| SW330 | 1 | 2 | 3 | 4 | 5 | - | - |
| SW355 | 6 | 7 | 8 | 9 | 10 | 11 | 12 |
| SW380 | 6 | 7 | 8 | 9 | 10 | 11 | 12 |
| SW405 | 6 | 7 | 8 | 9 | 10 | 11 | 12 |
| SW430 | 13 | 14 | 15 | 16 | 17 | 18 | 19 |
| SW455 | 13 | 14 | 15 | 16 | 17 | 18 | 19 |
| SW480 | 13 | 14 | 15 | 16 | 17 | 18 | 19 |
| SW505 | - | - | 20 | 21 | 22 | 23 | 24 |
| SW530 | - | - | 20 | 21 | 22 | 23 | 24 |

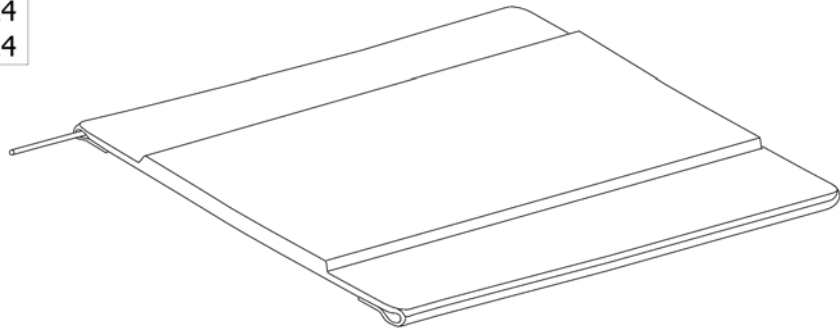

SPS1582177


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1582200 | Basic complete seat upholstery, SW305/330mm, SD350mm [SJ_1510] | | 1 |
| 2 | SP1582201 | Basic complete seat upholstery, SW305/330mm, SD375mm [SJ_1510] | | 1 |
| 3 | SP1582202 | Basic complete seat upholstery, SW305/330mm, SD400mm [SJ_1510] | | 1 |
| 4 | SP1582203 | Basic complete seat upholstery, SW305/330mm, SD425mm [SJ_1510] | | 1 |
| 5 | SP1582204 | Basic complete seat upholstery, SW305/330mm, SD450mm [SJ_1510] | | 1 |
| 6 | SP1582205 | Basic complete seat upholstery, SW355/405mm, SD350mm [SJ_1510] | | 1 |
| 7 | SP1582220 | Basic complete seat upholstery, SW355/405mm, SD375mm [SJ_1510] | | 1 |
| 8 | SP1582221 | Basic complete seat upholstery, SW355/405mm, SD400mm [SJ_1510] | | 1 |
| 9 | SP1582222 | Basic complete seat upholstery, SW355/405mm, SD425mm [SJ_1510] | | 1 |
| 10 | SP1582223 | Basic complete seat upholstery, SW355/405mm, SD450mm [SJ_1510] | | 1 |
| 11 | SP1582224 | Basic complete seat upholstery, SW355/405mm, SD475mm [SJ_1510] | | 1 |
| 12 | SP1582225 | Basic complete seat upholstery, SW355/405mm, SD500mm [SJ_1510] | | 1 |
| 13 | SP1582299 | Basic complete seat upholstery, SW430/480mm, SD355mm [SJ_1510] | | 1 |
| 14 | SP1582300 | Basic complete seat upholstery, SW430/480mm, SD375mm [SJ_1510] | | 1 |
| 15 | SP1582301 | Basic complete seat upholstery, SW430/480mm, SD400mm [SJ_1510] | | 1 |
| 16 | SP1582376 | Basic complete seat upholstery, SW430/480mm, SD425mm [SJ_1510] | | 1 |
| 17 | SP1582377 | Basic complete seat upholstery, SW430/480mm, SD450mm [SJ_1510] | | 1 |
| 18 | SP1582378 | Basic complete seat upholstery, SW430/480mm, SD475mm [SJ_1510] | | 1 |

Seat upholstery, basic

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 19 | SP1582379 | Basic complete seat upholstery, SW430/480mm, SD500mm <i>[SJ_1510]</i> | | 1 |
| 20 | SP1589983 | Basic complete seat Upholstery, SW505/530mm, SD400mm <i>[SJ_1510]</i> | | 1 |
| 21 | SP1582380 | Basic complete seat upholstery, SW505/530mm, SD425mm <i>[SJ_1510]</i> | | 1 |
| 22 | SP1582381 | Basic complete seat upholstery, SW505/530mm, SD450mm <i>[SJ_1510]</i> | | 1 |
| 23 | SP1582382 | Basic complete seat upholstery, SW505/530mm, SD475mm <i>[SJ_1510]</i> | | 1 |
| 24 | SP1582405 | Basic complete seat upholstery, SW505/530mm, SD500mm <i>[SJ_1510]</i> | | 1 |

Seat upholstery, tension & depth adjustable

| | SD350 SD400 | SD400 SD450 | SD450 SD500 |
|-------|----------------|----------------|----------------|
| SW305 | 1 | 2 | - |
| SW330 | 1 | 2 | - |
| SW355 | 3 | 4 | 5 |
| SW380 | 3 | 4 | 5 |
| SW405 | 6 | 7 | 8 |
| SW430 | 6 | 7 | 8 |
| SW455 | 9 | 10 | 11 |
| SW480 | 9 | 10 | 11 |
| SW505 | - | 12 | 13 |
| SW530 | - | 12 | 13 |

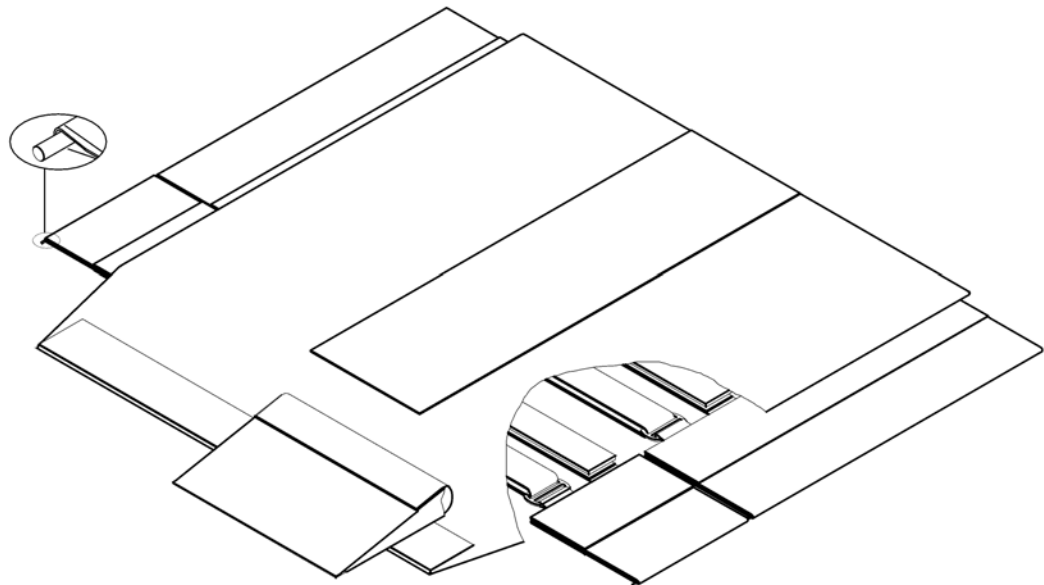





ISPS1582179

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1582596 | Seat upholstery tension adjustable, complete, SW305/330mm SD350/400mm <i>[SJ_1520]</i> | | 1 |
| 2 | SP1582597 | Seat upholstery tension adjustable, complete, SW305/330mm SD400/450mm <i>[SJ_1520]</i> | | 1 |
| 3 | SP1582598 | Seat upholstery tension adjustable, complete, SW355/380mm SD350/400mm <i>[SJ_1520]</i> | | 1 |
| 4 | SP1582599 | Seat upholstery tension adjustable, complete, SW355/380mm SD400/450mm <i>[SJ_1520]</i> | | 1 |
| 5 | SP1582600 | Seat upholstery tension adjustable, complete, SW355/380mm SD450/500mm <i>[SJ_1520]</i> | | 1 |
| 6 | SP1582601 | Seat upholstery tension adjustable, complete, SW405/430mm SD350/400mm <i>[SJ_1520]</i> | | 1 |
| 7 | SP1582773 | Seat upholstery tension adjustable, complete, SW405/430mm SD400/450mm <i>[SJ_1520]</i> | | 1 |
| 8 | SP1582774 | Seat upholstery tension adjustable, complete, SW405/430mm SD450/500mm <i>[SJ_1520]</i> | | 1 |
| 9 | SP1582775 | Seat upholstery tension adjustable, complete, SW455/480mm SD350/400mm <i>[SJ_1520]</i> | | 1 |
| 10 | SP1582776 | Seat upholstery tension adjustable, complete, SW455/480mm SD400/450mm <i>[SJ_1520]</i> | | 1 |
| 11 | SP1582777 | Seat upholstery tension adjustable, complete, SW455/480mm SD450/500mm <i>[SJ_1520]</i> | | 1 |
| 12 | SP1582778 | Seat upholstery tension adjustable, complete, SW505/530mm SD400/450mm <i>[SJ_1520]</i> | | 1 |

Seat upholstery, tension & depth adjustable

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 13 | SP1582779 | Seat upholstery tension adjustable, complete, SW505/530mm SD450/500mm <i>[SJ_1520]</i> | | 1 |






Backrest type

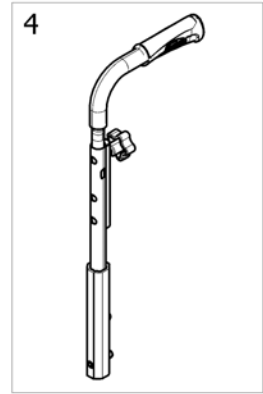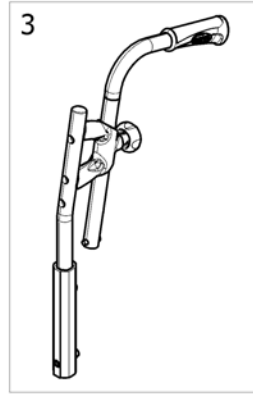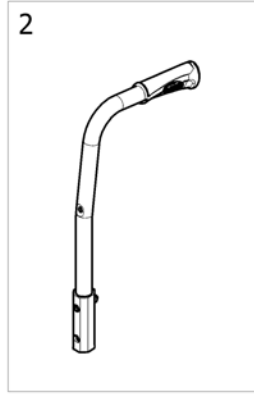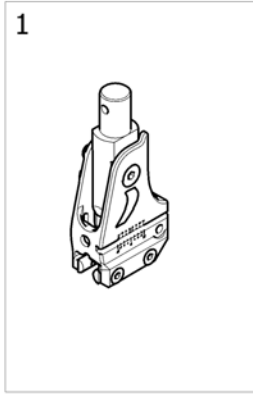

ISPS1601948 ~ <SP>ACTIONS

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Backrest attachment type <i>[SJ_2000/SJ_2010/SJ_2020]</i> | | 1 |
| 2 | - | Fixed backrest <i>[SJ_2130/SJ_2140/SJ_2143/SJ_2150/SJ_2153/SJ_2160/SJ_2163/SJ_2170/SJ_2173]</i> | | 1 |
| 3 | - | Backrest height adjustable 7°, without & rearsset pushhandle <i>[SJ_2300/SJ_2310/SJ_2400/SJ_2410]</i> | | 1 |
| 4 | - | Backrest height adjustable, Integrated pushhandle <i>[SJ_2300/SJ_2310/SJ_2420]</i> | | 1 |



Backrest attachment type

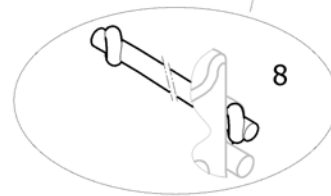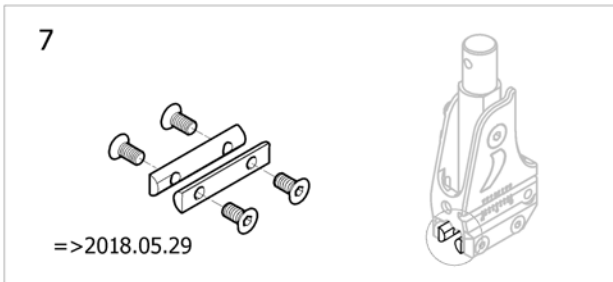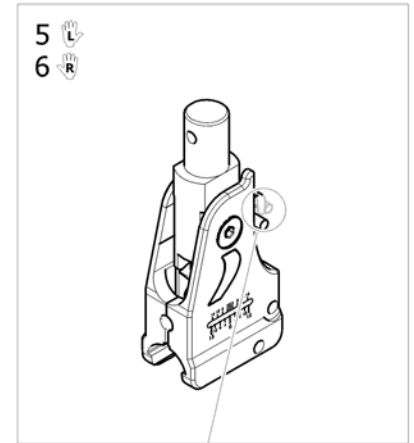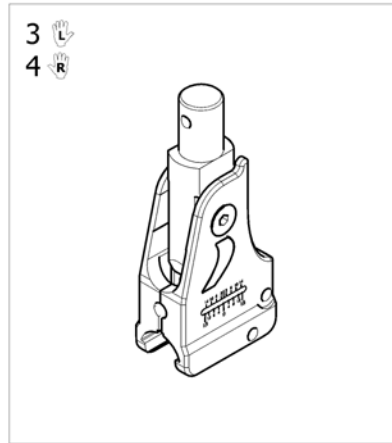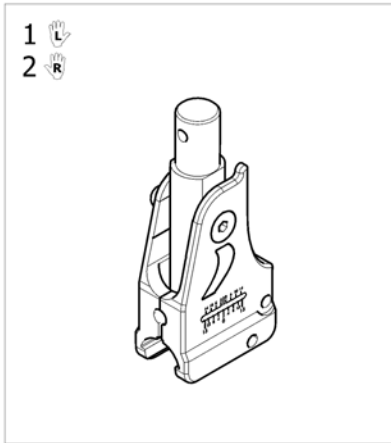

ISP51592182

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1645071 | Fixed angle backrest attachment Lh <i>[SJ_2000] ONLY FOR EUROPE - Replace SP1580830 Interchangeable</i> | 2018-05-30 → | 1 |
| 2 | SP1645072 | Fixed angle backrest attachment Rh <i>[SJ_2000] ONLY FOR EUROPE - Replace SP1580829 Interchangeable</i> | 2018-05-30 → | 1 |
| 3 | SP1645073 | Adjustable angle backrest attachment Lh <i>[SJ_2000] ONLY FOR EUROPE - Replace SP1580832 Interchangeable</i> | 2018-05-30 → | 1 |
| 4 | SP1645074 | Adjustable angle backrest attachment Rh <i>[SJ_2010] ONLY FOR EUROPE - Replace SP1580831 Interchangeable</i> | 2018-05-30 → | 1 |
| 5 | SP1645075 | Adjustable angle & fold down backrest attachment Lh <i>[SJ_2020] - Replace SP1583838 Interchangeable</i> | 2018-05-30 → | 1 |
| 6 | SP1645172 | Adjustable angle & fold down backrest attachment Rh <i>[SJ_2020] - Replace SP1583837 Interchangeable</i> | 2018-05-30 → | 1 |
| 7 | SP1589804 | Kit fixation for backrest attachment | → 2019-05-29 | 2 |
| 8 | SP1601900 | Cord for backbrace & angle/fold down backrest attachment <i>Replace SP1554427</i> | | 1 |


Fixed backrest

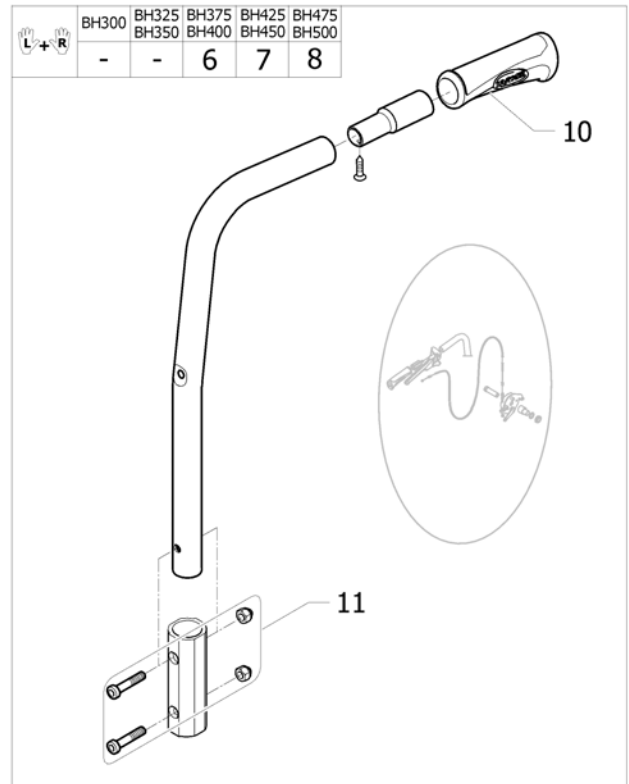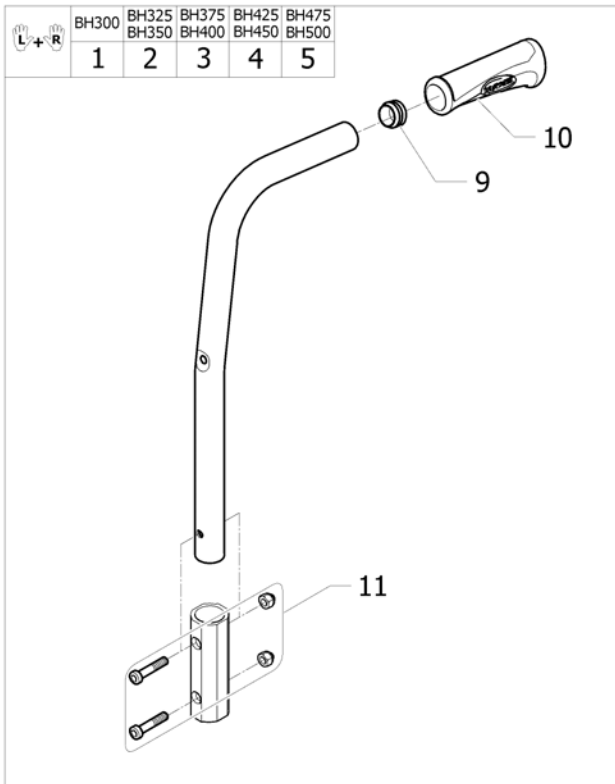

ISPS1606325


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1606320 | Backrest fixed, low, height 300mm + pushhandle | | 1 |
| 2 | SP1606321 | Backrest fixed, medium, height 325/350mm + pushhandle | | 1 |
| 3 | SP1606322 | Backrest fixed, medium, height 375/400mm + pushhandle | | 1 |
| 4 | SP1606323 | Backrest fixed, medium, height 425/450mm + pushhandle | | 1 |
| 5 | SP1606324 | Backrest fixed, medium, height 475/500mm + pushhandle | | 1 |
| 6 | SP1606326 | Backrest fixed with insert, medium, height 375/400mm + pushhandle | | 1 |
| 7 | SP1606327 | Backrest fixed with insert, medium, height 425/450mm + pushhandle | | 1 |
| 8 | SP1606328 | Backrest fixed with insert, medium, height 475/500mm + pushhandle | | 1 |
| 9 | 1418112 | Crossbrace back cap Ø22x15mm | | 2 |
| 10 | 1435420 | Handle Invacare for backrest Ø20mm | | 2 |
| 11 | SP1589826 | Kit fixation for fixed backrest | | 2 |

Backrest height adjustable 7°, without & rerset pushhandle

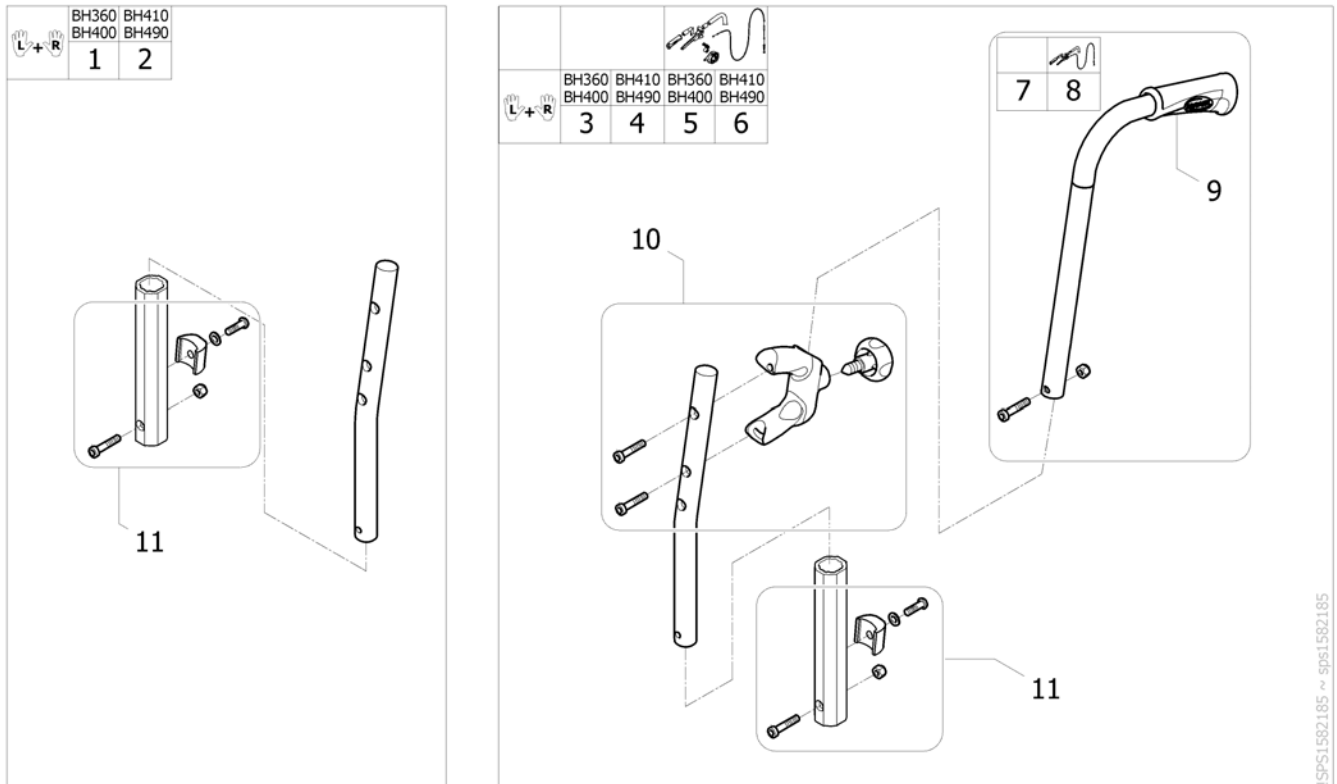





| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1584776 | Backrest 360-400mm, height adjustable, without pushhandle <i>[SJ_2300+SJ_2400]</i> | | 1 |
| 2 | SP1584777 | Backrest 410-490mm, height adjustable, without pushhandle <i>[SJ_2310+SJ_2400]</i> | | 1 |
| 3 | SP1638470 | Backrest 360-400mm, height adjustable, short, rerset pushhandle <i>[SJ_2300+SJ_2410]</i> | | 1 |
| 4 | SP1638471 | Backrest 410-490mm, height adjustable, short, rerset pushhandle <i>[SJ_2310+SJ_2410]</i> | | 1 |
| 5 | SP1584747 | Backrest 360-400mm, height adjustable, long, rerset pushhandle <i>[SJ_2300+SJ_2410]</i> | | 1 |
| 6 | SP1584764 | Backrest 410-490mm, height adjustable, long, rerset pushhandle <i>[SJ_2310+SJ_2410]</i> | | 1 |
| 7 | SP1638472 | Kit rerset pushhandle tube, short | | 2 |
| 8 | 1450574 | Kit rerset pushhandle tube long | | 2 |
| 9 | 1435420 | Handle Invacare for backrest Ø20mm | | 2 |
| 10 | 1451818 | Kit rerset pushhandle support | | 2 |
| 11 | SP1589825 | Kit adjustable backrest tube fixation | | 2 |

Backrest height adjustable, Integrated pushhandle

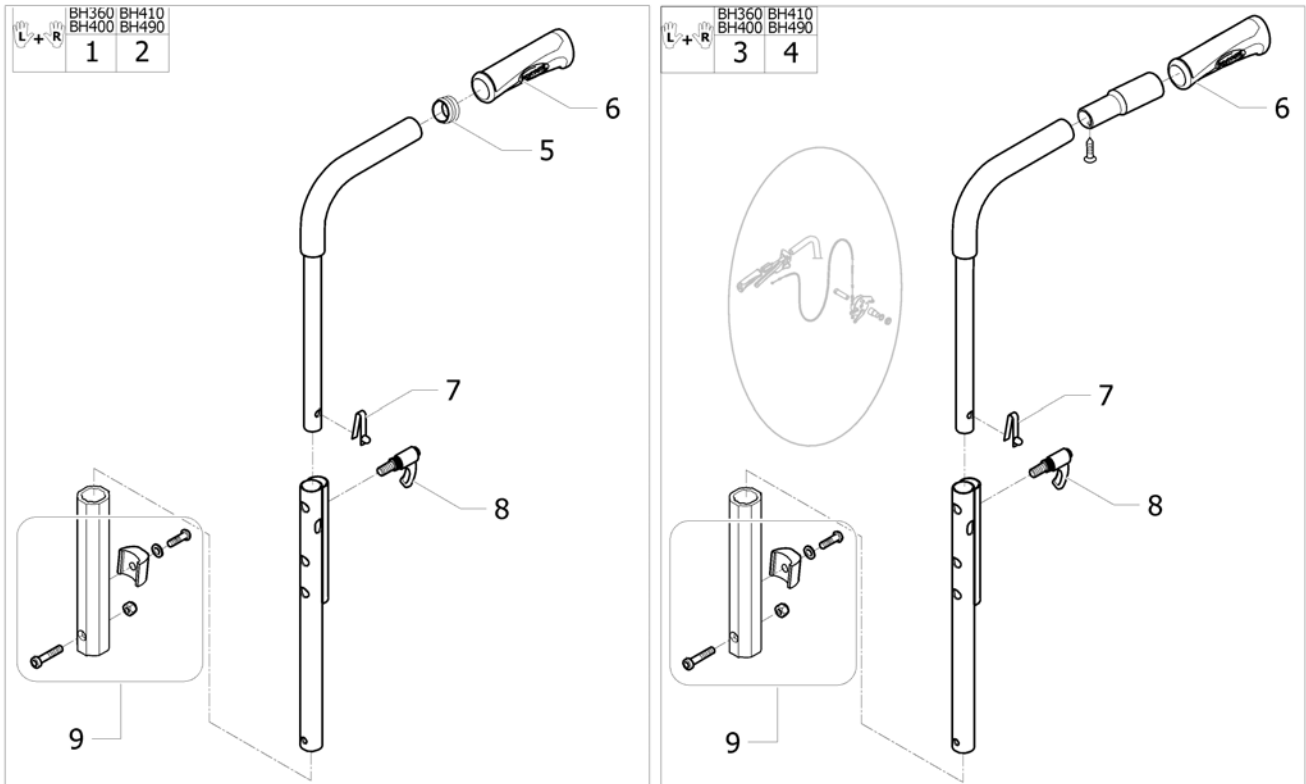


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1584778 | Backrest height adjustable, 360-400mm, Integrated pushhandle <i>[SJ_2300+SJ_2420]</i> | | 1 |
| 2 | SP1584779 | Backrest height adjustable, 410-490mm, Integrated pushhandle <i>[SJ_2310+SJ_2420]</i> | | 1 |
| 3 | SP1601903 | Backrest height adjustable with insert,360-400mm, Integrated pushhandle <i>[SJ_2300+SJ_2420+SJ_8160]</i> | | 1 |
| 4 | SP1601922 | Backrest height adjustable with insert,400-490mm, Integrated pushhandle <i>[SJ_2310+SJ_2420+SJ_8160]</i> | | 1 |
| 5 | 1418112 | Crossbrace back cap Ø22x15mm | | 2 |
| 6 | 1425473 | Handle Invacare for backrest Ø22mm | | 2 |
| 7 | 0027-10003 | Spring pin locked armrest | | 2 |
| 8 | 1443023 | Lever indexable M8x25 | | 2 |
| 9 | SP1589825 | Kit adjustable backrest tube fixation | | 2 |

Backrest Upholsteries

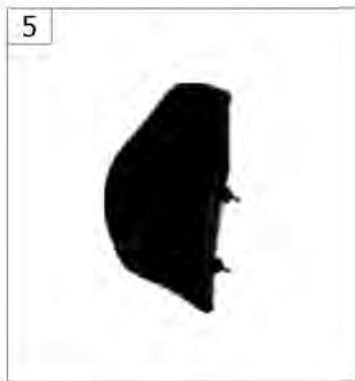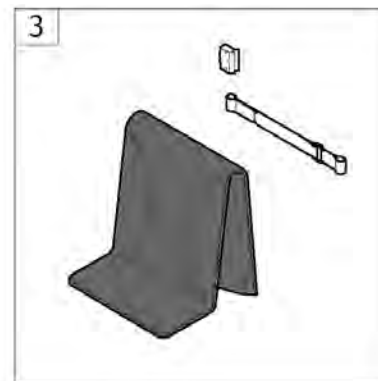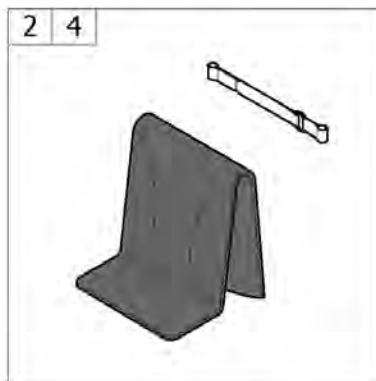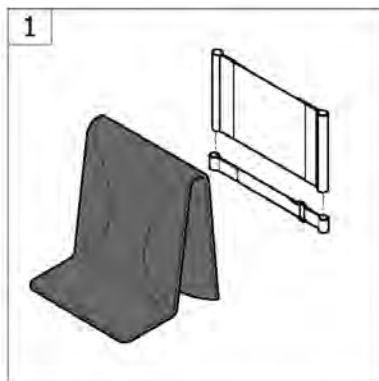

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Backrest upholstery fixed, for fixed & integrated pushhandle [SJ_2505] | | 1 |
| 2 | - | Backrest upholstery adjustable, for fixed & integrated pushhandle [SJ_2510] | | 1 |
| 3 | - | Backrest upholstery adjustable, for without & rearsset pushhandle [SJ_2510] | | 1 |
| 4 | - | Backrest Upholsteries adjustable télescopique [SJ_2510] | | 1 |
| 5 | - | Backrest upholstery MatrX [SJ_2460/SJ_2465/SJ_2470/SJ_2475] ONLY FOR EUROPE | | 1 |
| 6 | - | Mounting Hardware MatrX [SJ_2480] ONLY FOR EUROPE | | 1 |

Backrest upholstery fixed, for fixed & integrated pushhandle

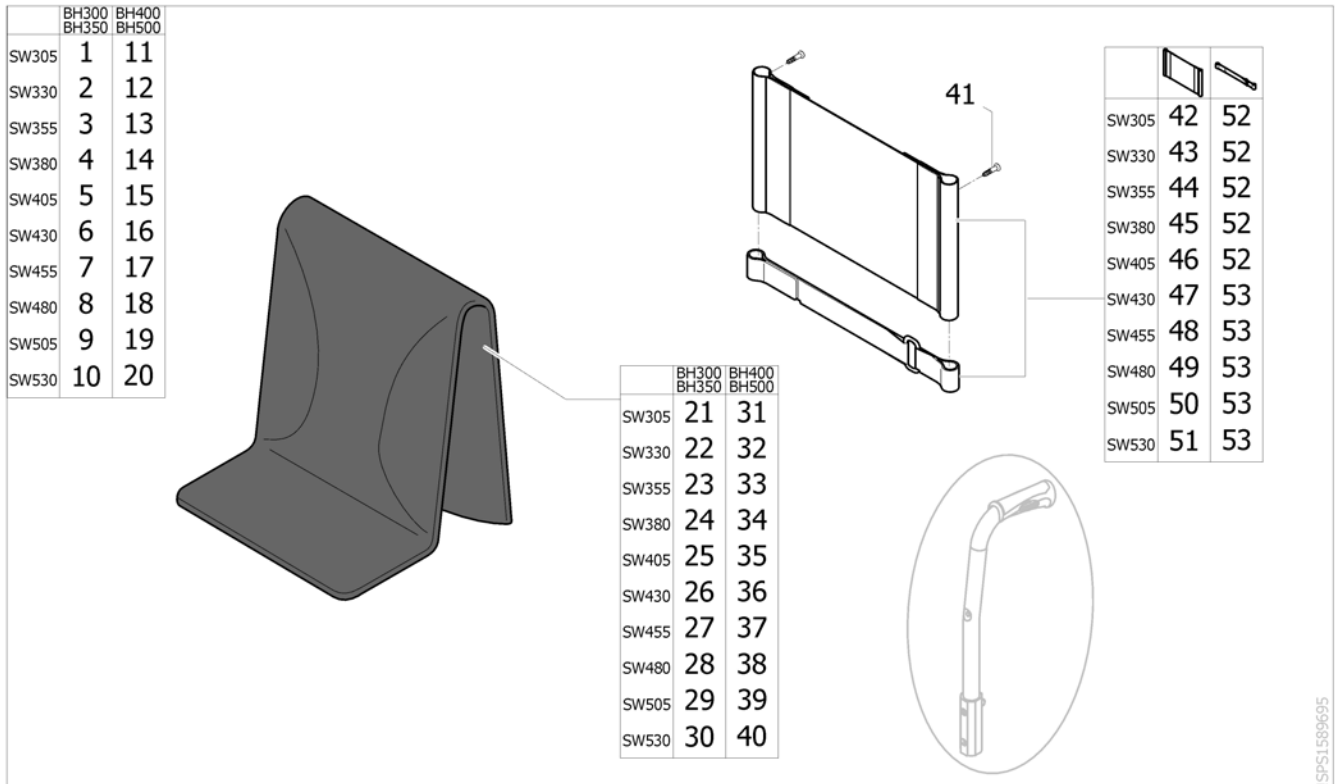


ISPS1589695

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1584647 | Backrest upholstery fixed, low, SW305mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 2 | SP1584648 | Backrest upholstery fixed, low, SW330mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 3 | SP1584649 | Backrest upholstery fixed, low, SW355mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 4 | SP1584650 | Backrest upholstery fixed, low, SW380mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 5 | SP1584651 | Backrest upholstery fixed, low, SW405mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 6 | SP1584652 | Backrest upholstery fixed, low, SW430mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 7 | SP1584653 | Backrest upholstery fixed, low, SW455mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 8 | SP1584654 | Backrest upholstery fixed, low, SW480mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 9 | SP1584655 | Backrest upholstery fixed, low, SW505mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 10 | SP1584656 | Backrest upholstery fixed, low, SW530mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 11 | SP1588315 | Backrest upholstery fixed, high, SW305mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 12 | SP1588316 | Backrest upholstery fixed, high, SW330mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 13 | SP1588317 | Backrest upholstery fixed, high, SW355mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 14 | SP1588318 | Backrest upholstery fixed, high, SW380mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 15 | SP1588339 | Backrest upholstery fixed, high, SW405mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 16 | SP1588340 | Backrest upholstery fixed, high, SW430mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 17 | SP1588341 | Backrest upholstery fixed, high, SW455mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 18 | SP1588412 | Backrest upholstery fixed, high, SW480mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |

Backrest upholstery fixed, for fixed & integrated pushhandle

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 19 | SP1588428 | Backrest upholstery fixed, high, SW505mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 20 | SP1588429 | Backrest upholstery fixed, high, SW530mm, fixed & integrated pushhandle <i>[SJ_2505+(SJ_2110/SJ_2113/SJ_2116/SJ_2120/SJ_2123/SJ_2126/SJ_2310)]</i> | | 1 |
| 21 | SP1586150 | Shapped cover, short, SW305 mm | | 1 |
| 22 | SP1586151 | Shapped cover, short, SW330 mm | | 1 |
| 23 | SP1586169 | Shapped cover, short, SW355 mm | | 1 |
| 24 | SP1586170 | Shapped cover, short, SW380 mm | | 1 |
| 25 | SP1602956 | Shapped cover, short, SW405 mm | | 1 |
| 26 | SP1602957 | Shapped cover, short, SW430 mm | | 1 |
| 27 | SP1602958 | Shapped cover, short, SW455 mm | | 1 |
| 28 | SP1586187 | Shapped cover, short, SW480 mm | | 1 |
| 29 | SP1586188 | Shapped cover, short, SW505 mm | | 1 |
| 30 | SP1586189 | Shapped cover, short, SW530 mm | | 1 |
| 31 | SP1586070 | Shapped cover, standard, SW305 mm | | 1 |
| 32 | SP1586071 | Shapped cover, standard, SW330 mm | | 1 |
| 33 | SP1586072 | Shapped cover, standard, SW355 mm | | 1 |
| 34 | SP1586073 | Shapped cover, standard, SW380 mm | | 1 |
| 35 | SP1602926 | Shapped cover, standard, SW405 mm | | 1 |
| 36 | SP1602927 | Shapped cover, standard, SW430 mm | | 1 |
| 37 | SP1602928 | Shapped cover, standard, SW455 mm | | 1 |
| 38 | SP1602929 | Shapped cover, standard, SW480 mm | | 1 |
| 39 | SP1586148 | Shapped cover, standard, SW505 mm | | 1 |
| 40 | SP1586149 | Shapped cover, standard, SW530 mm | | 1 |
| 41 | 1428603 | Screw Tfb 4,8x19 A2r Tx25 DIN7983 | | 2 |
| 42 | SP1584892 | Fix backrest muff SW305 BH 430 mm Action | | 1 |
| 43 | SP1584893 | Fix backrest muff SW330 BH B430 mm Action | | 1 |
| 44 | SP1584894 | Fix backrest muff SW355 BH 430 mm Action | | 1 |
| 45 | SP1532247 | Fix backrest muff SW380 BH 430 mm Action | | 1 |
| 46 | SP1532248 | Fix backrest muff SW405 BH 430 mm Action | | 1 |
| 47 | SP1532249 | Fix backrest muff SW430 BH 430 mm Action | | 1 |
| 48 | SP1532250 | Fix backrest muff SW455 BH 430 mm Action | | 1 |
| 49 | SP1532251 | Fix backrest muff SW480 BH 430 mm Action | | 1 |
| 50 | SP1532252 | Fix backrest muff SW505 BH 430 mm Action | | 1 |
| 51 | SP1533168 | Fix backrest muff SW530 BH 430 mm Action | | 1 |
| 52 | SP1601671 | Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i> | | 1 |
| 53 | SP1601712 | Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i> | | 1 |

Backrest upholstery adjustable, for fixed & integrated pushhandle

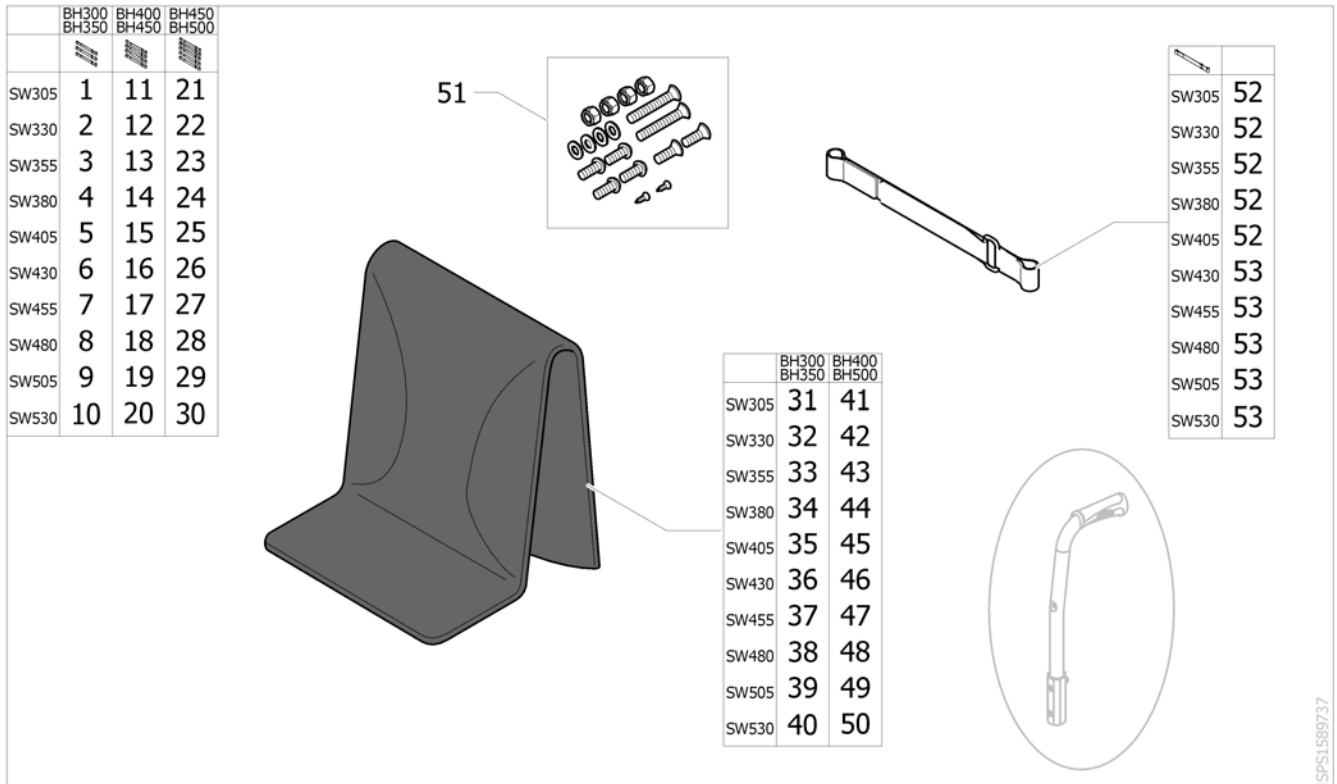

SPS1589737

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1589213 | Backrest upholstery adjustable, low, SW305mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 2 | SP1589214 | Backrest upholstery adjustable, low, SW330mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 3 | SP1589215 | Backrest upholstery adjustable, low, SW355mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 4 | SP1589216 | Backrest upholstery adjustable, low, SW380mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 5 | SP1589217 | Backrest upholstery adjustable, low, SW405mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 6 | SP1589218 | Backrest upholstery adjustable, low, SW430mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 7 | SP1589219 | Backrest upholstery adjustable, low, SW455mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 8 | SP1589220 | Backrest upholstery adjustable, low, SW480mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 9 | SP1589221 | Backrest upholstery adjustable, low, SW505mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 10 | SP1589222 | Backrest upholstery adjustable, low, SW530mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2100/SJ_2103/SJ_2106/SJ_2300)]</i> | | 1 |
| 11 | SP1602207 | Backrest upholstery adjustable, medium, SW305mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]</i> | | 1 |
| 12 | SP1602208 | Backrest upholstery adjustable, medium, SW330mm, fixed & integrated pushhandle <i>[SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)]</i> | | 1 |


Backrest upholstery adjustable, for fixed & integrated pushhandle

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 13 | SP1602209 | Backrest upholstery adjustable,medium,SW355mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 14 | SP1602210 | Backrest upholstery adjustable,medium,SW380mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 15 | SP1602211 | Backrest upholstery adjustable,medium,SW405mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 16 | SP1602212 | Backrest upholstery adjustable,medium,SW430mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 17 | SP1602213 | Backrest upholstery adjustable,medium,SW455mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 18 | SP1602214 | Backrest upholstery adjustable,medium,SW480mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 19 | SP1602215 | Backrest upholstery adjustable,medium,SW505mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 20 | SP1602216 | Backrest upholstery adjustable,medium,SW530mm, fixed & integrated pushhandle [SJ_2510+(SJ_2110/SJ_2113/SJ_2116/SJ_2300)] | | 1 |
| 21 | SP1589413 | Backrest upholstery adjustable, high, SW305mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 22 | SP1589414 | Backrest upholstery adjustable, high, SW330mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 23 | SP1589415 | Backrest upholstery adjustable, high, SW355mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 24 | SP1589416 | Backrest upholstery adjustable, high, SW380mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 25 | SP1589417 | Backrest upholstery adjustable, high, SW405mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 26 | SP1589418 | Backrest upholstery adjustable, high, SW430mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 27 | SP1589419 | Backrest upholstery adjustable, high, SW455mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 28 | SP1589420 | Backrest upholstery adjustable, high, SW480mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 29 | SP1589421 | Backrest upholstery adjustable, high, SW505mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 30 | SP1589422 | Backrest upholstery adjustable, high, SW530mm, fixed & integrated pushhandle [SJ_2510+(SJ_2120/SJ_2123/SJ_2126/SJ_2310)] | | 1 |
| 31 | SP1586150 | Shapped cover, short, SW305 mm | | 1 |
| 32 | SP1586151 | Shapped cover, short, SW330 mm | | 1 |
| 33 | SP1586169 | Shapped cover, short, SW355 mm | | 1 |
| 34 | SP1586170 | Shapped cover, short, SW380 mm | | 1 |
| 35 | SP1602956 | Shapped cover, short, SW405 mm | | 1 |
| 36 | SP1602957 | Shapped cover, short, SW430 mm | | 1 |
| 37 | SP1602958 | Shapped cover, short, SW455 mm | | 1 |
| 38 | SP1586187 | Shapped cover, short, SW480 mm | | 1 |
| 39 | SP1586188 | Shapped cover, short, SW505 mm | | 1 |
| 40 | SP1586189 | Shapped cover, short, SW530 mm | | 1 |
| 41 | SP1586070 | Shapped cover, standard, SW305 mm | | 1 |

Backrest upholstery adjustable, for fixed & integrated pushhandle

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 42 | SP1586071 | Shapped cover, standard, SW330 mm | | 1 |
| 43 | SP1586072 | Shapped cover, standard, SW355 mm | | 1 |
| 44 | SP1586073 | Shapped cover, standard, SW380 mm | | 1 |
| 45 | SP1602926 | Shapped cover, standard, SW405 mm | | 1 |
| 46 | SP1602927 | Shapped cover, standard, SW430 mm | | 1 |
| 47 | SP1602928 | Shapped cover, standard, SW455 mm | | 1 |
| 48 | SP1602929 | Shapped cover, standard, SW480 mm | | 1 |
| 49 | SP1586148 | Shapped cover, standard, SW505 mm | | 1 |
| 50 | SP1586149 | Shapped cover, standard, SW530 mm | | 1 |
| 51 | SP1601667 | Screws kit for adjustable backrest upholstery | | 1 |
| 52 | SP1601671 | Strap 475mm for tension adj. backrest (unit) [SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040] | | 5 |
| 53 | SP1601712 | Strap 600mm for tension adj. backrest (unit) [SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090] | | 5 |

Backrest upholstery adjustable, for without & rearsset pushhandle

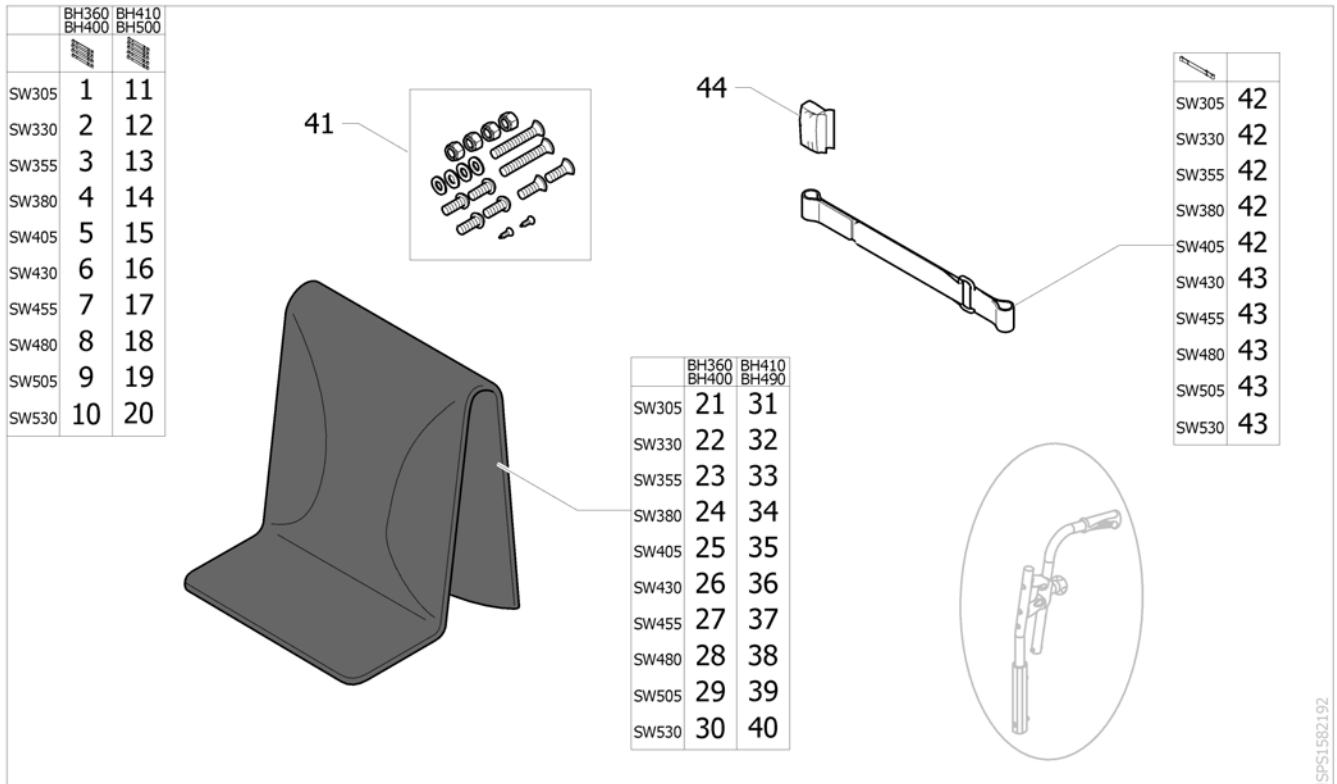


ISPS1582192

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1589223 | Backrest upholstery adjustable, SW305mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 2 | SP1589224 | Backrest upholstery adjustable, SW330mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 3 | SP1589327 | Backrest upholstery adjustable, SW355mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 4 | SP1589396 | Backrest upholstery adjustable, SW380mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 5 | SP1589397 | Backrest upholstery adjustable, SW405mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 6 | SP1589398 | Backrest upholstery adjustable, SW430mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 7 | SP1589399 | Backrest upholstery adjustable, SW455mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 8 | SP1589400 | Backrest upholstery adjustable, SW480mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 9 | SP1589401 | Backrest upholstery adjustable, SW505mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 10 | SP1589402 | Backrest upholstery adjustable, SW530mm, 360-400mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2300]</i> | | 1 |
| 11 | SP1589455 | Backrest upholstery adjustable, SW305mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 12 | SP1589456 | Backrest upholstery adjustable, SW330mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |

Backrest upholstery adjustable, for without & rearsset pushhandle

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 13 | SP1589457 | Backrest upholstery adjustable, SW355mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 14 | SP1589458 | Backrest upholstery adjustable, SW380mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 15 | SP1589459 | Backrest upholstery adjustable, SW405mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 16 | SP1589460 | Backrest upholstery adjustable, SW430mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 17 | SP1589461 | Backrest upholstery adjustable, SW455mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 18 | SP1589466 | Backrest upholstery adjustable, SW480mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 19 | SP1589467 | Backrest upholstery adjustable, SW505mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 20 | SP1589468 | Backrest upholstery adjustable, SW530mm, 410-490mm, w/o & rearsset pushhandle <i>[SJ_2510+SJ_2310]</i> | | 1 |
| 21 | SP1586150 | Shapped cover, short, SW305 mm | | 1 |
| 22 | SP1586151 | Shapped cover, short, SW330 mm | | 1 |
| 23 | SP1586169 | Shapped cover, short, SW355 mm | | 1 |
| 24 | SP1586170 | Shapped cover, short, SW380 mm | | 1 |
| 25 | SP1602956 | Shapped cover, short, SW405 mm | | 1 |
| 26 | SP1602957 | Shapped cover, short, SW430 mm | | 1 |
| 27 | SP1602958 | Shapped cover, short, SW455 mm | | 1 |
| 28 | SP1586187 | Shapped cover, short, SW480 mm | | 1 |
| 29 | SP1586188 | Shapped cover, short, SW505 mm | | 1 |
| 30 | SP1586189 | Shapped cover, short, SW530 mm | | 1 |
| 31 | SP1586070 | Shapped cover, standard, SW305 mm | | 1 |
| 32 | SP1586071 | Shapped cover, standard, SW330 mm | | 1 |
| 33 | SP1586072 | Shapped cover, standard, SW355 mm | | 1 |
| 34 | SP1586073 | Shapped cover, standard, SW380 mm | | 1 |
| 35 | SP1602926 | Shapped cover, standard, SW405 mm | | 1 |
| 36 | SP1602927 | Shapped cover, standard, SW430 mm | | 1 |
| 37 | SP1602928 | Shapped cover, standard, SW455 mm | | 1 |
| 38 | SP1602929 | Shapped cover, standard, SW480 mm | | 1 |
| 39 | SP1586148 | Shapped cover, standard, SW505 mm | | 1 |
| 40 | SP1586149 | Shapped cover, standard, SW530 mm | | 1 |
| 41 | SP1601667 | Screws kit for adjustable backrest upholstery | | 1 |
| 42 | SP1601671 | Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i> | | 5 |
| 43 | SP1601712 | Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i> | | 5 |
| 44 | SP1601713 | Kit protection for backrest without pushhandle | | 2 |

Backrest Upholsteries adjustable télecospique

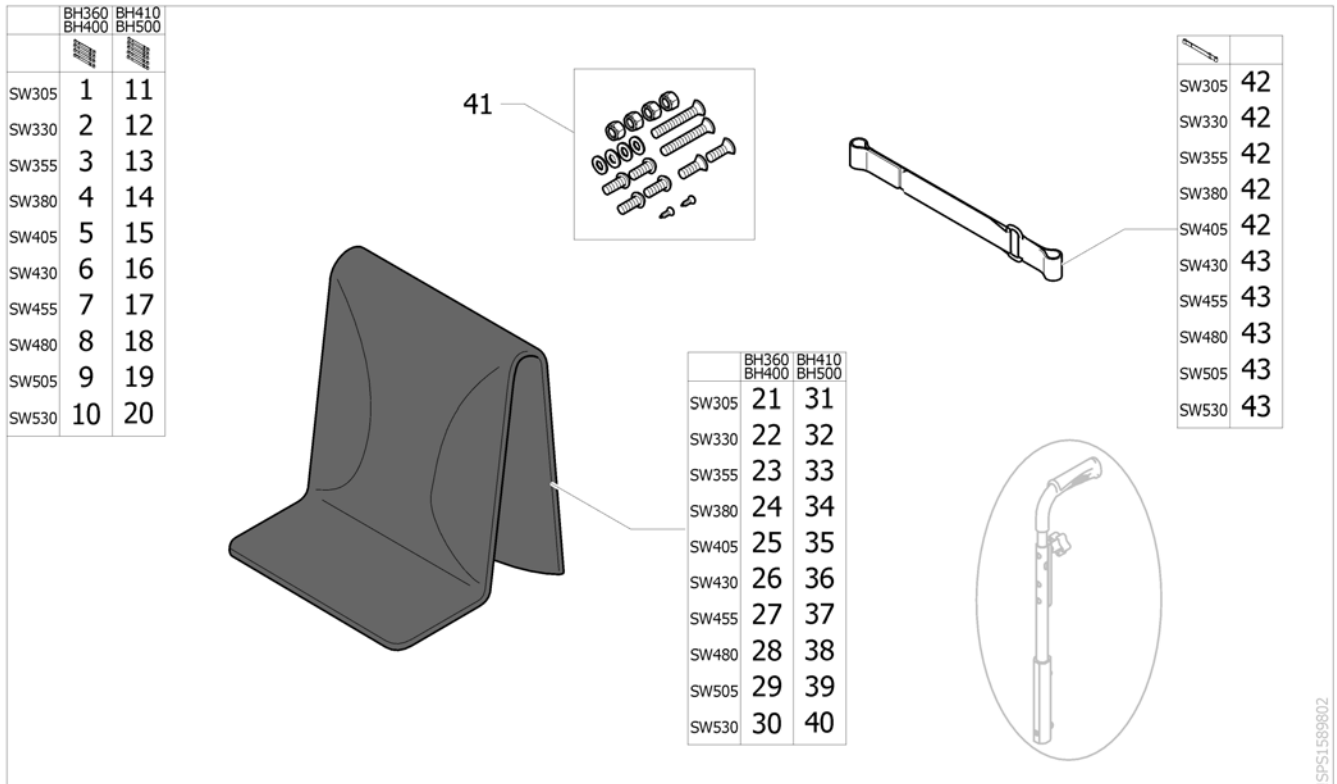

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1589403 | Back uphol. adj low integrated handle SW305 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 2 | SP1589404 | Back uphol. adj low integrated handle SW330 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 3 | SP1589405 | Back uphol. adj low integrated handle SW355 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 4 | SP1589406 | Back uphol. adj low integrated handle SW380 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 5 | SP1589407 | Back uphol. adj low integrated handle SW405 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 6 | SP1589408 | Back uphol. adj low integrated handle SW430 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 7 | SP1589409 | Back uphol. adj low integrated handle SW455 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 8 | SP1589410 | Back uphol. adj low integrated handle SW480 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 9 | SP1589411 | Back uphol. adj low integrated handle SW505 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 10 | SP1589412 | Back uphol. adj low integrated handle SW530 A5 <i>[SJ_2510+SJ_2420+SJ_2300]</i> | | 1 |
| 11 | SP1589423 | Back uphol. adj high integrated handle SW305 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 12 | SP1589446 | Back uphol. adj high integrated handle SW330 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 13 | SP1589447 | Back uphol. adj high integrated handle SW355 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 14 | SP1589448 | Back uphol. adj high integrated handle SW380 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 15 | SP1589449 | Back uphol. adj high integrated handle SW405 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 16 | SP1589450 | Back uphol. adj high integrated handle SW430 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 17 | SP1589451 | Back uphol. adj high integrated handle SW455 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 18 | SP1589452 | Back uphol. adj high integrated handle SW480 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |

Backrest Upholsteries adjustable télécospique

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 19 | SP1589453 | Back uphol. adj high integrated handle SW505 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 20 | SP1589454 | Back uphol. adj high integrated handle SW530 A5 <i>[SJ_2510+SJ_2420+SJ_2310]</i> | | 1 |
| 21 | SP1586150 | Shapped cover, short, SW305 mm | | 1 |
| 22 | SP1586151 | Shapped cover, short, SW330 mm | | 1 |
| 23 | SP1586169 | Shapped cover, short, SW355 mm | | 1 |
| 24 | SP1586170 | Shapped cover, short, SW380 mm | | 1 |
| 25 | SP1602956 | Shapped cover, short, SW405 mm | | 1 |
| 26 | SP1602957 | Shapped cover, short, SW430 mm | | 1 |
| 27 | SP1602958 | Shapped cover, short, SW455 mm | | 1 |
| 29 | SP1586188 | Shapped cover, short, SW505 mm | | 1 |
| 28 | SP1586187 | Shapped cover, short, SW480 mm | | 1 |
| 30 | SP1586189 | Shapped cover, short, SW530 mm | | 1 |
| 31 | SP1586070 | Shapped cover, standard, SW305 mm | | 1 |
| 32 | SP1586071 | Shapped cover, standard, SW330 mm | | 1 |
| 33 | SP1586072 | Shapped cover, standard, SW355 mm | | 1 |
| 34 | SP1586073 | Shapped cover, standard, SW380 mm | | 1 |
| 35 | SP1602926 | Shapped cover, standard, SW405 mm | | 1 |
| 36 | SP1602927 | Shapped cover, standard, SW430 mm | | 1 |
| 37 | SP1602928 | Shapped cover, standard, SW455 mm | | 1 |
| 38 | SP1602929 | Shapped cover, standard, SW480 mm | | 1 |
| 39 | SP1586148 | Shapped cover, standard, SW505 mm | | 1 |
| 40 | SP1586149 | Shapped cover, standard, SW530 mm | | 1 |
| 41 | SP1601667 | Screws kit for adjustable backrest upholstery | | 1 |
| 42 | SP1601671 | Strap 475mm for tension adj. backrest (unit) <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040]</i> | | 5 |
| 43 | SP1601712 | Strap 600mm for tension adj. backrest (unit) <i>[SJ_1050/SJ_1060/SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i> | | 5 |

Backrest upholstery MatrX




| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 1532093 | Backrest MatrX Elite W380 x H410 mm - without hardware | | 1 |
| 2 | 1515720 | Backrest MatrX Elite W410 x H410 mm - without hardware | | 1 |
| 3 | 1515722 | Backrest MatrX Elite W430 x H410 mm - without hardware | | 1 |
| 4 | 1515724 | Backrest MatrX Elite W360 x H410 mm - without hardware | | 1 |
| 5 | 1515726 | Backrest MatrX Elite W480 x H410 mm - without hardware | | 1 |
| 6 | 1532096 | Backrest MatrX Elite W510 x H410 mm - without hardware | | 1 |
| 7 | 1545100 | Backrest MatrX Elite W380 x H460 mm - without hardware | | 1 |
| 8 | 1545102 | Backrest MatrX Elite W410 x H460 mm - without hardware | | 1 |
| 9 | 1545105 | Backrest MatrX Elite W430 x H460 mm - without hardware | | 1 |
| 10 | 1545108 | Backrest MatrX Elite W460 x H460 mm - without hardware | | 1 |
| 11 | 1545111 | Backrest MatrX Elite W480 x H460 mm - without hardware | | 1 |
| 12 | 1545114 | Backrest MatrX Elite W510 x H460 mm - without hardware | | 1 |
| 13 | 1543509 | Backrest MatrX Elite Deep W380 x H410 mm - without hardware | | 1 |
| 14 | 1543513 | Backrest MatrX Elite Deep W410 x H410 mm - without hardware | | 1 |
| 15 | 1543517 | Backrest MatrX Elite Deep W430 x H410 mm - without hardware | | 1 |
| 16 | 1543521 | Backrest MatrX Elite Deep W460 x H410 mm - without hardware | | 1 |
| 17 | 1543525 | Backrest MatrX Elite Deep W480 x H410 mm - without hardware | | 1 |
| 18 | 1543989 | Backrest MatrX Elite Deep W510 x H410 mm - without hardware | | 1 |
| 19 | 1543510 | Backrest MatrX Elite Deep W380 x H460 mm - without hardware | | 1 |
| 20 | 1543514 | Backrest MatrX Elite Deep W410 x H460 mm - without hardware | | 1 |
| 21 | 1543518 | Backrest MatrX Elite Deep W430 x H460 mm - without hardware | | 1 |
| 22 | 1543522 | Backrest MatrX Elite Deep W460 x H460 mm - without hardware | | 1 |
| 23 | 1543526 | Backrest MatrX Elite Deep W480 x H460 mm - without hardware | | 1 |
| 24 | 1543990 | Backrest MatrX Elite Deep W510 x H460 mm - without hardware | | 1 |

Mounting Hardware MatrX




| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1545275 | PB Mounting Hardware for all MC Backs (pair) | | 1 |

Legrests

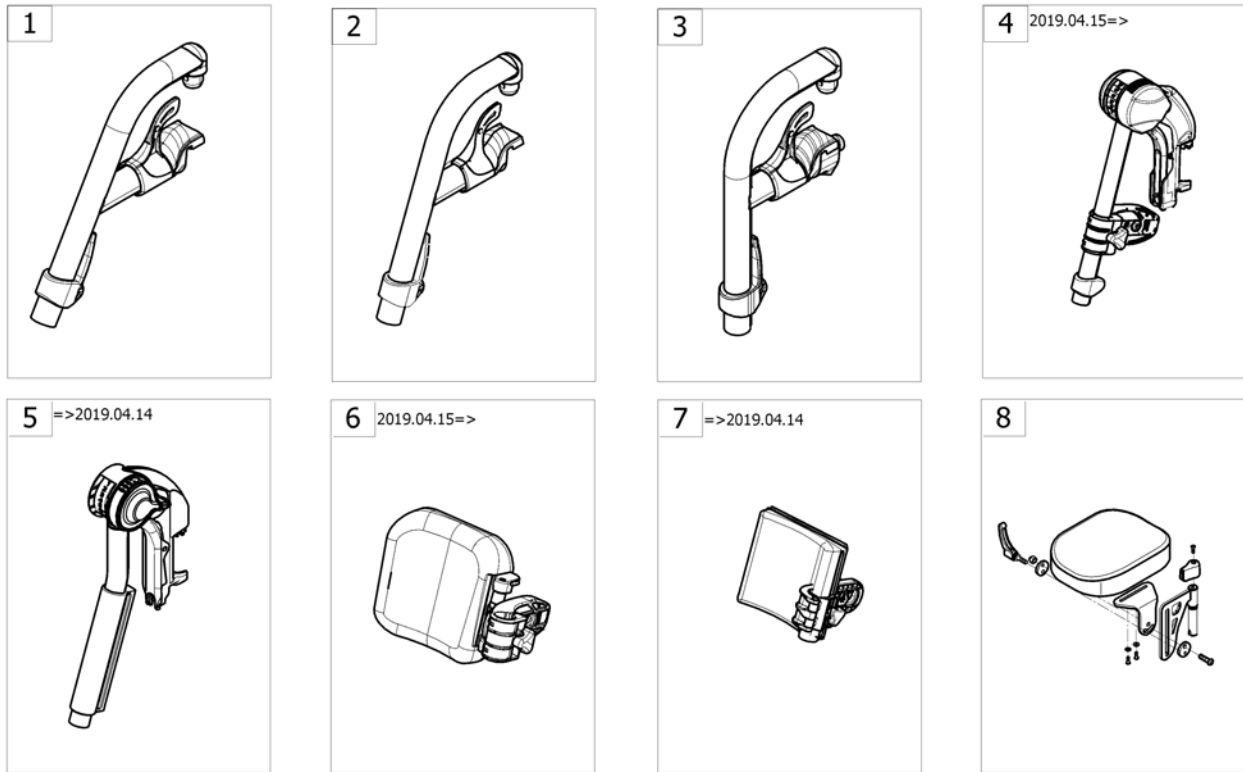


I->SFOACTION530

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | - | Hanger 70° [SJ_2810/SJ_3010/SJ_3110] | | 1 |
| 2 | - | Hanger 80° [SJ_2820/SJ_3020/SJ_3120] | | 1 |
| 3 | - | Hanger 90° [SJ_2830/SJ_3030/SJ_3130] | | 1 |
| 4 | - | Elevating Legrest Standard 2019.04.15 => | | 1 |
| 5 | - | Elevating legrest =>2019.04.14 [SJ_2840/SJ_3040/SJ_3140] | | 1 |
| 6 | - | Calf Pad Medium 2019.04.15 => | | 1 |
| 7 | - | Calf Pad =>2019.04.14 [SJ_2840/SJ_3040/SJ_3140] | | 1 |
| 8 | - | Amputee Legrest [SJ_2850/SJ_3050/SJ_3150] | | 1 |

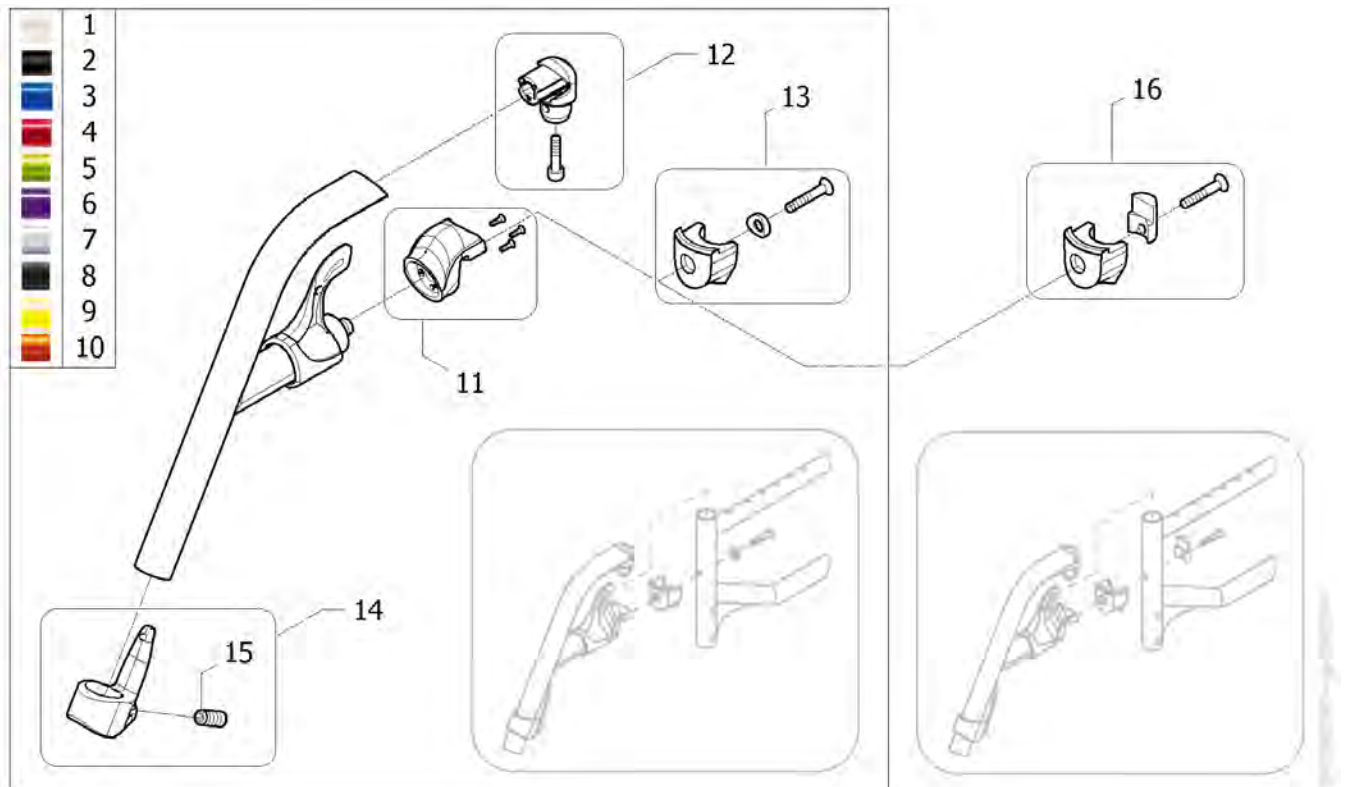Hanger 70°


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1584371-0637 | Hanger 70°, complete, Pearl White (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6000] | | 2 |
| 2 | SP1584371-0609 | Hanger 70°, complete, Anthracite (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6010] | | 2 |
| 3 | SP1584371-0643 | Hanger 70°, complete, Pop Blue (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6020] | | 2 |
| 4 | SP1584371-0607 | Hanger 70°, complete, Dark Red (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6030] | | 2 |
| 5 | SP1584371-0644 | Hanger 70°, complete, Electric Green (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6040] | | 2 |
| 6 | SP1584371-0645 | Hanger 70°, complete, Hypnotic Purple (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6050] | | 2 |
| 7 | SP1584371-0515 | Hanger 70°, complete, Pearl Grey (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6060] | | 2 |
| 8 | SP1584371-0646 | Hanger 70°, complete, Matt Black (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6070] | | 2 |
| 9 | SP1584371-0647 | Hanger 70°, complete, Intense Yellow (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6080] | | 2 |
| 10 | SP1584371-0648 | Hanger 70°, complete, Vibrating Orange (2015) [(SJ_2810/SJ_3010/SJ_3110)+SJ_6090] | | 2 |
| 11 | SP1584383 | Kit hanger locking nose (2015) | | 2 |
| 12 | SP1584381 | Kit hanger pivot (2015) | | 2 |
| 13 | SP1584380 | Kit hanger locked (2015) | | 2 |
| 14 | SP1584382 | Calf strap holder + ball screw (2015) | | 2 |
| 15 | 1424571 | Springscrew | | 2 |
| 16 | SP1587483 | Kit hanger stopper (2015) for Action NG | | 2 |

Hanger 80°

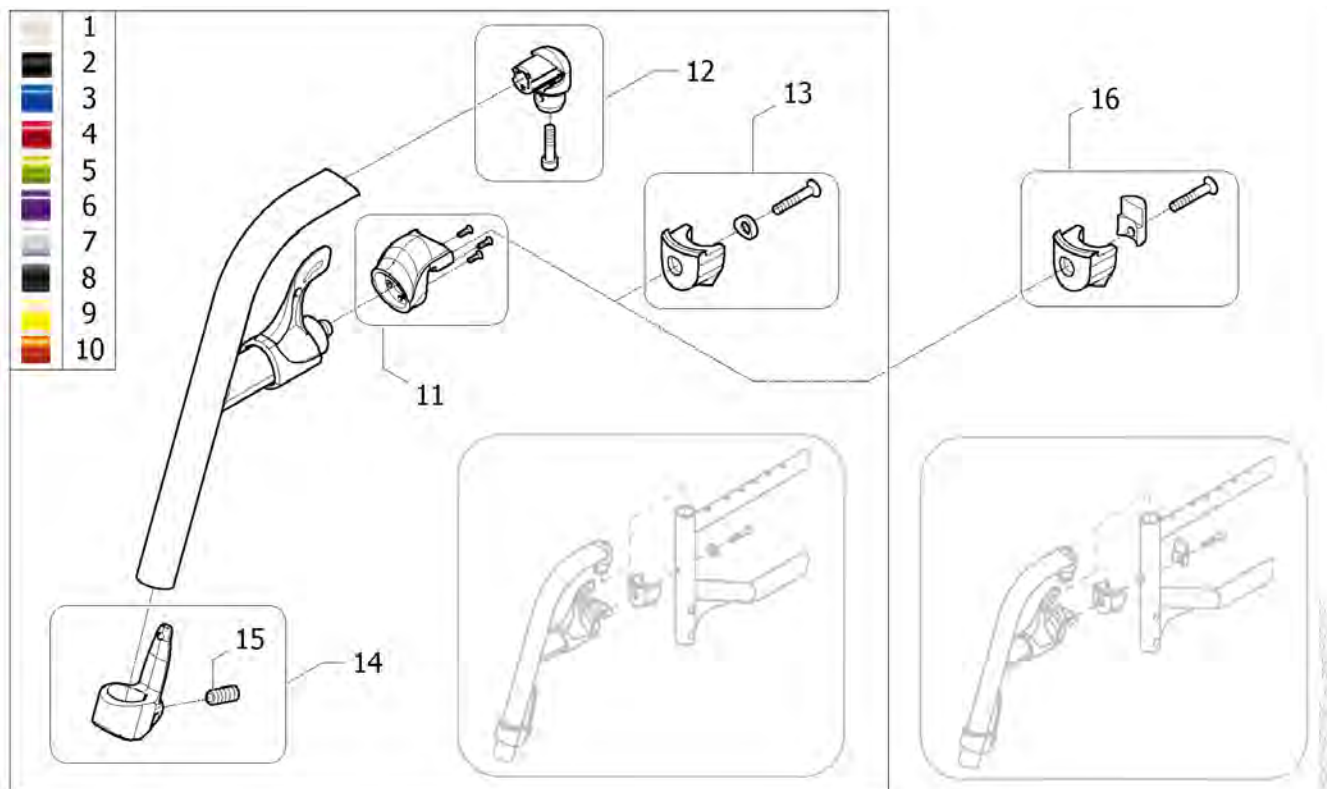


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 1 | SP1584369-0637 | Hanger 80° Alu, Pearl White (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6000] | | 2 |
| 2 | SP1584369-0609 | Hanger 80° Alu, Anthracite (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6010] | | 2 |
| 3 | SP1584369-0643 | Hanger 80°, complete, Pop Blue (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6020] | | 2 |
| 4 | SP1584369-0607 | Hanger 80° Alu, Dark Red (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6030] | | 2 |
| 5 | SP1584369-0644 | Hanger 80° Alu, Electric Green (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6040] | | 2 |
| 6 | SP1584369-0645 | Hanger 80°, complete, Hypnotic Purple (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6050] | | 2 |
| 7 | SP1584369-0515 | Hanger 80°, complete, Pearl Grey (2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6060] | | 2 |
| 8 | SP1584369-0646 | Hanger 80°, complete, Matt Black (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6070] | | 2 |
| 9 | SP1584369-0647 | Hanger 80°, complete, Intense Yellow (2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6080] | | 2 |
| 10 | SP1584369-0648 | Hanger 80°, complete, Vibrating Orange (v2015) [[SJ_2820/SJ_3020/SJ_3120)+SJ_6090] | | 2 |
| 11 | SP1584383 | Kit hanger locking nose (2015) | | 2 |
| 12 | SP1584381 | Kit hanger pivot (2015) | | 2 |
| 13 | SP1584380 | Kit hanger locked (2015) | | 2 |
| 14 | SP1584382 | Calf strap holder + ball screw (2015) | | 2 |
| 15 | 1424571 | Springscrew | | 2 |
| 16 | SP1587483 | Kit hanger stopper (2015) for Action NG | | 2 |

Hanger 90°

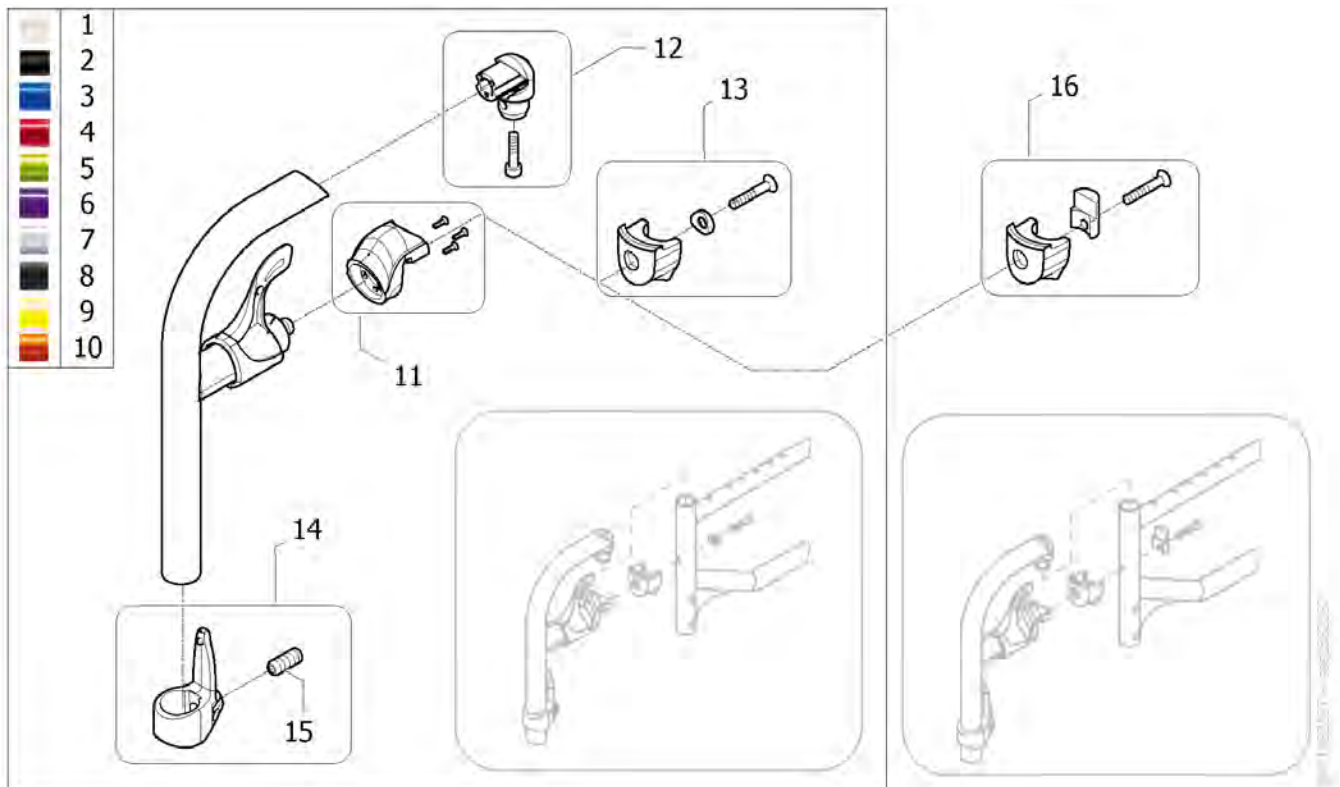


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1584370-0637 | Hanger 90°, complete, Pearl White (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6000] | | 2 |
| 2 | SP1584370-0609 | Hanger 90°, complete, Anthracite (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6010] | | 2 |
| 3 | SP1584370-0643 | Hanger 90°, complete, Pop Blue (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6020] | | 2 |
| 4 | SP1584370-0607 | Hanger 90°, complete, Dark Red (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6030] | | 2 |
| 5 | SP1584370-0644 | Hanger 90°, complete, Electric Green (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6040] | | 2 |
| 6 | SP1584370-0645 | Hanger 90°, complete, Hypnotic Purple (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6050] | | 2 |
| 7 | SP1584370-0515 | Hanger 90°, complete, Pearl Grey (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6060] | | 2 |
| 8 | SP1584370-0646 | Hanger 90°, complete, Matt Black (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6070] | | 2 |
| 9 | SP1584370-0647 | Hanger 90°, complete, Intense Yellow (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6080] | | 2 |
| 10 | SP1584370-0648 | Hanger 90°, complete, Vibrating Orange (2015) [[SJ_2830/SJ_3030/SJ_3130)+SJ_6090] | | 2 |
| 11 | SP1584383 | Kit hanger locking nose (2015) | | 2 |
| 12 | SP1584381 | Kit hanger pivot (2015) | | 2 |
| 13 | SP1584380 | Kit hanger locked (2015) | | 2 |
| 14 | SP1584382 | Calf strap holder + ball screw (2015) | | 2 |
| 15 | 1424571 | Springscrew | | 2 |
| 16 | SP1587483 | Kit hanger stopper (2015) for Action NG | | 2 |

Elevating Legrest Standard 2019.04.15 =>

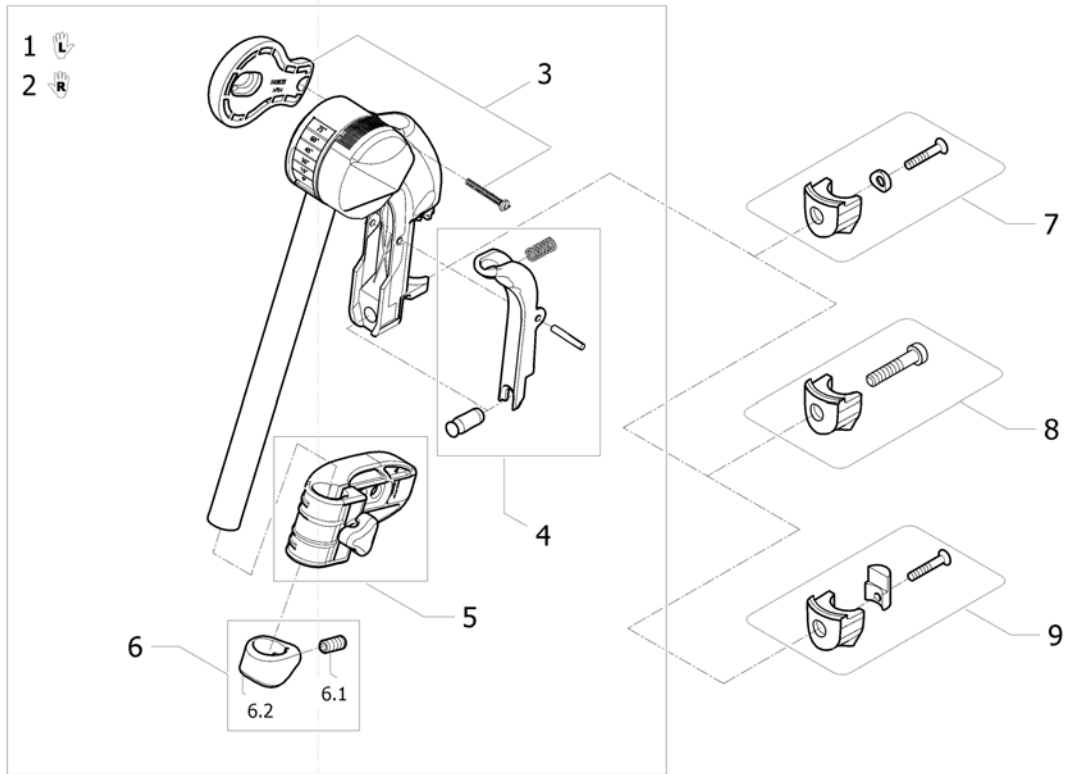

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1647258 | Elevating Legrest Lh + fixation | | 1 |
| 2 | SP1647259 | Elevating Legrest Rh + fixation | | 1 |
| 3 | SP1656819 | Kneepad Kit | | 2 |
| 4 | SP1656820 | Legrest Trigger Kit | | 2 |
| 5 | SP1602709 | Bracket assembly Calfpad Kit | | 2 |
| 6 | SP1651847 | Bridle Setting kit | | 2 |
| 6.1 | 1424571 | Springscrew | | 2 |
| 6.2 | SP1605203 | Screwsocket Legrest kit | | 2 |
| 7 | SP1584380 | Kit hanger locked (2015) | | 2 |
| 8 | SP1656824 | Kit hanger locked Passif | | 2 |
| 9 | SP1587483 | Kit hanger stopper (2015) for Action NG | | 2 |

Elevating legrest =>2019.04.14

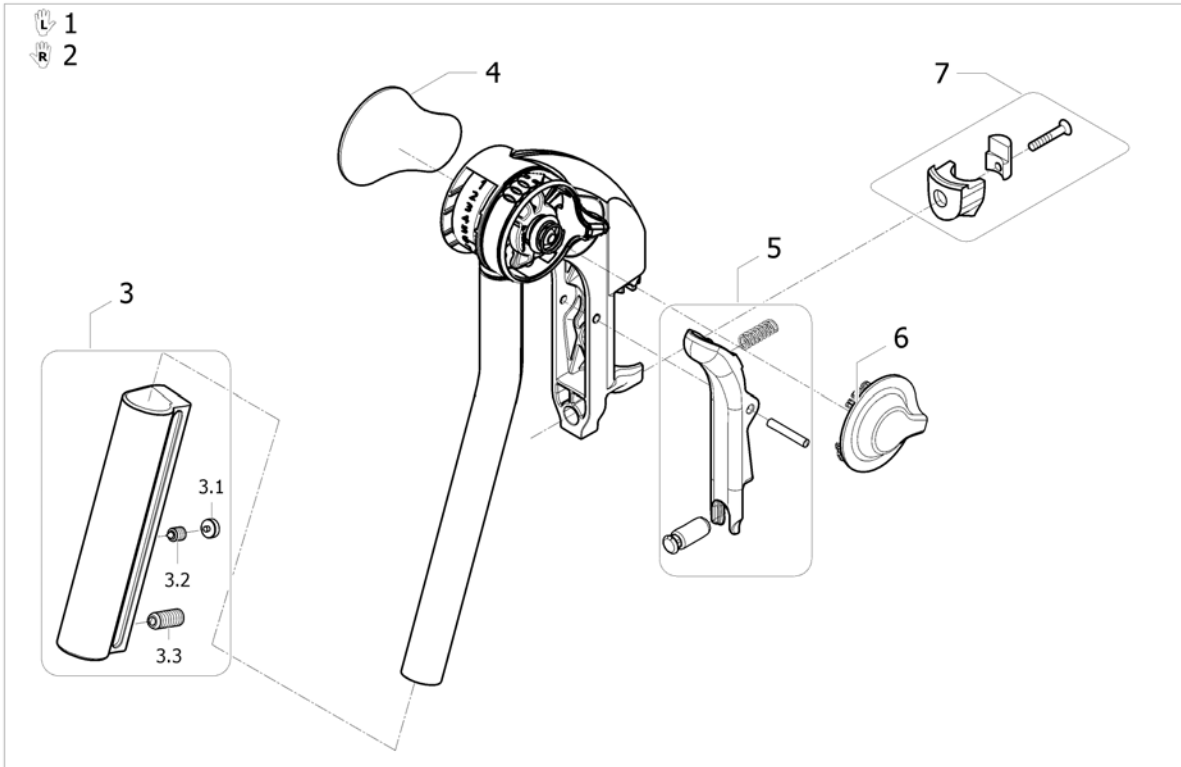

ISPS1535382

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1552881 | Elevating legrest Lh + fixation | | 1 |
| 2 | 1552882 | Elevating legrest Rh + fixation | | 1 |
| 3 | SP1651354 | Kit Fastener calf pad support | | 2 |
| 3.1 | 1435741 | Plug Ikpt 8-5,5 S | | 2 |
| 3.2 | 1424572 | Stop Screw S6Ss M10x12 DIN914 DIN914/ISO4027 | | 2 |
| 3.3 | 1424571 | Springscrew | | 2 |
| 4 | 1428336 | Kneepad | | 2 |
| 5 | 1430429 | Legrest Trigger Kit | | 2 |
| 6 | 1508943 | Cap, Alu2008 Legrest | | 2 |
| 7 | SP1584380 | Kit hanger locked (2015) <i>Replace 1430427</i> | | 2 |


Calf Pad Medium 2019.04.15 =>

| SW | Action 5 | | | | Action 4 NG Esprit A4NG | | | | Action 2 NG Action 3 NG* | | | |
|-------|----------|---|----|---|----------------------------|----|----|---|-----------------------------|----|-----|---|
| | L+R | L | R | | L+R | L | R | | L+R | L | R | |
| SW305 | - | - | - | - | - | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - | - | - | - | - | - |
| SW380 | 1 | 5 | 6 | | 1 | 5 | 6 | | 1 | 5 | 6 | |
| SW405 | 1 | 5 | 6 | | 1 | 5 | 6 | | 1 | 5 | 6 | |
| SW430 | 2 | 7 | 8 | | 2 | 7 | 8 | | 2 | 7 | 8 | |
| SW455 | 2 | 7 | 8 | | 2 | 7 | 8 | | 2 | 7 | 8 | |
| SW480 | 3 | 9 | 10 | | 3 | 9 | 10 | | 3 | 9 | 10 | |
| SW505 | 3 | 9 | 10 | | 3 | 9 | 10 | | 3* | 9* | 10* | |
| SW530 | 3 | 9 | 10 | | - | - | - | | - | - | - | |
| SW555 | - | - | - | | 4 | 11 | 12 | | - | - | - | |
| SW605 | - | - | - | | 4 | 11 | 12 | | - | - | - | |

| SW | L | R |
|-------|----|----|
| SW380 | 13 | 14 |
| SW405 | 13 | 14 |
| SW430 | 15 | 16 |
| SW455 | 15 | 16 |
| SW480 | 17 | 18 |
| SW505 | 17 | 18 |
| SW530 | 17 | 18 |
| SW555 | 19 | 20 |
| SW605 | 19 | 20 |

SPS1651965


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-------------------------------------|--------------------|-----|
| 1 | SP1651927 | Pair of Calfpad cplt 120 cpl | | 1 |
| 2 | SP1651928 | Pair of Calfpad cplt 145 cpl | | 1 |
| 3 | SP1651929 | Pair of Calfpad cplt 170 cpl | | 1 |
| 4 | SP1651937 | Pair of Calfpad cplt 195 cpl | | 1 |
| 5 | SP1605182 | Calfpad and Bracket cplt 120 Left | | 1 |
| 6 | SP1605181 | Calfpad and Bracket cplt 120 Right | | 1 |
| 7 | SP1605192 | Calfpad and Bracket cplt 145 Left | | 1 |
| 8 | SP1605013 | Calfpad and Bracket cplt 145 Right | | 1 |
| 9 | SP1605195 | Calfpad and Bracket cplt 170 Left | | 1 |
| 10 | SP1605193 | Calfpad and Bracket cplt 170 Right | | 1 |
| 11 | SP1605199 | Calfpad and Bracket cplt 195 Left | | 1 |
| 12 | SP1605198 | Calfpad and Bracket cplt 195 Right | | 1 |
| 13 | SP1605830 | Linkplate, Calfpad 120 Left | | 1 |
| 14 | SP1605828 | Linkplate, Calfpad 120 Right | | 1 |
| 15 | SP1605831 | Linkplate, Calfpad 145 Left | | 1 |
| 16 | SP1603256 | Linkplate, Calfpad 145 Right | | 1 |
| 17 | SP1605832 | Linkplate, Calfpad 170 Left | | 1 |
| 18 | SP1605829 | Linkplate, Calfpad 170 Right | | 1 |
| 19 | SP1605833 | Linkplate, Calfpad 195 Left | | 1 |
| 20 | SP1605834 | Linkplate, Calfpad 195 Right | | 1 |
| 21 | SP1602709 | Bracket assembly Calfpad Kit | | 2 |
| 22 | SP1651847 | Bridle Setting kit | | 2 |
| 22.1 | 1424571 | Springscrew | | 2 |
| 22.2 | SP1605203 | Screwsocket Legrest kit | | 2 |
| 23 | SP1647254 | Screw BHC M8x12 ISO7380-2 8.8 Black | | 2 |

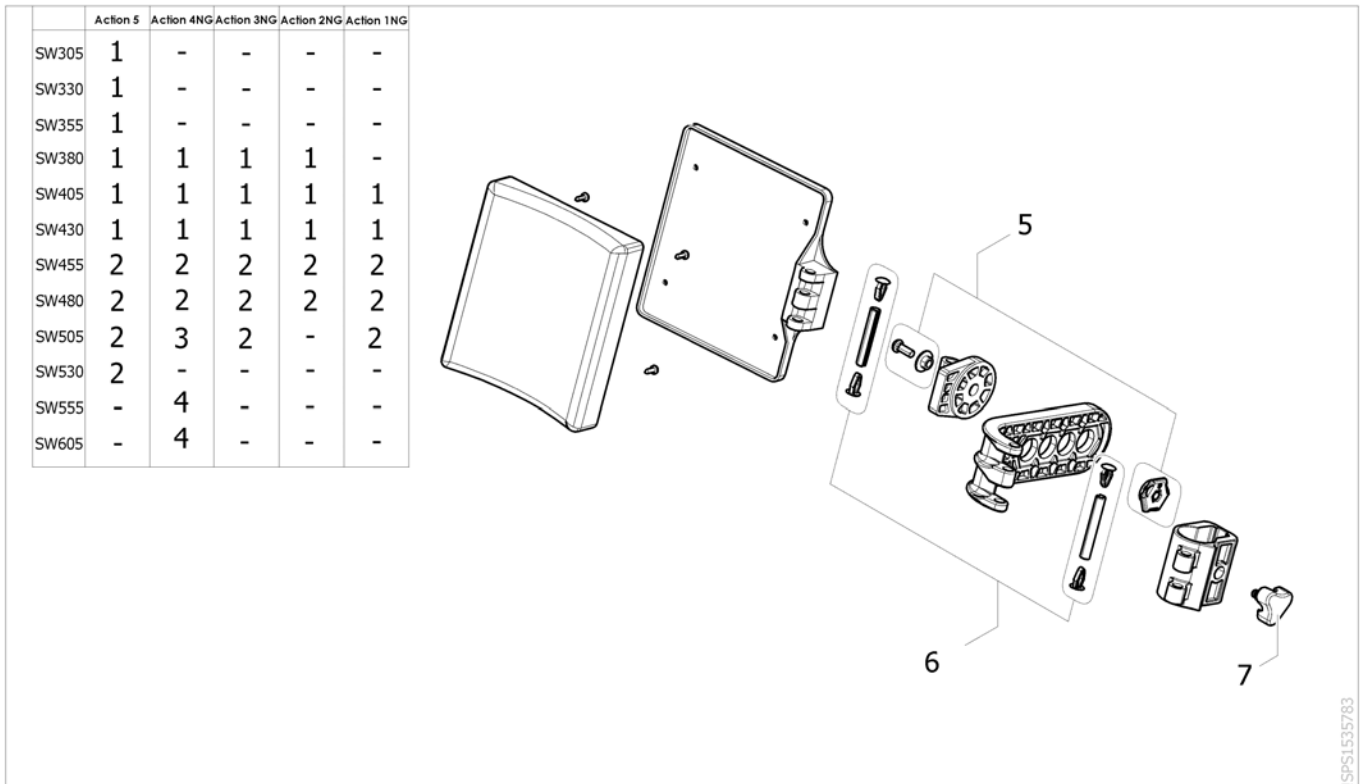

ISPS1535783

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 1450932 | Complete Polyurethane calf pad (cushion with support), SW380/405/430 mm | | 2 |
| 2 | 1450933 | Complete Polyurethane calf pad (cushion with support), SW455/480/505 mm | | 2 |
| 3 | 1450935 | Complete Dartex calf pad (cushion with support), SW480/505 mm | | 2 |
| 4 | 1450936 | Complete Dartex calf pad (cushion with support), SW555/605 mm | | 2 |
| 5 | 1440132 | Kit fixation angle adjustable calf-pad | | 2 |
| 6 | 1430432 | Kit articulation pin calf pad | | 2 |
| 7 | 1440943 | Locking star knob | | 2 |


Amputee Legrest

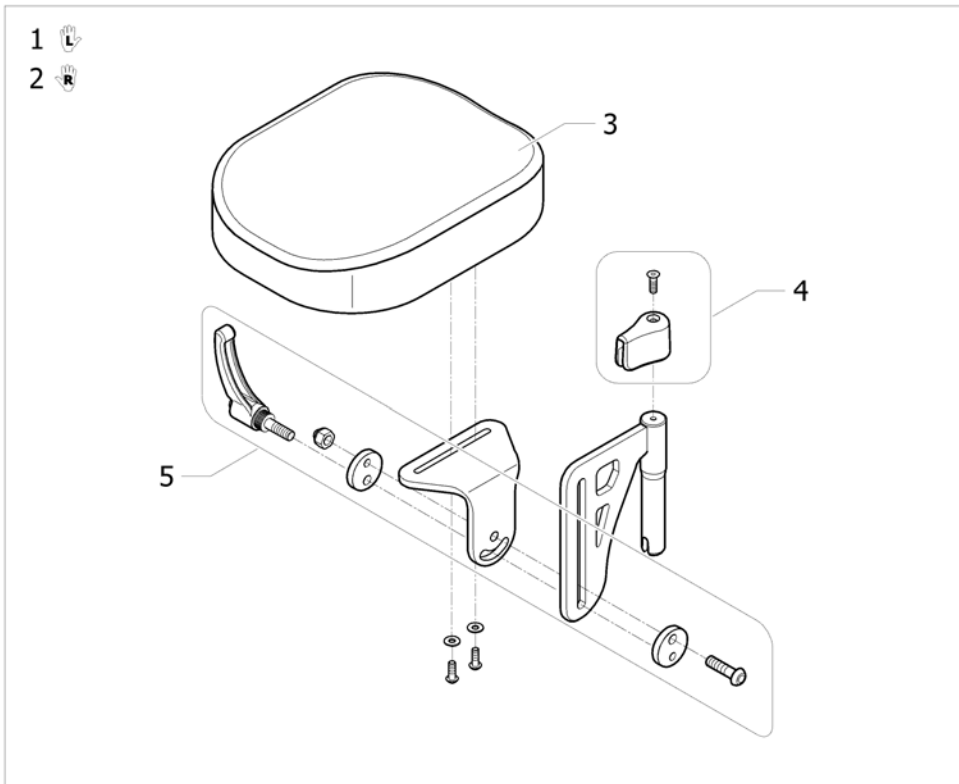

ISP51495897


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1442991 | Legrest, Amputee, Lh | | 1 |
| 2 | 1442990 | Legrest, Amputee, Rh | | 1 |
| 3 | 1497450 | Cushion amputee legrest | | 2 |
| 4 | 1446556 | Amputee legrest stop + fixation | | 2 |
| 5 | 1446557 | Amputee legrest slide fixation | | 2 |
| 6 | SP1584380 | Kit hanger locked (2015) <i>Replace 1430427</i> | | 2 |

Footrests

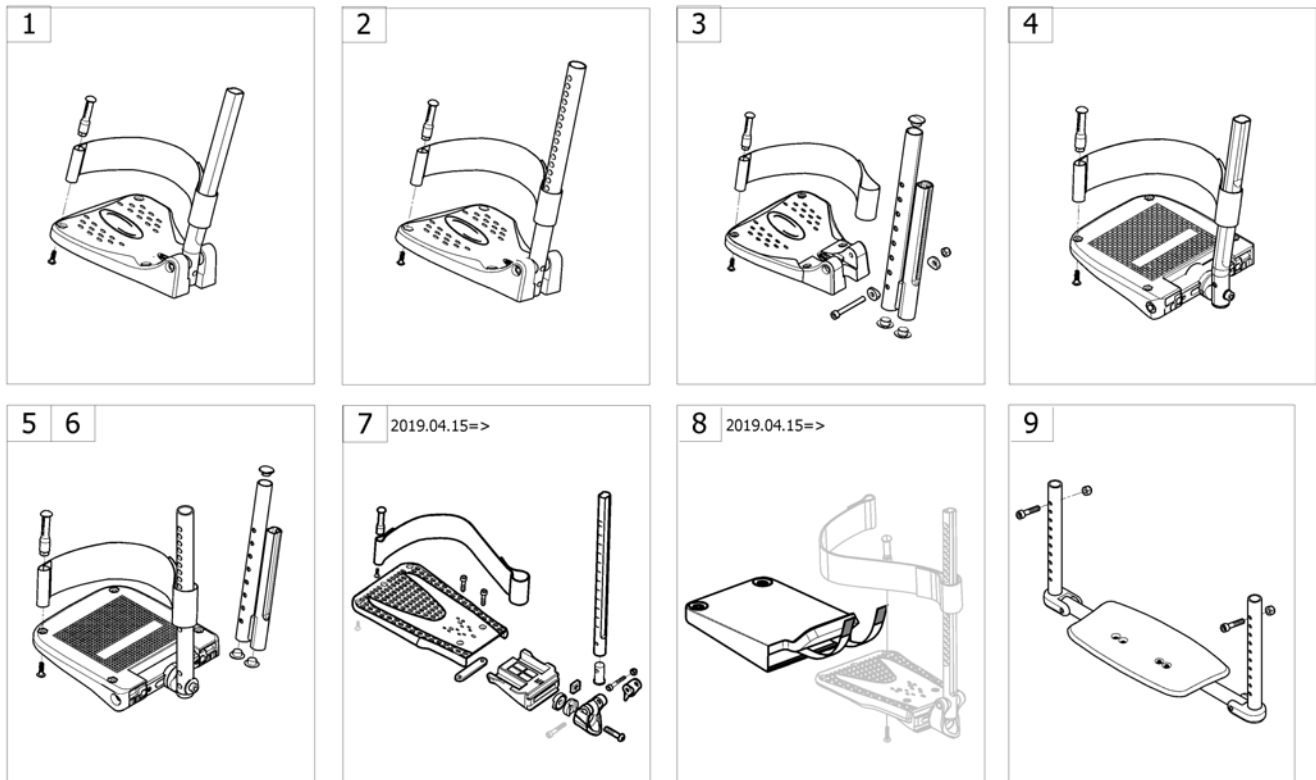

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Footrest fixed, swing away frame [SJ_0200+(SJ_3170/SJ_3200/SJ_3300)+(SJ_3420/SJ_3430)] | | 1 |
| 2 | - | Footrest fixed, fixed frame [SJ_0210+(SJ_3170/SJ_3200/SJ_3300)] | | 1 |
| 3 | - | Footrest fixed, high mounted tube, swing away frame [(SJ_3170/SJ_3200/SJ_3300)+SJ_3400] | | 1 |
| 4 | - | Footrest angle & depth adjustable, swing away frame [SJ_0200+(SJ_3175/SJ_3210/SJ_3310)+(SJ_3410/SJ_3420/SJ_3430)] | | 1 |
| 5 | - | Footrest angle & depth adjustable, fixed frame [SJ_0210+(SJ_3175/SJ_3210/SJ_3310)+(SJ_3440/SJ_3450)] | | 1 |
| 6 | - | Footrest angle & depth adjustable, high mounted tube, swing away frame [(SJ_3175/SJ_3210/SJ_3310)+SJ_3400] | | 1 |
| 7 | - | Angle and depth adjustable footrest AA-Legrest(Medium) 2019.04.15 => | | 1 |
| 8 | - | Footplate cover Medium 2019.04.15 => | | 1 |
| 9 | - | One piece footplate for fixed frame [SJ_0210+(SJ_3185/SJ_3230)+(SJ_3440/SJ_3450)] | | 1 |

Footrest fixed, swing away frame

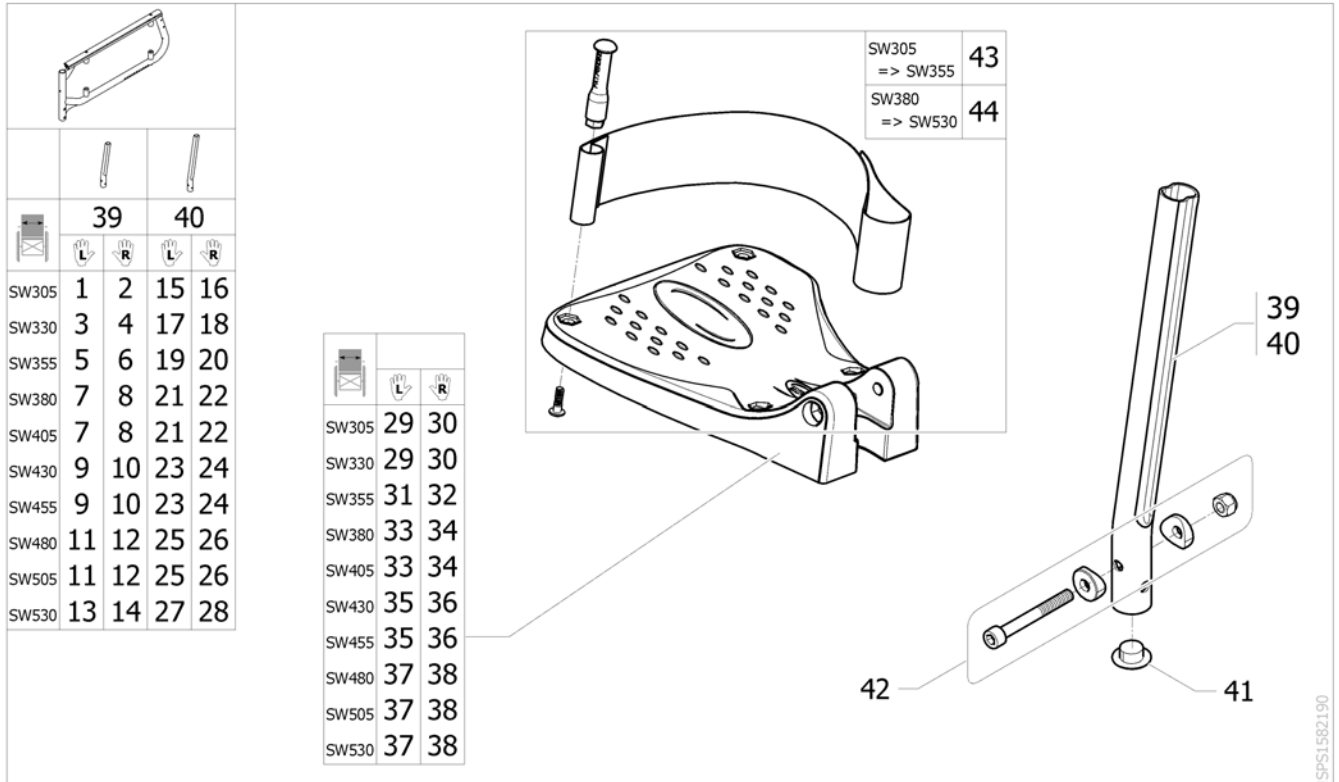


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1586899 | Footrest fixed, Lh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 2 | SP1586900 | Footrest fixed, Rh, SW305mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 3 | SP1586901 | Footrest fixed, Lh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 4 | SP1587031 | Footrest fixed, Rh, SW330mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 5 | SP1587179 | Footrest fixed, Lh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 6 | SP1587215 | Footrest fixed, Rh, SW355mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 7 | 1530620 | Footrest fixed, Lh, SW380-405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 8 | 1530614 | Footrest fixed, Rh, SW380-405mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 9 | 1530622 | Footrest fixed, Lh, SW430-455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 10 | 1530616 | Footrest fixed, Rh, SW430-455mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 11 | 1530623 | Footrest fixed, Lh, SW480-505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 12 | 1530617 | Footrest fixed, Rh, SW480-505mm, standard tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 13 | SP1601715 | Footrest fixed, Lh, SW530mm, standard tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3420]</i> | | 1 |
| 14 | SP1601716 | Footrest fixed, Rh, SW530mm, standard tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3420]</i> | | 1 |
| 15 | SP1587273 | Footrest fixed, Lh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 16 | SP1587274 | Footrest fixed, Rh, SW305mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 17 | SP1587275 | Footrest fixed, Lh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |

Footrest fixed, swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|--------------|---|--------------------|-----|
| 18 | SP1587312 | Footrest fixed, Rh, SW330mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 19 | SP1587313 | Footrest fixed, Lh, SW355mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 20 | SP1587405 | Footrest fixed, Rh, SW355mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 21 | 1530633 | Footrest fixed, Lh, SW380-405mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 22 | 1530630 | Footrest fixed, Rh, SW380-405mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 23 | 1530634 | Footrest fixed, Lh, SW330-455mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 24 | 1530631 | Footrest fixed, Rh, SW330-455mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 25 | 1530635 | Footrest fixed, Lh, SW480-505mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 26 | 1530632 | Footrest fixed, Rh, SW480-505mm, long tube, swing away frame <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 27 | SP1601762 | Footrest fixed, Lh, SW530mm, long tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3300)+SJ_3430]</i> | | 1 |
| 28 | SP1601763 | Footrest fixed, Rh, SW530mm, long tube, swing away frame + Heel strap <i>[SJ_0200+(SJ_3170/SJ_3200)+SJ_3430]</i> | | 1 |
| 29 | 0572-04433 | Footplate, fixed, Lh, 133mm | | 1 |
| 30 | 0572-04447 | Footplate, fixed, Rh, 133mm | | 1 |
| 31 | 0572-04434 | Footplate, fixed, Lh, 163mm | | 1 |
| 32 | 0572-04448 | Footplate, fixed, Rh, 163mm | | 1 |
| 33 | 1529643 | Footplate, fixed, Lh, 174mm (V2009) | | 1 |
| 34 | 1529642 | Footplate, fixed, Rh, 174mm (V2009) | | 1 |
| 35 | 1529645 | Footplate, fixed, Lh, 199mm (V2009) | | 1 |
| 36 | 1529644 | Footplate, fixed, Rh, 199mm (V2009) | | 1 |
| 37 | 1529647 | Footplate, fixed, Lh, 224mm (V2009) | | 1 |
| 38 | 1529646 | Footplate, fixed, Rh, 224mm (V2009) | | 1 |
| 39 | 1434561-0010 | Teleskopic tube short for legrest Compact Attract <i>[SJ_0200+SJ_3420]</i> | | 2 |
| 40 | 1434563-0010 | Footrest Tube, Long <i>[SJ_0200+SJ_3430]</i> | | 2 |
| 41 | 1424576 | Plastic cap black Ø20mm | | 2 |
| 42 | 1430433 | Screw Kit, Footrest Fixed | | 2 |
| 43 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |
| 44 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Footrest fixed, fixed frame

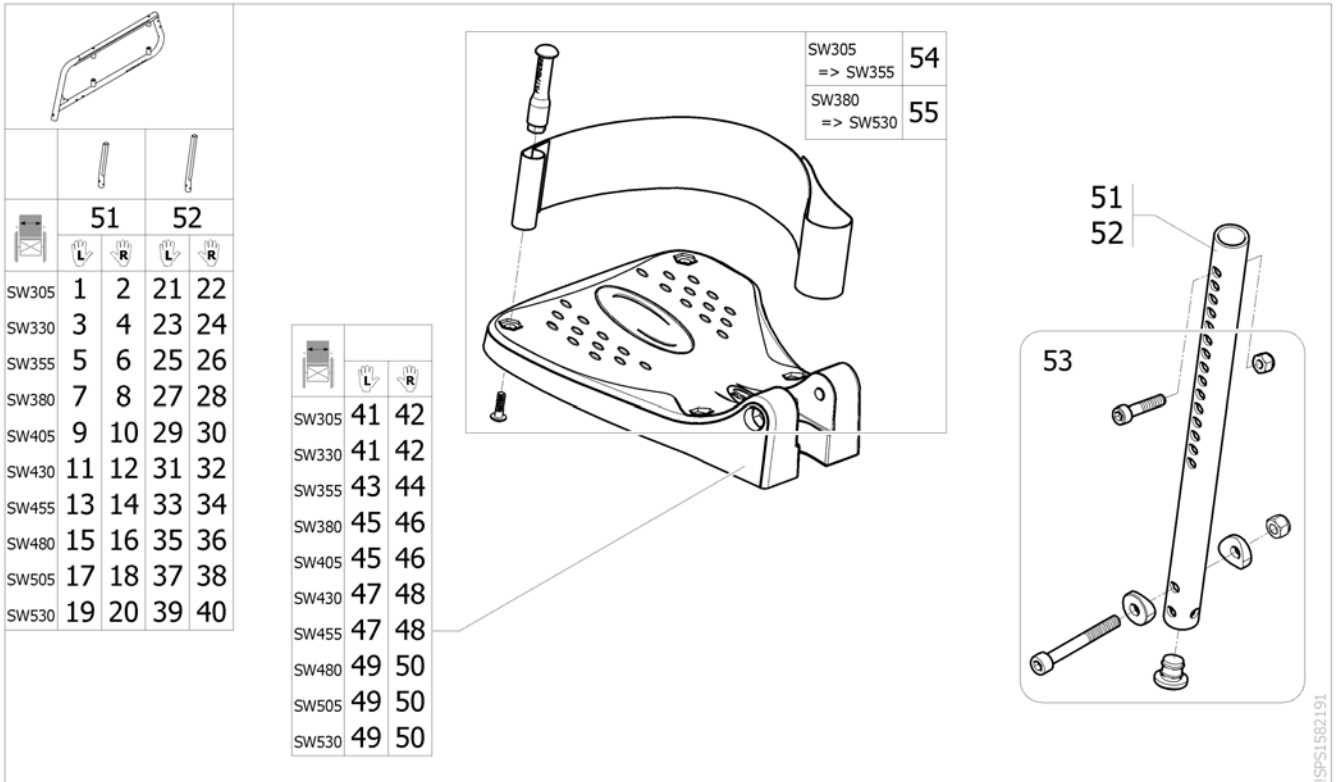


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1590821 | Footrest fixed, Lh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 2 | SP1590822 | Footrest fixed, Rh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 3 | SP1590823 | Footrest fixed, Lh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 4 | SP1590829 | Footrest fixed, Rh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 5 | SP1590830 | Footrest fixed, Lh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 6 | SP1590831 | Footrest fixed, Rh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 7 | SP1590846 | Footrest fixed, Lh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 8 | SP1590848 | Footrest fixed, Rh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 9 | SP1586809 | Footrest fixed, Lh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 10 | SP1586810 | Footrest fixed, Rh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 11 | SP1586811 | Footrest fixed, Lh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 12 | SP1586812 | Footrest fixed, Rh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 13 | SP1586813 | Footrest fixed, Lh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 14 | SP1586814 | Footrest fixed, Rh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 15 | SP1586815 | Footrest fixed, Lh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 16 | SP1586816 | Footrest fixed, Rh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 17 | SP1586817 | Footrest fixed, Lh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 18 | SP1586818 | Footrest fixed, Rh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |

Footrest fixed, fixed frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 19 | SP1601764 | Footrest fixed, Lh, SW530mm, standard tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3440]</i> | | 1 |
| 20 | SP1601783 | Footrest fixed, Rh, SW530mm, standard tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3440]</i> | | 1 |
| 21 | SP1590970 | Footrest fixed, Lh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 22 | SP1590971 | Footrest fixed, Rh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 23 | SP1590980 | Footrest fixed, Lh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 24 | SP1590981 | Footrest fixed, Rh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 25 | SP1590982 | Footrest fixed, Lh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 26 | SP1590989 | Footrest fixed, Rh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 27 | SP1590990 | Footrest fixed, Lh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 28 | SP1590991 | Footrest fixed, Rh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 29 | SP1590992 | Footrest fixed, Lh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 30 | SP1590993 | Footrest fixed, Rh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 31 | SP1590994 | Footrest fixed, Lh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 32 | SP1590995 | Footrest fixed, Rh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 33 | SP1590996 | Footrest fixed, Lh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 34 | SP1590997 | Footrest fixed, Rh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 35 | SP1590998 | Footrest fixed, Lh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 36 | SP1591018 | Footrest fixed, Rh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 37 | SP1591019 | Footrest fixed, Lh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 38 | SP1591031 | Footrest fixed, Rh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 39 | SP1601784 | Footrest fixed, Lh, SW530mm, long tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3300)+SJ_3450]</i> | | 1 |
| 40 | SP1601785 | Footrest fixed, Rh, SW530mm, long tube, fixed frame + Heel strap <i>[SJ_0210+(SJ_3170/SJ_3200)+SJ_3450]</i> | | 1 |
| 41 | 0572-04433 | Footplate, fixed, Lh, 133mm | | 1 |
| 42 | 0572-04447 | Footplate, fixed, Rh, 133mm | | 1 |
| 43 | 0572-04434 | Footplate, fixed, Lh, 163mm | | 1 |
| 44 | 0572-04448 | Footplate, fixed, Rh, 163mm | | 1 |
| 45 | 1529643 | Footplate, fixed, Lh, 174mm (V2009) | | 1 |
| 46 | 1529642 | Footplate, fixed, Rh, 174mm (V2009) | | 1 |
| 47 | 1529645 | Footplate, fixed, Lh, 199mm (V2009) | | 1 |
| 48 | 1529644 | Footplate, fixed, Rh, 199mm (V2009) | | 1 |
| 49 | 1529647 | Footplate, fixed, Lh, 224mm (V2009) | | 1 |
| 50 | 1529646 | Footplate, fixed, Rh, 224mm (V2009) | | 1 |
| 51 | SP1511849-0010 | Footrest tube standard straight for fixed frame <i>[SJ_0210+SJ_3440]</i> | | 2 |
| 52 | SP1506244-0699 | Footrest tube long straight for fixed frame <i>[SJ_0210+SJ_3450]</i> | | 2 |
| 53 | 1434612 | Kit Fixation Footrest Tube | | 2 |
| 54 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |
| 55 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Footrest fixed, high mounted tube, swing away frame

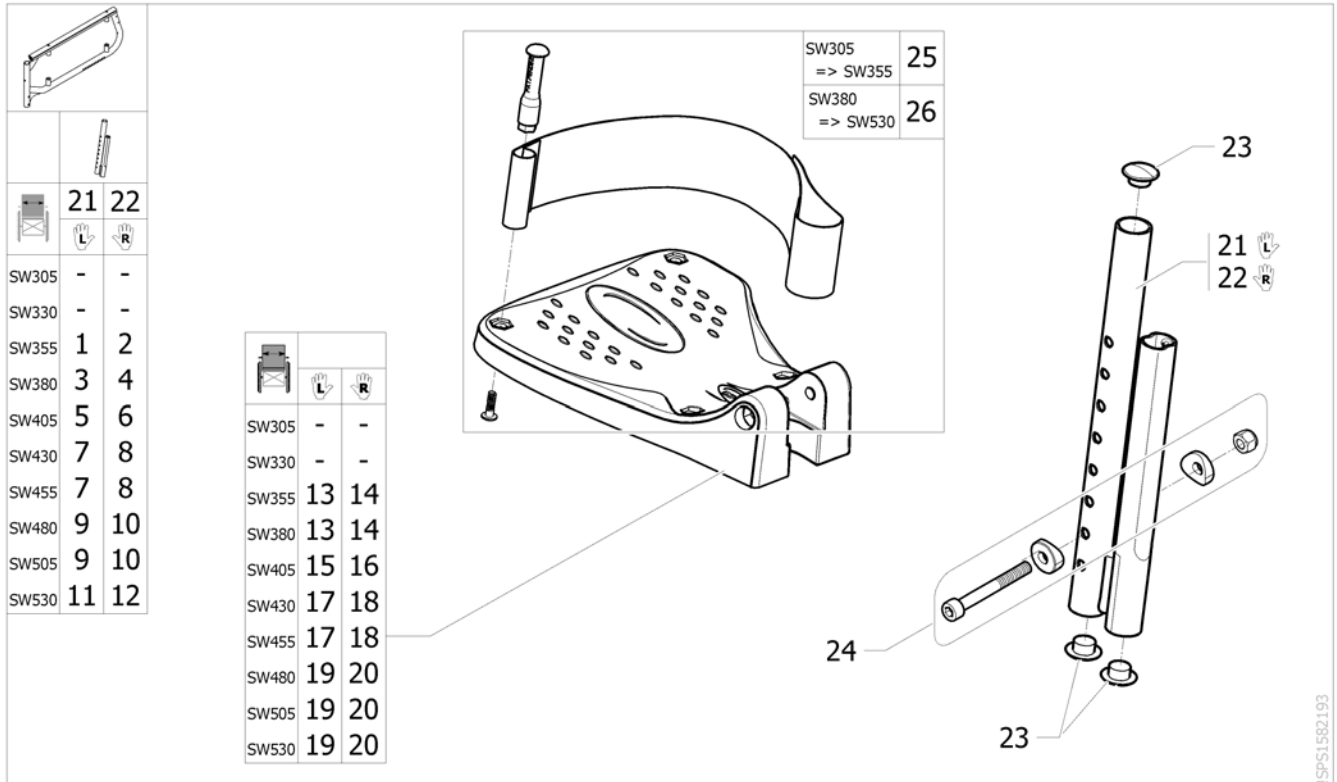


| Pos. | Part number | Description | Valid from → until | Qty |
|------|--------------|--|--------------------|-----|
| 1 | SP1587842 | Footrest fixed, Lh, SW355mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 2 | SP1587843 | Footrest fixed, Rh, SW355mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 3 | SP1587844 | Footrest fixed, Lh, SW380mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 4 | SP1587845 | Footrest fixed, Rh, SW380mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 5 | SP1588156 | Footrest fixed, Lh, SW405mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 6 | SP1588157 | Footrest fixed, Rh, SW405mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 7 | 1532070 | Footrest fixed, Lh, SW430-455mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 8 | 1532066 | Footrest fixed, Rh, SW430-455mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 9 | 1532072 | Footrest fixed, Lh, SW480-505mm, high mounted tube [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 10 | 1532068 | Footrest fixed, Rh, SW480-505mm, high mounted tube [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 11 | SP1601786 | Footrest fixed, Lh, SW530mm, high mounted tube + Heel strap [(SJ_3170/SJ_3300)+SJ_3400] | | 1 |
| 12 | SP1601787 | Footrest fixed, Rh, SW530mm, high mounted tube + Heel strap [(SJ_3170/SJ_3200)+SJ_3400] | | 1 |
| 13 | 0572-04433 | Footplate, fixed, Lh, 133mm | | 1 |
| 14 | 0572-04447 | Footplate, fixed, Rh, 133mm | | 1 |
| 15 | 0572-04434 | Footplate, fixed, Lh, 163mm | | 1 |
| 16 | 0572-04448 | Footplate, fixed, Rh, 163mm | | 1 |
| 17 | 1529643 | Footplate, fixed, Lh, 174mm (V2009) | | 1 |
| 18 | 1529642 | Footplate, fixed, Rh, 174mm (V2009) | | 1 |
| 19 | 1529645 | Footplate, fixed, Lh, 199mm (V2009) | | 1 |
| 20 | 1529644 | Footplate, fixed, Rh, 199mm (V2009) | | 1 |
| 21 | 1531202-0699 | Footrest tube high mounted Lh [SJ_3400] | | 1 |
| 23 | 1424576 | Plastic cap black Ø20mm | | 6 |

Footrest fixed, high mounted tube, swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|--------------|---|--------------------|-----|
| 22 | 1531203-0699 | Footrest tube high mounted Rh <i>[SJ_3400]</i> | | 1 |
| 24 | 1430433 | Screw Kit, Footrest Fixed | | 2 |
| 25 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |
| 26 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Footrest angle & depth adjustable, swing away frame

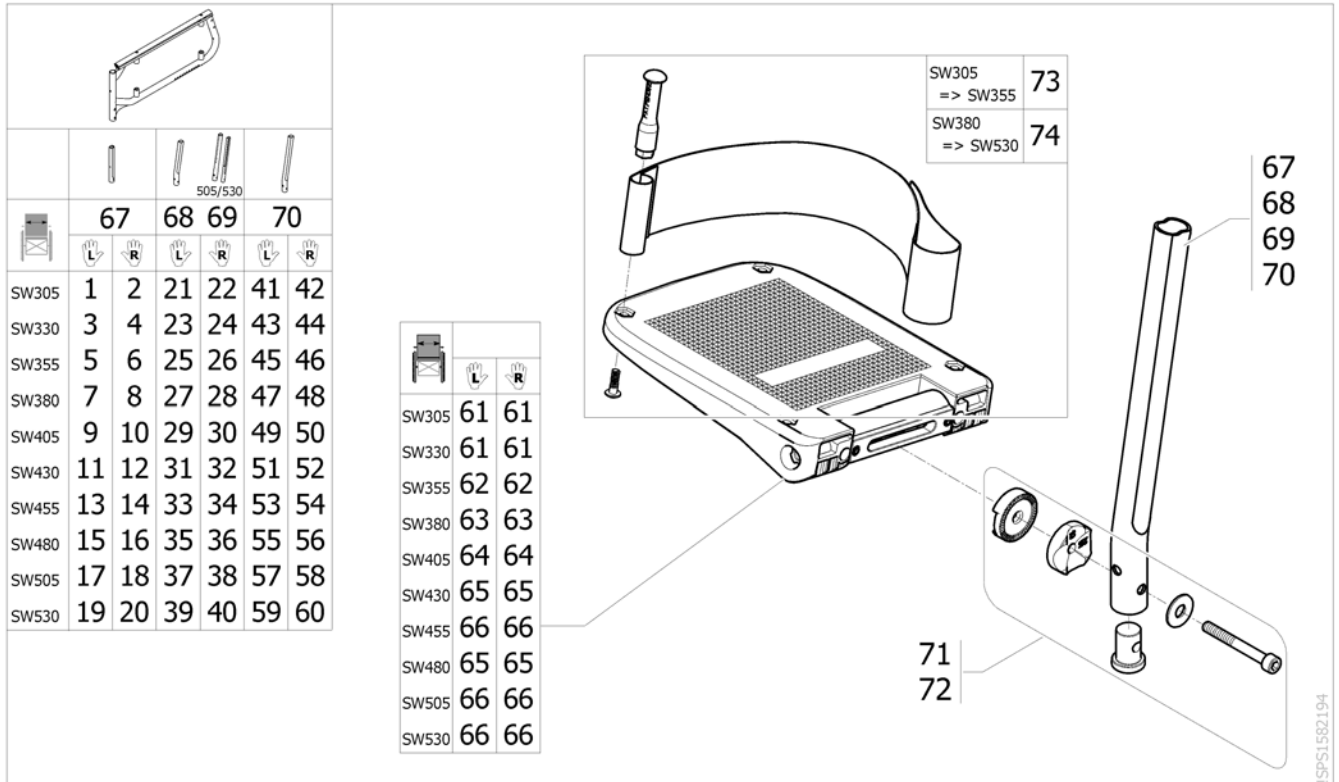

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1587834 | Footrest a&d adjustable, Lh, SW305mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 2 | SP1587835 | Footrest a&d adjustable, Rh, SW305mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 3 | SP1587836 | Footrest a&d adjustable, Lh, SW330mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 4 | SP1587837 | Footrest a&d adjustable, Rh, SW330mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 5 | SP1587838 | Footrest a&d adjustable, Lh, SW355mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 6 | SP1587839 | Footrest a&d adjustable, Rh, SW355mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 7 | 1548477 | Footrest a&d adjustable, Lh, SW380mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 8 | 1548478 | Footrest a&d adjustable, Rh, SW380mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 9 | 1548487 | Footrest a&d adjustable, Lh, SW405mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 10 | 1548488 | Footrest a&d adjustable, Rh, SW405mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 11 | 1548489 | Footrest a&d adjustable, Lh, SW430mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 12 | 1548490 | Footrest a&d adjustable, Rh, SW430mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 13 | 1548491 | Footrest a&d adjustable, Lh, SW455mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 14 | 1548492 | Footrest a&d adjustable, Rh, SW455mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 15 | 1548518 | Footrest a&d adjustable, Lh, SW480mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 16 | 1548519 | Footrest a&d adjustable, Rh, SW480mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |
| 17 | 1548520 | Footrest a&d adjustable, Lh, SW505mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3310)+SJ_3410]</i> | | 1 |
| 18 | 1548521 | Footrest a&d adjustable, Rh, SW505mm, short tube, swing away frame <i>[SJ_0200+(SJ_3175/SJ_3210)+SJ_3410]</i> | | 1 |


Footrest angle & depth adjustable, swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 19 | SP1601788 | Footrest a&d adjustable, Lh, SW530mm, short tube, s/a frame + heel [SJ_0200+(SJ_3175/SJ_3310)+SJ_3410] | | 1 |
| 20 | SP1601789 | Footrest a&d adjustable, Rh, SW530mm, short tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3210)+SJ_3410] | | 1 |
| 21 | SP1587406 | Footrest a&d adjustable, Lh, SW305mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 22 | SP1587438 | Footrest a&d adjustable, Rh, SW305mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 23 | SP1587439 | Footrest a&d adjustable, Lh, SW330mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 24 | SP1587440 | Footrest a&d adjustable, Rh, SW330mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 25 | SP1587441 | Footrest a&d adjustable, Lh, SW355mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 26 | SP1587442 | Footrest a&d adjustable, Rh, SW355mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 27 | 1428657 | Footrest a&d adjustable, Lh, SW380mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 28 | 1428651 | Footrest a&d adjustable, Rh, SW380mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 29 | 1428658 | Footrest a&d adjustable, Lh, SW405mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 30 | 1428652 | Footrest a&d adjustable, Rh, SW405mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 31 | 1428659 | Footrest a&d adjustable, Lh, SW430mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 32 | 1428653 | Footrest a&d adjustable, Rh, SW430mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 33 | 1428660 | Footrest a&d adjustable, Lh, SW455mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 34 | 1428654 | Footrest a&d adjustable, Rh, SW455mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 35 | 1428661 | Footrest a&d adjustable, Lh, SW480mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 36 | 1428655 | Footrest a&d adjustable, Rh, SW480mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 37 | 1532964 | Footrest a&d adjustable, Lh, SW505mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 38 | 1532963 | Footrest a&d adjustable, Rh, SW505mm, standard tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 39 | SP1601790 | Footrest a&d adjustable, Lh, SW530mm, standard tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3310)+SJ_3420] | | 1 |
| 40 | SP1601791 | Footrest a&d adjustable, Rh, SW530mm, standard tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3210)+SJ_3420] | | 1 |
| 41 | SP1587443 | Footrest a&d adjustable, Lh, SW305mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 42 | SP1587444 | Footrest a&d adjustable, Rh, SW305mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 43 | SP1587445 | Footrest a&d adjustable, Lh, SW330mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 44 | SP1587446 | Footrest a&d adjustable, Rh, SW330mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 45 | SP1587447 | Footrest a&d adjustable, Lh, SW355mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |

Footrest angle & depth adjustable, swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|--------------|--|--------------------|-----|
| 46 | SP1587448 | Footrest a&d adjustable, Rh, SW355mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 47 | 1440816 | Footrest a&d adjustable, Lh, SW380mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 48 | 1440810 | Footrest a&d adjustable, Rh, SW380mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 49 | 1440817 | Footrest a&d adjustable, Lh, SW405mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 50 | 1440811 | Footrest a&d adjustable, Rh, SW405mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 51 | 1440818 | Footrest a&d adjustable, Lh, SW430mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 52 | 1440812 | Footrest a&d adjustable, Rh, SW430mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 53 | 1440819 | Footrest a&d adjustable, Lh, SW455mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 54 | 1440813 | Footrest a&d adjustable, Rh, SW455mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 55 | 1440820 | Footrest a&d adjustable, Lh, SW480mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 56 | 1440814 | Footrest a&d adjustable, Rh, SW480mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 57 | 1440821 | Footrest a&d adjustable, Lh, SW505mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 58 | 1440815 | Footrest a&d adjustable, Rh, SW505mm, long tube, swing away frame [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 59 | SP1601792 | Footrest a&d adjustable, Lh, SW530mm, long tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3310)+SJ_3430] | | 1 |
| 60 | SP1601793 | Footrest a&d adjustable, Rh, SW530mm, long tube, s/a frame + heel strap [SJ_0200+(SJ_3175/SJ_3210)+SJ_3430] | | 1 |
| 61 | 5106371 | Footplate, angle and depht adjustable, 106 mm | | 2 |
| 62 | 5106373 | Footplate, angle and depth adjustable, 131 mm | | 2 |
| 63 | 5106374 | Footplate, angle and depth adjustable, 141 mm | | 2 |
| 64 | 5106375 | Footplate, angle and depth adjustable, 156 mm | | 2 |
| 65 | 5106594 | Footplate, angle and depth adjustable, 166 mm | | 1 |
| 66 | 5106376 | Footplate, angle and depth adjustable, 176 mm | | 1 |
| 67 | 1541994 | Short footrest tube for a&d adjustable footrest [SJ_0200+SJ_3410] | | 2 |
| 68 | 1558855-0010 | Standard tube for a&d adjustable footrest [SJ_0200+(SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040/SJ_1050/SJ_1060/SJ_1070)+SJ_3420] | | 2 |
| 69 | 1533719 | Standard footrest tube with heavy bar [SJ_0200+(SJ_1075/SJ_1078/SJ_1080/SJ_1090)+SJ_3420] | | 1 |
| 70 | 1434563-0010 | Footrest Tube, Long [SJ_0200+SJ_3430] | | 2 |
| 71 | 1430839 | Screw Kit, Footrest aa | | 2 |
| 72 | 1430840 | Screw Kit, Footrest aa Ext | | 2 |
| 73 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |
| 74 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Footrest angle & depth adjustable, fixed frame

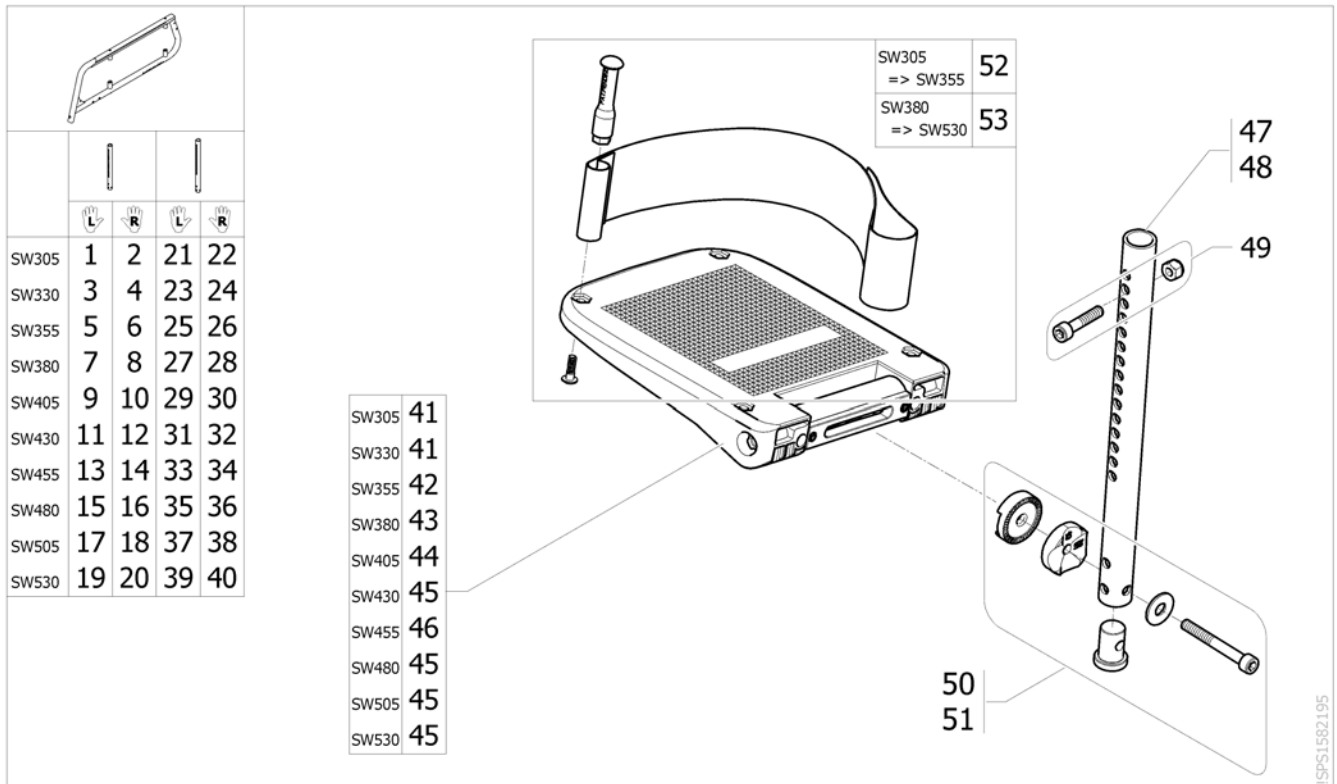

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1591045 | Footrest a&d adjustable, Lh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 2 | SP1591046 | Footrest a&d adjustable, Rh, SW305mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 3 | SP1591047 | Footrest a&d adjustable, Lh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 4 | SP1591052 | Footrest a&d adjustable, Rh, SW330mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 5 | SP1591053 | Footrest a&d adjustable, Lh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 6 | SP1591054 | Footrest a&d adjustable, Rh, SW355mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 7 | SP1591055 | Footrest a&d adjustable, Lh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 8 | SP1591056 | Footrest a&d adjustable, Rh, SW380mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 9 | SP1587513 | Footrest a&d adjustable, Lh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 10 | SP1587604 | Footrest a&d adjustable, Rh, SW405mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 11 | SP1587615 | Footrest a&d adjustable, Lh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 12 | SP1587616 | Footrest a&d adjustable, Rh, SW430mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 13 | SP1587617 | Footrest a&d adjustable, Lh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 14 | SP1587698 | Footrest a&d adjustable, Rh, SW455mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 15 | SP1587699 | Footrest a&d adjustable, Lh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 16 | SP1587747 | Footrest a&d adjustable, Rh, SW480mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 17 | SP1587748 | Footrest a&d adjustable, Lh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 18 | SP1587749 | Footrest a&d adjustable, Rh, SW505mm, standard tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |


Footrest angle & depth adjustable, fixed frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 19 | SP1590816 | Footrest a&d adjustable, Lh, SW530mm, standard tube, fixed frame +heel strap <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3440]</i> | | 1 |
| 20 | SP1590817 | Footrest a&d adjustable, Rh, SW530mm, standard tube, fixed frame +heel strap <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3440]</i> | | 1 |
| 21 | SP1587451 | Footrest a&d adjustable, Lh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 22 | SP1587452 | Footrest a&d adjustable, Rh, SW305mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 23 | SP1587453 | Footrest a&d adjustable, Lh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 24 | SP1587454 | Footrest a&d adjustable, Rh, SW330mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 25 | SP1587455 | Footrest a&d adjustable, Lh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 26 | SP1587456 | Footrest a&d adjustable, Rh, SW355mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 27 | SP1587457 | Footrest a&d adjustable, Lh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 28 | SP1587458 | Footrest a&d adjustable, Rh, SW380mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 29 | SP1587459 | Footrest a&d adjustable, Lh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 30 | SP1587460 | Footrest a&d adjustable, Rh, SW405mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 31 | SP1587461 | Footrest a&d adjustable, Lh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 32 | SP1587462 | Footrest a&d adjustable, Rh, SW430mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 33 | SP1587463 | Footrest a&d adjustable, Lh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 34 | SP1587464 | Footrest a&d adjustable, Rh, SW455mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 35 | SP1587465 | Footrest a&d adjustable, Lh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 36 | SP1587466 | Footrest a&d adjustable, Rh, SW480mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 37 | SP1587467 | Footrest a&d adjustable, Lh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 38 | SP1587468 | Footrest a&d adjustable, Rh, SW505mm, long tube, fixed frame <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 39 | SP1601794 | Footrest a&d adjustable, Lh, SW530mm, long tube, fixed frame + heel strap <i>[SJ_0210+(SJ_3175/SJ_3310)+SJ_3450]</i> | | 1 |
| 40 | SP1601795 | Footrest a&d adjustable, Rh, SW530mm, long tube, fixed frame + heel strap <i>[SJ_0210+(SJ_3175/SJ_3210)+SJ_3450]</i> | | 1 |
| 41 | 5106371 | Footplate, angle and depth adjustable, 106 mm | | 2 |
| 42 | 5106373 | Footplate, angle and depth adjustable, 131 mm | | 2 |
| 43 | 5106374 | Footplate, angle and depth adjustable, 141 mm | | 2 |
| 44 | 5106375 | Footplate, angle and depth adjustable, 156 mm | | 2 |
| 45 | 5106594 | Footplate, angle and depth adjustable, 166 mm | | 2 |
| 46 | 5106376 | Footplate, angle and depth adjustable, 176 mm | | 2 |
| 47 | SP1511849-0010 | Footrest tube standard straight for fixed frame <i>[SJ_0210+SJ_3440]</i> | | 2 |
| 48 | SP1506244-0699 | Footrest tube long straight for fixed frame <i>[SJ_0210+SJ_3450]</i> | | 2 |
| 49 | SP1602122 | Kit fixation, footrest adj tube with fixed frame | | 2 |
| 50 | SP1641026 | Kit fixation, adjustable footplate, Short spacer <i>[SJ_1000/SJ_1010/SJ_1020/SJ_1030/SJ_1040/SJ_1050/SJ_1060]</i> | | 2 |
| 51 | 1430840 | Screw Kit, Footrest aa Ext <i>[SJ_1070/SJ_1075/SJ_1078/SJ_1080/SJ_1090]</i> | | 2 |
| 52 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |

Footrest angle & depth adjustable, fixed frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-----------------------------------|--------------------|-----|
| 53 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Footrest angle & depth adjustable, high mounted tube, swing away frame

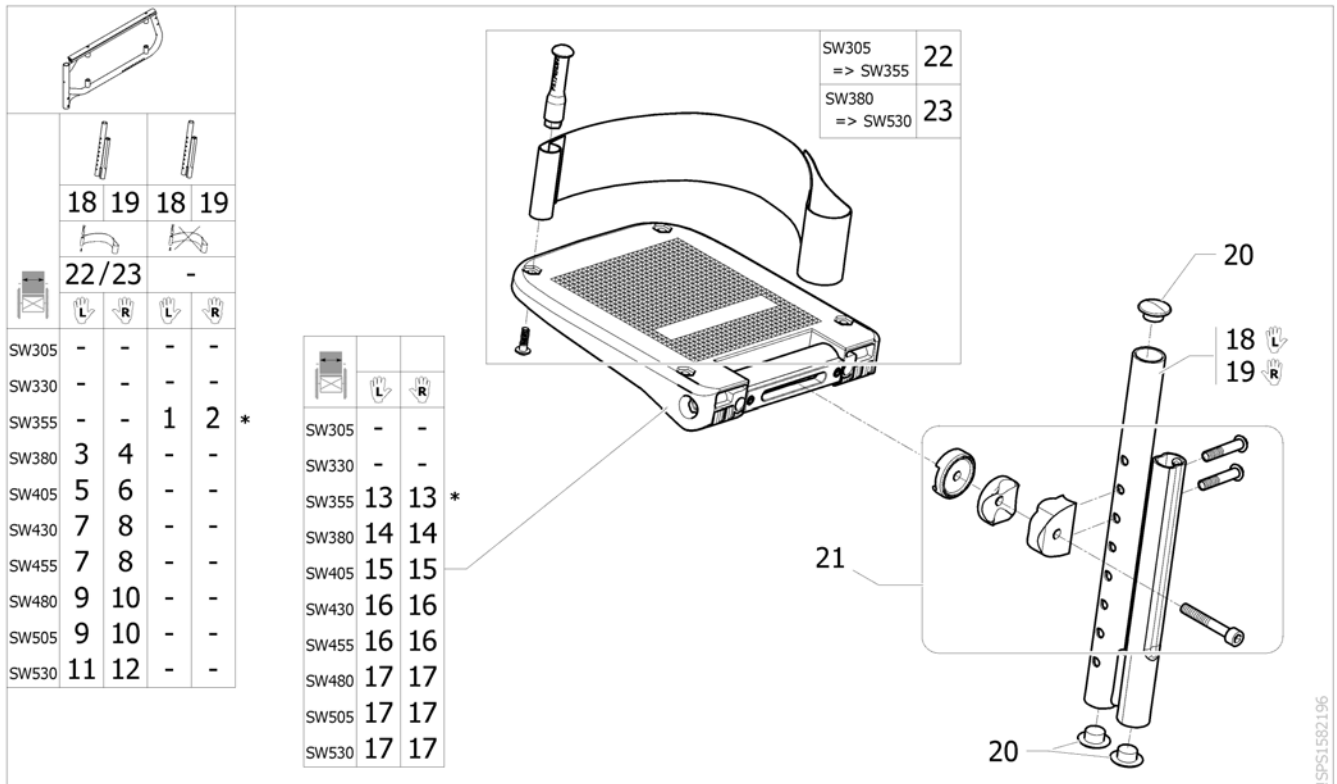


| Pos. | Part number | Description | Valid from → until | Qty |
|------|--------------|---|--------------------|-----|
| 1 | SP1588277 | Footrest a&d adjustable, Lh, SW355mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400] / Without Heel strap | | 1 |
| 2 | SP1588278 | Footrest a&d adjustable, Rh, SW355mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400] / Without Heel strap | | 1 |
| 3 | SP1586054 | Footrest a&d adjustable, Lh, SW380mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400] | | 1 |
| 4 | SP1586052 | Footrest a&d adjustable, Rh, SW380mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400] | | 1 |
| 5 | SP1586068 | Footrest a&d adjustable, Lh, SW405mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400] | | 1 |
| 6 | SP1586053 | Footrest a&d adjustable, Rh, SW405mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400] | | 1 |
| 7 | 1532078 | Footrest a&d adjustable, Lh, SW430-455mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400] | | 1 |
| 8 | 1532074 | Footrest a&d adjustable, Rh, SW430-455mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400] | | 1 |
| 9 | 1532080 | Footrest a&d adjustable, Lh, SW480-505mm, high mounted tube [(SJ_3175/SJ_3310)+SJ_3400] | | 1 |
| 10 | 1532076 | Footrest a&d adjustable, Rh, SW480-505mm, high mounted tube [(SJ_3175/SJ_3210)+SJ_3400] | | 1 |
| 11 | SP1601796 | Footrest a&d adjustable, Lh, SW530mm, high mounted tube + heel strap [(SJ_3175/SJ_3310)+SJ_3400] | | 1 |
| 12 | SP1601797 | Footrest a&d adjustable, Rh, SW530mm, high mounted tube + heel strap [(SJ_3175/SJ_3210)+SJ_3400] | | 1 |
| 13 | 1518638 | Footplate, angle and depth adjustable, 82 mm | | 1 |
| 14 | 5106371 | Footplate, angle and depth adjustable, 106 mm | | 1 |
| 15 | 5106373 | Footplate, angle and depth adjustable, 131 mm | | 1 |
| 16 | 5106374 | Footplate, angle and depth adjustable, 141 mm | | 1 |
| 17 | 5106594 | Footplate, angle and depth adjustable, 166 mm | | 1 |
| 18 | 1531202-0699 | Footrest tube high mounted Lh [SJ_3400] | | 1 |
| 19 | 1531203-0699 | Footrest tube high mounted Rh [SJ_3400] | | 1 |
| 20 | 1424576 | Plastic cap black Ø20mm | | 6 |
| 21 | 1533714 | Kit fixation for angle & depth adjustable footrest, high mounted tube | | 2 |

Footrest angle & depth adjustable, high mounted tube, swing away frame


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-----------------------------------|--------------------|-----|
| 22 | 0587-05201 | Kit heelstrap, adjustable, small | | 2 |
| 23 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |

Angle and depth adjustable footrest AA-Legrest(Medium) 2019.04.15 =>

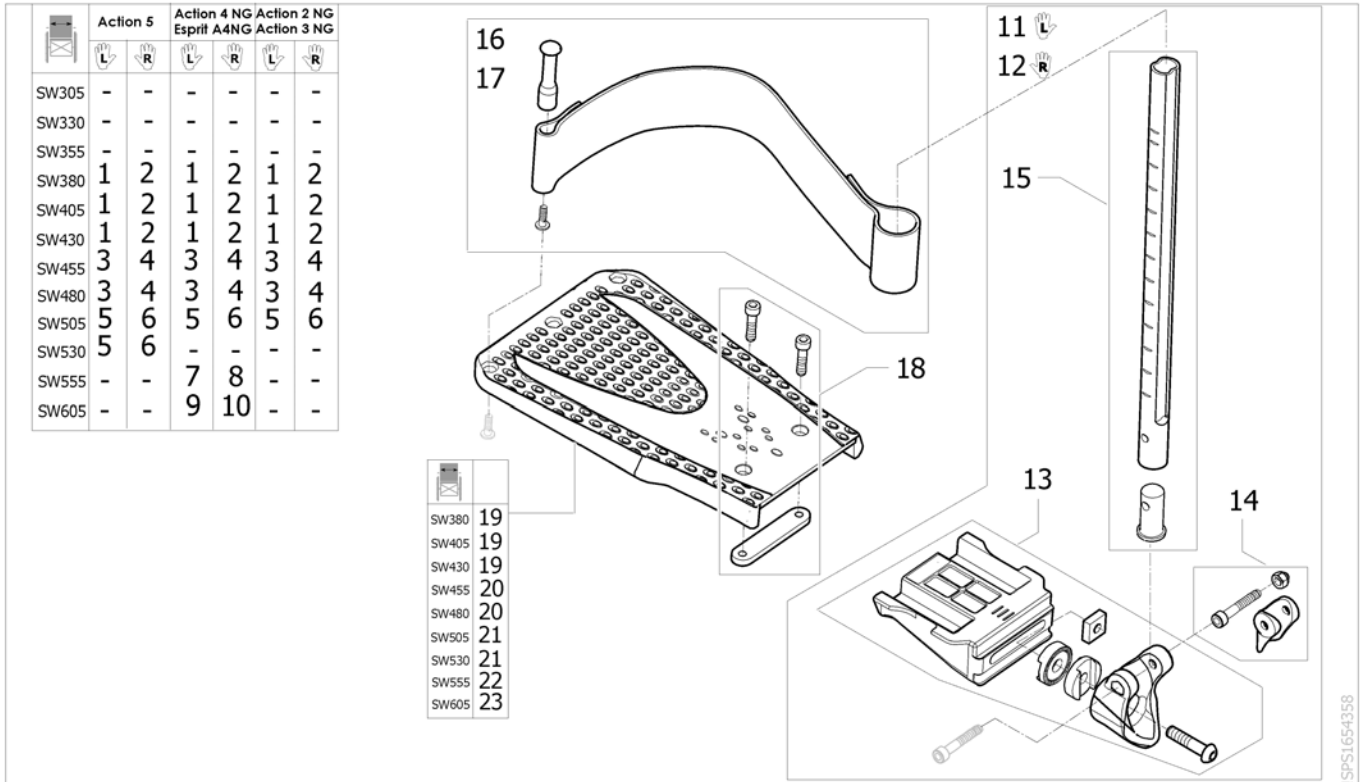


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1605950 | Footrest Lh inclinable Cplt AA Legrest(Footplate 147 mm) | | 1 |
| 2 | SP1605949 | Footrest Rh inclinable Cplt AA Legrest(Footplate 147 mm) | | 1 |
| 3 | SP1603263 | Footrest Lh inclinable Cplt AA Legrest(Footplate 172 mm) | | 1 |
| 4 | SP1603259 | Footrest Rh inclinable Cplt AA Legrest(Footplate 172 mm) | | 1 |
| 5 | SP1603264 | Footrest Lh inclinable Cplt AA Legrest(Footplate 197 mm) | | 1 |
| 6 | SP1603260 | Footrest Rh inclinable Cplt AA Legrest(Footplate 197 mm) | | 1 |
| 7 | SP1603265 | Footrest Lh inclinable Cplt AA Legrest(Footplate 222 mm) | | 1 |
| 8 | SP1603261 | Footrest Rh inclinable Cplt AA Legrest(Footplate 222 mm) | | 1 |
| 9 | SP1603373 | Footrest Lh inclinable Cplt AA Legrest(Footplate 247 mm) | | 1 |
| 10 | SP1603262 | Footrest Rh inclinable Cplt AA Legrest(Footplate 247 mm) | | 1 |
| 11 | SP1603257 | Left Footrest Without Footplate | | 1 |
| 12 | SP1603258 | Right Footrest Without Footplate | | 1 |
| 13 | SP60121862 | Footplate joint kit | | 2 |
| 14 | SP1654361 | Kit plastic Sleeve | | 2 |
| 15 | SP1657123 | tube with cap insert for adjustable footrest | | 2 |
| 16 | 1492315 | Kit heelstrap, adjustable, medium | | 2 |
| 17 | SP1654360 | Kit heelstrap, adjustable, 760 mm AA Legrest | | 1 |
| 18 | SP1654388 | Kit fixation for adjustable footrest AA Legrest | | 2 |
| 19 | SP1605149 | Footplate 147 mm lower legrest | | 2 |
| 20 | SP1603896 | Footplate 172 mm lower legrest | | 2 |
| 21 | SP1603897 | Footplate 197 mm lower legrest | | 2 |
| 22 | SP1603898 | Footplate 222 mm lower legrest | | 2 |
| 23 | SP1603899 | Footplate 247 mm lower legrest | | 1 |

Footplate cover Medium 2019.04.15 =>

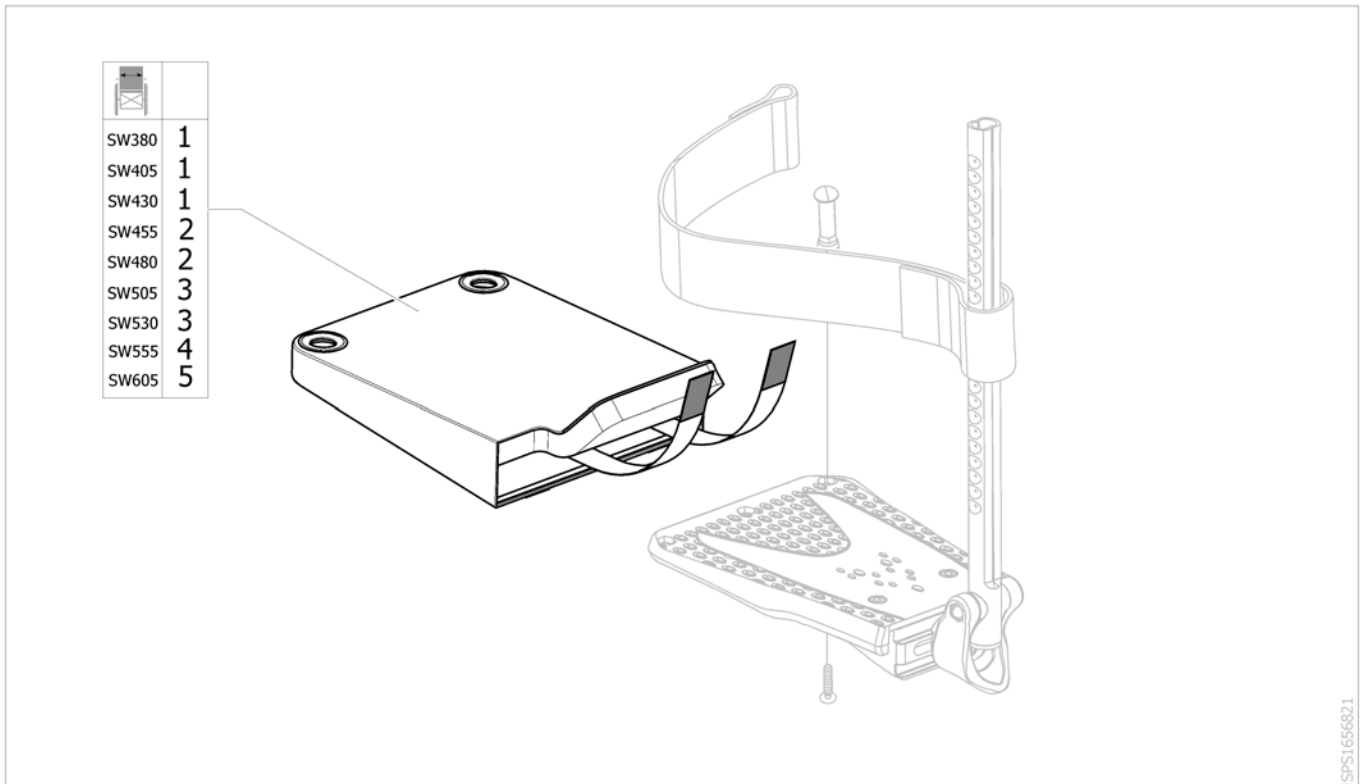

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-------------------------|--------------------|-----|
| 1 | SP1650354 | Cover, Footplate 147 mm | | 2 |
| 2 | SP1650355 | Cover, Footplate 172 mm | | 2 |
| 3 | SP1650356 | Cover, Footplate 197 mm | | 2 |
| 4 | SP1650357 | Cover, Footplate 222 mm | | 2 |
| 5 | SP1650358 | Cover, Footplate 247 mm | | 2 |

One piece footplate for fixed frame

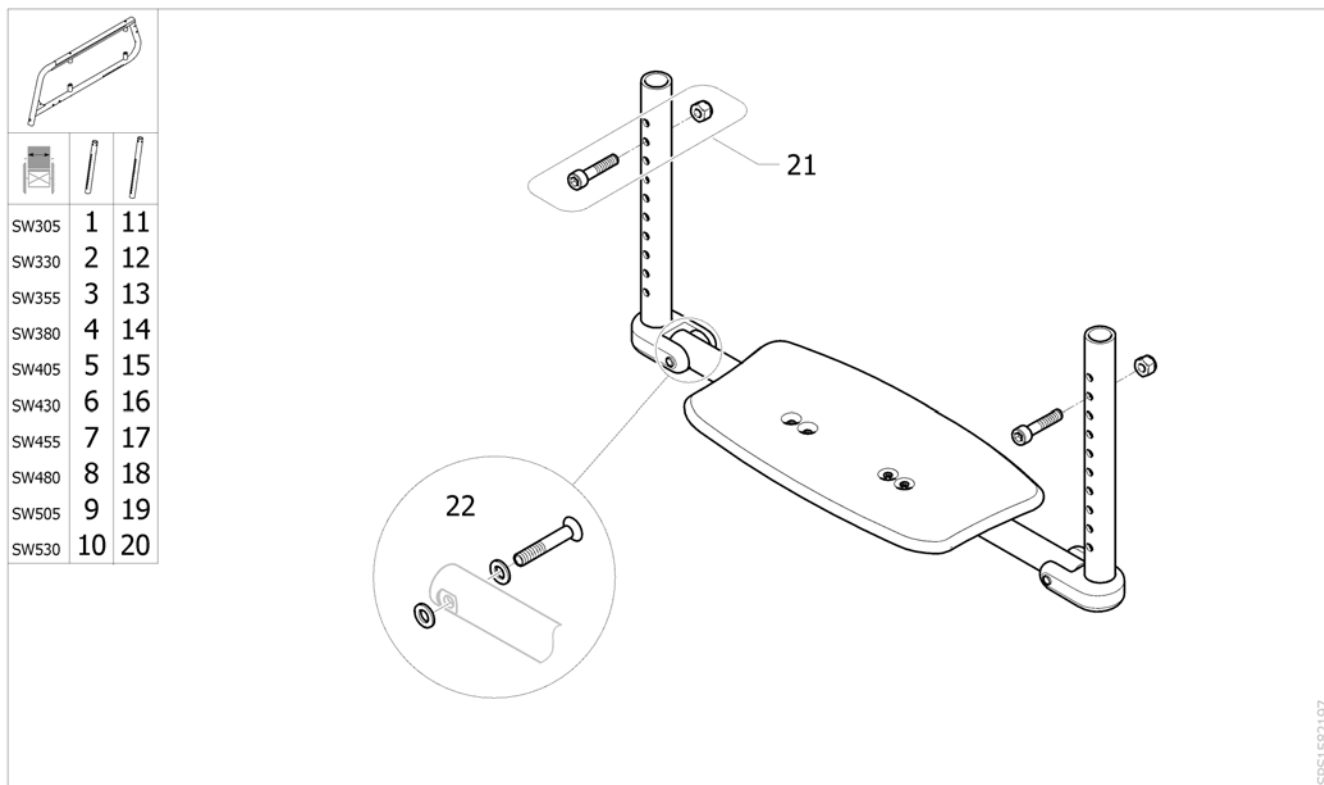


SPS1582197

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1587750 | One piece footplate, standard tube, SW305 mm Action 5 <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 2 | SP1587751 | One piece footplate, standard tube, SW330 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 3 | SP1587767 | One piece footplate, standard tube, SW355 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 4 | SP1587768 | One piece footplate, standard tube, SW380 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 5 | SP1587769 | One piece footplate, standard tube, SW405 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 6 | SP1587770 | One piece footplate, standard tube, SW430 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 7 | SP1587771 | One piece footplate, standard tube, SW455 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 8 | SP1587813 | One piece footplate, standard tube, SW480 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 9 | SP1587814 | One piece footplate, standard tube, SW505 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 10 | SP1587818 | One piece footplate, standard tube, SW530 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3440]</i> | | 1 |
| 11 | SP1587824 | One piece footplate, long tube, SW305 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 12 | SP1587825 | One piece footplate, long tube, SW330 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 13 | SP1587826 | One piece footplate, long tube, SW355 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 14 | SP1587827 | One piece footplate, long tube, SW380 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 15 | SP1587828 | One piece footplate, long tube, SW405 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 16 | SP1587829 | One piece footplate, long tube, SW430 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 17 | SP1587830 | One piece footplate, long tube, SW455 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 18 | SP1587831 | One piece footplate, long tube, SW480 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |

One piece footplate for fixed frame



| Pos. | Part number | Description | Valid from → until | Qty |
|-------------|--------------------|--|---------------------------|------------|
| 19 | SP1587832 | One piece footplate, long tube, SW505 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 20 | SP1587833 | One piece footplate, long tube, SW530 mm <i>[SJ_0210+(SJ_3185/SJ_3230)+SJ_3450]</i> | | 1 |
| 21 | SP1602218 | Kit fixation, footrest Monobloc with fixed frame | | 2 |
| 22 | SP1606180 | Fixing kit for support footrest monobloc | | 2 |

Siderests

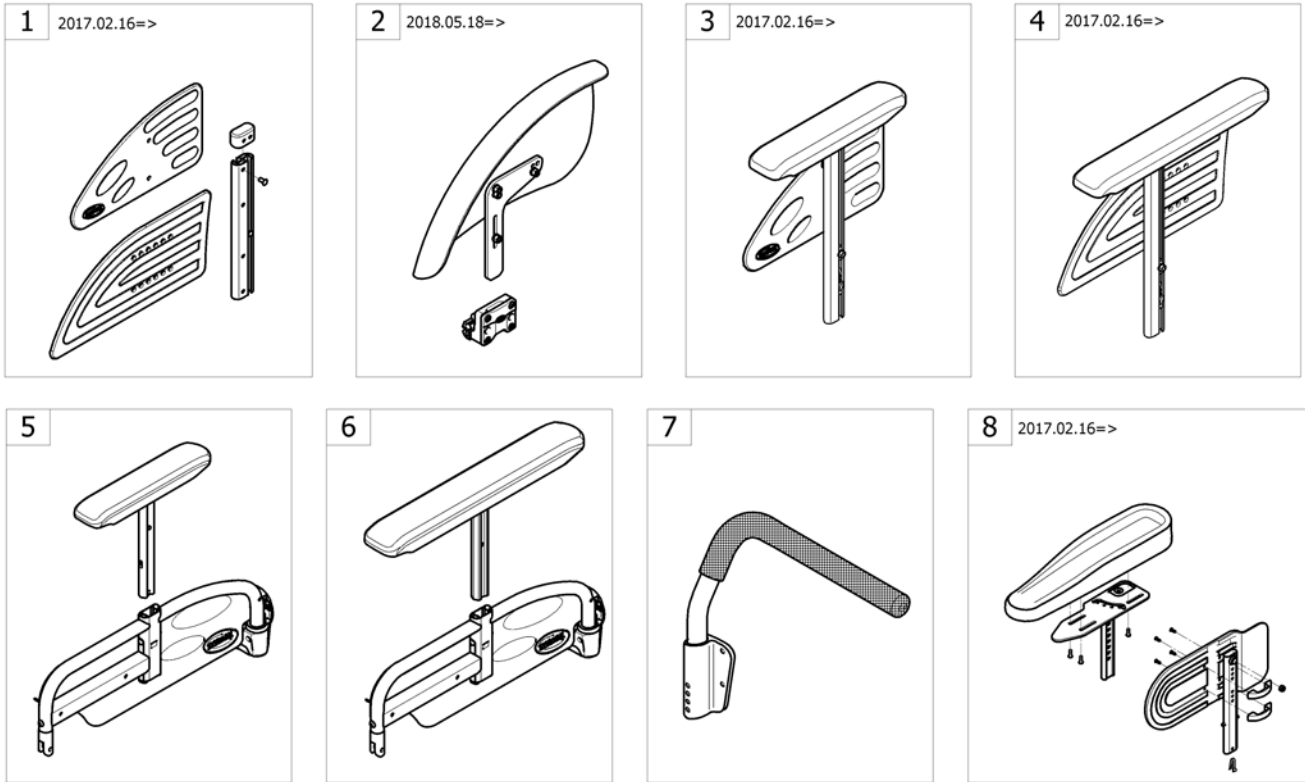

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Side guard 2017.02.16=> [SJ_3750] | 2017-01-16 → | 1 |
| 2 | - | Mudguard 2018.05.18=> | 2018-05-15 → | 1 |
| 3 | - | Armrest n°1, Removable, height adjustable T-arm, desk armpad 280mm 2017.02.16=> [SJ_3910/SJ_4010/SJ_4110] | 2017-01-16 → | 1 |
| 4 | - | Armrest n°1, Removable, height adjustable T-arm, full lenght armpad 380mm 2017.02.16=> [SJ_3920/SJ_4020/SJ_4120] | 2017-02-16 → | 1 |
| 5 | - | Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm) [SJ_3930/SJ_4030/SJ_4130] | | 1 |
| 6 | - | Armrest n°3, Swing away detachable & height adjustable, full length armpad (380mm) [SJ_3940/SJ_4040/SJ_4140] | | 1 |
| 7 | - | Tubular Armrest [SJ_3950/SJ_4050/SJ_4150] | | 1 |
| 8 | - | Hemi Dartex armpad multiposition standard 2017.02.16=> [SJ_3960/SJ_4060/SJ_4160] | | 1 |


Side guard 2017.02.16=>

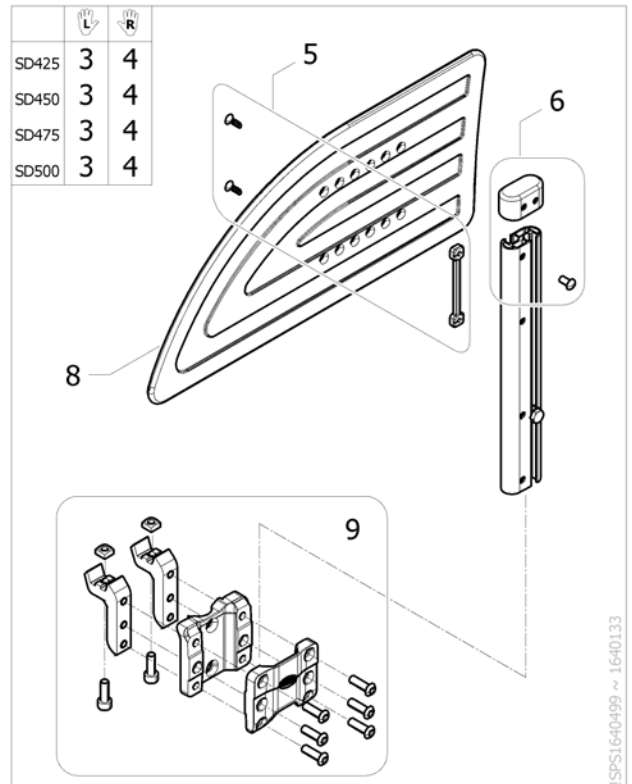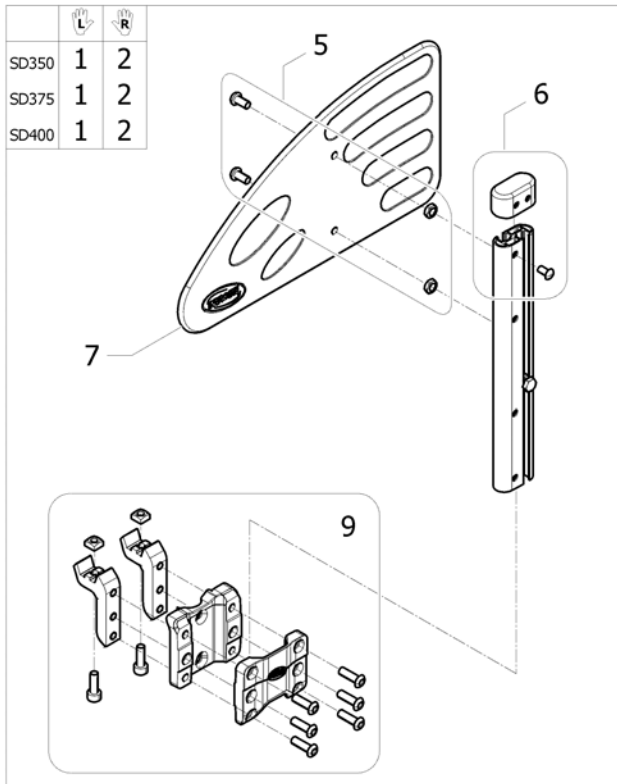

ISPS1640499 - 1640133

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1639977 | Side Guard, Lh, large, SD350-400 mm (v2017) <i>[SJ_3750]</i> | | 1 |
| 2 | SP1639978 | Side Guard, Rh, large, SD350-400mm (v2017) <i>[SJ_3750]</i> | | 1 |
| 3 | SP1640133 | Side Guard, Lh, extra large, SD425-500mm (v2017) | | 1 |
| 4 | SP1640134 | Side Guard, Rh, extra large, SD425-500mm (v2017) | | 1 |
| 5 | SP1601494 | Kit fixation skirtguard & armpad | | 1 |
| 6 | 1513515 | Kit cap + rivet for skirt guard | | 2 |
| 7 | 1425601 | Skirtguard black | | 2 |
| 8 | 1434725 | Skirtguard Rio height adjustable | | 2 |
| 9 | SP1640067 | Side Guard support (v2017) | | 2 |


Mudguard 2018.05.18=>

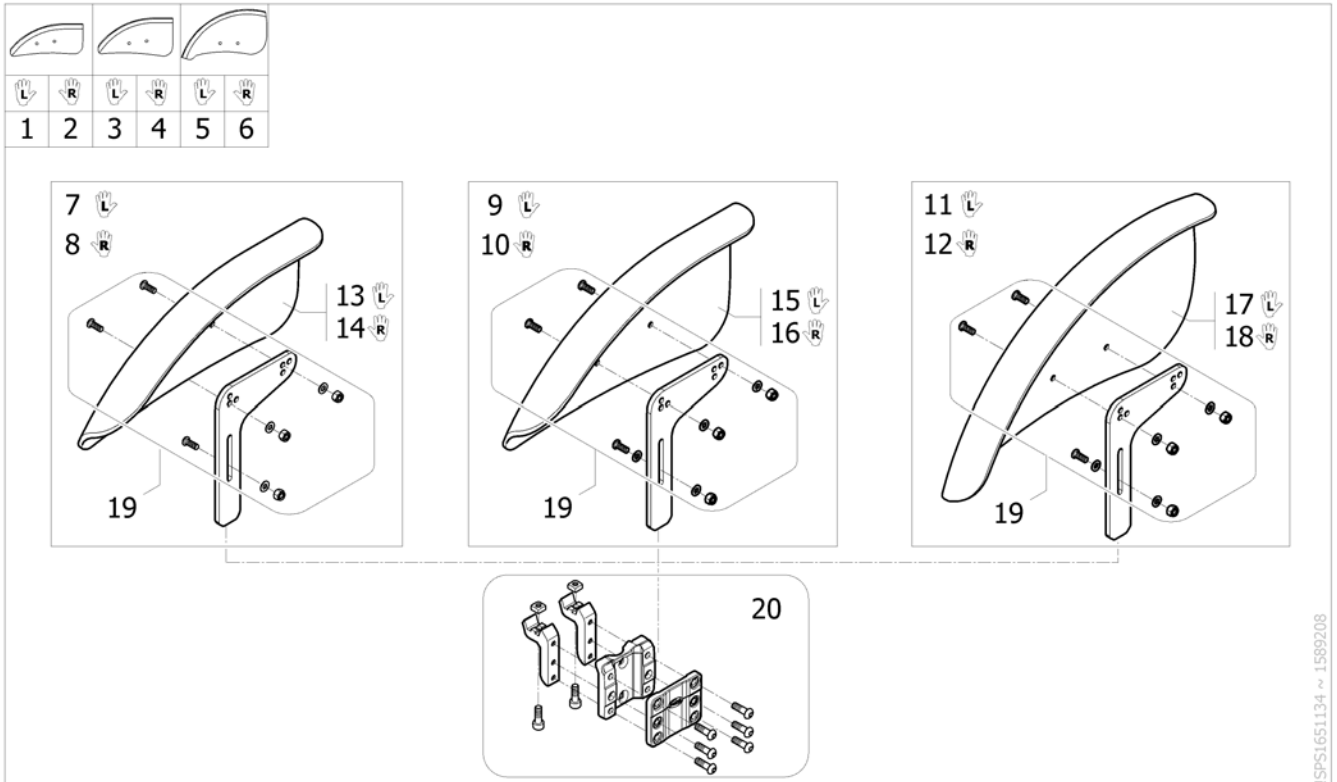

ISPS1651134 ~ 1599208


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1651086 | Mudguard, medium, complete, Lh (v2017) <i>Replace SP1590444</i> | | 1 |
| 2 | SP1651087 | Mudguard, medium, complete, Rh (v2017) <i>Replace SP1590445</i> | | 1 |
| 3 | SP1651088 | Mudguard, large, complete, Lh (v2017) <i>Replace SP1590446</i> | | 1 |
| 4 | SP1651089 | Mudguard, large, complete, Rh (v2017) <i>Replace SP1590447</i> | | 1 |
| 5 | SP1651090 | Mudguard, extra large, complete, Lh (v2017) <i>Replace SP1590448</i> | | 1 |
| 6 | SP1651091 | Mudguard, extra large, complete, Rh (v2017) <i>Replace SP1590449</i> | | 1 |
| 7 | SP1601477 | Kit mudguard, medium, Lh, + L support | | 1 |
| 8 | SP1601478 | Kit mudguard, medium, Rh, + L support | | 1 |
| 9 | SP1601479 | Kit mudguard, large, Lh, + L support | | 1 |
| 10 | SP1601480 | Kit mudguard, large, Rh, + L support | | 1 |
| 11 | SP1601481 | Kit mudguard, extra large, Lh, + L support | | 1 |
| 12 | SP1601482 | Kit mudguard, extra large, Rh, + L support | | 1 |
| 13 | SP1586515 | Mudguard, medium, Lh | | 1 |
| 14 | SP1586516 | Mudguard, medium, Rh | | 1 |
| 15 | SP1586517 | Mudguard, large, Lh | | 1 |
| 16 | SP1586518 | Mudguard, large, Rh | | 1 |
| 17 | SP1584373 | Mudguard, extra large, Lh | | 1 |
| 18 | SP1584372 | Mudguard, extra large, Rh | | 1 |
| 19 | SP1601485 | Kit fixation mudguard with L support | | 1 |
| 20 | SP1651135 | Mudguard support (v2017) <i>Replace SP1601486</i> | | 1 |

Armrest n°1, Removable, height adjustable T-arm, desk armpad 280mm 2017.02.16=>

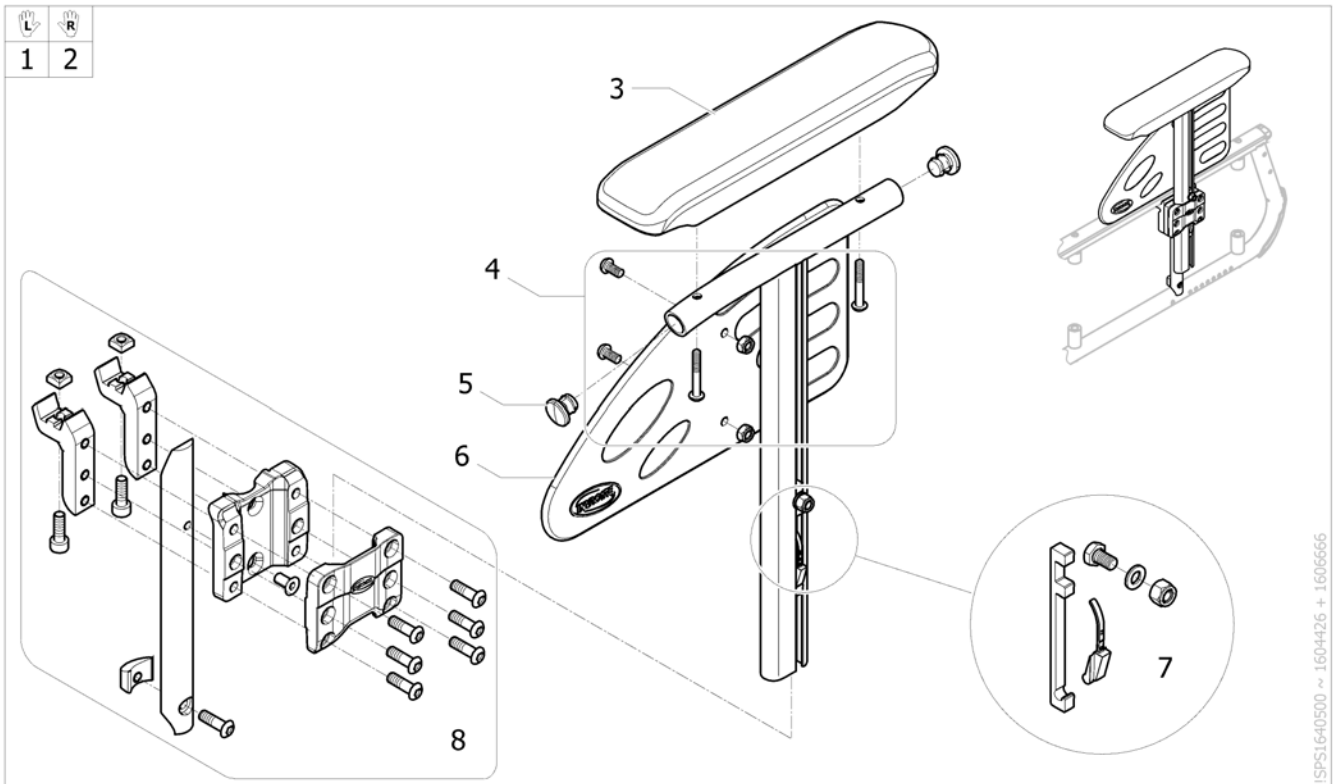

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1639973 | Armrest 1 Lh, removable and height adjustable, desk armpad 280 mm (v2017) <i>[SJ_3910/SJ_4110]</i> | | 1 |
| 2 | SP1639974 | Armrest 1 Rh, removable and height adjustable, desk armpad 280 mm (v2017) <i>[SJ_3910/SJ_4010]</i> | | 1 |
| 3 | 1543220 | Short armpad 280mm + 2 screws | | 2 |
| 4 | SP1601494 | Kit fixation skirtguard & armpad | | 2 |
| 5 | 0021-20003 | Black Ø18 x 1.5 mm cylindrical cap | | 4 |
| 6 | 1425601 | Skirtguard black | | 2 |
| 7 | SP1589904 | Kit locking system for armrest 1 | | 2 |
| 8 | SP1606666 | Plastic support of armrest 1 & 4 | | 2 |


Armrest n°1, Removable, height adjustable T-arm, full lenght armpad 380mm 2017.02.16=>

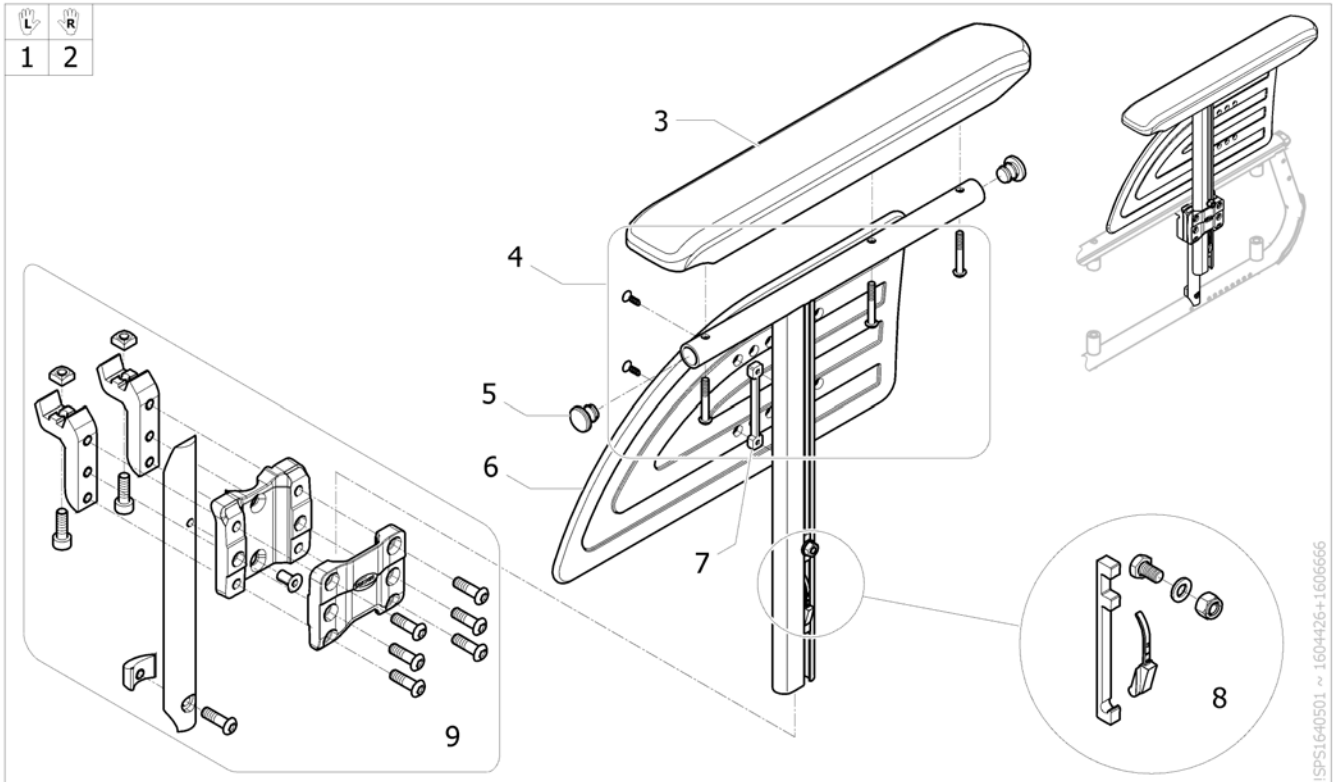

ISPS1640501 ~ 1604426+1606666

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1639975 | Armrest 1 Lh, removable and height adjustable, full lenght armpad 380 mm (v2017) <i>[SJ_3920/SJ_4120]</i> | | 1 |
| 2 | SP1639976 | Armrest 1 Rh, removable and height adjustable, full lenght armpad 380 mm (v2017) <i>[SJ_3920/SJ_4020]</i> | | 1 |
| 3 | 1543247 | Kit long armpad 380 mm + 3 screws | | 2 |
| 4 | SP1601494 | Kit fixation skirtguard & armpad | | 2 |
| 5 | 0021-20003 | Black Ø18 x 1.5 mm cylindrical cap | | 4 |
| 6 | 1434725 | Skirtguard Rio height adjustable | | 2 |
| 7 | 1435388 | Double-nut | | 2 |
| 8 | SP1589904 | Kit locking system for armrest 1 | | 2 |
| 9 | SP1606666 | Plastic support of armrest 1 & 4 | | 2 |

Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm)

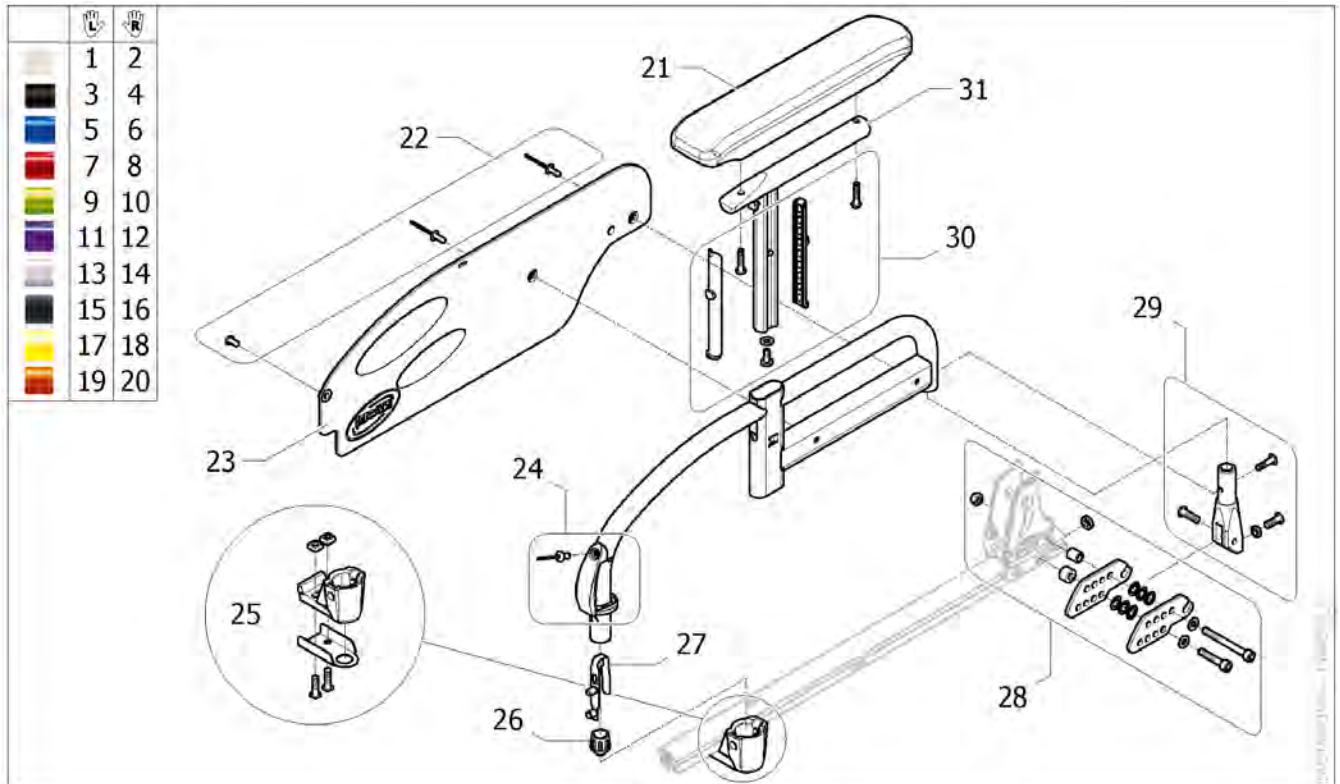


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1584889-0637 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl White [[SJ_3930/SJ_4130)+SJ_6000] | | 1 |
| 2 | SP1584888-0637 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl White [[SJ_3930/SJ_4030)+SJ_6000] | | 1 |
| 3 | SP1584889-0609 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Anthracite [[SJ_3930/SJ_4130)+SJ_6010] | | 1 |
| 4 | SP1584888-0609 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Anthracite [[SJ_3930/SJ_4030)+SJ_6010] | | 1 |
| 5 | SP1584889-0643 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pop Blue [[SJ_3930/SJ_4130)+SJ_6020] | | 1 |
| 6 | SP1584888-0643 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pop Blue [[SJ_3930/SJ_4030)+SJ_6020] | | 1 |
| 7 | SP1584889-0607 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Dark Red [[SJ_3930/SJ_4130)+SJ_6030] | | 1 |
| 8 | SP1584888-0607 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Dark Red [[SJ_3930/SJ_4030)+SJ_6030] | | 1 |
| 9 | SP1584889-0644 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Electric Green [[SJ_3930/SJ_4130)+SJ_6040] | | 1 |
| 10 | SP1584888-0644 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Electric Green [[SJ_3930/SJ_4030)+SJ_6040] | | 1 |
| 11 | SP1584889-0645 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Hypnotic Purple [[SJ_3930/SJ_1130)+SJ_6050] | | 1 |
| 12 | SP1584888-0645 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Hypnotic Purple [[SJ_3930/SJ_4030)+SJ_6050] | | 1 |

Armrest n°3, Swing away detachable & height adjustable, desk armpad (280mm)

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 13 | SP1584889-0515 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl Grey [(SJ_3930/SJ_4130)+SJ_6060] | | 1 |
| 14 | SP1584888-0515 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Pearl Grey [(SJ_3930/SJ_4030)+SJ_6060] | | 1 |
| 15 | SP1584889-0646 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Matt Black [(SJ_3930/SJ_4130)+SJ_6070] | | 1 |
| 16 | SP1584888-0646 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Matt Black [(SJ_3930/SJ_4030)+SJ_6070] | | 1 |
| 17 | SP1584889-0647 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Intense Yellow [(SJ_3930/SJ_4130)+SJ_6080] | | 1 |
| 18 | SP1584888-0647 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Intense Yellow [(SJ_3930/SJ_4030)+SJ_6080] | | 1 |
| 19 | SP1584889-0648 | Armrest 3, Lh, swing away detachable & height adjustable, desk armpad (280 mm), Vibrating Orange [(SJ_3930/SJ_4130)+SJ_6090] | | 1 |
| 20 | SP1584888-0648 | Armrest 3, Rh, swing away detachable & height adjustable, desk armpad (280 mm), Vibrating Orange [(SJ_3930/SJ_4030)+SJ_6090] | | 1 |
| 21 | SP1606082 | Kit short armpad 280 mm + 2 screw for armrest 3 | | 2 |
| 22 | SP1601924 | Fixing skirtguard kit armrest 3 | | 2 |
| 23 | 1426725 | Skirtguard black for armrest 3 | | 2 |
| 24 | SP1601925 | Unlocking button kit armrest 3 | | 2 |
| 25 | SP1601926 | Locking kit armrest3 | | 2 |
| 26 | 0572-03405 | Short conical cap | | 2 |
| 27 | 1018225 | Double stud spring | | 2 |
| 28 | SP1601927 | Armpad support kit armrest 3 | | 2 |
| 29 | SP1601928 | Articulation & crew kit armrest 3 | | 2 |
| 30 | SP1601929 | Plastic insert & screw kit armrest 3 | | 2 |
| 31 | SP1601930 | T support kit, short armpad, armrest 3 | | 2 |

Armrest n°3, Swing away detachable & height adjustable, full length armpad (380mm)

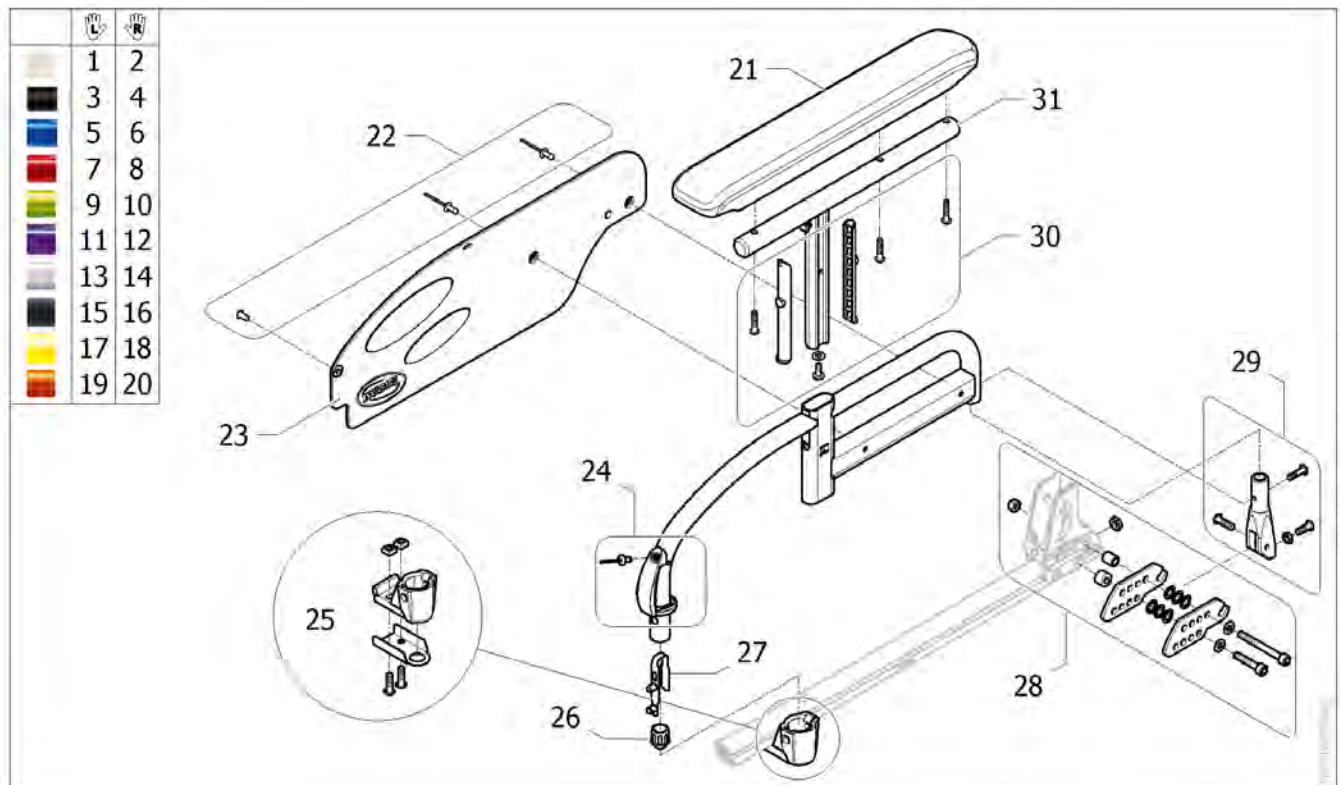


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1605688-0637 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Pearl White | | 1 |
| 2 | SP1604893-0637 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Pearl White | | 1 |
| 3 | SP1605688-0609 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Anthracite | | 1 |
| 4 | SP1604893-0609 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Anthracite | | 1 |
| 5 | SP1605688-0643 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Pop Blue | | 1 |
| 6 | SP1604893-0643 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Pop Blue | | 1 |
| 7 | SP1605688-0607 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Dark Red | | 1 |
| 8 | SP1604893-0607 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Dark Red | | 1 |
| 9 | SP1605688-0644 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Electric Green | | 1 |
| 10 | SP1604893-0644 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Electric Green | | 1 |
| 11 | SP1605688-0645 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Hypnotic Purple | | 1 |
| 12 | SP1604893-0645 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Hypnotic Purple | | 1 |
| 13 | SP1605688-0515 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Pearl Grey | | 1 |
| 14 | SP1604893-0515 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Pearl Grey | | 1 |
| 15 | SP1605688-0646 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Matt Black | | 1 |
| 16 | SP1604893-0646 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Matt Black | | 1 |
| 17 | SP1605688-0647 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Intense Yellow | | 1 |
| 18 | SP1604893-0647 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Intense Yellow | | 1 |

Armrest n°3, Swing away detachable & height adjustable, full length armpad (380mm)

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 19 | SP1605688-0648 | Armrest 3, Lh, swing away detachable & height adjustable, full length armpad (380 mm), Vibrating Orange | | 1 |
| 20 | SP1604893-0648 | Armrest 3, Rh, swing away detachable & height adjustable, full length armpad (380 mm), Vibrating Orange | | 1 |
| 21 | SP1606083 | Kit long armpad 380 mm + 3 screw for armrest 3 | | 2 |
| 22 | SP1601924 | Fixing skirtguard kit armrest 3 | | 2 |
| 23 | 1426725 | Skirtguard black for armrest 3 | | 2 |
| 24 | SP1601925 | Unlocking button kit armrest 3 | | 2 |
| 25 | SP1601926 | Locking kit armrest3 | | 2 |
| 26 | 0572-03405 | Short conical cap | | 2 |
| 27 | 1018225 | Double stud spring | | 2 |
| 28 | SP1601927 | Armpad support kit armrest 3 | | 2 |
| 29 | SP1601928 | Articulation & crew kit armrest 3 | | 2 |
| 30 | SP1601929 | Plastic insert & screw kit armrest 3 | | 2 |
| 31 | SP1606077 | T support kit, long armpad, armrest3 | | 2 |


Tubular Armrest

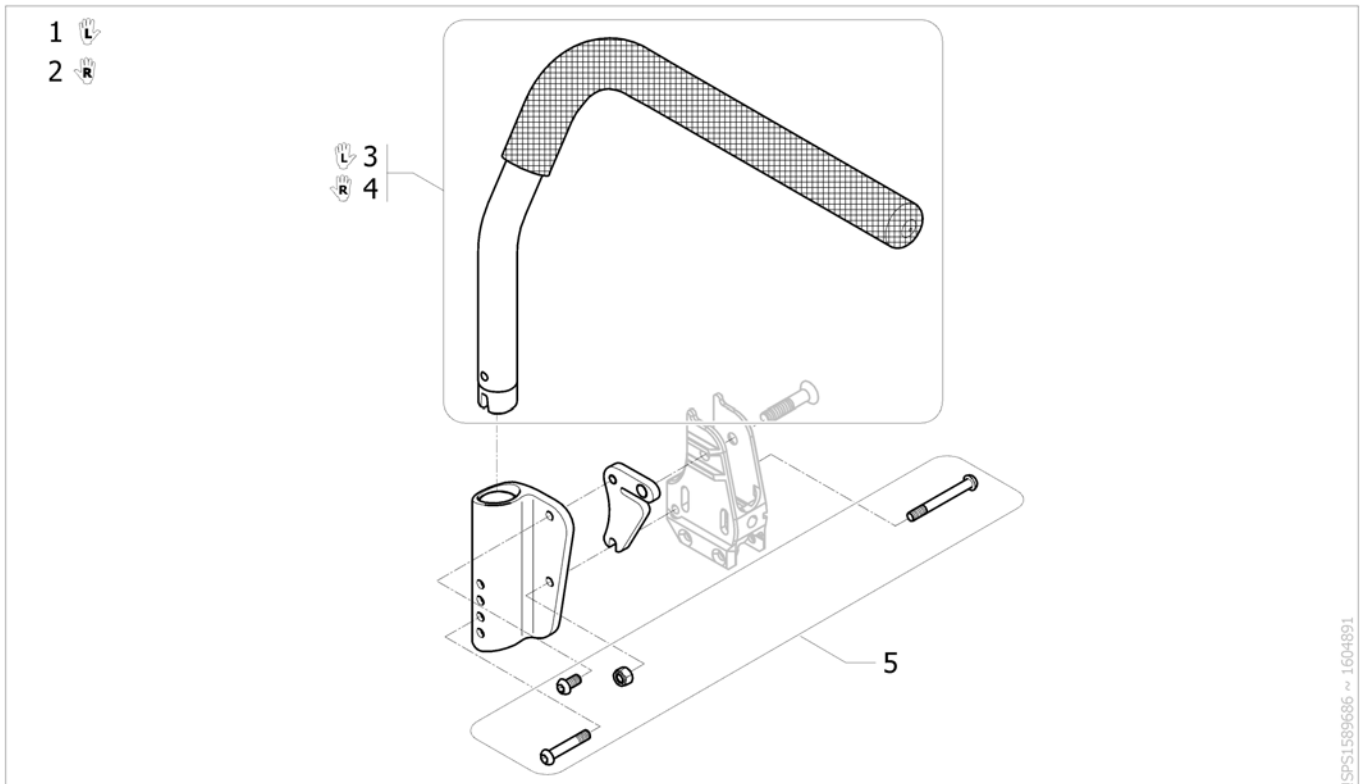

ISP51589686 ~ 1604891

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1583753 | Tubular armrest LH, Swing away padded tube | | 1 |
| 2 | SP1583726 | Tubular armrest RH, Swing away padded tube | | 1 |
| 3 | SP1581267 | Left tubular armrest assembly | | 1 |
| 4 | SP1581505 | Right tubular armrest assembly | | 1 |
| 5 | SP1606078 | Fixing tubular armrest kit | | 2 |


Hemi Dartex armpad multiposition standard 2017.02.16=>

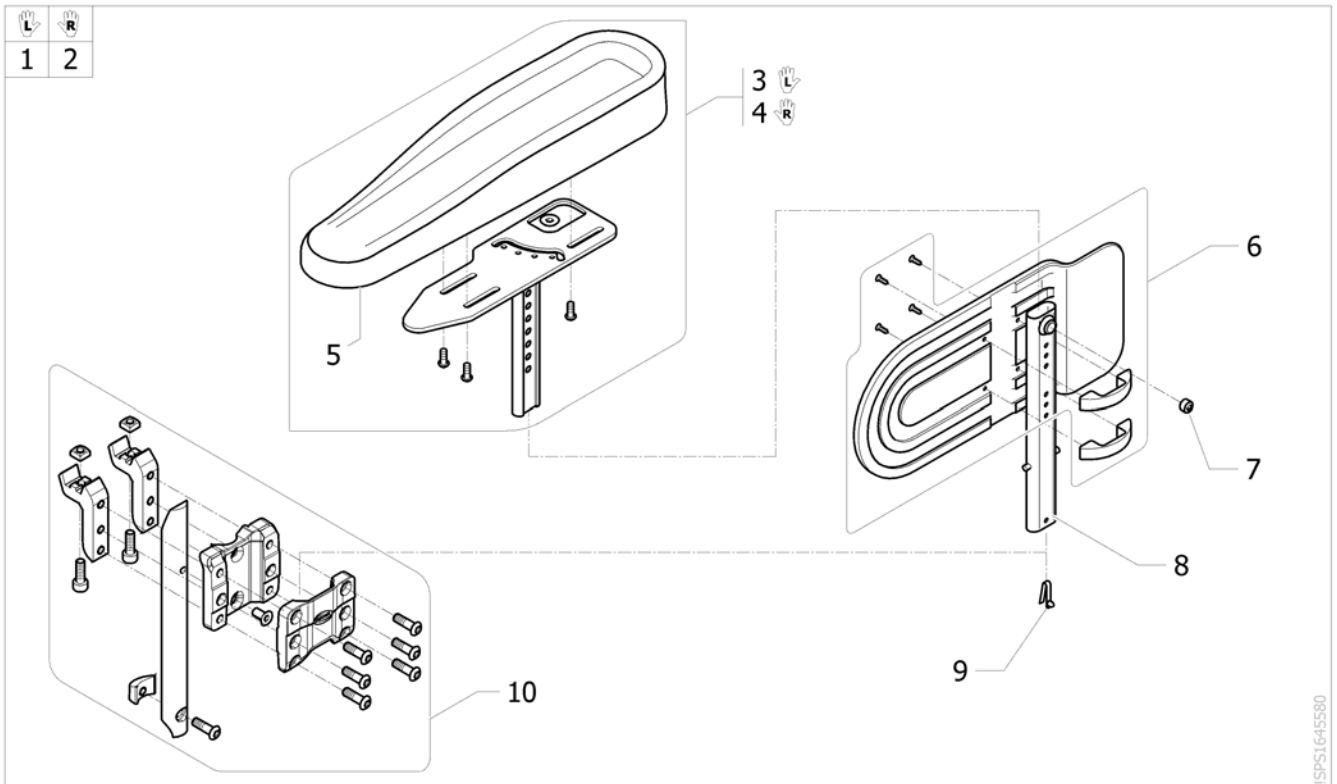

SPS1645580

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1645578 | Lh hemi armrest N°4 multiposition (v2017) | | 1 |
| 2 | SP1645579 | Rh hemi armrest N°4 multiposition (v2017) | | 1 |
| 3 | 1493220 | Lh hemi dartex armpad complete | | 1 |
| 4 | 1493219 | Rh hemi dartex armpad complete | | 1 |
| 5 | 1493251 | Armpad, Hemi Armrest, Azalea | | 2 |
| 6 | 1430425 | Skirtguard Armrest N°4 Fixation Kit | | 2 |
| 7 | 5106533 | Locking Screw M12x10 | | 2 |
| 8 | 1443058 | Lowest Arm Rest Tube Support N°4 | | 2 |
| 9 | 0027-10003 | Spring pin locked armrest | | 2 |
| 10 | SP1606666 | Plastic support of armrest 1 & 4 | | 2 |


Front wheels

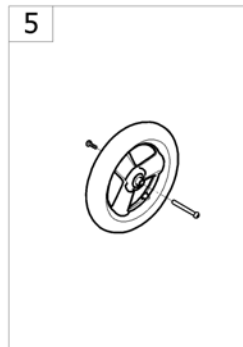

-<SPCACTION550

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Forks for swing away frame <i>[SJ_0200]</i> | | 1 |
| 2 | - | Forks for fixed frame <i>[SJ_0210]</i> | | 1 |
| 3 | - | Forks support for swing away frame <i>[SJ_0200]</i> | | 1 |
| 4 | - | Forks support for fixed frame <i>[SJ_0210]</i> | | 1 |
| 5 | - | Front wheels | | 1 |
| 6 | - | Forks and Front wheels Adjustment | | 1 |

Forks for swing away frame

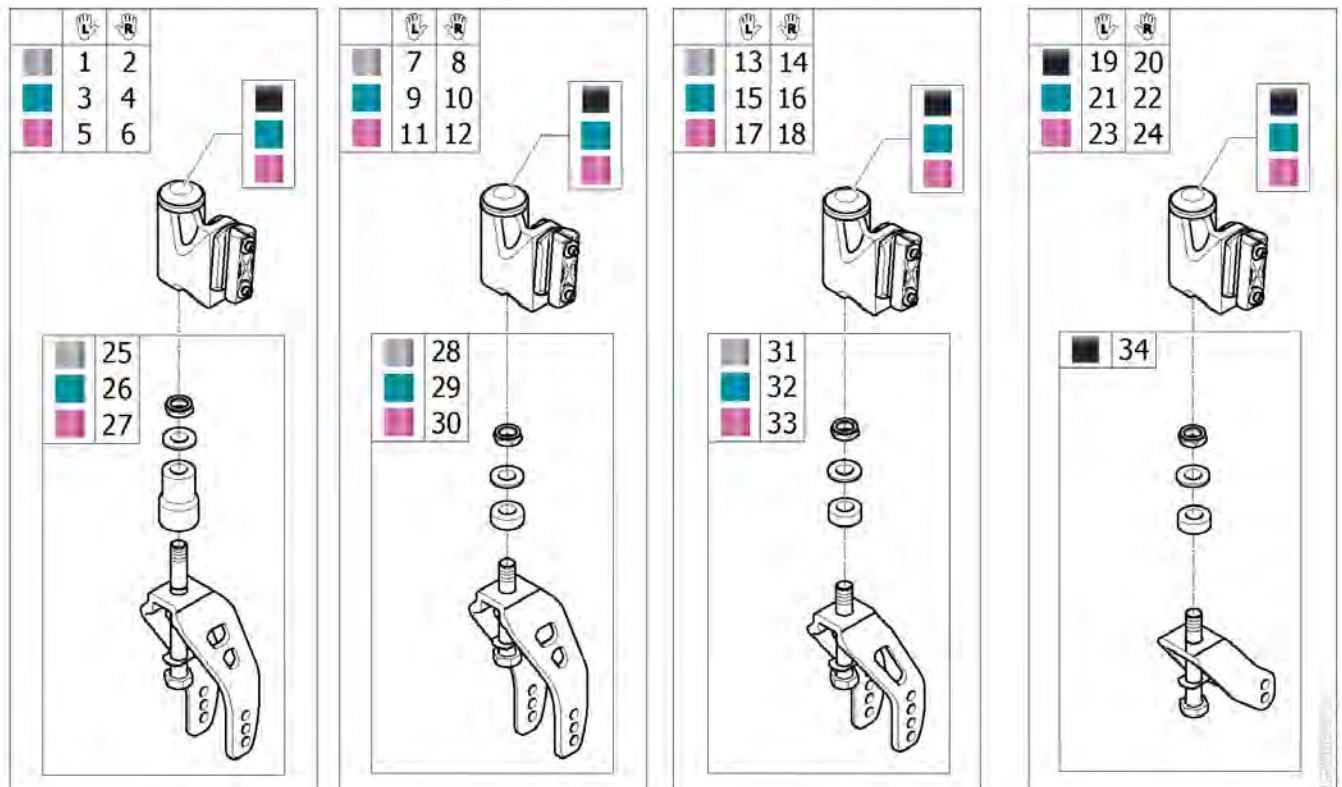


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1583759-0005 | Kit fork medium, high, Lh, without accent, swing away frame [SJ_0200+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6100] | | 1 |
| 2 | SP1586262-0005 | Kit fork medium, high, Rh, without accent, swing away frame [SJ_0200+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6100] | | 1 |
| 3 | SP1583759-0641 | Kit fork medium, high, Lh, accent Blue, swing away frame [SJ_0200+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6110] | | 1 |
| 4 | SP1586262-0641 | Kit fork medium, high, Rh, accent Blue, swing away frame [SJ_0200+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6110] | | 1 |
| 5 | SP1583759-0642 | Kit fork medium, high, Lh, accent Purple, swing away frame [SJ_0200+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6200] | | 1 |
| 6 | SP1586262-0642 | Kit fork medium, high, Rh, accent Purple, swing away frame [SJ_0200+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0200+(SJ_1260/SJ_1270)+SJ_6200] | | 1 |
| 7 | SP1583757-0005 | Kit fork medium, low, Lh, without accent, swing away frame [SJ_0200+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 8 | SP1586259-0005 | Kit fork medium, low, Rh, without accent, swing away frame [SJ_0200+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 9 | SP1583757-0641 | Kit fork medium, low, Lh, accent Blue, swing away frame [SJ_0200+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 10 | SP1586259-0641 | Kit fork medium, low, Rh, accent Blue, swing away frame [SJ_0200+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 11 | SP1583757-0642 | Kit fork medium, low, Lh, accent Purple, swing away frame [SJ_0200+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 12 | SP1586259-0642 | Kit fork medium, low, Rh, accent Purple, swing away frame [SJ_0200+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 13 | SP1583758-0005 | Kit fork short, Lh, without accent, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |

Forks for swing away frame

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 14 | SP1586260-0005 | Kit fork short, Rh, without accent, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 15 | SP1583758-0641 | Kit fork short, Lh, accent Blue, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 16 | SP1586260-0641 | Kit fork short, Rh, accent Blue, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 17 | SP1583758-0642 | Kit fork short, Lh, accent Purple, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 18 | SP1586260-0642 | Kit fork short, Rh, accent Purple, swing away frame [SJ_0200+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0200+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 19 | SP1583761-0005 | Kit fork extra short, Lh, without accent, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6100] | | 1 |
| 20 | SP1586270-0005 | Kit fork extra short, Rh, without accent, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6100] | | 1 |
| 21 | SP1583761-0641 | Kit fork extra short, Lh, accent Blue, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6110] | | 1 |
| 22 | SP1586270-0641 | Kit fork extra short, Rh, accent Blue, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6110] | | 1 |
| 23 | SP1583761-0642 | Kit fork extra short, Lh, accent Purple, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6200] | | 1 |
| 24 | SP1586270-0642 | Kit fork extra short, Rh, accent Purple, swing away frame [SJ_0200+(SJ_1200/SJ_1210)+SJ_6200] | | 1 |
| 25 | SP1584027-0005 | Kit fork medium, high, without accent [SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6100] | | 2 |
| 26 | SP1584027-0641 | Kit fork medium, high, accent Blue [SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6110] | | 2 |
| 27 | SP1584027-0642 | Kit fork medium, high, accent Purple [SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6200] | | 2 |
| 28 | SP1584025-0005 | Kit fork medium, low, without accent [SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 29 | SP1584025-0641 | Kit fork medium, low, accent Blue [SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 30 | SP1584025-0642 | Kit fork medium, low, accent Purple [SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 31 | SP1584026-0005 | Kit fork short, low, without accent [(SJ_1220/SJ_1230)+SJ_6100] [(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 32 | SP1584026-0641 | Kit fork short, low, accent Blue [(SJ_1220/SJ_1230)+SJ_6110] [(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 33 | SP1584026-0642 | Kit fork short, low, accent Purple [(SJ_1220/SJ_1230)+SJ_6200] [(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 34 | SP1584029-0601 | Kit fork extra short, low, black [SJ_1200/SJ_1210] | | 2 |

Forks for fixed frame

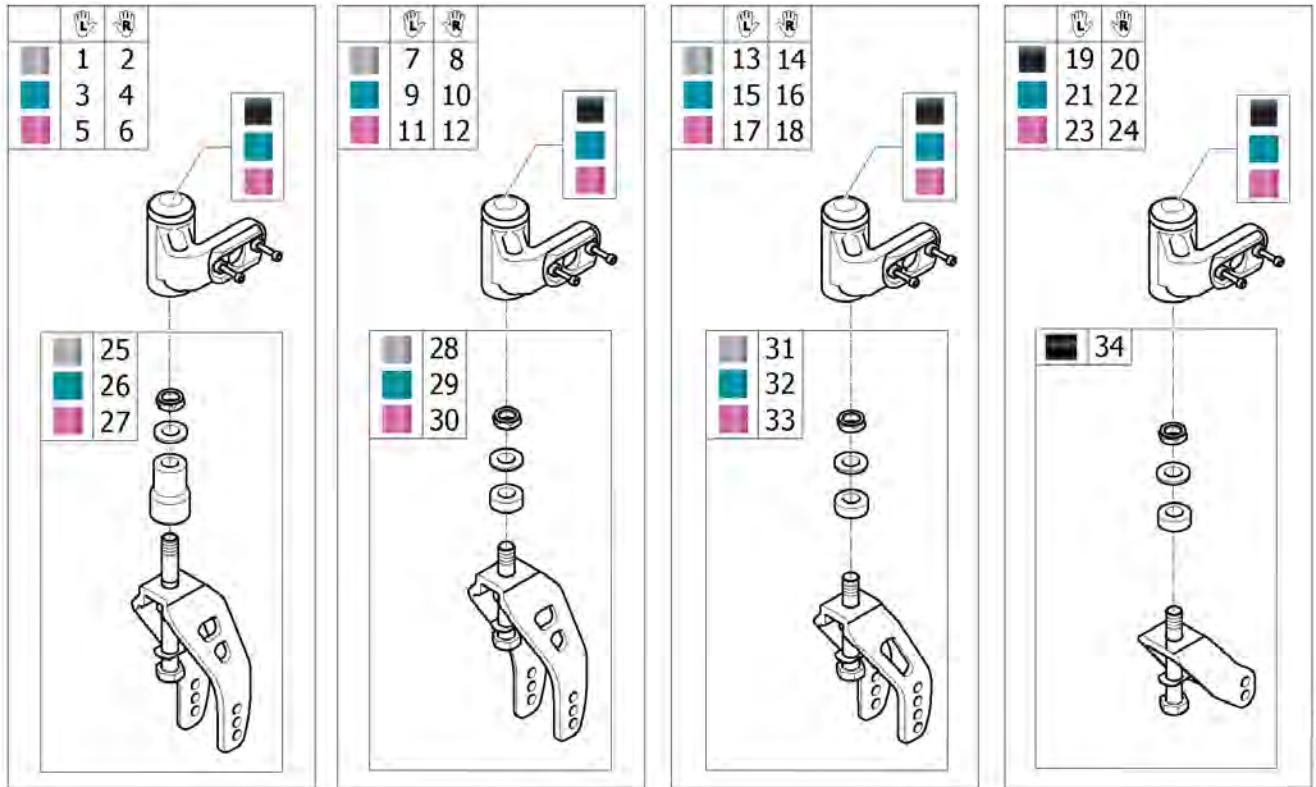


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 1 | SP1583854-0005 | Kit fork medium, high, Lh, without accent, fixed frame [SJ_0210+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6100] | | 1 |
| 2 | SP1586290-0005 | Kit fork medium, high, Rh, without accent, fixed frame [SJ_0210+SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6100] | | 1 |
| 3 | SP1583854-0641 | Kit fork medium, high, Lh, accent Blue, fixed frame [SJ_0210+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6110] | | 1 |
| 4 | SP1586290-0641 | Kit fork medium, high, Rh, accent Blue, fixed frame [SJ_0210+SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6110] | | 1 |
| 5 | SP1583854-0642 | Kit fork medium, high, Lh, accent Purple, fixed frame [SJ_0210+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6200] | | 1 |
| 6 | SP1586290-0642 | Kit fork medium, high, Rh, accent Purple, fixed frame [SJ_0210+SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [SJ_0210+(SJ_1260/SJ_1270)+SJ_6200] | | 1 |
| 7 | SP1583852-0005 | Kit fork medium, low, Lh, without accent, fixed frame [SJ_0210+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 8 | SP1586286-0005 | Kit fork medium, low, Rh, without accent, fixed frame [SJ_0210+SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 9 | SP1583852-0641 | Kit fork medium, low, Lh, accent Blue, fixed frame [SJ_0210+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 10 | SP1586286-0641 | Kit fork medium, low, Rh, accent Blue, fixed frame [SJ_0210+SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 11 | SP1583852-0642 | Kit fork medium, low, Lh, accent Purple, fixed frame [SJ_0210+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 12 | SP1586286-0642 | Kit fork medium, low, Rh, accent Purple, fixed frame [SJ_0210+SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 1 |
| 13 | SP1583853-0005 | Kit fork short, Lh, without accent, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |

Forks for fixed frame


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 14 | SP1586288-0005 | Kit fork short, Rh, without accent, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6100] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 15 | SP1583853-0641 | Kit fork short, Lh, accent Blue, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 16 | SP1586288-0641 | Kit fork short, Rh, accent Blue, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6110] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 17 | SP1583853-0642 | Kit fork short, Lh, accent Purple, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 18 | SP1586288-0642 | Kit fork short, Rh, accent Purple, fixed frame [SJ_0210+(SJ_1220/SJ_1230)+SJ_6200] [SJ_0210+(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 1 |
| 19 | SP1583762-0005 | Kit fork extra short, Lh, without accent, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6100] | | 1 |
| 20 | SP1586292-0005 | Kit fork extra short, Rh, without accent, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6100] | | 1 |
| 21 | SP1583762-0641 | Kit fork extra short, Lh, accent Blue, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6110] | | 1 |
| 22 | SP1586292-0641 | Kit fork extra short, Rh, accent Blue, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6110] | | 1 |
| 23 | SP1583762-0642 | Kit fork extra short, Lh, accent Purple, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6200] | | 1 |
| 24 | SP1586292-0642 | Kit fork extra short, Rh, accent Purple, fixed frame [SJ_0210+(SJ_1200/SJ_1210)+SJ_6200] | | 1 |
| 25 | SP1584027-0005 | Kit fork medium, high, without accent [SJ_1250+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6100] | | 2 |
| 26 | SP1584027-0641 | Kit fork medium, high, accent Blue [SJ_1250+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6110] | | 2 |
| 27 | SP1584027-0642 | Kit fork medium, high, accent Purple [SJ_1250+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] [(SJ_1260/SJ_1270)+SJ_6200] | | 2 |
| 28 | SP1584025-0005 | Kit fork medium, low, without accent [SJ_1240+SJ_6100+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 29 | SP1584025-0641 | Kit fork medium, low, accent Blue [SJ_1240+SJ_6110+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 30 | SP1584025-0642 | Kit fork medium, low, accent Purple [SJ_1240+SJ_6200+(SJ_7000/SJ_7020/SJ_7030/SJ_7040/SJ_7050/SJ_7055/SJ_7060/SJ_7070)] | | 2 |
| 31 | SP1584026-0005 | Kit fork short, low, without accent [(SJ_1220/SJ_1230)+SJ_6100] [(SJ_1240/SJ_1250)+SJ_6100+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 32 | SP1584026-0641 | Kit fork short, low, accent Blue [(SJ_1220/SJ_1230)+SJ_6110] [(SJ_1240/SJ_1250)+SJ_6110+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 33 | SP1584026-0642 | Kit fork short, low, accent Purple [(SJ_1220/SJ_1230)+SJ_6200] [(SJ_1240/SJ_1250)+SJ_6200+(SJ_7080/SJ_7090/SJ_7100/SJ_7110)] | | 2 |
| 34 | SP1584029-0601 | Kit fork extra short, low, black [SJ_1200/SJ_1210] | | 2 |

Forks support for swing away frame

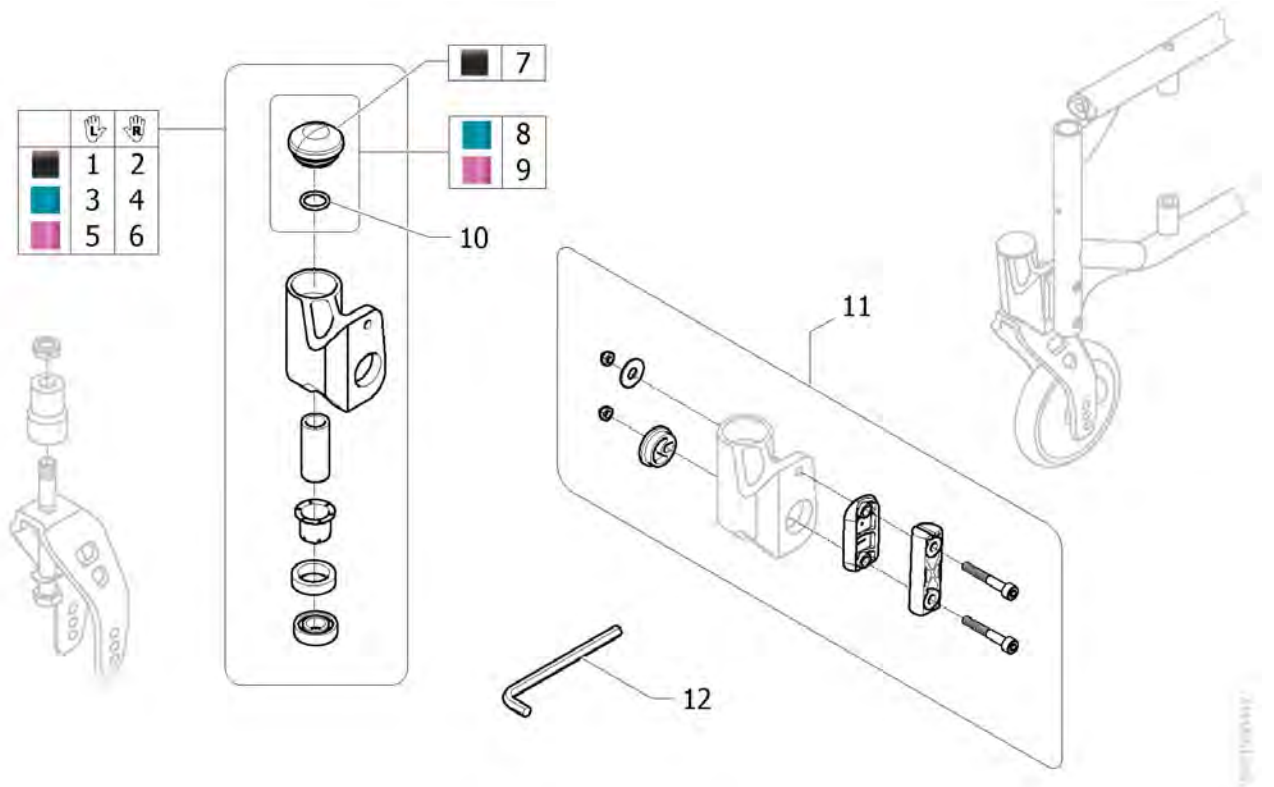


| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1584023-0005 | Kit fork support, Lh, without accent, swing away frame <i>[SJ_0200+SJ_6100]</i> | | 1 |
| 2 | SP1586261-0005 | Kit fork support, Rh, without accent, swing away frame <i>[SJ_0200+SJ_6100]</i> | | 1 |
| 3 | SP1584023-0641 | Kit fork support, Lh, accent Blue, swing away frame <i>[SJ_0200+SJ_6110]</i> | | 1 |
| 4 | SP1586261-0641 | Kit fork support, Rh, accent Blue, swing away frame <i>[SJ_0200+SJ_6110]</i> | | 1 |
| 5 | SP1584023-0642 | Kit fork support, Lh, accent Purple, swing away frame <i>[SJ_0200+SJ_6200]</i> | | 1 |
| 6 | SP1586261-0642 | Kit fork support, Rh, accent Purple, swing away frame <i>[SJ_0200+SJ_6200]</i> | | 1 |
| 7 | SP1589193 | Castor housing cap, Black <i>Replace SP1585342-0005</i> | 2015-12-23 → | 2 |
| 8 | SP1585342-0641 | Castor housing cap, accent Blue <i>[SJ_6110]</i> | | 2 |
| 9 | SP1585342-0642 | Castor housing cap, accent Purple <i>[SJ_6200]</i> | | 2 |
| 10 | SP1483872 | O-ring 15,6 x 2,4 NITRILE 70 SH | | 2 |
| 11 | SP1584024 | Kit adjustment fork support, swing away frame | | 2 |
| 12 | SP1586506 | Short Metric Hex Key (5mm) | → 2018-04-08 | 1 |

Forks support for fixed frame

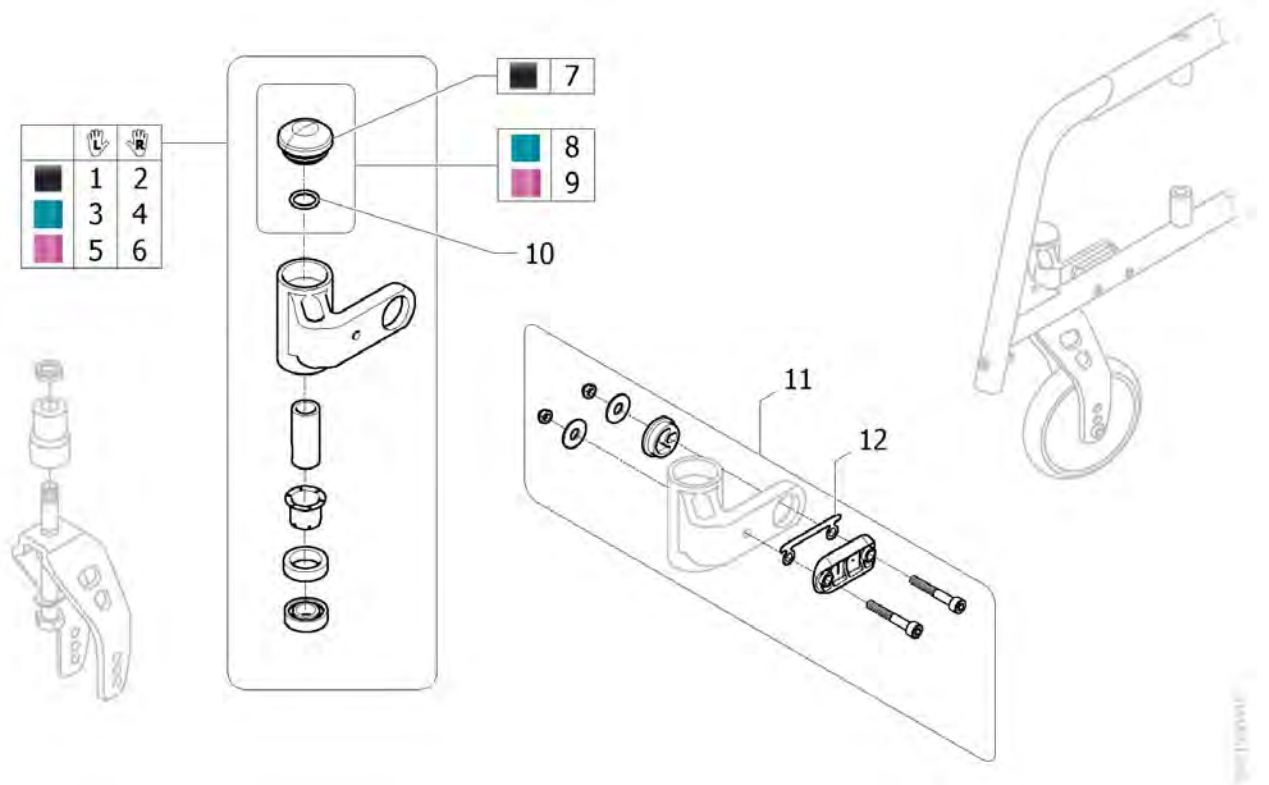

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1586271-0005 | Kit fork support, Lh, without accent, fixed frame [SJ_0210+SJ_6100] | | 1 |
| 2 | SP1586285-0005 | Kit fork support, Rh, without accent, fixed frame [SJ_0210+SJ_6100] | | 1 |
| 3 | SP1586271-0641 | Kit fork support, Lh, accent Blue, fixed frame [SJ_0210+SJ_6110] | | 1 |
| 4 | SP1586285-0641 | Kit fork support, Rh, accent Blue, fixed frame [SJ_0210+SJ_6110] | | 1 |
| 5 | SP1586271-0642 | Kit fork support, Lh, accent Purple, fixed frame [SJ_0210+SJ_6200] | | 1 |
| 6 | SP1586285-0642 | Kit fork support, Rh, accent Purple, fixed frame [SJ_0210+SJ_6200] | | 1 |
| 7 | SP1589193 | Castor housing cap, Black Replace SP1585342-0005 | 2015-12-23 → | 2 |
| 8 | SP1585342-0641 | Castor housing cap, accent Blue [SJ_6110] | | 2 |
| 9 | SP1585342-0642 | Castor housing cap, accent Purple [SJ_6200] | | 2 |
| 10 | SP1483872 | O-ring 15,6 x 2,4 NITRILE 70 SH | | 2 |
| 11 | SP1586287 | Kit adjustment fork support, fixed frame | | 2 |
| 12 | SP1604465 | Kit adjusting part front wheel A5 (pair) | | 1 |


Front wheels

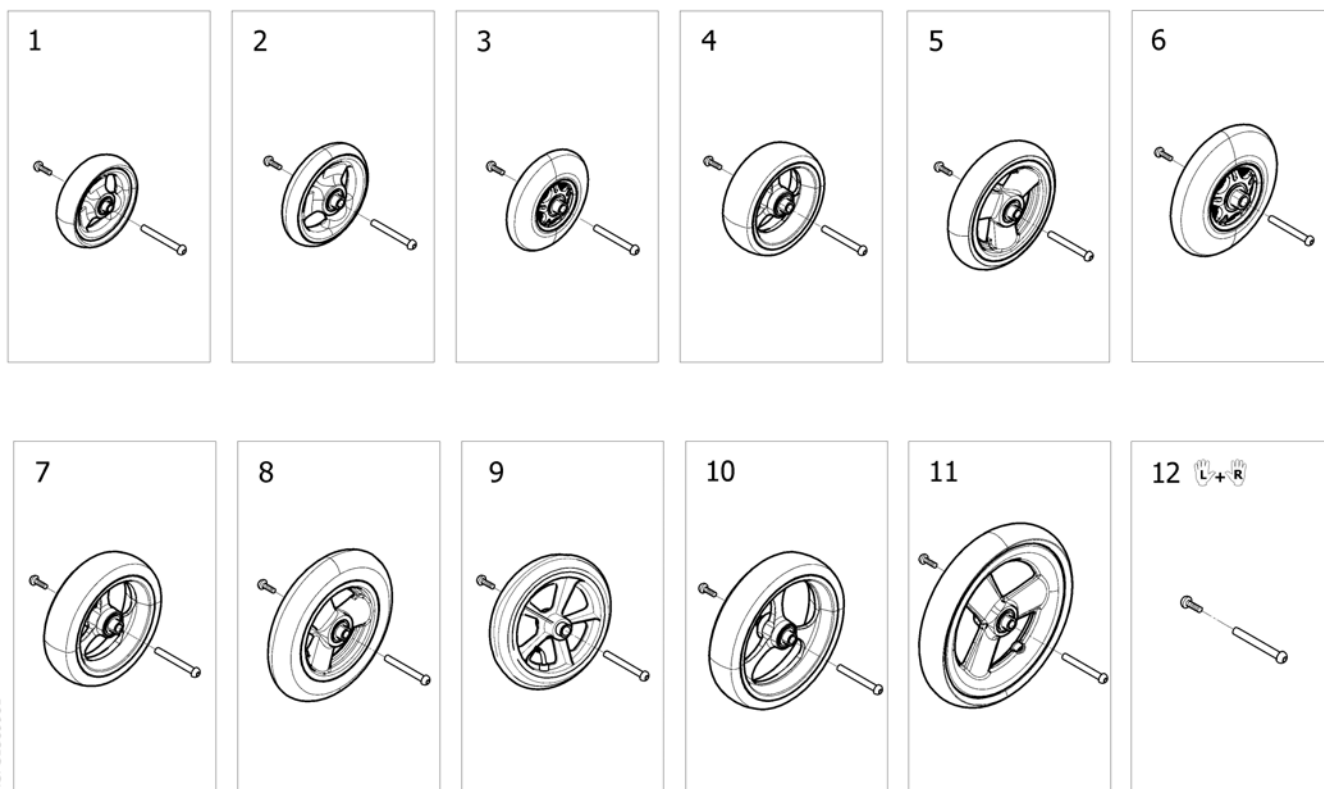

ISP51589961

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1584356 | Front wheel, 4", softroll for fork W40mm [SJ_7000] | | 2 |
| 2 | SP1584357 | Front wheel, 4" x 1", solid for fork W40mm [SJ_7020] | | 2 |
| 3 | SP1584358 | Front wheel, 4", solid lights for fork W40mm [SJ_7030] | | 2 |
| 4 | SP1584359 | Front wheel, 4", alu softroll for fork W40mm [SJ_7040] | | 2 |
| 5 | SP1584354 | Front wheel, 5" x 1", solid for fork W40mm [SJ_7055] | | 2 |
| 6 | SP1584355 | Front wheel, 5", solid lights for fork W40mm [SJ_7060] | | 2 |
| 7 | SP1584360 | Front wheel, 5", alu softroll for fork W40mm [SJ_7070] | | 2 |
| 8 | SP1584238 | Front wheel, 6" x 1", solid for fork W40mm [SJ_7080] | | 2 |
| 9 | SP1584239 | Front wheel, 6" x 1", pneumatic for fork W40mm [SJ_7090] | | 2 |
| 10 | SP1584361 | Front wheel, 6", alu softroll for fork W40mm [SJ_7100] | | 2 |
| 11 | SP1584236 | Front wheel, 7" x 1", solid for fork W40mm [SJ_7110] | | 2 |
| 12 | SP1570168 | Kit axle M8 for forks L40 (Pair) | | 1 |

Rear Wheels

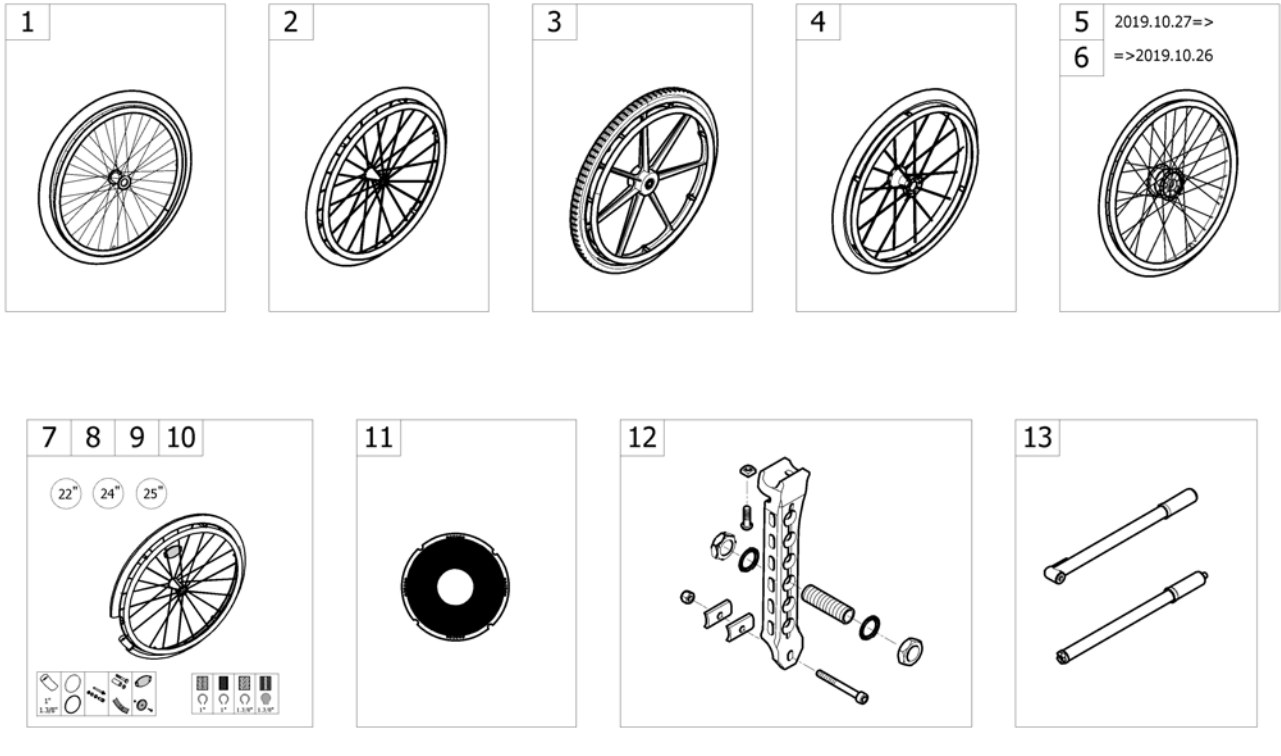

-<SPACTION555


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | 22"/24" HD Cross Spoked wheels [SJ_7200/SJ_7230] | | 1 |
| 2 | - | 22"/24"/25" Lightweight [SJ_7210/SJ_7240/SJ_7250] | | 1 |
| 3 | - | 22"/24" Composite wheels [SJ_7220/SJ_7242] | | 1 |
| 4 | - | 24"/25" Spinerger Wheels [SJ_7244/SJ_7260] | | 1 |
| 5 | - | 22"/24" Hub brake Wheels (New) 2019.10.27=> | | 1 |
| 6 | - | 22"/24" Hub Brake Wheels =>2019.10.26 [(SJ_7200/SJ_7230)+SJ_8160] | | 1 |
| 7 | - | Standard & hub brake wheels accessories [SJ_7200/SJ_7230] | | 1 |
| 8 | - | Lightweight wheels accessories [SJ_7210/SJ_7240/SJ_7250] | | 1 |
| 9 | - | Composite wheels accessories [SJ_7220/SJ_7242] | | 1 |
| 10 | - | Spinerger wheels accessories [SJ_7244/SJ_7260] | | 1 |
| 11 | - | Spoke protector [SJ_8000] | | 1 |
| 12 | - | Rear wheel support | | 1 |
| 13 | - | Pump [SJ_8020/SJ_8030] | | 1 |
| 14 | - | Bracket set for rear wheel support Twion & E-motion | | 1 |

22"/24" HD Cross Spoked wheels

| | | | |
|-----|----|--------|--------|
| | | | |
| | 1" | 1.3/8" | 1.3/8" |
| 22" | 1 | 2 | 3 |
| 24" | 4 | 5 | 6 |

| | | | |
|-----|----|--------|--------|
| | | | |
| | 1" | 1.3/8" | 1.3/8" |
| 22" | 7 | 8 | 9 |
| 24" | 10 | 11 | 12 |

| | | | |
|-----|----|--------|--------|
| | | | |
| | 1" | 1.3/8" | 1.3/8" |
| 22" | - | - | - |
| 24" | 13 | 14 | 15 |

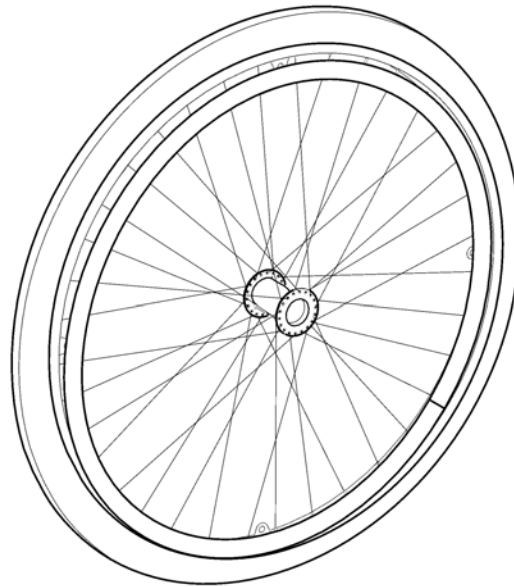

ISPS1590798

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1588645 | Rear wheel 22" HD, Cross spoked wheel, HP Pneumatic tyre Grey & Black 1", Aluminium handrim <i>[SJ_7200+SJ_7310+SJ_7400]</i> | 2018-06-30 → | 2 |
| 2 | 1498493 | Rear wheel 22"x 1.3/8", Heavy Duty, Spoked wheel, Grey Pneumatic tyre, Aluminium handrim <i>[SJ_7200+SJ_7305+SJ_7400]</i> | | 2 |
| 3 | SP1588643 | Rear wheel 22" HD, Cross spoked wheel, Solid tyre Grey 1.3/8", Aluminium handrim <i>[SJ_7200+SJ_7300+SJ_7400]</i> | | 2 |
| 4 | SP1588711 | Rear wheel 24" HD, Cross spoked wheel, HP Pneumatic tyre Grey & Black 1", Aluminium handrim <i>[SJ_7230+SJ_7310+SJ_7400]</i> | | 2 |
| 5 | 1498486 | Rear wheel 24"x 1.3/8", Heavy Duty, Spoked rim, Grey Pneumatic tyre, Aluminium handrim <i>[SJ_7230+SJ_7305+SJ_7400]</i> | | 2 |
| 6 | 1498467 | Rear wheel 24"x 1.3/8", Heavy Duty, Spoked rim, Grey Solid tyre, Aluminium handrim <i>[SJ_7230+SJ_7300+SJ_7400]</i> | | 2 |
| 7 | SP1588647 | Rear wheel 22" HD, Cross spoked wheel, HP pneumatic tyre Grey & Black 1", Anti-slip handrim <i>[SJ_7200+SJ_7310+SJ_7420]</i> | 2018-06-30 → | 2 |
| 8 | 1498494 | Rear wheel 22"x 1.3/8", Heavy Duty, Spoke wheel, Grey Pneumatic tyre, Anti-slip handrim <i>[SJ_7200+SJ_7305+SJ_7420]</i> | | 2 |
| 9 | SP1588644 | Rear wheel 22" HD, Cross spoked wheel, Solid tyre Grey 1.3/8", Anti-slip handrim <i>[SJ_7200+SJ_7300+SJ_7420]</i> | | 2 |
| 10 | SP1588742 | Rear wheel 24" HD, Cross spoked wheel, HP Pneumatic tyre Grey & Black 1", Anti-slip handrim <i>[SJ_7230+SJ_7310+SJ_7420]</i> | | 2 |
| 11 | 1498492 | Rear wheel 24"x 1.3/8", Heavy Duty, Spoked rim, Pneumatic tyre Grey, Anti-slip handrim <i>[SJ_7230+SJ_7305+SJ_7420]</i> | | 2 |
| 12 | 1498478 | Rear wheel 24"x1.3/8", Heavy Duty, Spoked rim, Grey Solid tyre, Anti-slip handrim <i>[SJ_7230+SJ_7300+SJ_7420]</i> | | 2 |


22"/24" HD Cross Spoked wheels

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 13 | SP1588744 | Rear wheel 24" HD, Cross spoked wheel, HP Pneumatic tyre Black & Grey 1", Stainless Steel handrim <i>[SJ_7230+SJ_7310+SJ_7410] ONLY FOR EUROPE</i> | | 2 |
| 14 | SP1588651 | Rear wheel 24" HD, Cross spoked wheel, Pneumatic tyre Grey 1.3/8", Stainless Steel handrim <i>[SJ_7230+SJ_7305+SJ_7410] ONLY FOR EUROPE</i> | | 2 |
| 15 | SP1588649 | Rear wheel 24" HD, Cross spoked wheel, Solid tyre Grey 1.3/8", Stainless Steel handrim <i>[SJ_7230+SJ_7300+SJ_7410] ONLY FOR EUROPE</i> | | 2 |

22"/24"/25" Lightweight

| | | | | |
|-----|----|----|--------|--------|
| | | | | |
| | 1" | 1" | 1.3/8" | 1.3/8" |
| 22" | 1 | - | 2 | 3 |
| 24" | 4 | 5 | 6 | 7 |
| 25" | 8 | 9 | - | - |

| | | | | |
|-----|----|----|--------|--------|
| | | | | |
| | 1" | 1" | 1.3/8" | 1.3/8" |
| 22" | 10 | - | 11 | 12 |
| 24" | 13 | 14 | 15 | 16 |
| 25" | 17 | 18 | - | - |

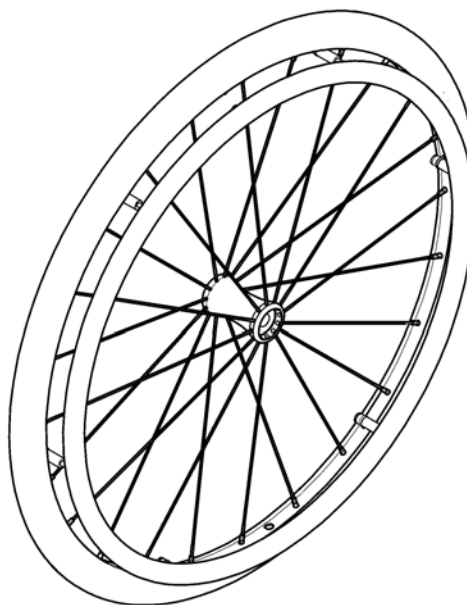





ISPS1590800 ~ sps1588755





| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1588896 | Rear wheel, 22", lightweight, pneumatic Right Run Plus HP 1", aluminium handrim <i>[SJ_7210+SJ_7310+SJ_7400]</i> | 2018-06-10 → | 2 |
| 2 | SP1588890 | Rear wheel, 22", lightweight, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7210+SJ_7305+SJ_7400]</i> | | 2 |
| 3 | SP1588888 | Rear wheel, 22", lightweight, solid tyre 1.3/8", aluminium handrim <i>[SJ_7210+SJ_7300+SJ_7400]</i> | | 2 |
| 4 | SP1588754 | Rear wheel, 24", lightweight, pneumatic Right Run Plus HP 1", aluminium handrim <i>[SJ_7240+SJ_7310+SJ_7400]</i> | | 2 |
| 5 | SP1588756 | Rear wheel, 24", lightweight, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7240+SJ_7320+SJ_7400]</i> | | 2 |
| 6 | SP1588752 | Rear wheel, 24", lightweight, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7240+SJ_7305+SJ_7400]</i> | | 2 |
| 7 | SP1588750 | Rear wheel, 24", lightweight, solid tyre 1.3/8", aluminium handrim <i>[SJ_7240+SJ_7300+SJ_7400]</i> | | 2 |
| 8 | SP1588746 | Rear wheel, 25", lightweight, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7250+SJ_7310+SJ_7400]</i> | 2018-06-10 → | 2 |
| 9 | SP1588748 | Rear wheel, 25", lightweight, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7250+SJ_7320+SJ_7400]</i> | | 2 |
| 10 | SP1588897 | Rear wheel, 22", lightweight, pneumatic Right Run Plus, anti-slip handrim <i>[SJ_7210+SJ_7310+SJ_7420]</i> | 2018-06-10 → | 2 |
| 11 | SP1588895 | Rear wheel, 22", lightweight, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7210+SJ_7305+SJ_7420]</i> | | 2 |
| 12 | SP1588889 | Rear wheel, 22", lightweight, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7210+SJ_7300+SJ_7420]</i> | | 2 |
| 13 | SP1588755 | Rear wheel, 24", lightweight, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7240+SJ_7310+SJ_7420]</i> | | 2 |
| 14 | SP1588887 | Rear wheel, 24", lightweight, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7240+SJ_7320+SJ_7420]</i> | | 2 |
| 15 | SP1588753 | Rear wheel, 24", lightweight, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7240+SJ_7305+SJ_7420]</i> | | 2 |
| 16 | SP1588751 | Rear wheel, 24", lightweight, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7240+SJ_7300+SJ_7420]</i> | | 2 |
| 17 | SP1588747 | Rear wheel, 25", lightweight, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7250+SJ_7310+SJ_7420]</i> | 2018-06-10 → | 2 |


Invacare® Action 5 | Rear Wheels
22"/24"/25" Lightweight

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 18 | SP1588749 | Rear wheel, 25", lightweight, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7250+SJ_7320+SJ_7420]</i> | | 2 |

22"/24" Composite wheels




| | | | |
|---|---|---|---|
|  |  |  |  |
| | 1.3/8" | 1.3/8" | 1.3/8" |
| 22" | 1 | 2 | 3 |
| 24" | 4 | 5 | 6 |
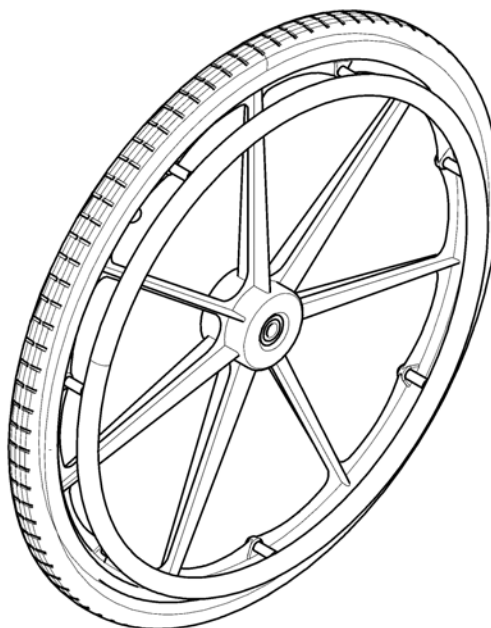
| | | | |
|---|---|---|---|
|  |  |  |  |
| | 1.3/8" | 1.3/8" | 1.3/8" |
| 22" | 7 | 8 | 9 |
| 24" | 10 | 11 | 12 |




ISP51590799 ~ sp51588904

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1588902 | Rear wheel, 22", composite, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7220+SJ_7305+SJ_7400]</i> | | 2 |
| 2 | SP1588900 | Rear wheel, 22", composite, solid tyre 1.3/8", aluminium handrim <i>[SJ_7220+SJ_7300+SJ_7400]</i> | | 2 |
| 3 | SP1589156 | Rear wheel, 22", composite, flat free insert, aluminium handrim <i>[SJ_7220+SJ_7330+SJ_7400]</i> | | 2 |
| 4 | SP1588906 | Rear wheel, 24", composite, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7242+SJ_7305+SJ_7400]</i> | | 2 |
| 5 | SP1588904 | Rear wheel, 24", composite, solid tyre 1.3/8", aluminium handrim <i>[SJ_7242+SJ_7300+SJ_7400]</i> | | 2 |
| 6 | SP1589158 | Rear wheel, 24", composite, flat free insert, aluminium handrim <i>[SJ_7242+SJ_7330+SJ_7400]</i> | | 2 |
| 7 | SP1588903 | Rear wheel, 22", composite, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7220+SJ_7305+SJ_7420]</i> | | 2 |
| 8 | SP1588901 | Rear wheel, 22", composite, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7220+SJ_7300+SJ_7420]</i> | | 2 |
| 9 | SP1589157 | Rear wheel, 22", composite, flat free insert, anti-slip handrim <i>[SJ_7220+SJ_7330+SJ_7420]</i> | | 2 |
| 10 | SP1588923 | Rear wheel, 24", composite, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7242+SJ_7305+SJ_7420]</i> | | 2 |
| 11 | SP1588905 | Rear wheel, 24", composite, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7242+SJ_7300+SJ_7420]</i> | | 2 |
| 12 | SP1589159 | Rear wheel, 24", composite, flat free insert, anti-slip handrim <i>[SJ_7242+SJ_7330+SJ_7420]</i> | | 2 |

| | | |
|---|---|---|
|  |  |  |
| | 1" | 1" |
| 24" | 1 | 2 |
| 25" | 3 | 4 |

| | | |
|---|---|---|
|  |  |  |
| | 1" | 1" |
| 24" | 5 | 6 |
| 25" | - | - |

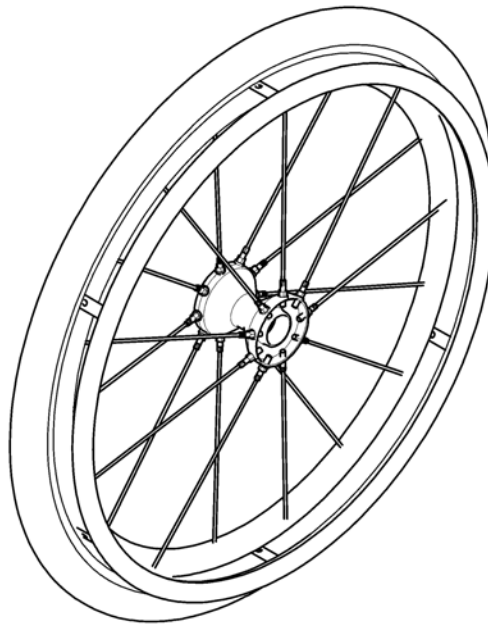


ISP51590801 ~ sps1588928

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1588927 | Rear wheel, 24", Spinergy, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7244+SJ_7310+SJ_7400]</i> | | 2 |
| 2 | SP1588929 | Rear wheel, 24", Spinergy, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7244+SJ_7320+SJ_7400]</i> | | 2 |
| 3 | SP1589150 | Rear wheel, 25", Spinergy, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7260+SJ_7310+SJ_7400]</i> | 2018-06-30 → | 2 |
| 4 | SP1589151 | Rear wheel, 25", Spinergy, pneumatic tyre Evo 1", aluminium handrim <i>[SJ_7260+SJ_7320+SJ_7400]</i> | | 2 |
| 5 | SP1588928 | Rear wheel, 24", Spinergy, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7244+SJ_7310+SJ_7420]</i> | | 2 |
| 6 | SP1589149 | Rear wheel, 24", Spinergy, pneumatic tyre Evo 1", anti-slip handrim <i>[SJ_7244+SJ_7320+SJ_7420]</i> | | 2 |

22"/24" Hub brake Wheels (New) 2019.10.27=>

| Action 5 | | | |
|----------|---|---|---|
| 22" | | | |
| | 1 | 3 | 5 |
| | 2 | 4 | 6 |

| Action 5 | | | |
|----------|---|----|----|
| 24" | | | |
| | 7 | 10 | 13 |
| | 8 | 11 | 14 |
| | 9 | 12 | 15 |

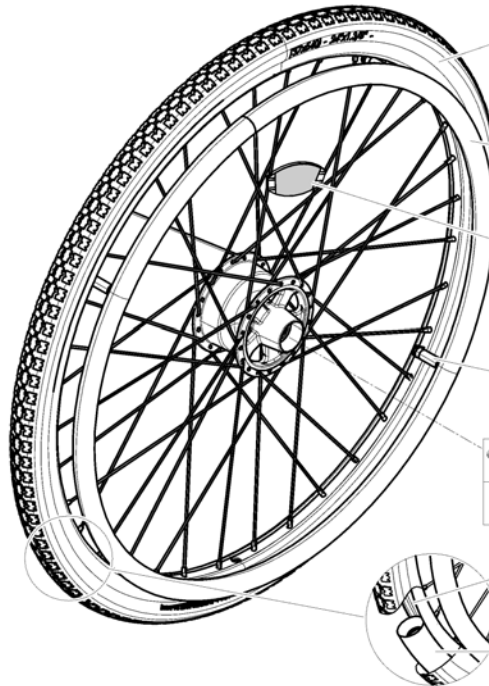

| 22" | | | 24" | | |
|-----|----|----|-----|----|----|
| | | | | | |
| 16 | 17 | 18 | 19 | 20 | 21 |

| 22" | | | 24" | | |
|-----|----|----|-----|----|--|
| | | | | | |
| 22 | 23 | 24 | 25 | 26 | |

| | |
|--|----|
| | 27 |
|--|----|

| | |
|--|----|
| | 28 |
|--|----|

| | |
|--|----|
| | 35 |
|--|----|

| 22" | 24" |
|-----|-----|
| | |
| 29 | 30 |

| 22" | | 24" | |
|-----|----|-----|----|
| | | | |
| 31 | 32 | 33 | 34 |





ISPS1657898





| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1661724 | Rear wheel 22"x 1", Spoked rim, Hub Brake, Grey/Black Right Run Plus Pneumatic tyre, Aluminium handrim <i>Replace SP1588646</i> | | 2 |
| 2 | SP1661725 | Rear wheel 22"x 1", Spoked rim, Hub Brake, Grey/Black Right Run Plus Pneumatic tyre, Anti-slip handrim <i>Replace SP1588648</i> | | 2 |
| 3 | SP1661687 | Rear wheel 22"x 1.3/8", Spoked rim, Hub Brake, Grey Pneumatic tyre, Aluminium handrim <i>Replace 1498466</i> | | 2 |
| 4 | SP1661713 | Rear wheel 22"x 1.3/8", Spoked rim, Hub Brake, Grey Pneumatic tyre, Anti-slip handrim <i>Replace 1557823</i> | | 2 |
| 5 | SP1661686 | Rear wheel 22"x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Aluminium handrim <i>Replace 1498461</i> | | 2 |
| 6 | SP1661688 | Rear wheel 22"x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Anti-slip handrim <i>Replace 1557801</i> | | 2 |
| 7 | SP1661731 | Rear wheel 24"x 1", Spoked rim, Hub Brake, Grey/Black Right Run Plus Pneumatic tyre, Aluminium handrim <i>Replace SP1588712</i> | | 2 |
| 8 | SP1661732 | Rear wheel 24"x 1", Spoked rim, Hub Brake, Grey/Black Right Run Plus Pneumatic tyre, Anti-slip handrim <i>Replace SP1588743</i> | | 2 |
| 9 | SP1661733 | Rear wheel 24"x 1", Spoked rim, Hub Brake, Grey/Black Right Run Plus Pneumatic tyre, Stainless handrim <i>Replace SP1588745</i> | | 2 |
| 10 | SP1661685 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Pneumatic tyre, Aluminium handrim <i>Replace 1498428</i> | | 2 |
| 11 | SP1661718 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Pneumatic tyre, Anti-slip handrim <i>Replace 1573481</i> | | 2 |
| 12 | SP1661730 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Pneumatic tyre, Stainless handrim <i>Replace SP1588652</i> | | 2 |





22"/24" Hub brake Wheels (New) 2019.10.27=>


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 13 | SP1661757 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Aluminium handrim <i>Replace 1498427</i> | | 2 |
| 14 | SP1661723 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Anti-slip handrim <i>Replace 1573482</i> | | 2 |
| 15 | SP1661729 | Rear wheel 24"x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Stainless handrim <i>Replace SP1588650</i> | | 2 |
| 16 | 1531456 | Pneumatic Right Run Plus, 22" x 1" (25-501), Grey/Black | | 2 |
| 17 | 0036-30035 | Pneumatic tyre 22" x 1.3/8", Grey | | 2 |
| 18 | 0036-10014 | Solid tyre 22" x 1.3/8", Grey | | 2 |
| 19 | SP1585334 | Pneumatic tyre, 24"x 1", HP Grey and Black | | 2 |
| 20 | 0036-30001 | Pneumatic Tyre, 24"x 1.3/8", Grey | | 2 |
| 21 | 0036-10013 | Solid tyre, 24" x 1.3/8", Grey | | 2 |
| 22 | 1505299 | Kit handrim 22", Aluminium Anodized, 4 fixations | | 2 |
| 23 | 1505920 | Kit anti-slip handrim 22", 4 fixations | | 2 |
| 24 | 1505291 | Kit Handrim 24" Aluminum 4 fixations | | 2 |
| 25 | 1505298 | Kit Handrim 24" Plastic 4 fixations | | 2 |
| 26 | SP1591282 | Kit Handrim 24" Stainless 4 fixations | | 2 |
| 27 | N90003 | Kit reflector for spoked wheel | | 2 |
| 29 | 5750029 | Rim tape 22" | | 2 |
| 30 | 0036-40003 | Rim tape 24" | | 2 |
| 31 | 1446040 | Inner tube, 22" x 1", with straight car valve | | 2 |
| 32 | 0036-20007 | Inner tube, 22" x 1.3/8" | | 2 |
| 33 | 1032240 | Inner tube, 24" x 1" | | 2 |
| 34 | 0036-20001 | Inner Tube 24"x 1.3/8" | | 2 |
| 35 | SP1661878 | Kit quick release axle for hub brake Action5 <i>Replace SP1591281</i> | | 2 |

22"/24" Hub Brake Wheels =>2019.10.26

| | | | |
|---|---|---|---|
|  |  |  |  |
| 22" | 1 | 2 | 3 |
| 24" | 4 | 5 | 6 |

| | | | |
|---|---|---|---|
|  |  |  |  |
| 22" | 7 | 8 | 9 |
| 24" | 10 | 11 | 12 |


| | | | |
|---|---|---|---|
|  |  |  |  |
| 22" | - | - | - |
| 24" | 13 | 14 | 15 |

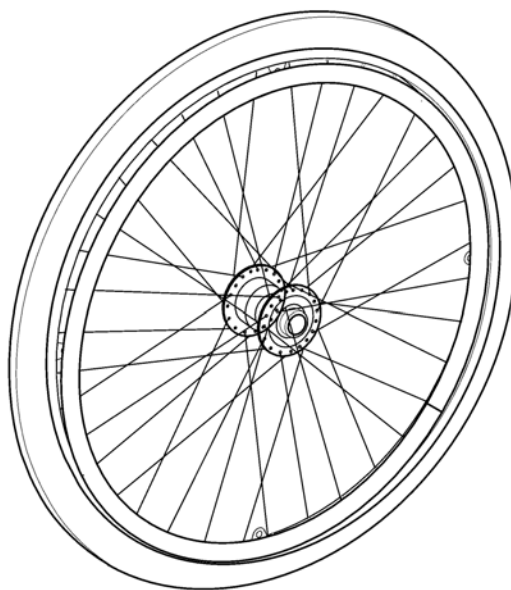

ISPS159080Z

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1588646 | Rear wheel, 22", hub brake, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7200+SJ_7310+SJ_7400+SJ_8160]</i> | 2018-06-30 → | 2 |
| 2 | 1498466 | Rear wheel 22" x 1.3/8", Spoke rim, Hub brake, Grey Pneumatic tyre, Aluminium handrim <i>[SJ_7200+SJ_7305+SJ_7400+SJ_8160]</i> | | 2 |
| 3 | 1498461 | Rear wheel 22" x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Aluminium handrim <i>[SJ_7200+SJ_7300+SJ_7400+SJ_8160]</i> | | 2 |
| 4 | SP1588712 | Rear wheel, 24", hub brake, pneumatic tyre HP 1", aluminium handrim <i>[SJ_7230+SJ_7310+SJ_7400+SJ_8160]</i> | | 2 |
| 5 | 1498428 | Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7305+SJ_7400+SJ_8160]</i> | | 2 |
| 6 | 1498427 | Rear wheel, 24", hub brake, solid tyre 1.3/8", aluminium handrim <i>[SJ_7230+SJ_7300+SJ_7400+SJ_8160]</i> | | 2 |
| 7 | SP1588648 | Rear wheel, 22", hub brake, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7200+SJ_7310+SJ_7420+SJ_8160]</i> | 2018-06-30 → | 2 |
| 8 | 1557823 | Rear wheel 22" x 1.3/8", Spoke rim, Hub Brake, Grey Pneumatic tyre, Anti-slip handrim <i>[SJ_7200+SJ_7305+SJ_7420+SJ_8160]</i> | | 2 |
| 9 | 1557801 | Rear wheel 22" x 1.3/8", Spoked rim, Hub Brake, Grey Solid tyre, Anti-slip handrim <i>[SJ_7200+SJ_7300+SJ_7420+SJ_8160]</i> | | 2 |
| 10 | SP1588743 | Rear wheel, 24", hub brake, pneumatic tyre HP 1", anti-slip handrim <i>[SJ_7230+SJ_7310+SJ_7420+SJ_8160]</i> | | 2 |
| 11 | 1573481 | Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7305+SJ_7420+SJ_8160]</i> | | 2 |
| 12 | 1573482 | Rear wheel, 24", hub brake, solid tyre 1.3/8", anti-slip handrim <i>[SJ_7230+SJ_7300+SJ_7420+SJ_8160]</i> | | 2 |
| 13 | SP1588745 | Rear wheel, 24", hub brake, pneumatic tyre HP 1", stainless handrim <i>[SJ_7230+SJ_7310+SJ_7410+SJ_8160] ONLY FOR EUROPE</i> | | 2 |
| 14 | SP1588652 | Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7305+SJ_7410+SJ_8160] ONLY FOR EUROPE</i> | | 2 |
| 15 | SP1588650 | Rear wheel, 24", hub brake, solid tyre 1.3/8", stainless handrim <i>[SJ_7230+SJ_7300+SJ_7410+SJ_8160] ONLY FOR EUROPE</i> | | 2 |
| 16 | 1505290 | Kit QR axle for hub brake spoked wheels | → 2019-10-26 | 2 |

Standard & hub brake wheels accessories

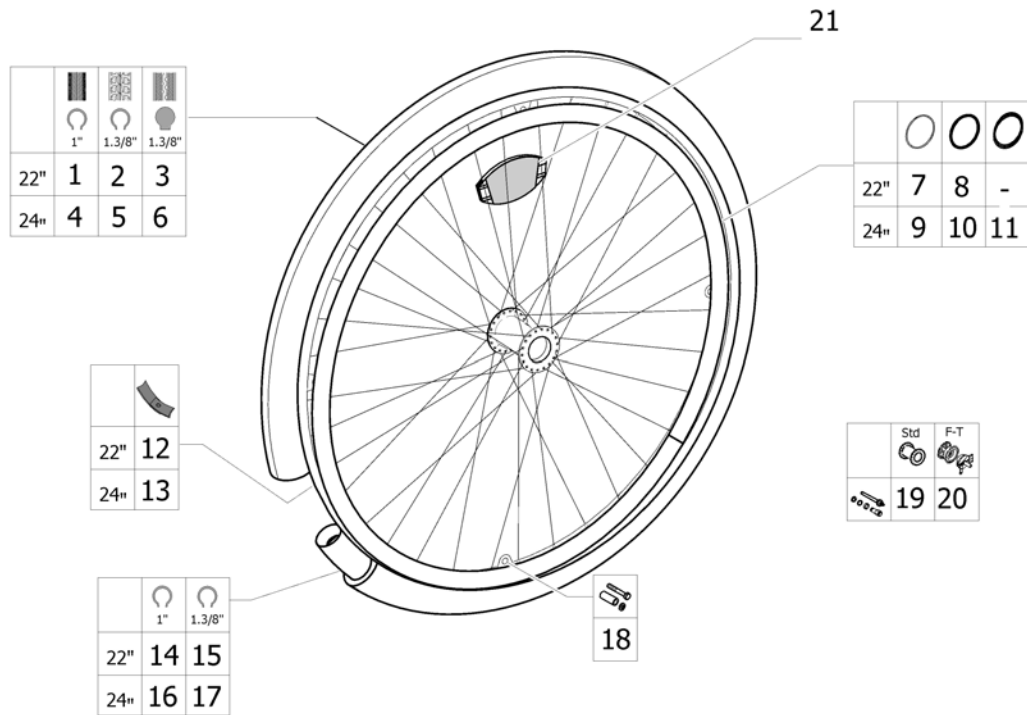


ISPS1590003

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 1531456 | Pneumatic Right Run Plus, 22"x 1" (25-501), Grey/Black [SJ_7200+SJ_7310] | 2018-06-30 → | 2 |
| 2 | 0036-30035 | Pneumatic tyre 22" x 1.3/8", Grey [SJ_7200+SJ_7305] | | 2 |
| 3 | 0036-10014 | Solid tyre 22" x 1.3/8", Grey [SJ_7200+SJ_7300] | | 2 |
| 4 | SP1585334 | Pneumatic tyre, 24"x 1", HP Grey and Black [SJ_7230+SJ_7310] | 2017-02-27 → | 2 |
| 5 | 0036-30001 | Pneumatic Tyre, 24"x 1.3/8", Grey [SJ_7230+SJ_7305] | | 2 |
| 6 | 0036-10013 | Solid tyre, 24" x 1.3/8", Grey [SJ_7230+SJ_7300] | | 2 |
| 7 | 1505299 | Kit handrim 22", Aluminium Anodized, 4 fixations [SJ_7200+SJ_7400] | | 2 |
| 8 | 1505920 | Kit anti-slip handrim 22", 4 fixations [SJ_7200+SJ_7420] | | 2 |
| 9 | 1505291 | Kit Handrim 24" Aluminum 4 fixations [SJ_7230+SJ_7400] | | 2 |
| 10 | 1505298 | Kit Handrim 24" Plastic 4 fixations [SJ_7230+SJ_7420] | | 2 |
| 11 | SP1591282 | Kit Handrim 24" Stainless 4 fixations [SJ_7230+SJ_7410] ONLY FOR EUROPE | | 2 |
| 12 | 5750029 | Rim tape 22" [SJ_7200] | | 2 |
| 13 | 0036-40003 | Rim tape 24" [SJ_7230] | | 2 |
| 14 | 1446040 | Inner tube, 22" x 1", with straight car valve [SJ_7200+SJ_7310] - replaces 1061214 | | 2 |
| 15 | 0036-20007 | Inner tube, 22" x 1.3/8" [SJ_7200+SJ_7305] | | 2 |
| 16 | 1032240 | Inner tube, 24" x 1" [SJ_7230+SJ_7310] | | 2 |
| 17 | 0036-20001 | Inner Tube 24"x 1.3/8" [SJ_7230+SJ_7305] | | 2 |
| 18 | 1573626 | Kit handrim fixation for spoked wheel, 4 pts | | 2 |
| 19 | SP1591280 | Kit quick release axle | | 2 |

Standard & hub brake wheels accessories

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 20 | SP1591281 | Kit quick release axle for hub brake <i>[SJ_8160] - This kit axle is only for OLD Rear Wheel Hub Brake</i> | → 2019-10-26 | 2 |
| 21 | N90003 | Kit reflector for spoked wheel <i>[SJ_8600]</i> | | 2 |

Lightweight wheels accessories

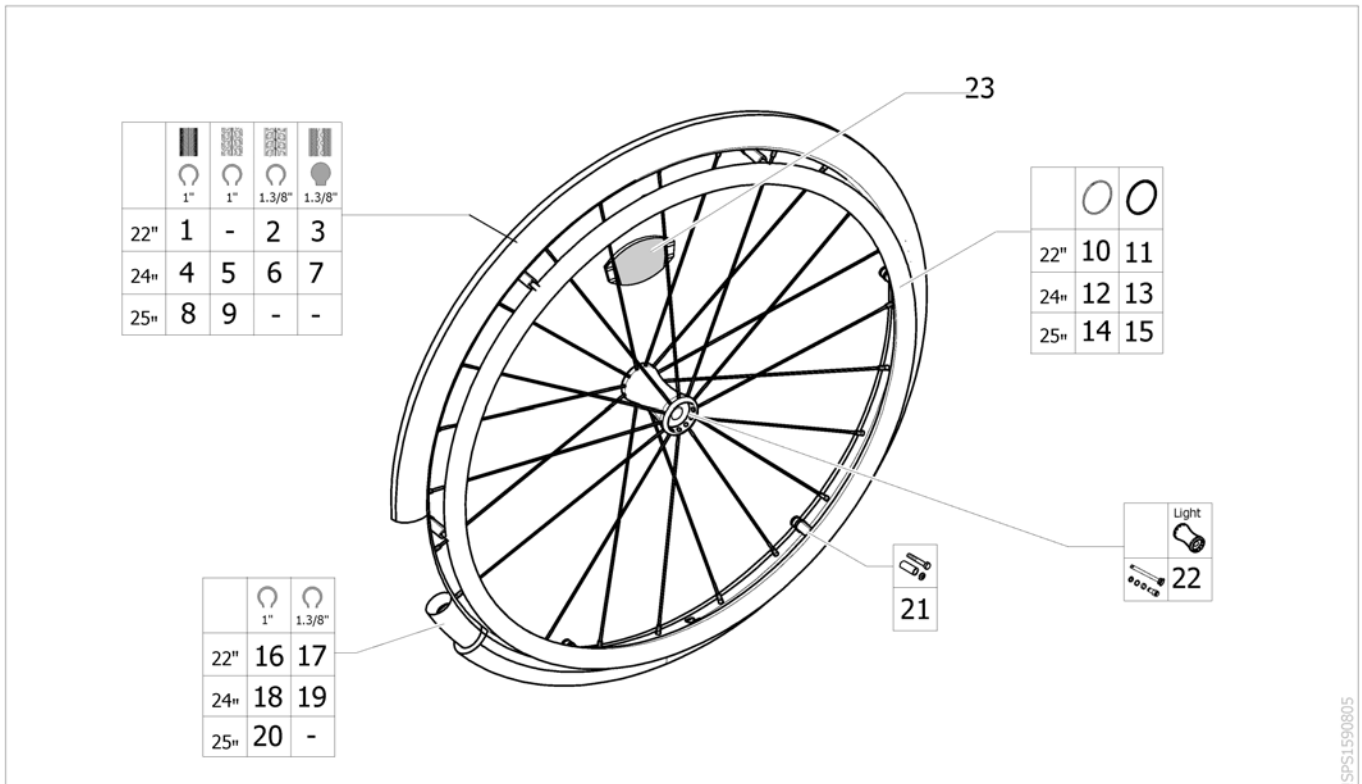


ISPS1590805

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1531456 | Pneumatic Right Run Plus, 22"x 1" (25-501), Grey/Black <i>[SJ_7210+SJ_7310]</i> | 2018-06-30 → | 2 |
| 2 | 0036-30035 | Pneumatic tyre 22" x 1.3/8", Grey <i>[SJ_7210+SJ_7305]</i> | | 2 |
| 3 | 0036-10014 | Solid tyre 22" x 1.3/8", Grey <i>[SJ_7210+SJ_7300]</i> | | 2 |
| 4 | SP1585334 | Pneumatic tyre, 24"x 1", HP Grey and Black <i>[SJ_7240+SJ_7310]</i> | 2017-02-27 → | 2 |
| 5 | SP1585331 | Pneumatic tyre, 24" x 1", Marathon Plus Evo <i>[SJ_7240+SJ_7320]</i> | | 2 |
| 6 | 0036-30001 | Pneumatic Tyre, 24"x 1.3/8", Grey <i>[SJ_7240+SJ_7305]</i> | | 2 |
| 7 | 0036-10013 | Solid tyre, 24" x 1.3/8", Grey <i>[SJ_7240+SJ_7300]</i> | | 2 |
| 8 | SP1585335 | Pneumatic tyre, 25" x 1", Right Run Plus grey Black <i>[SJ_7250+SJ_7310]</i> | 2018-06-10 → | 2 |
| 9 | SP1585332 | Pneumatic tyre, 25" x 1", Marathon Plus Evo <i>[SJ_7250+SJ_7320]</i> | | 2 |
| 10 | SP1601462 | Kit aluminium handrim 22", 6 fixations, Lightweight <i>[SJ_7210+SJ_7400]</i> | | 2 |
| 11 | SP1601463 | Kit anti-slip handrim 22", 6 fixations, Lightweight <i>[SJ_7210+SJ_7420]</i> | | 2 |
| 12 | SP1601464 | Kit aluminium handrim 24", 6 fixations, Lightweight <i>[SJ_7240+SJ_7400]</i> | | 2 |
| 13 | SP1601465 | Kit anti-slip handrim 24", 6 fixations, Lightweight <i>[SJ_7240+SJ_7420]</i> | | 2 |
| 14 | SP1601466 | Kit aluminium handrim 25", 6 fixations, Lightweight <i>[SJ_7250+SJ_7400]</i> | | 2 |
| 15 | SP1601467 | Kit anti-slip handrim 25", 6 fixations, Lightweight <i>[SJ_7250+SJ_7420]</i> | | 2 |
| 16 | 1446040 | Inner tube, 22" x 1", with straight car valve <i>[SJ_7210+SJ_7310] - replaces 1061214</i> | | 2 |
| 17 | 0036-20007 | Inner tube, 22" x 1.3/8" <i>[SJ_7210+SJ_7305]</i> | | 2 |
| 18 | 1032240 | Inner tube, 24" x 1" <i>[SJ_7240+SJ_7310/SJ_7320]</i> | | 2 |

Lightweight wheels accessories

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 19 | 0036-20001 | Inner Tube 24"x 1.3/8" <i>[SJ_7240+SJ_7305]</i> | | 2 |
| 20 | SP1430886 | Inner tube, 25", for High-Performance wheel <i>[SJ_7250+SJ_7310/SJ_7320]</i> | | 2 |
| 21 | SP1601472 | Kit handrim fixation for Lightweight wheel, 6 pts | | 2 |
| 22 | SP1601461-0005 | Kit quick release axle with Grey cap <i>[SJ_6100]</i> | | 2 |
| 22 | SP1601461-0641 | Kit quick release axle with Blue cap <i>[SJ_6110]</i> | | 2 |
| 22 | SP1601461-0642 | Kit quick release axle with Purple cap <i>[SJ_6200]</i> | | 2 |
| 23 | N90003 | Kit reflector for spoked wheel <i>[SJ_8600]</i> | | 2 |


Composite wheels accessories

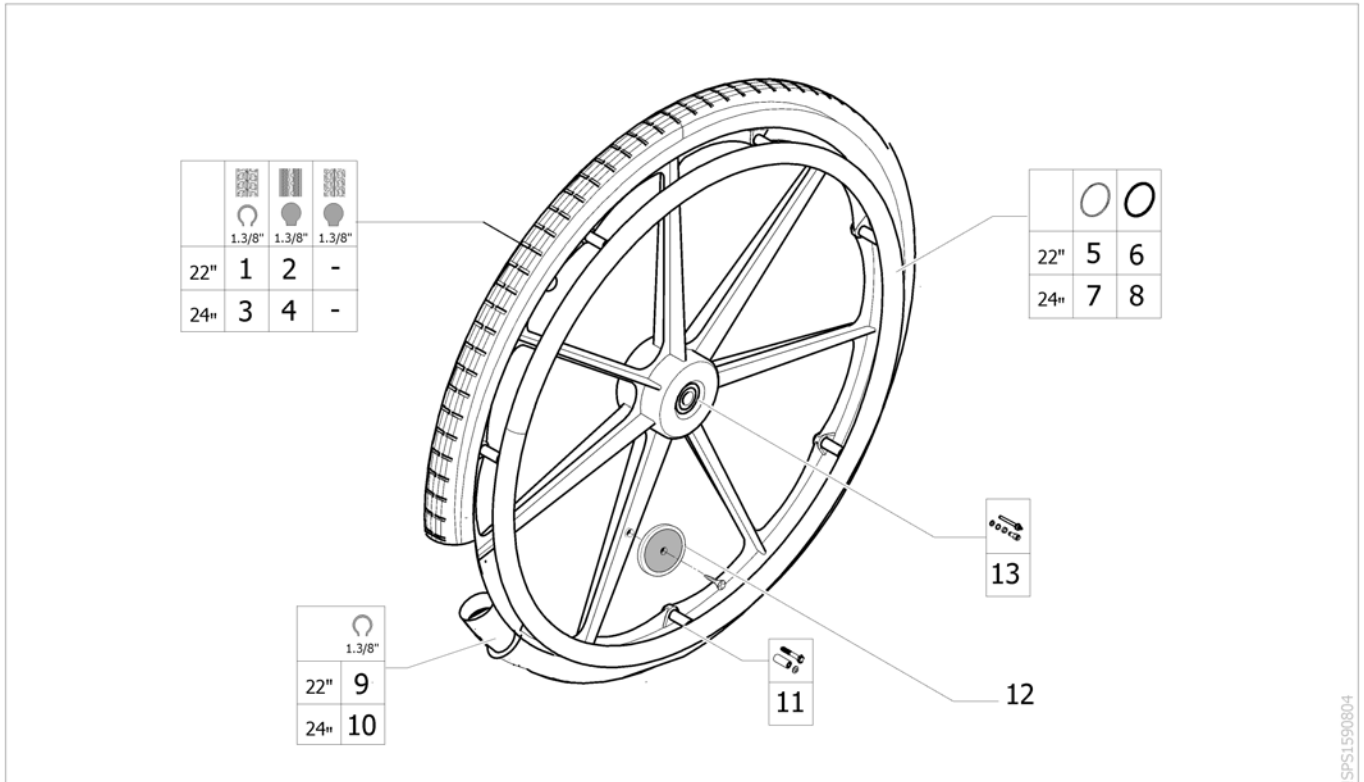

ISPS1590804

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 0036-30035 | Pneumatic tyre 22" x 1.3/8", Grey [SJ_7220+SJ_7305] | | 2 |
| 2 | 0036-10014 | Solid tyre 22" x 1.3/8", Grey [SJ_7220+SJ_7300] | | 2 |
| 3 | 0036-30001 | Pneumatic Tyre, 24"x 1.3/8", Grey [SJ_7242+SJ_7305] | | 2 |
| 4 | 0036-10013 | Solid tyre, 24" x 1.3/8", Grey [SJ_7242+SJ_7300] | | 2 |
| 5 | SP1601839 | Kit aluminium handrim 22", 7 fixations, composite wheel [SJ_7220+SJ_7400] | | 2 |
| 6 | SP1601840 | Kit anti-slip handrim 22", 7 fixations, composite wheel [SJ_7220+SJ_7420] | | 2 |
| 7 | SP1601841 | Kit aluminium handrim 24", 7 fixations, composite wheel [SJ_7242+SJ_7400] | | 2 |
| 8 | SP1601842 | Kit anti-slip handrim 24", 7 fixations, composite wheel [SJ_7242+SJ_7420] | | 2 |
| 9 | 0036-20007 | Inner tube, 22" x 1.3/8" [SJ_7220+SJ_7305] | | 2 |
| 10 | 0036-20001 | Inner Tube 24"x 1.3/8" [SJ_7242+SJ_7305] | | 2 |
| 11 | SP1601484 | Kit handrim fixation for composite wheel, 7 pts | | 2 |
| 12 | 9037-30001 | Kit reflector for composite wheel [(SJ_7220/SJ_7242)+SJ_8600] | | 2 |
| 13 | SP1591280 | Kit quick release axle | | 2 |



Spinergy wheels accessories

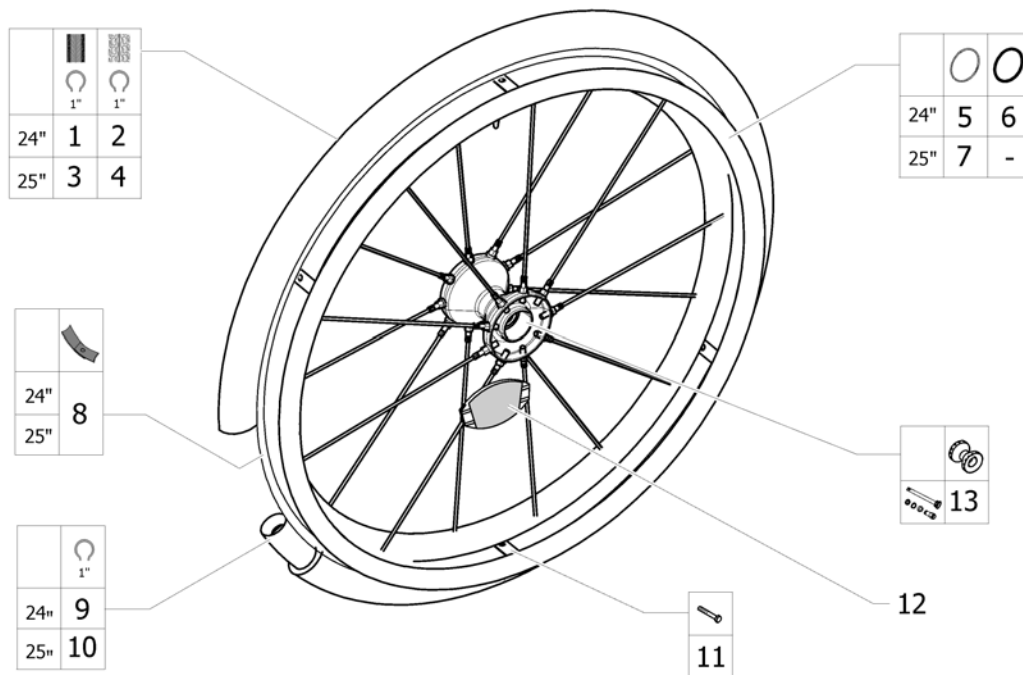

ISPS1590806

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|---|--------------------|-----|
| 1 | SP1585334 | Pneumatic tyre, 24"x 1", HP Grey and Black <i>[SJ_7244+SJ_7310]</i> | 2017-02-27 → | 2 |
| 2 | SP1585331 | Pneumatic tyre, 24" x 1", Marathon Plus Evo <i>[SJ_7244+SJ_7320]</i> | | 2 |
| 3 | SP1585335 | Pneumatic tyre, 25" x 1", Right Run Plus grey Black <i>[SJ_7260+SJ_7310]</i> | 2018-06-30 → | 2 |
| 4 | SP1585332 | Pneumatic tyre, 25" x 1", Marathon Plus Evo <i>[SJ_7260+SJ_7320]</i> | | 2 |
| 5 | SP1601468 | Kit aluminium handrim 24", 6 fixations, Spinergy <i>[SJ_7244+SJ_7400]</i> | | 2 |
| 6 | SP1601469 | Kit anti-slip handrim 24", 6 fixations Spinergy <i>[SJ_7244+SJ_7420]</i> | | 2 |
| 7 | SP1601470 | Kit aluminium handrim 25", 6 fixations, Spinergy <i>[SJ_7260+SJ_7400]</i> | | 2 |
| 8 | 0036-40003 | Rim tape 24" | | 2 |
| 9 | 1032240 | Inner tube, 24" x 1" <i>[SJ_7244+(SJ_7310/SJ_7320)]</i> | | 2 |
| 10 | SP1430886 | Inner tube, 25", for High-Performance wheel <i>[SJ_7260+(SJ_7310/SJ_7320)]</i> | | 2 |
| 11 | SP1601502 | Kit handrim fixation for Spinergy wheel, 6pts | | 2 |
| 12 | N90003 | Kit reflector for spoked wheel <i>[SJ_8600]</i> | | 2 |
| 13 | SP1603952-0005 | Kit quick release axle with Grey cap Wheel Spinergy <i>[SJ_6100]</i> | | 2 |
| 13 | SP1603952-0641 | Kit quick release axle with Blue cap Wheel Spinergy <i>[SJ_6100]</i> | | 2 |
| 13 | SP1603952-0642 | Kit quick release axle with Purple cap Wheel Spinergy <i>[SJ_6100]</i> | | 2 |


Spoke protector

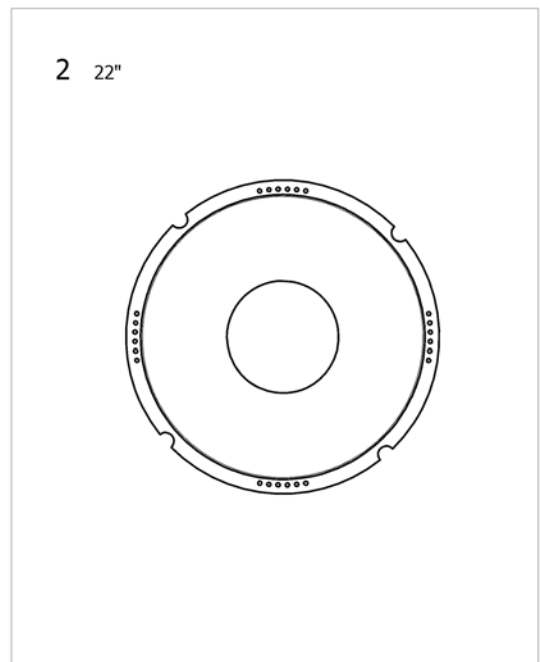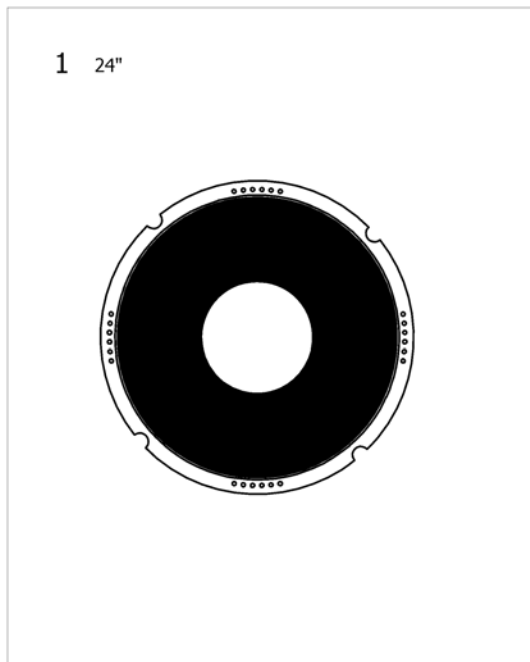

ISPS1590810


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 0789-04400 | Spoke guard 24" Black (pair) <i>[SJ_7230+SJ_8000]</i> | | 2 |
| 2 | 1556990 | Spoke guard 22" Transparent (Pair) <i>[SJ_7200+SJ_8000]</i> | | 2 |

Rear wheel support

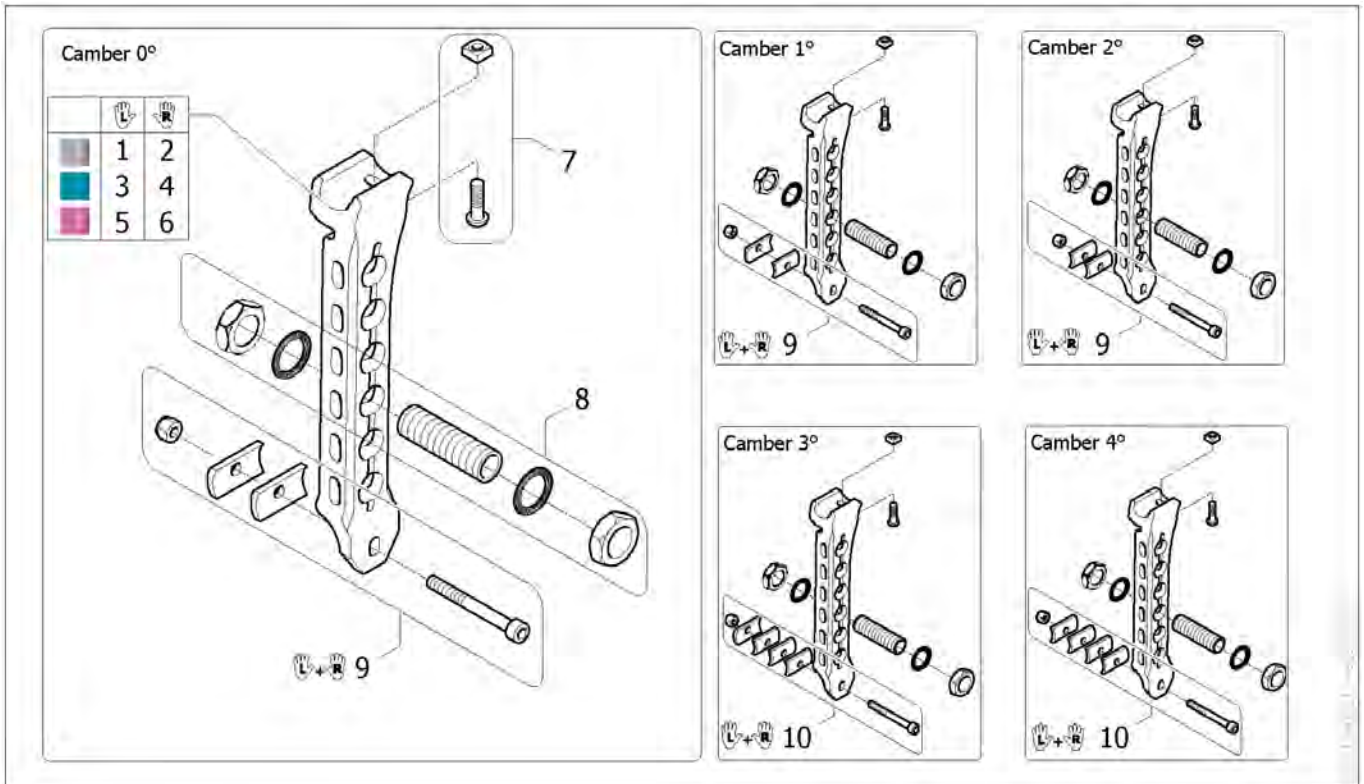

| Pos. | Part number | Description | Valid from → until | Qty |
|------|----------------|--|--------------------|-----|
| 1 | SP1554604-0005 | Kit rear wheel support, Lh, without accent <i>[SJ_6100]</i> | | 1 |
| 2 | SP1555921-0005 | Kit rear wheel support, Rh, without accent <i>[SJ_6100]</i> | | 1 |
| 3 | SP1554604-0641 | Kit rear wheel support, Lh, accent Blue <i>[SJ_6110]</i> | | 1 |
| 4 | SP1555921-0641 | Kit rear wheel support, Rh, accent Blue <i>[SJ_6110]</i> | | 1 |
| 5 | SP1554604-0642 | Kit rear wheel support, Lh, accent Purple <i>[SJ_6200]</i> | | 1 |
| 6 | SP1555921-0642 | Kit rear wheel support, Rh, accent Purple <i>[SJ_6200]</i> | | 1 |
| 7 | SP1601541 | Kit fixation rear wheel support | | 2 |
| 8 | SP1601542 | Kit rear wheel bearing | | 2 |
| 9 | SP1584797 | Kit camber 0° to 2° (pair) <i>[SJ_7700/SJ_7710/SJ_7720]</i> | | 1 |
| 10 | SP1584798 | Kit camber 3° to 4° (pair) <i>[SJ_7730/SJ_7740]</i> | | 1 |


Pump

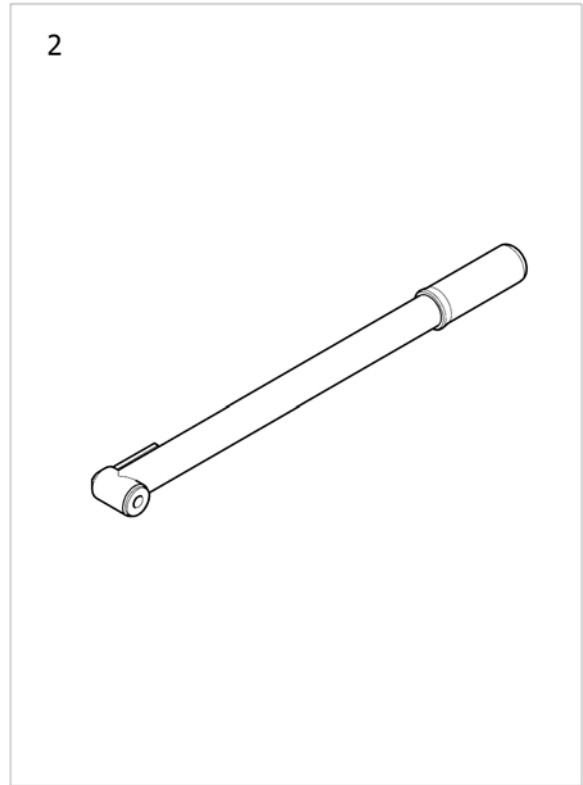

ISPS1590812




| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1590818 | Pump - Low pressure <i>[SJ_8020]</i> | | 1 |
| 2 | SP1590819 | Pump - High pressure <i>[SJ_8030]</i> | | 1 |

Bracket set for rear wheel support Twion & E-motion

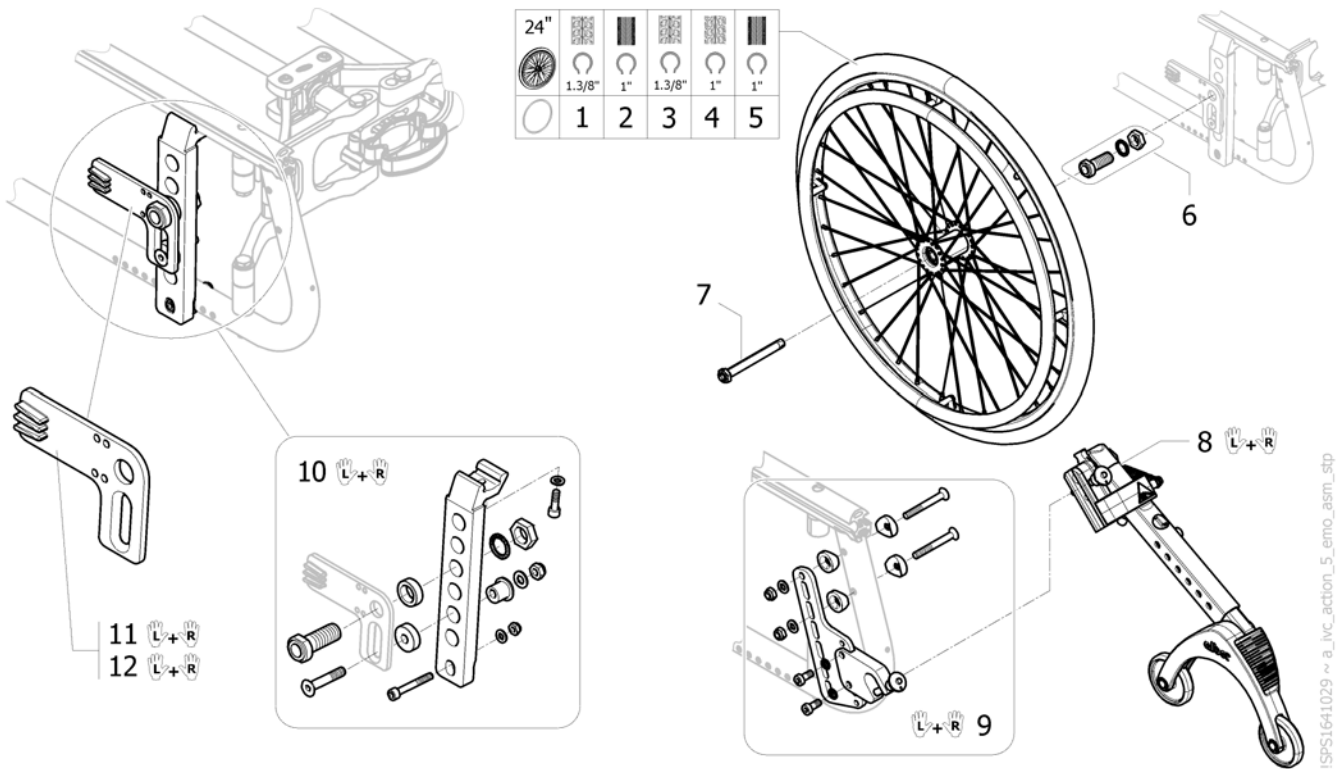

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1656499 | Wheel 24x1 3/8" std, cross spoked, grey pneu tyre, block pattern, aluminium handrim, for 12,7mm axle | 2019-01-01 → | 2 |
| 2 | SP1641140 | Rear wheel 24"x 1"Pneumatic Right Run Plus, Alu handrim, for 12,7 axle | | 2 |
| 3 | SP1656479 | Wheel 24x1 3/8" lightweight, pneu grey tyre, block pattern, aluminium handrim, for 12,7mm axle | 2019-01-01 → | 2 |
| 4 | SP1652088 | Wheel 24x1", Spoked, Lightweight pneumatic marathon Aluminium Handrim for 12.7 mm axle | 2018-07-01 → | 2 |
| 5 | SP1656467 | wheel 24x1" lightweight radial spoked, black and grey pneu tyre, Aluminium handrim for 12,7mm axle | 2019-01-01 → | 2 |
| 6 | 1463606 | Axle Receiver 1/2, Length=50,8Mm Incl. Nord-Lock And Nut Incl. Nord-Lock And Nut | | 1 |
| 7 | 1456261 | Stub Axle 1/2, Le=104.05 Mm L=123.00 Mm | | 2 |
| 8 | 1593206 | black anti-tippers with jack up function (pair) <i>replaces 1489214</i> | 2018-01-01 → | 1 |
| 9 | SP1566600 | Fixation Anti-tipper kit for E-Motion and Twion (pair) | 2018-01-01 → | 1 |
| 10 | SP1566599 | Bracket Set Twion (pair) | | 1 |
| 11 | 1565787 | Torque receiver with angle for Twion & E-Motion M25 (pair) | | 1 |
| 12 | 1463605 | Torque receiver with angle for E-Motion M15 (pair) | 2018-01-01 → | 1 |


Brakes

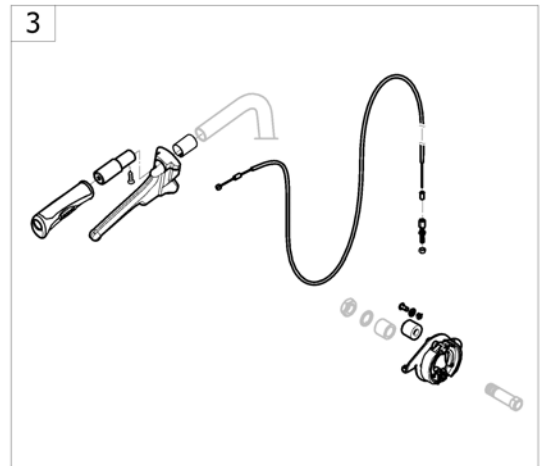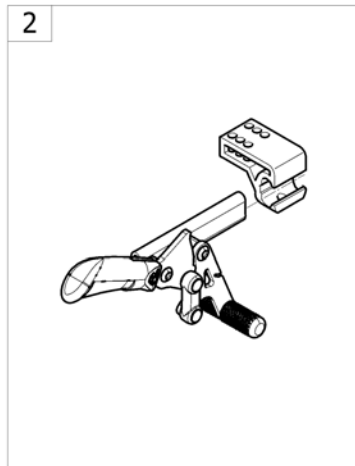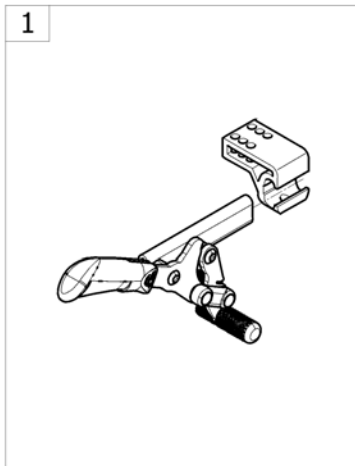

-<SPOACTION565

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Push Brakes <i>[SJ_8100][SJ_8120]</i> | | 1 |
| 2 | - | Pull Brakes <i>[SJ_8110][SJ_8130]</i> | | 1 |
| 3 | - | Attendant Hub Brake <i>[SJ_8160]</i> | | 1 |


Push Brakes

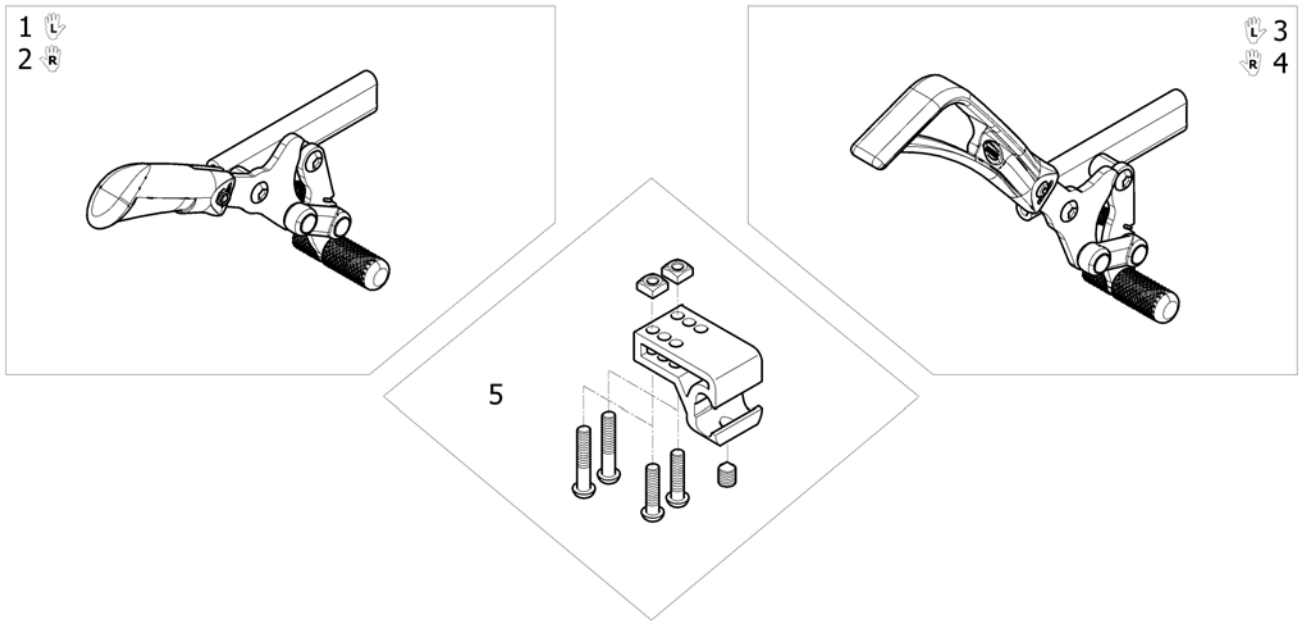

ISPS1589977

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1581538 | Push brake, Lh <i>[SJ_8100]</i> | | 1 |
| 2 | SP1581537 | Push brake, Rh <i>[SJ_8100]</i> | | 1 |
| 3 | SP1581546 | Push brake, Lh, extension handle <i>[SJ_8120]</i> | | 1 |
| 4 | SP1581545 | Push brake, Rh, extension handle <i>[SJ_8120]</i> | | 1 |
| 5 | SP1589978 | Kit brakes clamp | | 2 |


Pull Brakes

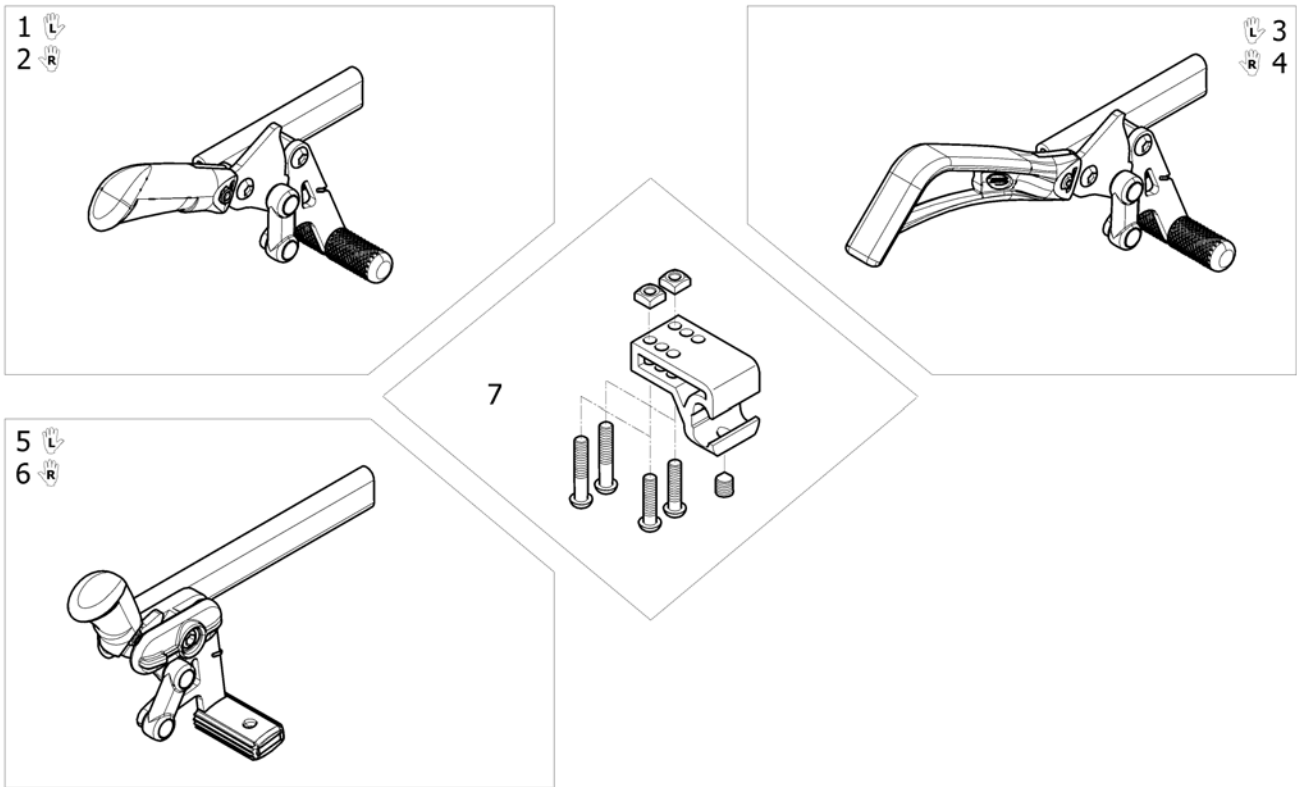

ISPS1590815





| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1581534 | Pull brake, Lh <i>[SJ_8110]</i> | | 1 |
| 2 | SP1581533 | Pull brake, Rh <i>[SJ_8110]</i> | | 1 |
| 3 | SP1581542 | Pull brake, Lh, extension handle <i>[SJ_8130]</i> | | 1 |
| 4 | SP1581541 | Pull brake, Rh, extension handle <i>[SJ_8130]</i> | | 1 |
| 5 | SP1559151 | Left pull brake long support | | 1 |
| 6 | SP1559150 | Right pull brake long support | | 1 |
| 7 | SP1589978 | Kit brakes clamp | | 2 |

Attendant Hub Brake

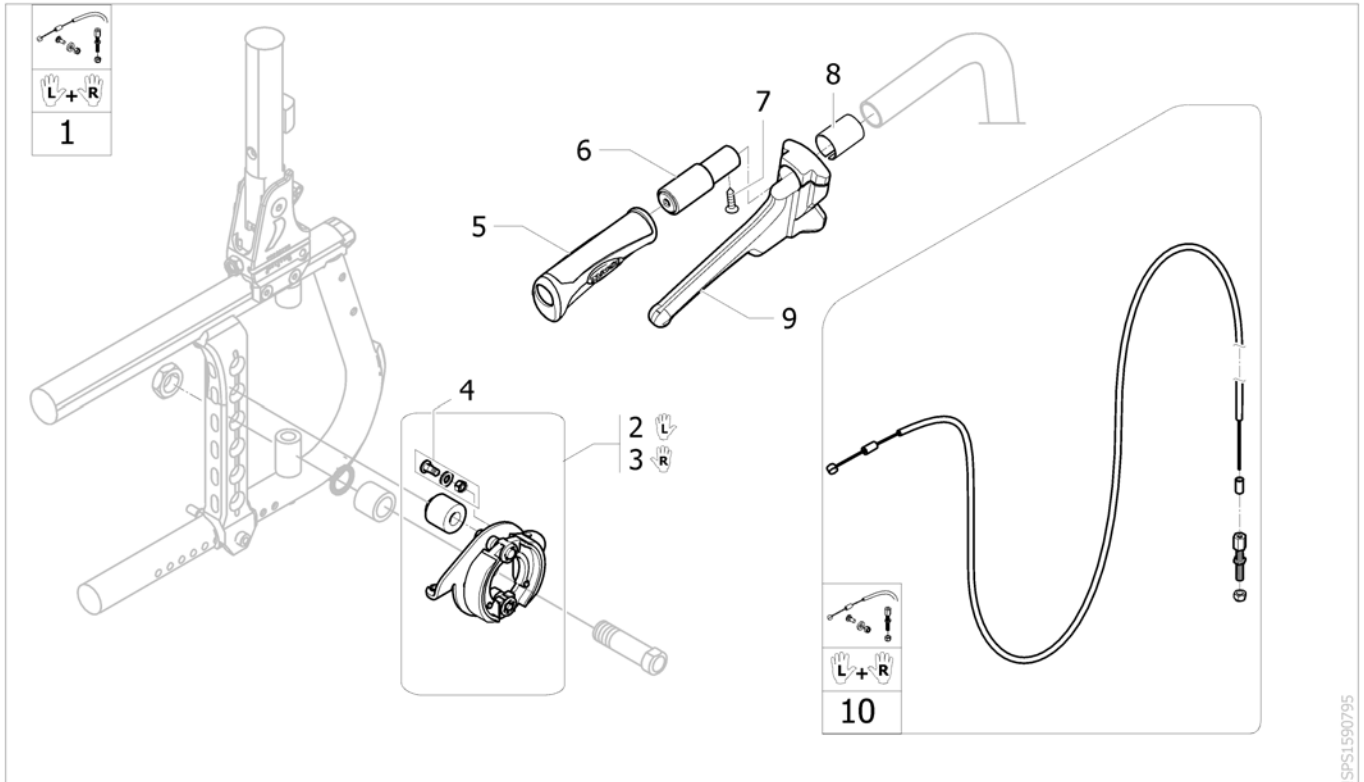

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1601543 | Kit attendant hub brake complete (without wheels) <i>[SJ_8160]</i> | | 1 |
| 2 | SP1601544 | Kit hub brake plate Lh with spacer | | 1 |
| 3 | SP1601545 | Kit hub brake plate Rh with spacer | | 1 |
| 4 | M30071 | Cable Clamp Bolt c/wNut& Wash | | 2 |
| 5 | 1425473 | Handle Invacare for backrest Ø22mm | | 2 |
| 6 | 1425530 | Insert push handle | | 2 |
| 7 | 1428603 | Screw Tfb 4,8x19 A2r Tx25 DIN7983 | | 2 |
| 8 | 5221363 | Fixed Backrest Hub Brake Adaptation | | 2 |
| 9 | 1420527 | Lockable handle for drum brake | | 2 |
| 10 | 1564821 | Complete cable for attendant hub brake (pair) | | 1 |


Safety Options

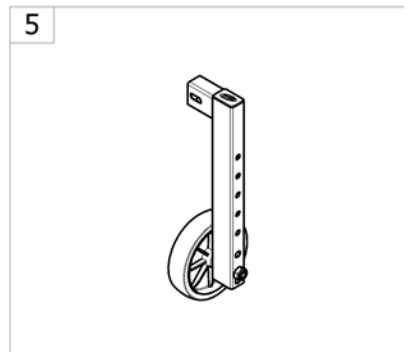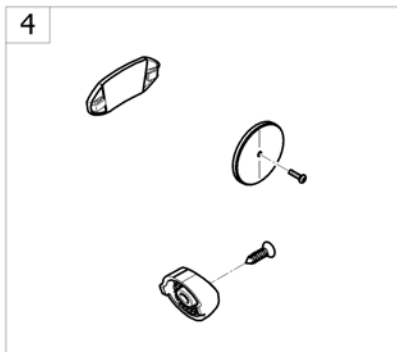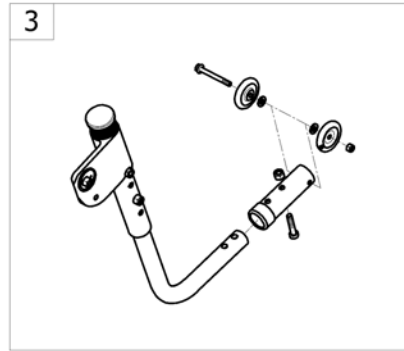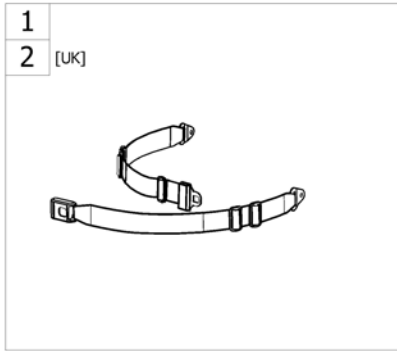

->SPCACTIONS90


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | - | Buckled belt <i>[SJ_8300]</i> | | 1 |
| 2 | - | Buckled belt [UK] <i>[SJ_8300] ONLY FOR UK</i> | | 1 |
| 3 | - | Anti-tipper, Angle adjustable <i>[SJ_8500/SJ_8550]</i> | | 1 |
| 4 | - | Reflectors <i>[SJ_8600]</i> | | 1 |
| 5 | - | Kit anti barrier <i>[SJ_8900] ONLY FOR ITALY</i> | | 1 |

Buckled belt

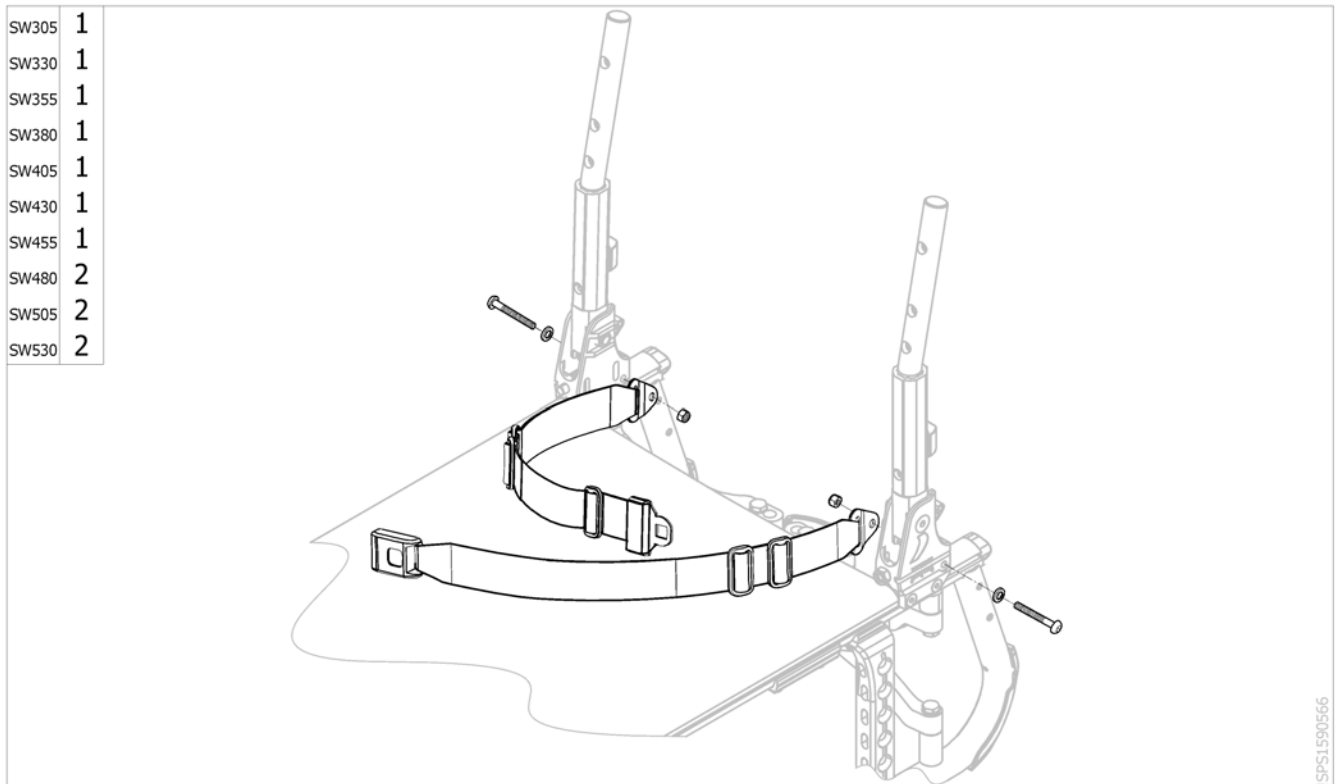


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1590426 | Kit buckle belt metal SW305-455mm <i>[SJ_8300]</i> | | 1 |
| 2 | SP1590427 | Kit buckle belt metal SW480-530mm <i>[SJ_8300]</i> | | 1 |

Buckled belt [UK]

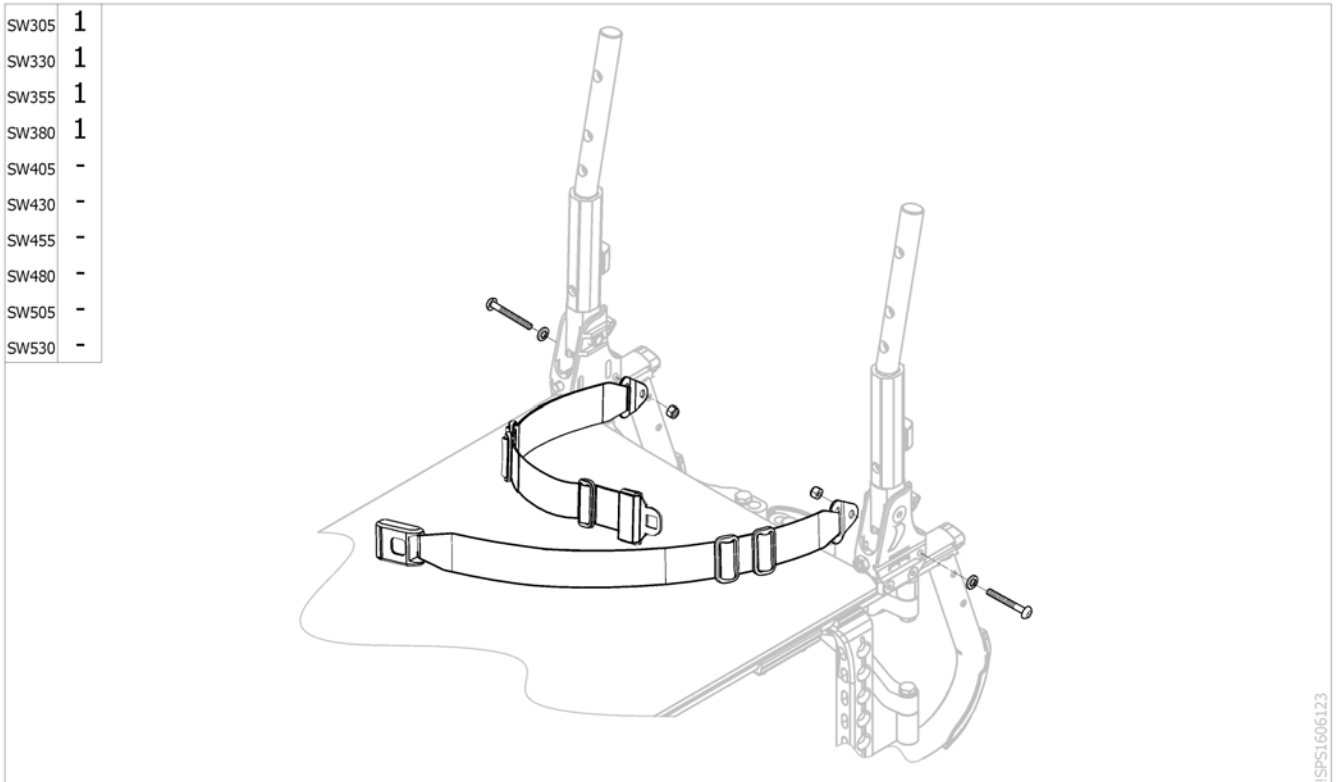

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1606088 | Kit buckled belt SW305-380mm UK <i>[SJ_8300] for UK</i> | | 1 |




Anti-tipper, Angle adjustable

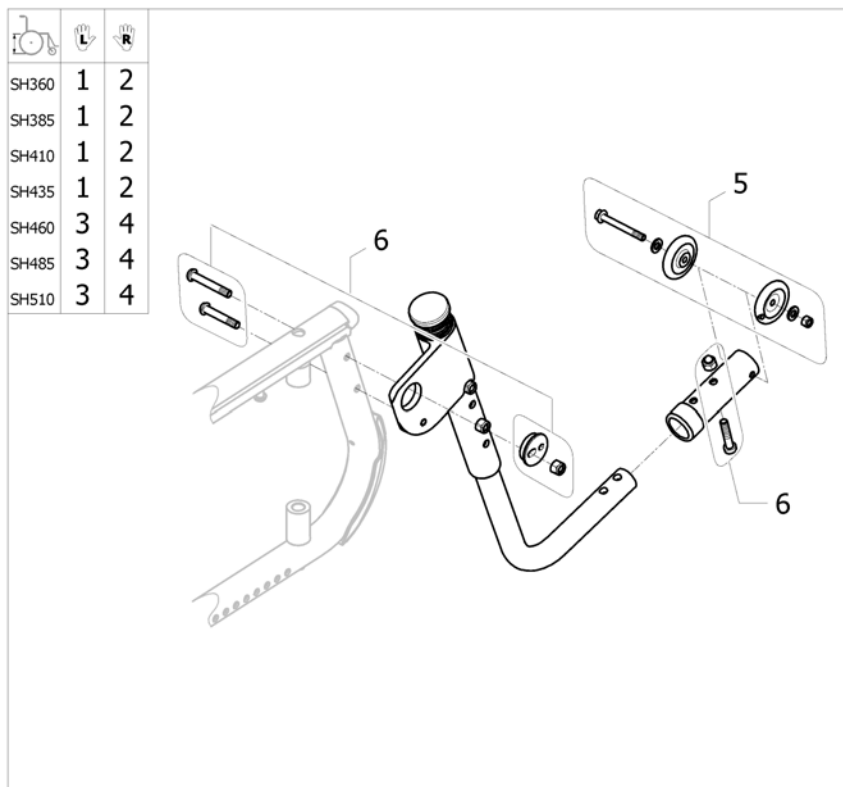

ISPS1607191 ~ 1607180_35m

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1607127 | Anti-tipper active angle adjustable, Lh, short tube, rear seat height 360 to 435 mm <i>Interchangeable - Replace SP1586894-0004</i> | | 1 |
| 2 | SP1607128 | Anti-tipper active angle adjustable, Rh, short tube, rear seat height 360 to 435 mm <i>Interchangeable - Replace SP1584735-0004</i> | | 1 |
| 3 | SP1607180 | Anti-tipper active angle adjustable, Lh, long tube, rear seat height 460 to 510 mm <i>Interchangeable - Replace SP1586896-0004</i> | | 1 |
| 4 | SP1607181 | Anti-tipper active angle adjustable, Rh, long tube, rear seat height 460 to 510 mm <i>Interchangeable - Replace SP1584737-0004</i> | | 1 |
| 5 | 1548802 | Kit wheels + attachment anti-tippers | | 1 |
| 6 | SP1607192 | Fixing kit and setting anti-tipper | | 1 |


Reflectors

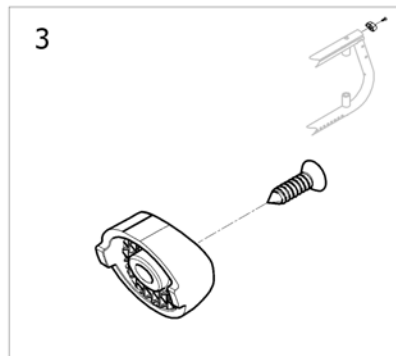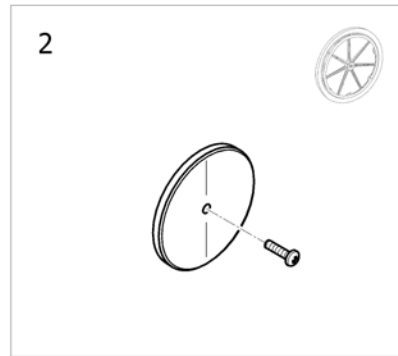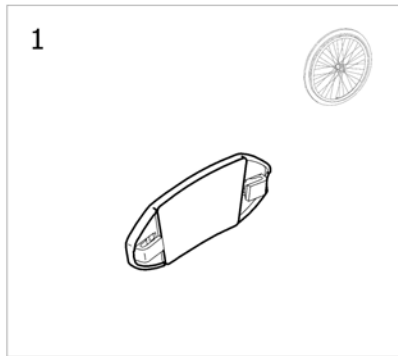

ISPS1582181


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | N90003 | Kit reflector for spoked wheel [[SJ_7200/SJ_7210/SJ_7230/SJ_7240/SJ_7244/SJ_7250/SJ_7260)+SJ_8600] | | 2 |
| 2 | 9037-30001 | Kit reflector for composite wheel [SJ_7242+SJ_8600] | | 2 |
| 3 | SP1589683 | Kit rear cap reflector for frame | | 2 |

Kit anti barrier

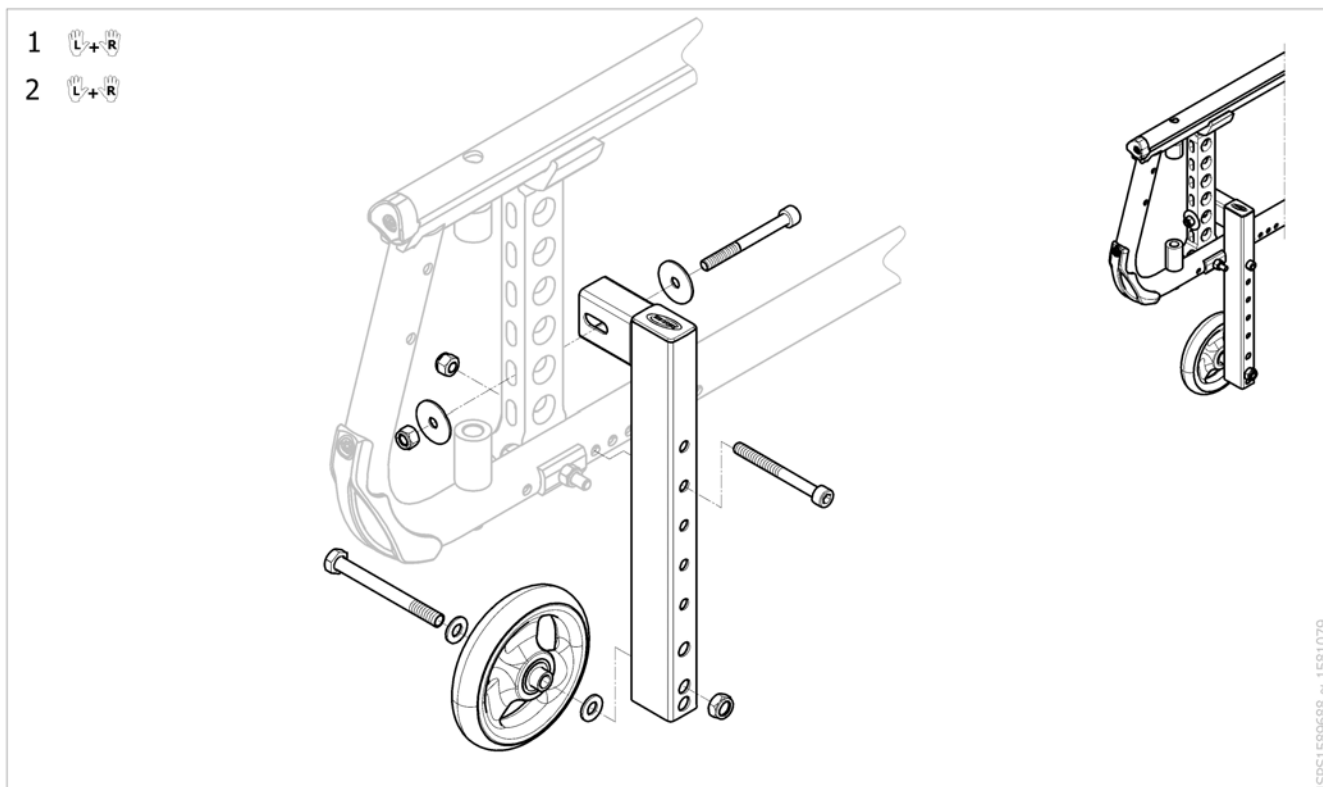

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-----------------------|--------------------|-----|
| 1 | SP1590532 | Kit anti barrier low | | 1 |
| 2 | SP1590533 | Kit anti barrier high | | 1 |

Seating Options

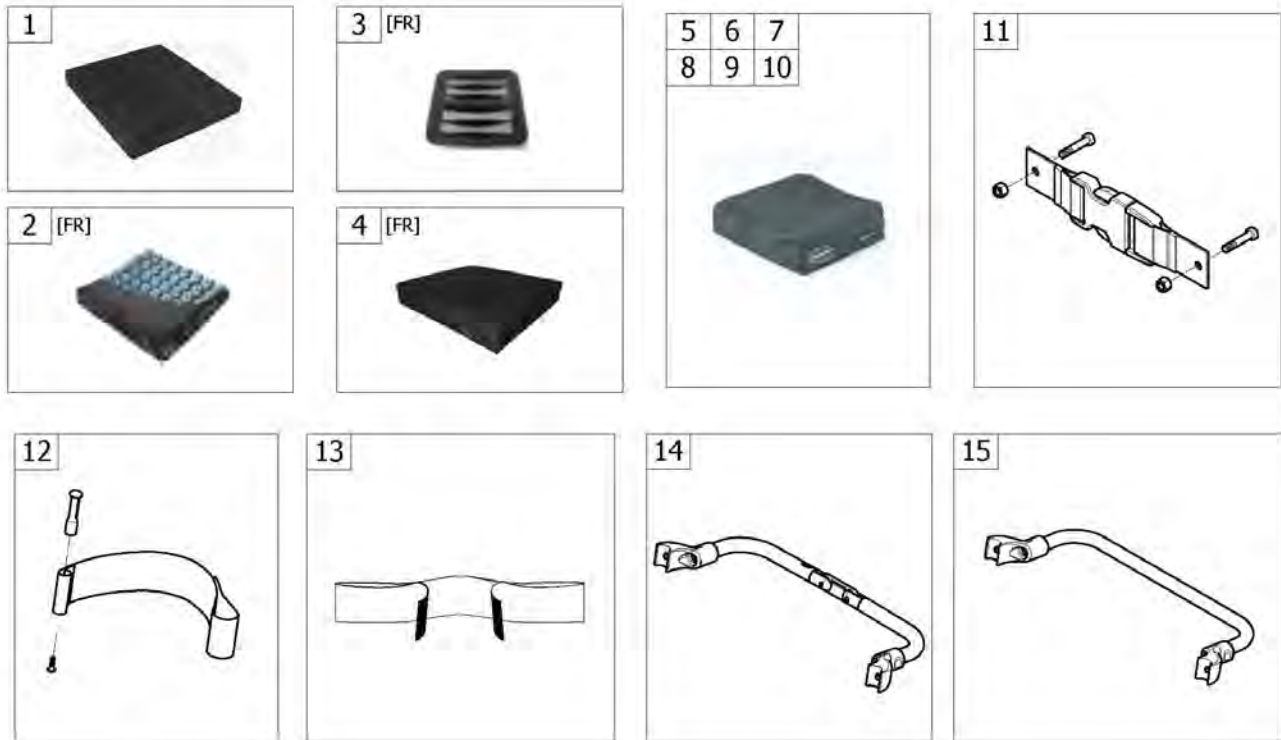

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | - | Cushion foam Nylon & anti-slip cover <i>[SJ_5000]</i> | | 1 |
| 2 | - | Profil Air Cushions [FR] <i>[SJ_5005] ONLY FOR FRANCE</i> | | 1 |
| 3 | - | Rigidizer [FR] <i>[SJ_5090] ONLY FOR FRANCE / ONLY WITH CUSHION TYPE</i> | | 1 |
| 4 | - | Classic Visco Cushions [FR] <i>[SJ_5006][SJ_5007][SJ_5008][SJ_5009] ONLY FOR FRANCE</i> | | 1 |
| 5 | - | Matrx Flo-Tech CONTOUR VISCO cushions <i>[SJ_5010]</i> | | 1 |
| 6 | - | Matrx Flo-Tech IMAGE cushions <i>[SJ_5020]</i> | | 1 |
| 7 | - | Matrx Flo-Tech LITE cushions <i>[SJ_5030]</i> | | 1 |
| 8 | - | Matrx Flo-Tech XTRA cushions <i>[SJ_5040]</i> | | 1 |
| 9 | - | Matrx VI Cushions <i>[SJ_5042]</i> | | 1 |
| 10 | - | Matrx LIBRA Cushions <i>[SJ_5044]</i> | | 1 |
| 11 | - | Clip to keep the chair folded <i>[SJ_5100]</i> | | 1 |
| 12 | - | Heels strap <i>[SJ_5300]</i> | | 1 |
| 13 | - | Calf strap <i>[SJ_5400]</i> | | 1 |
| 14 | - | Folding backbrace <i>[SJ_5500]</i> | | 1 |
| 15 | - | Backbrace for rigid chassis <i>[SJ_5600]</i> | | 1 |

Cushion foam Nylon & anti-slip cover

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | 1 | 5 | 5 | 10 | 10 | - | - |
| SW330 | 2 | 5 | 5 | 10 | 10 | - | - |
| SW355 | 3 | 6 | 6 | 7 | 7 | 8 | 8 |
| SW380 | 4 | 6 | 6 | 7 | 7 | 8 | 8 |
| SW405 | - | 7 | 7 | 11 | 11 | 12 | 12 |
| SW430 | - | 7 | 7 | 11 | 11 | 12 | 12 |
| SW455 | - | 8 | 8 | 12 | 12 | 14 | 14 |
| SW480 | - | 8 | 8 | 12 | 12 | 14 | 14 |
| SW505 | - | - | 9 | 13 | 13 | 15 | 15 |
| SW530 | - | - | 9 | 13 | 13 | 15 | 15 |



ISPS:1569738

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1500971 | Essential Junior cushion (I), SW305 x SD275/325 mm <i>[SJ_5000]</i> | | 1 |
| 2 | 1500973 | Essential Junior cushion (K), SW330 x SD275/325 mm <i>[SJ_5000]</i> | | 1 |
| 3 | 1500975 | Essential Junior cushion (M), SW355 x SD275/325 mm <i>[SJ_5000]</i> | | 1 |
| 4 | 1500977 | Essential Junior cushion (O), SW380 x SD275/325 mm <i>[SJ_5000]</i> | | 1 |
| 5 | 1564260 | Foam Cushion 312x362 mm <i>[SJ_5000]</i> | | 1 |
| 6 | 1564263 | Foam Cushion 362x362 mm <i>[SJ_5000]</i> | | 1 |
| 7 | 1564264 | Foam Cushion 362x412 mm <i>[SJ_5000]</i> | | 1 |
| 8 | 1564265 | Foam Cushion 362x462 mm <i>[SJ_5000]</i> | | 1 |
| 9 | 1564266 | Foam Cushion 362x500 mm <i>[SJ_5000]</i> | | 1 |
| 10 | 1564261 | Foam Cushion 312x412 mm <i>[SJ_5000]</i> | | 1 |
| 11 | 1564267 | Foam Cushion 412x412 mm <i>[SJ_5000]</i> | | 1 |
| 12 | 1564268 | Foam Cushion 412x462 mm <i>[SJ_5000]</i> | | 1 |
| 13 | 1564269 | Foam Cushion 412x500 mm <i>[SJ_5000]</i> | | 1 |
| 14 | 1564270 | Foam Cushion 462x462 mm <i>[SJ_5000]</i> | | 1 |
| 15 | 1564271 | Foam Cushion 462x500 mm <i>[SJ_5000]</i> | | 1 |

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | - | - | - | - | - |
| SW405 | - | - | 1 | 1 | 2 | 2 | - |
| SW430 | - | - | 2 | 2 | 3 | 3 | - |
| SW455 | - | - | 2 | 2 | 3 | 3 | - |
| SW480 | - | - | - | - | - | - | - |
| SW505 | - | - | - | - | - | - | - |
| SW530 | - | - | - | - | - | - | - |



KSPS1626702

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-----------------------------------|--------------------|-----|
| 1 | 1627195 | Cushion Profil Air W400 x D400 mm | | 1 |
| 2 | 1627196 | Cushion Profil Air W400 x D450 mm | | 1 |
| 3 | 1627197 | Cushion Profil Air W450 x D450 mm | | 1 |


Rigidizer [FR]

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | - | - | - | - | - |
| SW405 | - | - | 1 | 1 | 3 | 3 | - |
| SW430 | - | - | 1 | 1 | 3 | 3 | - |
| SW455 | - | - | 2 | 2 | 4 | 4 | - |
| SW480 | - | - | - | - | - | - | - |
| SW505 | - | - | - | - | - | - | - |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 1494497 | Rigidizer W395 x D380 - for FloTech cushion | | 1 |
| 2 | 1494500 | Rigidizer W445 x D380 - For FloTech cushion | | 1 |
| 3 | 1494498 | Rigidizer W395 x D430 - For FloTech cushion | | 1 |
| 4 | 1494501 | Rigidizer W445 x D430 - For FloTech cushion | | 1 |

Classic Visco Cushions [FR]

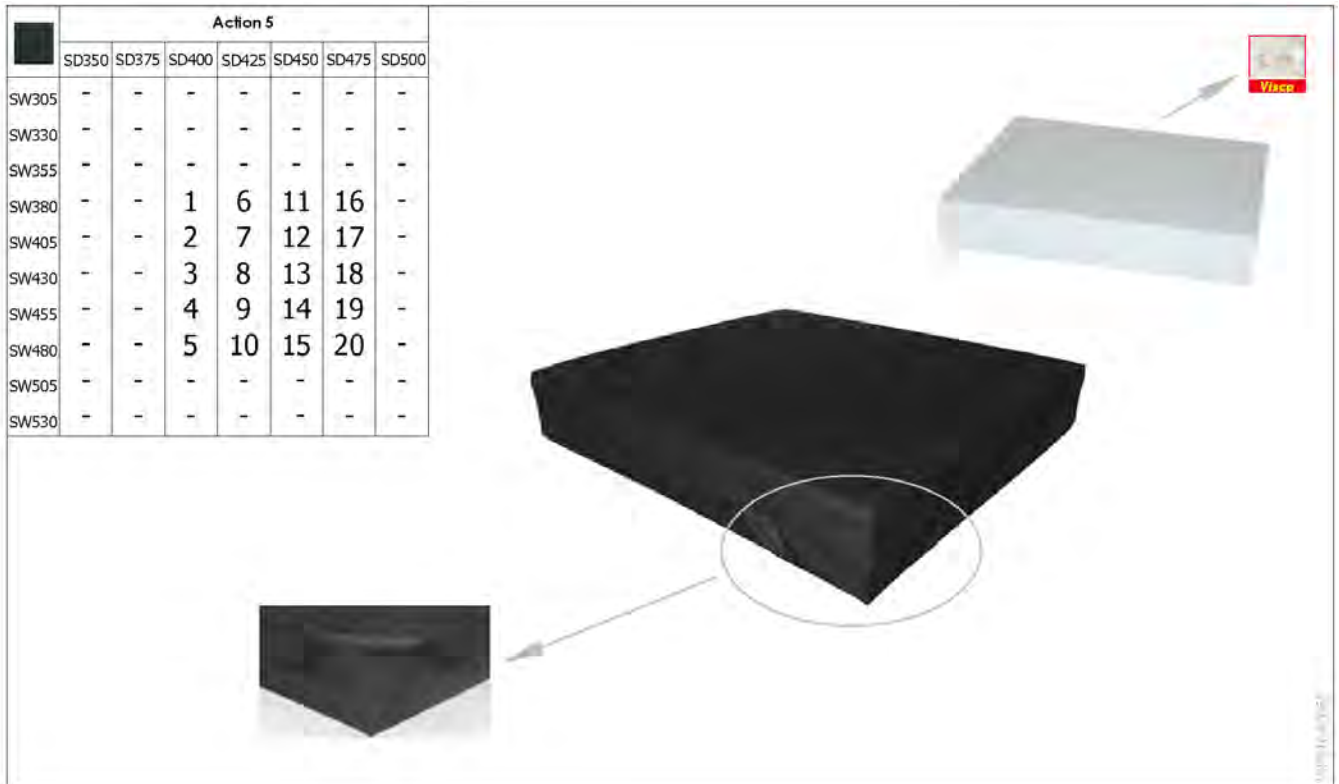

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 2 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 3 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 3 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 3 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 4 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 4 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 4 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 5 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 6 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 6 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 7 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 7 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 8 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 8 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 8 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 9 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 10 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 11 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 11 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 12 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 12 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 12 | 1627253 | Classic Visco Cushion 375x400mm Inc Extra Cover | | 1 |
| 13 | 1627254 | Classic Visco Cushion 405x400mm Inc Extra Cover | | 1 |
| 13 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 14 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 14 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 15 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 16 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 17 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 17 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |

Classic Visco Cushions [FR]

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 18 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 18 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 19 | 1627256 | Classic Visco Cushion 455x400mm Inc Extra Cover | | 1 |
| 19 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |
| 20 | 1627255 | Classic Visco Cushion 430x430mm Inc Extra Cover | | 1 |

Matrix Flo-Tech CONTOUR VISCO cushions

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | 1 | 7 | 7 | - |
| SW405 | - | - | 2 | 2 | 8 | 8 | - |
| SW430 | - | - | 3 | 3 | 9 | 9 | - |
| SW455 | - | - | 4 | 4 | 10 | 10 | - |
| SW480 | - | - | 5 | 5 | 11 | 11 | - |
| SW505 | - | - | 6 | 6 | 12 | 12 | - |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1559590 | Cushion Contour Visco, SW380 x SD405 x H90mm MUW:100Kg (FCSB15/16/LMM/SGL) <i>[SJ_5010]</i> | | 1 |
| 2 | 1559659 | Cushion Contour Visco, SW405 x SD405 x H90mm MUW:100Kg (FCSB16/16/LMM/SGL) <i>[SJ_5010]</i> | | 1 |
| 3 | 1559730 | Cushion Contour Visco, SW430 x SD405 x H90mm MUW:100Kg (FCSB17/16/LMM/SGL) <i>[SJ_5010]</i> | | 1 |
| 4 | 1501114 | Cushion Contour Visco, SW455 x SD405 x H90mm MUW:100Kg (FCSB18/16/LMF/SGL) <i>[SJ_5010]</i> | | 1 |
| 5 | 1501116 | Cushion Contour Visco, SW480 x SD405 x H90mm MUW:100Kg (FCSB19/16/LMF/SGL) <i>[SJ_5010]</i> | | 1 |
| 6 | 1501118 | Cushion Contour Visco, SW505 x SD405 x H90mm MUW:125Kg (FCSB20/16/LMV/SGL) <i>[SJ_5010]</i> | | 1 |
| 7 | 1559657 | Cushion Contour Visco, SW380 x SD455 x H90mm MUW:100Kg (FCSB15/18/LMM/SGL) <i>[SJ_5010]</i> | | 1 |
| 8 | 1526594 | Cushion Contour Visco, SW405 x SD455 x H90mm MUW:100Kg (FCSB16/18/LMF/SGL) <i>[SJ_5010]</i> | | 1 |
| 9 | 1526595 | Cushion Contour Visco, SW430 x SD455 x H90mm MUW:100Kg (FCSB17/18/LMF/SGL) <i>[SJ_5010]</i> | | 1 |
| 10 | 1526608 | Cushion Contour Visco, SW455 x SD455 x H90mm MUW:100Kg (FCSB18/18/LMF/SGL) <i>[SJ_5010]</i> | | 1 |
| 11 | 1525020 | Cushion Contour Visco, SW480 x SD455 x H90mm MUW:125Kg (FCSB19/18/LMV/SGL) <i>[SJ_5010]</i> | | 1 |
| 12 | 1525021 | Cushion Contour Visco, SW505 x SD455 x H90mm MUW:125Kg (FCSB20/18/LMV/SGL) <i>[SJ_5010]</i> | | 1 |

Matrix Flo-Tech IMAGE cushions

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | 1 | 7 | 7 | - |
| SW405 | - | - | 2 | 2 | 8 | 8 | - |
| SW430 | - | - | 3 | 3 | 9 | 9 | - |
| SW455 | - | - | 4 | 4 | 10 | 10 | - |
| SW480 | - | - | 5 | 5 | 11 | 11 | - |
| SW505 | - | - | 6 | 6 | 12 | 12 | - |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1575728 | Cushion Image, SW380 x SD405 x H78mm, MUW-140kg (FISB15/16) [SJ_5020] | | 1 |
| 2 | 1511196 | Cushion Image, SW405 x SD405 x H78mm, MUW-140kg (FISB16/16) [SJ_5020] | | 1 |
| 3 | 1511197 | Cushion Image, SW430 x SD405 x H78mm, MUW-140kg (FISB17/16) [SJ_5020] | | 1 |
| 4 | 1511198 | Cushion Image, SW455 x SD405 x H78mm, MUW-140kg (FISB18/16) [SJ_5020] | | 1 |
| 5 | 1511199 | Cushion Image, SW480 x SD405 x H78mm, MUW-140kg (FISB19/16) [SJ_5020] | | 1 |
| 6 | 1526417 | Cushion Image, SW505 x SD405 x H78mm, MUW-140kg (FISB20/16) [SJ_5020] | | 1 |
| 7 | 1575729 | Cushion Image, SW380 x SD455 x H78mm, MUW-140kg (FISB15/18) [SJ_5020] | | 1 |
| 8 | 1526412 | Cushion Image, SW405 x SD455 x H78mm, MUW-140kg (FISB16/18) [SJ_5020] | | 1 |
| 9 | 1526413 | Cushion Image, SW430 x SD455 x H78mm, MUW-140kg (FISB17/18) [SJ_5020] | | 1 |
| 11 | 1526415 | Cushion Image, SW480 x SD455 x H78mm, MUW-140kg (FISB19/18) [SJ_5020] | | 1 |
| 10 | 1526414 | Cushion Image, SW455 x SD455 x H78mm, MUW-140kg (FISB18/18) [SJ_5020] | | 1 |
| 12 | 1526416 | Cushion Image, SW505 x SD455 x H78mm, MUW-140kg (FISB20/18) [SJ_5020] | | 1 |

Matrx Flo-Tech LITE cushions

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | 1 | 7 | 7 | - |
| SW405 | - | - | 2 | 2 | 8 | 8 | - |
| SW430 | - | - | 3 | 3 | 9 | 9 | - |
| SW455 | - | - | 4 | 4 | 10 | 10 | - |
| SW480 | - | - | 5 | 5 | 11 | 11 | - |
| SW505 | - | - | 6 | 6 | 12 | 12 | - |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 8 | 1575610 | Cushion Lite, SW405 x SD450mm <i>[SJ_5030]</i> | | 1 |
| 1 | 1575607 | Cushion Lite, SW380 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 2 | 1575609 | Cushion Lite, SW405 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 3 | 1575611 | Cushion Lite, SW430 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 4 | 1575613 | Cushion Lite, SW455 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 5 | 1575615 | Cushion Lite, SW480 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 6 | 1575617 | Cushion Lite, SW505 x SD400mm <i>[SJ_5030]</i> | | 1 |
| 7 | 1575608 | Cushion Lite, SW380 x SD450mm <i>[SJ_5030]</i> | | 1 |
| 9 | 1575612 | Cushion Lite, SW430 x SD450mm <i>[SJ_5030]</i> | | 1 |
| 10 | 1575614 | Cushion Lite, SW455 x SD450mm <i>[SJ_5030]</i> | | 1 |
| 11 | 1575616 | Cushion Lite, SW480 x SD450mm <i>[SJ_5030]</i> | | 1 |
| 12 | 1575618 | Cushion Lite, SW505 x SD450mm <i>[SJ_5030]</i> | | 1 |

Matrix Flo-Tech XTRA cushions

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | 1 | 7 | 7 | - |
| SW405 | - | - | 2 | 2 | 8 | 8 | - |
| SW430 | - | - | 3 | 3 | 9 | 9 | - |
| SW455 | - | - | 4 | 4 | 10 | 10 | - |
| SW480 | - | - | 5 | 5 | 11 | 11 | - |
| SW505 | - | - | 6 | 6 | 12 | 12 | - |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | 1494430 | Cushion Solution Xtra, SW380 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 2 | 1487145 | Cushion Solution Xtra, SW405 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 3 | 1494441 | Cushion Solution Xtra, SW430 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 4 | 1487147 | Cushion Solution Xtra, SW455 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 5 | 1494451 | Cushion Solution Xtra, SW480 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 6 | 1552058 | Cushion Soutlion Xtra, SW505 x SD405mm <i>[SJ_5040]</i> | | 1 |
| 7 | 1494432 | Cushion Solution Xtra, SW380 x SD455mm <i>[SJ_5040]</i> | | 1 |
| 8 | 1551914 | Cushion Solution Xtra, SW405 x SD455mm <i>[SJ_5040]</i> | | 1 |
| 9 | 1551922 | Cushion Solution Xtra, SW430 x SD455mm <i>[SJ_5040]</i> | | 1 |
| 10 | 1551925 | Cushion Solution Xtra, SW455 x SD455mm <i>[SJ_5040]</i> | | 1 |
| 11 | 1552055 | Cushion Solution Xtra, SW480 x SD455mm <i>[SJ_5040]</i> | | 1 |
| 12 | 1552060 | Cushion Solution Xtra, SW505 x SD455mm <i>[SJ_5040]</i> | | 1 |

Matrx VI Cushions

| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | - | 5 | - | 9 |
| SW405 | - | - | 1 | - | 5 | - | 9 |
| SW430 | - | - | 2 | - | 6 | - | 10 |
| SW455 | - | - | 3 | - | 7 | - | 11 |
| SW480 | - | - | 4 | - | 8 | - | 12 |
| SW505 | - | - | 4 | - | 8 | - | 12 |
| SW530 | - | - | - | - | - | - | - |



| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---------------------------------------|--------------------|-----|
| 1 | 1579786 | Matrx Flo-Tech VI cushions 410x410 mm | | 1 |
| 2 | 1579829 | Matrx Flo-Tech VI cushions 430x410 mm | | 1 |
| 3 | 1579920 | Matrx Flo-Tech VI cushions 460x410 mm | | 1 |
| 4 | 1579923 | Matrx Flo-Tech VI cushions 510x410 mm | | 1 |
| 5 | 1579787 | Matrx Flo-Tech VI cushions 410x460 mm | | 1 |
| 6 | 1579918 | Matrx Flo-Tech VI cushions 430x460 mm | | 1 |
| 7 | 1579921 | Matrx Flo-Tech VI cushions 460x460 mm | | 1 |
| 8 | 1579924 | Matrx Flo-Tech VI cushions 510x460 mm | | 1 |
| 9 | 1579788 | Matrx Flo-Tech VI cushions 410x510 mm | | 1 |
| 10 | 1579919 | Matrx Flo-Tech VI cushions 430x510 mm | | 1 |
| 11 | 1579922 | Matrx Flo-Tech VI cushions 460x510 mm | | 1 |
| 12 | 1579925 | Matrx Flo-Tech VI cushions 510x510 mm | | 1 |

Matrx LIBRA Cushions


| | SD350 | SD375 | SD400 | SD425 | SD450 | SD475 | SD500 |
|-------|-------|-------|-------|-------|-------|-------|-------|
| SW305 | - | - | - | - | - | - | - |
| SW330 | - | - | - | - | - | - | - |
| SW355 | - | - | - | - | - | - | - |
| SW380 | - | - | 1 | - | 7 | - | 13 |
| SW405 | - | - | 2 | - | 8 | - | 14 |
| SW430 | - | - | 3 | - | 9 | - | 15 |
| SW455 | - | - | 4 | - | 10 | - | 16 |
| SW480 | - | - | 5 | - | 11 | - | 17 |
| SW505 | - | - | 6 | - | 12 | - | 17 |
| SW530 | - | - | - | - | - | - | - |



0101100304


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---------------------------------|--------------------|-----|
| 1 | LC1516 | Matrx Libra cushions 380x410 mm | | 1 |
| 2 | LC1616 | Matrx Libra cushions 410x410 mm | | 1 |
| 3 | LC1716 | Matrx Libra cushions 430x410 mm | | 1 |
| 4 | LC1816 | Matrx Libra cushions 460x410 mm | | 1 |
| 5 | LC1916 | Matrx Libra cushions 480x410 mm | | 1 |
| 6 | LC2016 | Matrx Libra cushions 510x410 mm | | 1 |
| 7 | LC1518 | Matrx Libra cushions 380x460 mm | | 1 |
| 8 | LC1618 | Matrx Libra cushions 410x460 mm | | 1 |
| 9 | LC1718 | Matrx Libra cushions 430x460 mm | | 1 |
| 10 | LC1818 | Matrx Libra cushions 460x460 mm | | 1 |
| 11 | LC1918 | Matrx Libra cushions 480x460 mm | | 1 |
| 12 | LC2018 | Matrx Libra cushions 510x460 mm | | 1 |
| 13 | LC1520 | Matrx Libra cushions 380x510 mm | | 1 |
| 14 | LC1620 | Matrx Libra cushions 410x510 mm | | 1 |
| 15 | LC1720 | Matrx Libra cushions 430x510 mm | | 1 |
| 16 | LC1820 | Matrx Libra cushions 380x510 mm | | 1 |
| 17 | LC1920 | Matrx Libra cushions 480x510 mm | | 1 |
| 18 | LC2020 | Matrx Libra cushions 510x510 mm | | 1 |

Clip to keep the chair folded

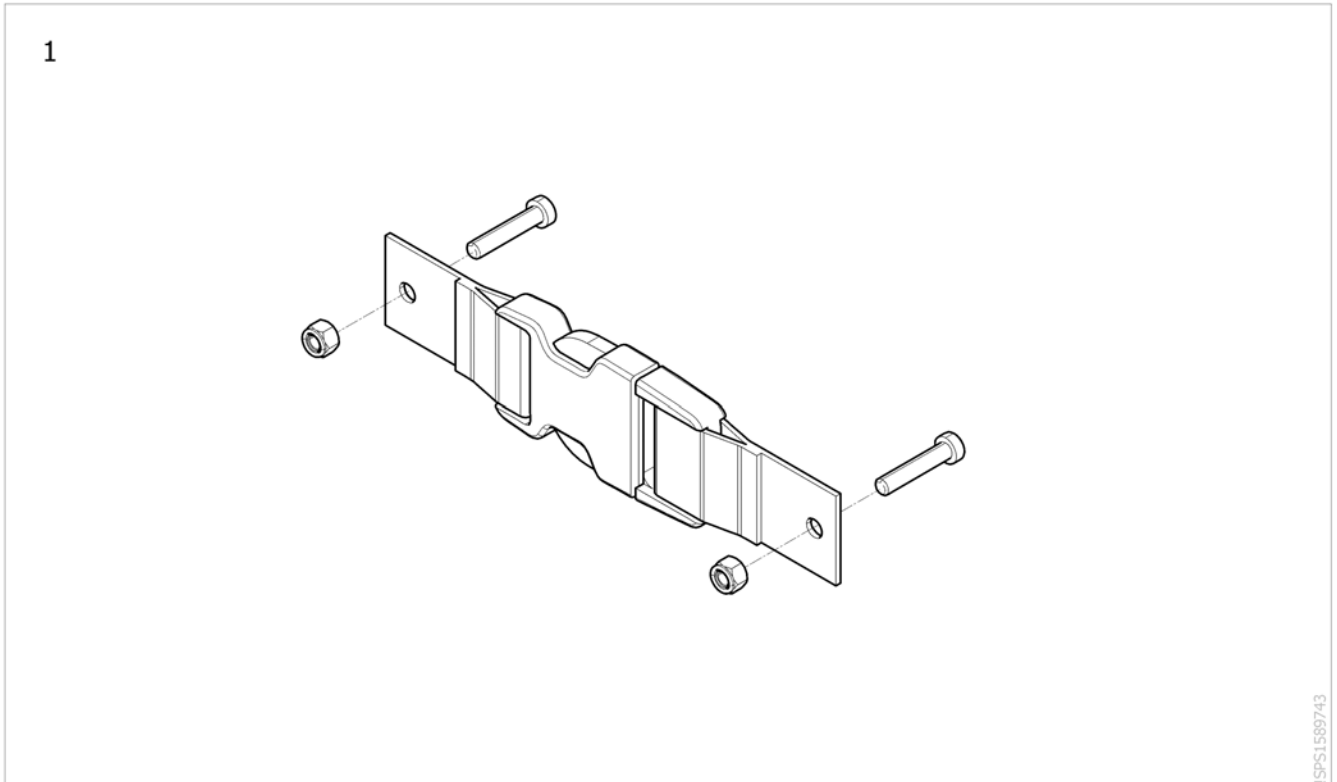


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | SP1590820 | Kit clip to keep the chair folded <i>[SJ_5100]</i> | | 1 |

Heels strap

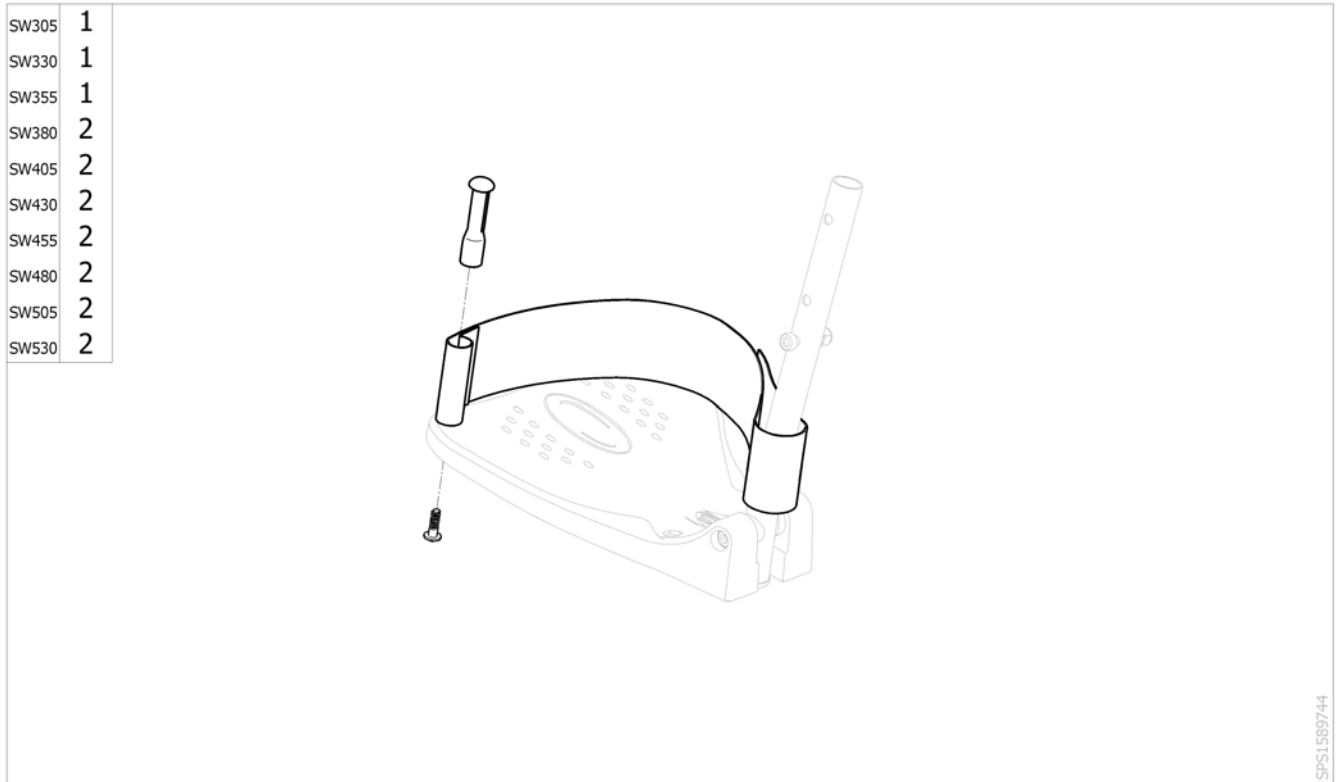


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 0587-05201 | Kit heelstrap, adjustable, small <i>[SJ_5300]</i> | | 2 |
| 2 | 1492315 | Kit heelstrap, adjustable, medium <i>[SJ_5300]</i> | | 2 |

Calf strap

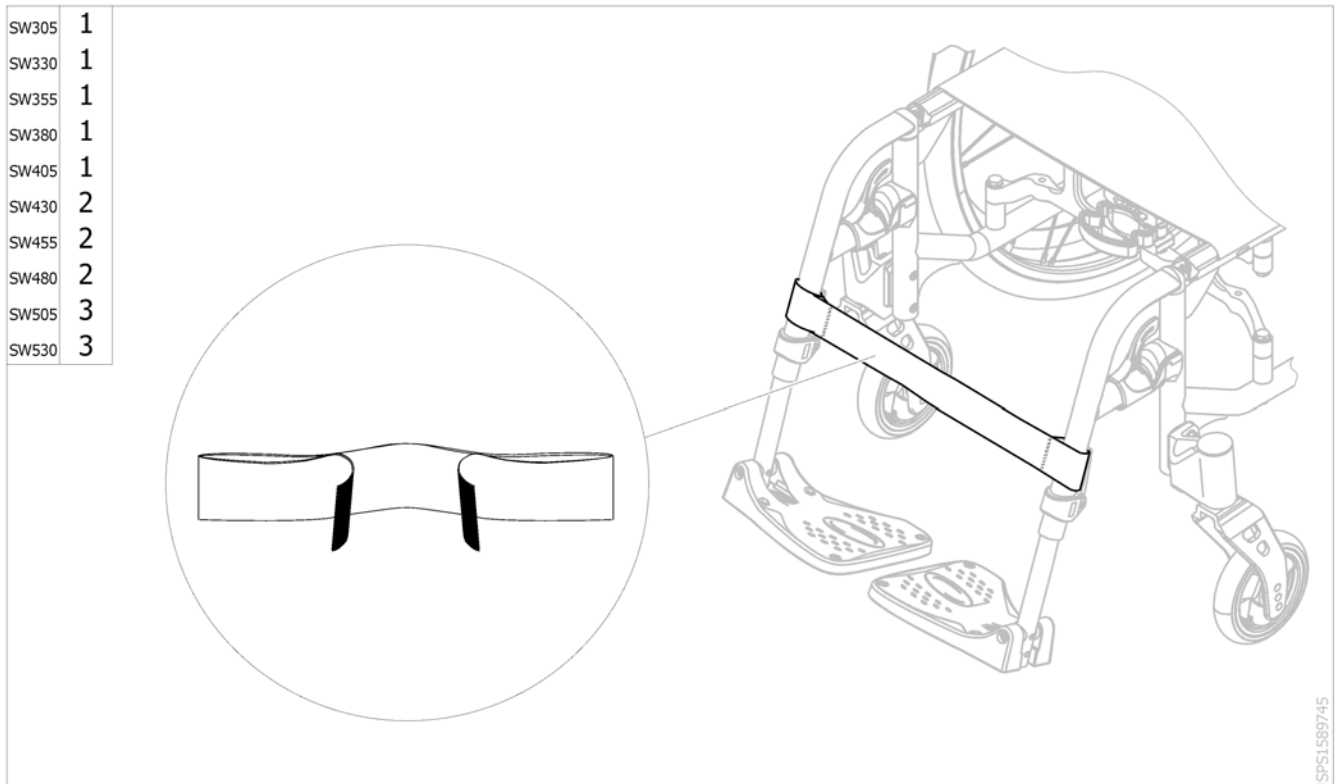


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---|--------------------|-----|
| 1 | 0572-05794 | Calf strap adjustable, small, SW305-405mm <i>[SJ_5400]</i> | | 1 |
| 2 | 1433580 | Calf strap adjustable, Medium <i>[SJ_5400]</i> | | 1 |
| 3 | 1497574 | Calf strap adjustable, Large <i>[SJ_5400]</i> | | 1 |

Folding backbrace

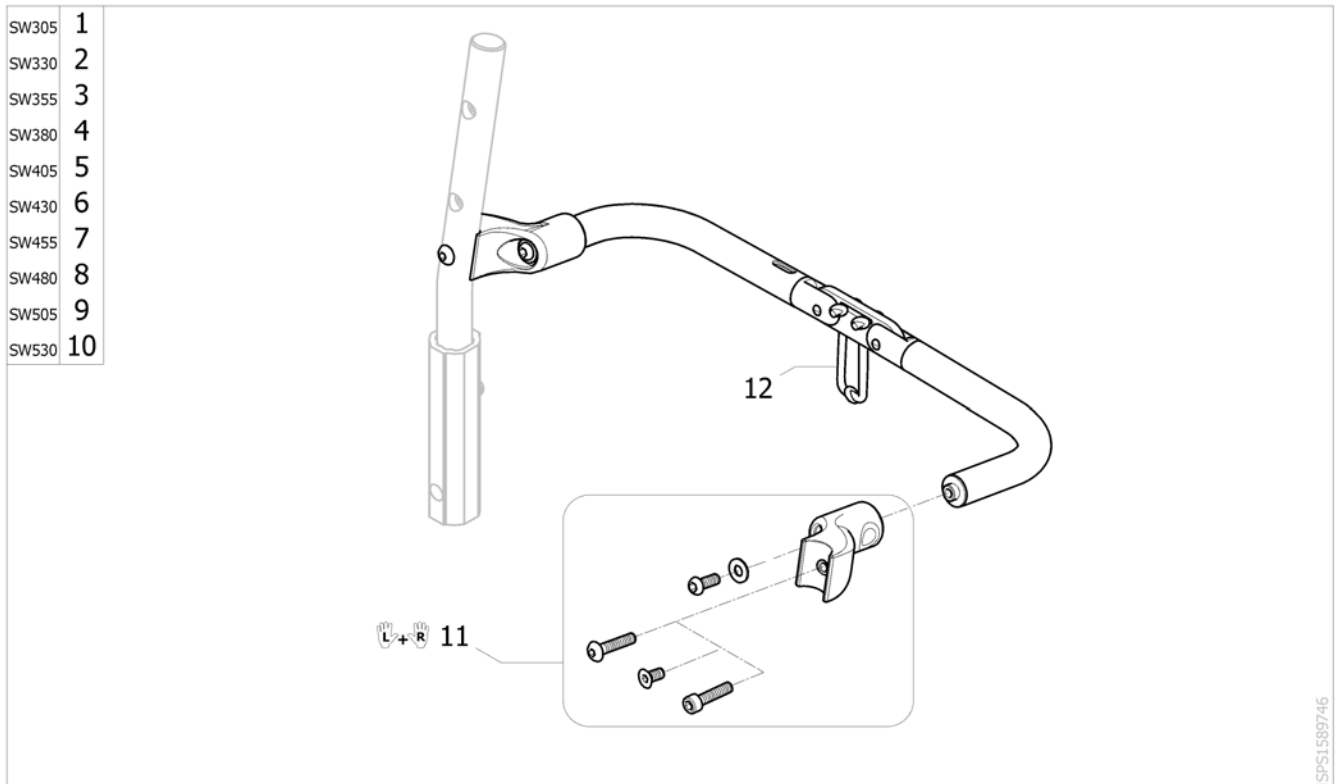

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1583662 | Kit folding backbrace, SW305mm [SJ_5500] | | 1 |
| 2 | SP1583663 | Kit folding backbrace, SW330mm [SJ_5500] | | 1 |
| 3 | SP1583664 | Kit folding backbrace, SW355mm [SJ_5500] | | 1 |
| 4 | SP1583665 | Kit folding backbrace, SW380mm [SJ_5500] | | 1 |
| 5 | SP1583666 | Kit folding backbrace, SW405mm [SJ_5500] | | 1 |
| 6 | SP1583667 | Kit folding backbrace, SW430mm [SJ_5500] | | 1 |
| 7 | SP1583668 | Kit folding backbrace, SW455mm [SJ_5500] | | 1 |
| 8 | SP1583669 | Kit folding backbrace, SW480mm [SJ_5500] | | 1 |
| 9 | SP1583724 | Kit folding backbrace, SW505mm [SJ_5500] | | 1 |
| 10 | SP1583725 | Kit folding backbrace, SW530mm [SJ_5500] | | 1 |
| 11 | SP1589801 | Kit fixation for backbrace (pair) | | 1 |
| 12 | SP1601900 | Cord for backbrace & angle/fold down backrest attachment | | 1 |





Backbrace for rigid chassis

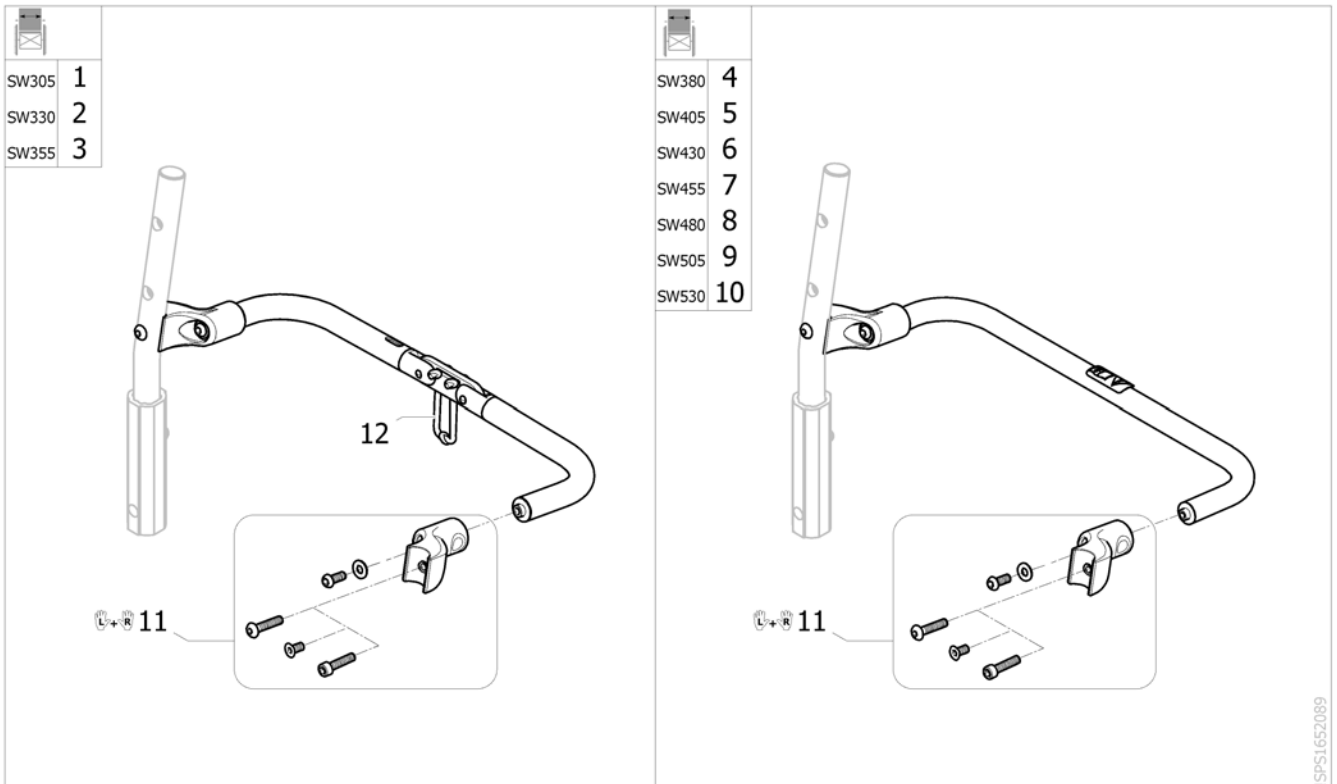

ISPS1652089

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1583662 | Kit folding backbrace, SW305mm [SJ_5600] | | 1 |
| 2 | SP1583663 | Kit folding backbrace, SW330mm [SJ_5600] | | 1 |
| 3 | SP1583664 | Kit folding backbrace, SW355mm [SJ_5600] | | 1 |
| 4 | SP1651848 | Kit fixed backbrace, SW380 mm [SJ_5600] | | 1 |
| 5 | SP1651849 | Kit fixed backbrace, SW405 mm [SJ_5600] | | 1 |
| 6 | SP1651850 | Kit fixed backbrace, SW430 mm [SJ_5600] | | 1 |
| 7 | SP1651851 | Kit fixed backbrace, SW455 mm [SJ_5600] | | 1 |
| 8 | SP1651893 | Kit fixed backbrace, SW480 mm [SJ_5600] | | 1 |
| 9 | SP1651894 | Kit fixed backbrace, SW505 mm [SJ_5600] | | 1 |
| 10 | SP1651895 | Kit fixed backbrace, SW530 mm [SJ_5600] | | 1 |
| 11 | SP1589801 | Kit fixation for backbrace (pair) | | 1 |
| 12 | SP1601900 | Cord for backbrace & angle/fold down backrest attachment | | 1 |


Other Options & Accessories

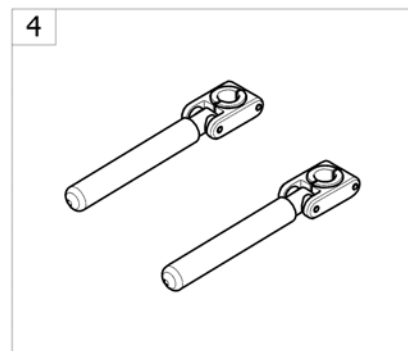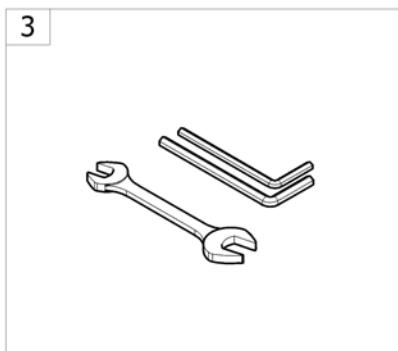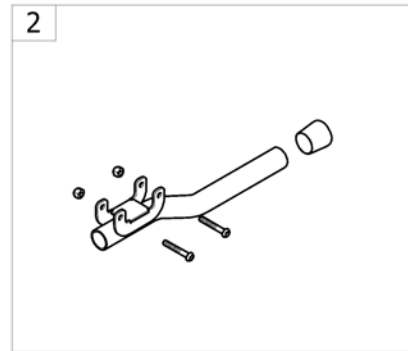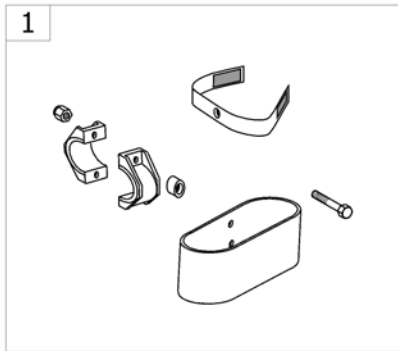

->SPOACTION598


| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|----------------------------------|--------------------|-----|
| 1 | - | Cane holder [SJ_9000] | | 1 |
| 2 | - | Tipping aid [SJ_9100/SJ_9200] | | 1 |
| 3 | - | Tool Kit [SJ_9300] | | 1 |
| 4 | - | Case holder [SJ_9400] | | 1 |

Cane holder

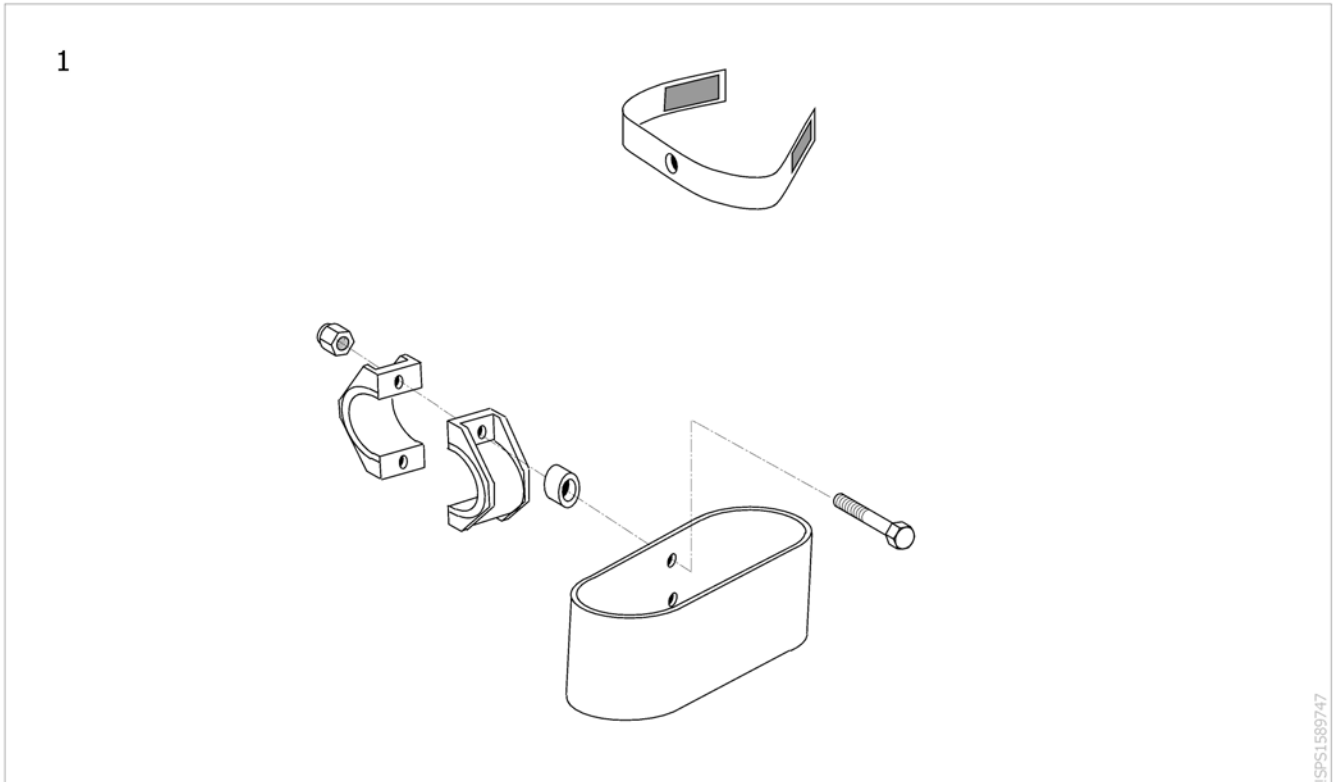

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--------------------------|--------------------|-----|
| 1 | 0572-02056 | Cane holder [SJ_9000] | | 1 |



Tipping aid

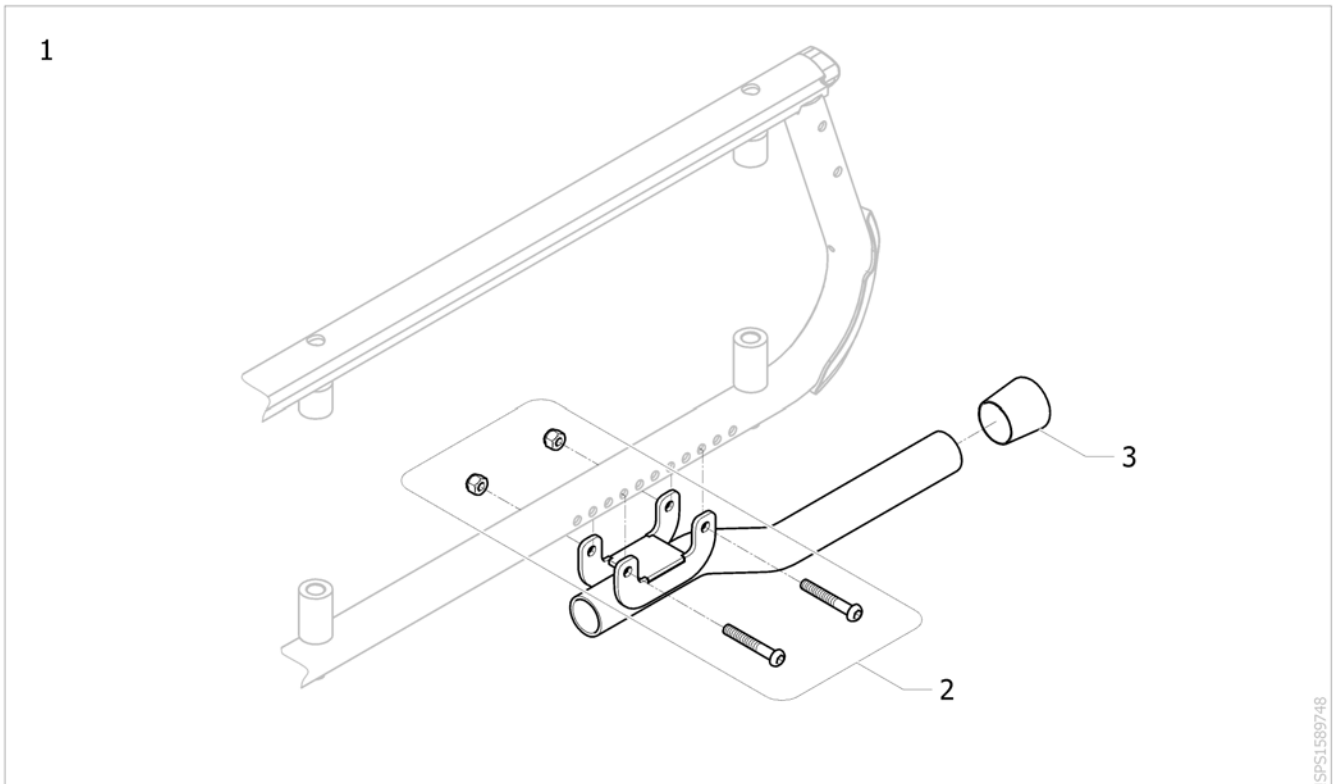

ISPS1589748

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|--|--------------------|-----|
| 1 | SP1581062 | Tipping aid complete <i>[SJ_9100/SJ_9200]</i> | | 1 |
| 2 | SP1589693 | Kit fixation tipping aid | | 1 |
| 3 | SP1589694 | Plastic cap for tipping aid | | 1 |


Tool Kit

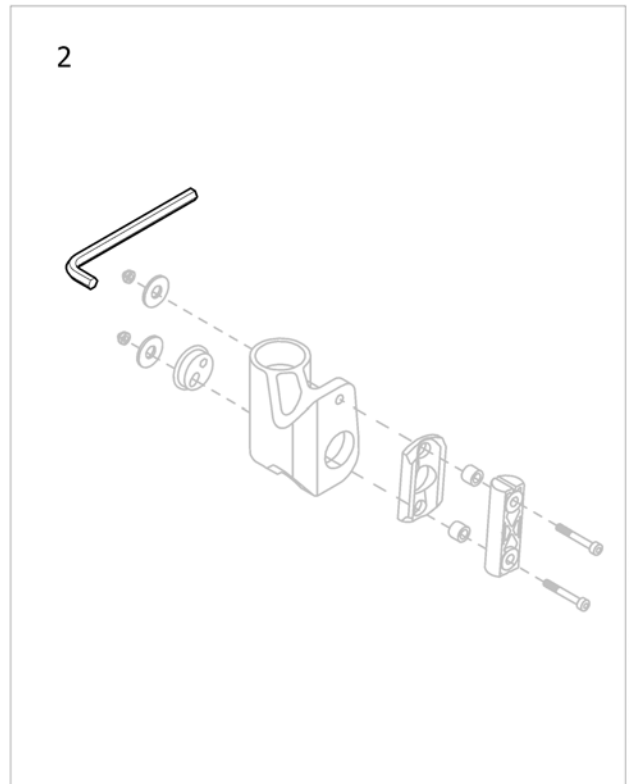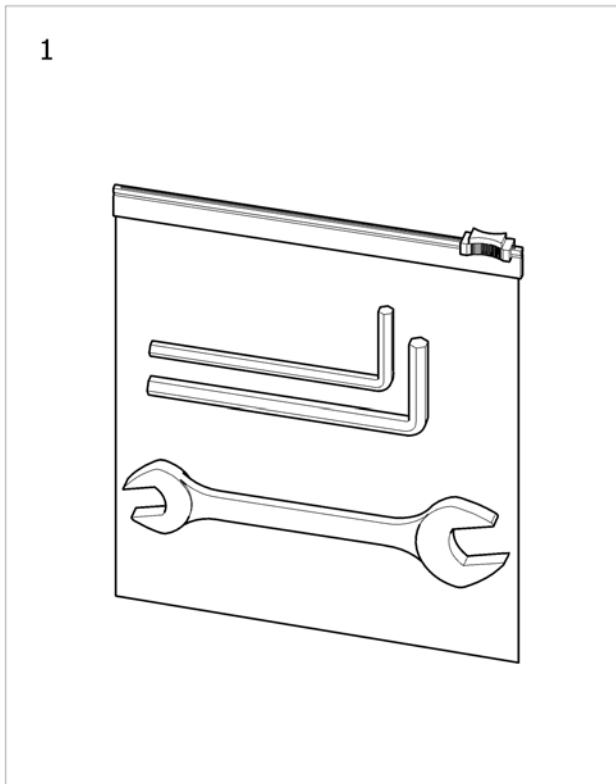

SPS1604323 ~ 1603974 ~ 1638806

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|-----------------------------|--------------------|-----|
| 1 | SP1603974 | Tool Kit | | 1 |
| 2 | SP1586506 | Short Metric Hex Key (5mm) | | 1 |

Case holder

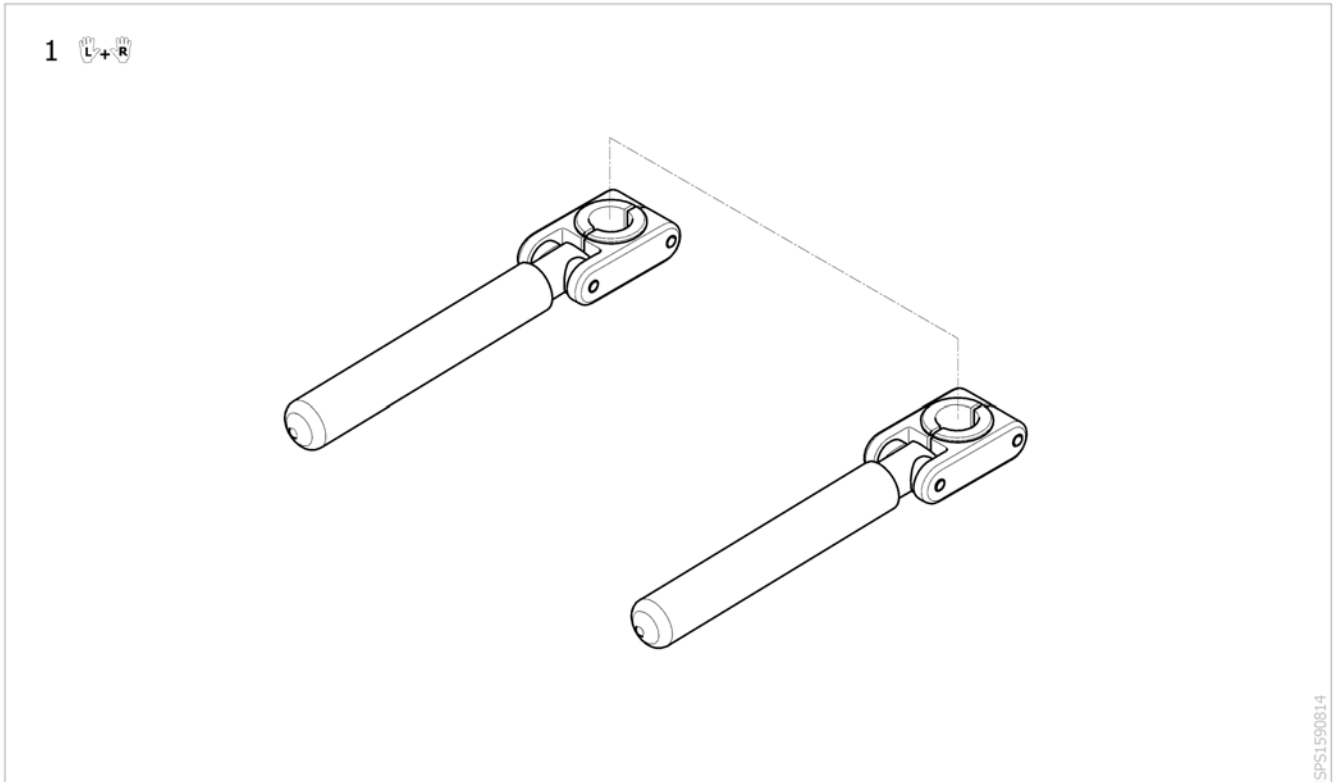

| Pos. | Part number | Description | Valid from → until | Qty |
|------|-------------|---------------------------------|--------------------|-----|
| 1 | SP1587821 | Case holder (pair) [SJ_9400] | | 1 |



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
eda@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46147, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånåsa
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk