

Spare Parts Catalogue

Version: 2018/04/25c EN



Invacare® Colibri™



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

User Manuals

User Manuals Colibri	5
----------------------------	---

Chassis & Shrouds

Front Chassis 3-Wheel	6
Front Chassis & Front Axis 4-Wheel	7
Front Shroud 3-Wheel	8
Colour Shrouds 3-Wheel	9
Front Shroud 4-Wheel	10
Colour Shrouds 4-Wheel	11
Rear Chassis	12
Rear Shroud	13
Batteries	14
Steering & Tiller Shrouds 3-Wheel	15
Colour Shrouds 3-Wheel	9
Electronics & Controller	16
Steering & Tiller Shrouds 4-Wheel	17
Colour Shrouds 4-Wheel	11
Electronics & Controller	16
Colour Shrouds 3-Wheel	9
Colour Shrouds 4-Wheel	11

Seat

Seat & Armpads	18
Options & Accessories	19

Wheels

Front Wheels 3-Wheel	20
Steering & Tiller Shrouds 3-Wheel	15
Colour Shrouds 3-Wheel	9
Electronics & Controller	16
Front Wheels 4-Wheel	21
Rear Wheels	22

Propulsion

Motor & Transaxle	23
-------------------------	----

Electronics & Controller

Electronics & Controller	16
--------------------------------	----

Batteries, Cables & Chargers

Batteries	14
Cables	24
Chargers	25

Option & Accessories

Options & Accessories	19
-----------------------------	----

User Manuals Colibri


languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1576371	User Manual Colibri - English		1
CS	SP1576468	User Manual Colibri - Czech		1
DA	SP1576469	User Manual Colibri - Danish		1
DE	SP1576470	User Manual Colibri - German		1
ES	SP1576471	User Manual Colibri - Spanish		1
ET	SP1576472	User Manual Colibri - Estonian		1
FI	SP1576514	User Manual Colibri - Finnish		1
FR	SP1576515	User Manual Colibri - French		1
HR	SP1576516	User Manual Colibri - Croatian		1
IT	SP1576517	User Manual Colibri - Italian		1
LT	SP1576518	User Manual Colibri - Lithuanian		1
NL	SP1576519	User Manual Colibri - Dutch		1
PL	SP1576520	User Manual Colibri - Polish		1
PT	SP1576521	User Manual Colibri - Portuguese		1
RO	SP1576522	User Manual Colibri - Romanian		1
SL	SP1576523	User Manual Colibri - Slovenian		1
SV	SP1576524	User Manual Colibri - Swedish		1
UK	SP1579821	User Manual Colibri - Ukrainian		1


Front Chassis 3-Wheel

1606742.iss

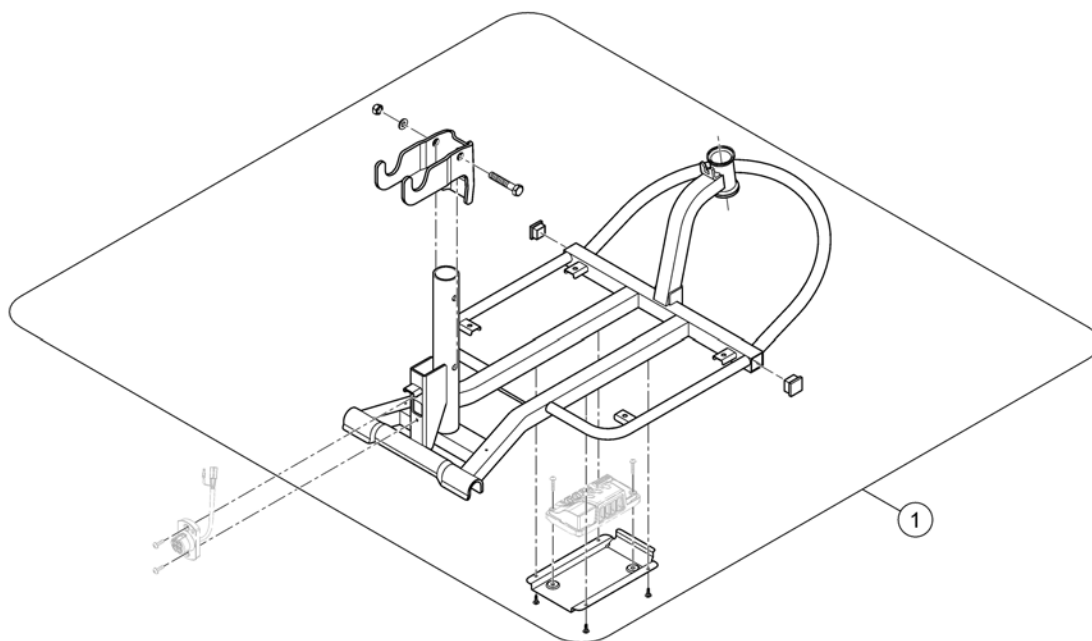

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577514	Front Frame 3-Wheel		1


Front Chassis & Front Axis 4-Wheel

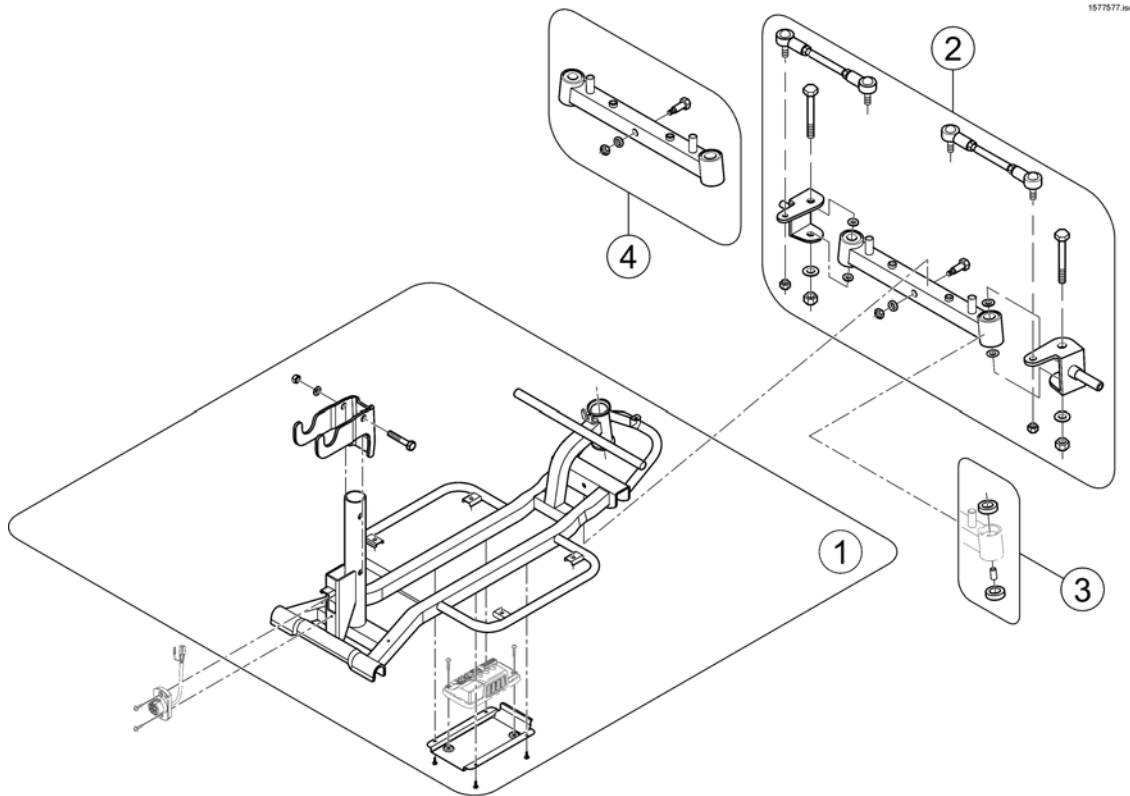


Pos.	Part number	Description	Valid from → until	Qty
1	SP1577405	Front Frame 4-Wheel		1
2	SP1577424	Front Axis, cpl.		1
3	SP1577517	Bearing Kit Front Axis, cpl.		1
4	SP1577539	Swing Assembly		1

Front Shroud 3-Wheel

1505745 iso

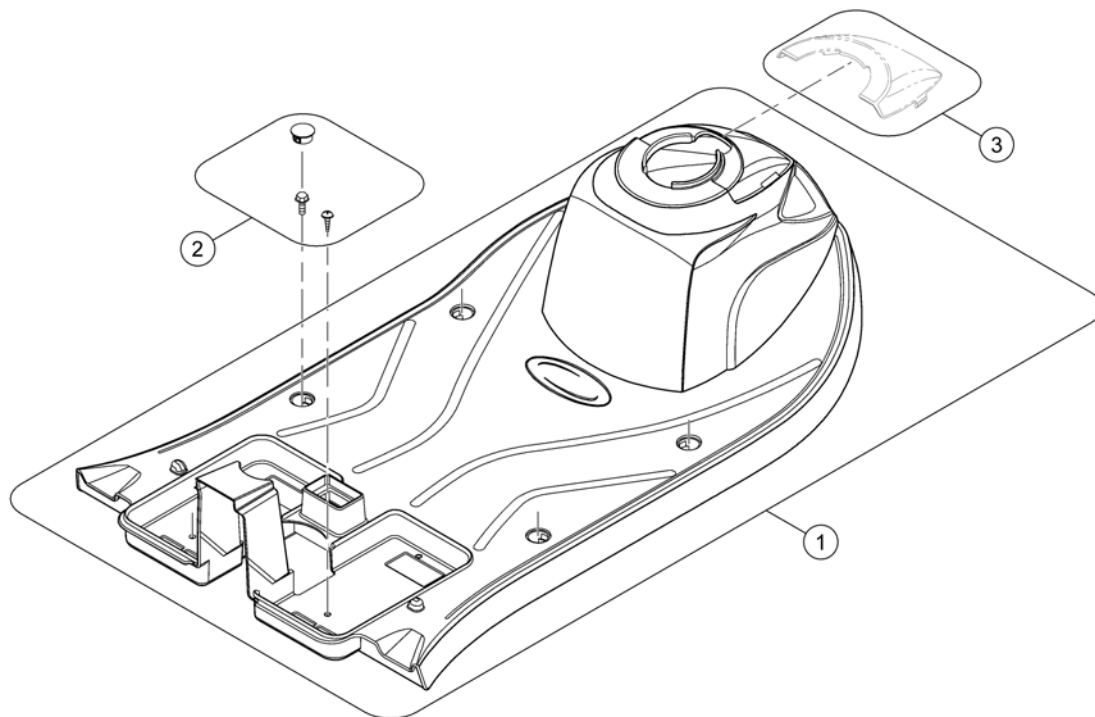


Pos.	Part number	Description	Valid from → until	Qty
-	SP1585425	Front Reflector Kit 3-Wheel		1
-	SP1585423	Side Reflector Kit		1
1	SP1577515	Front Shroud 3-Wheel		1
2	SP1577422	Mounting Kit Front Shroud		1
3	-	Colour Shrouds 3-Wheel		1

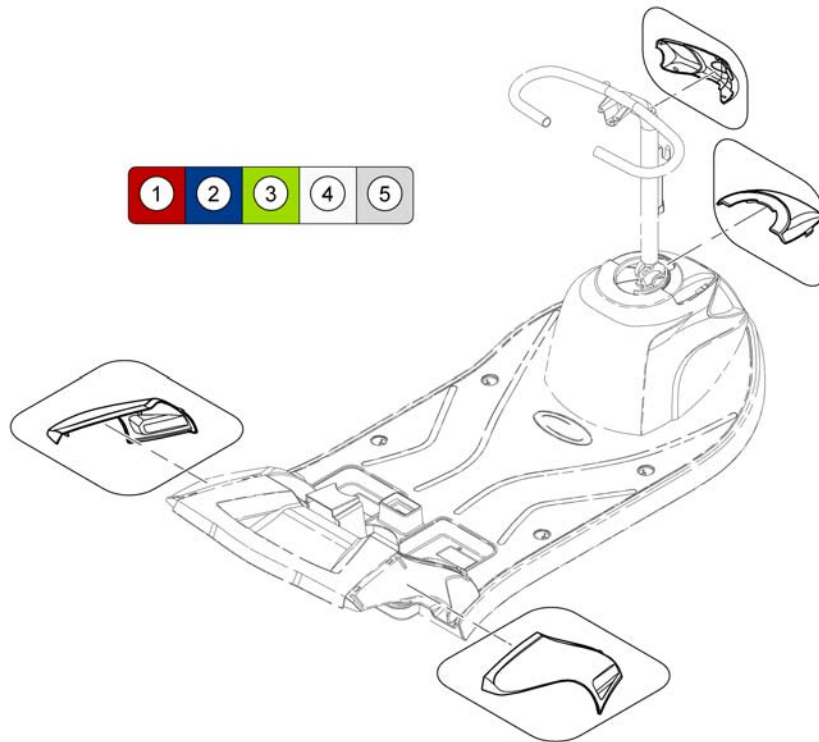

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604313	Colour Shrouds Kit 3-Wheel - Jasper Red		1
2	SP1604351	Colour Shrouds Kit 3-Wheel - Sapphire Blue		1
3	SP1604314	Colour Shrouds Kit 3-Wheel - Jade Green		1
4	SP1604312	Colour Shrouds Kit 3-Wheel - Marble White		1
5	SP1604352	Colour Shrouds Kit 3-Wheel - Diamond Silver		1


Front Shroud 4-Wheel

1577578.iso

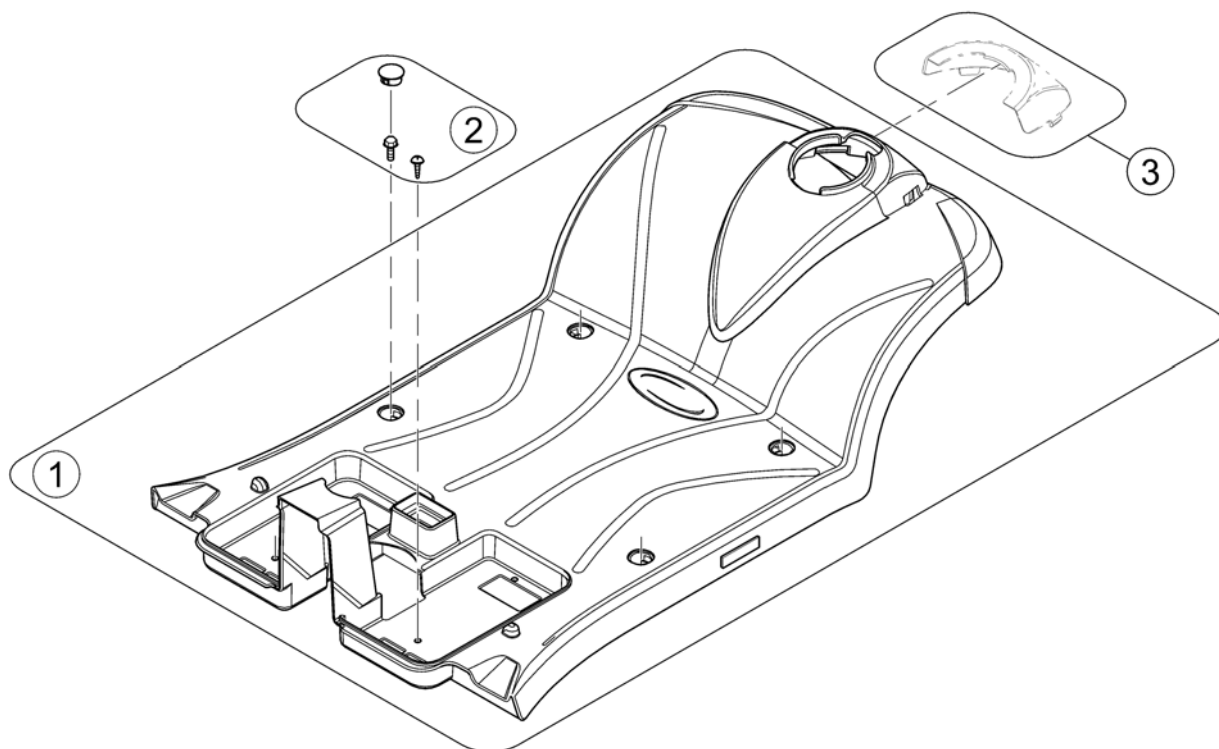

Pos.	Part number	Description	Valid from → until	Qty
-	SP1585424	Front Reflector Kit 4-Wheel		1
-	SP1585423	Side Reflector Kit		1
1	SP1577419	Front Shroud 4-Wheel		1
2	SP1577422	Mounting Kit Front Shroud		1
3	-	Colour Shrouds 4-Wheel		0


1577619.kim

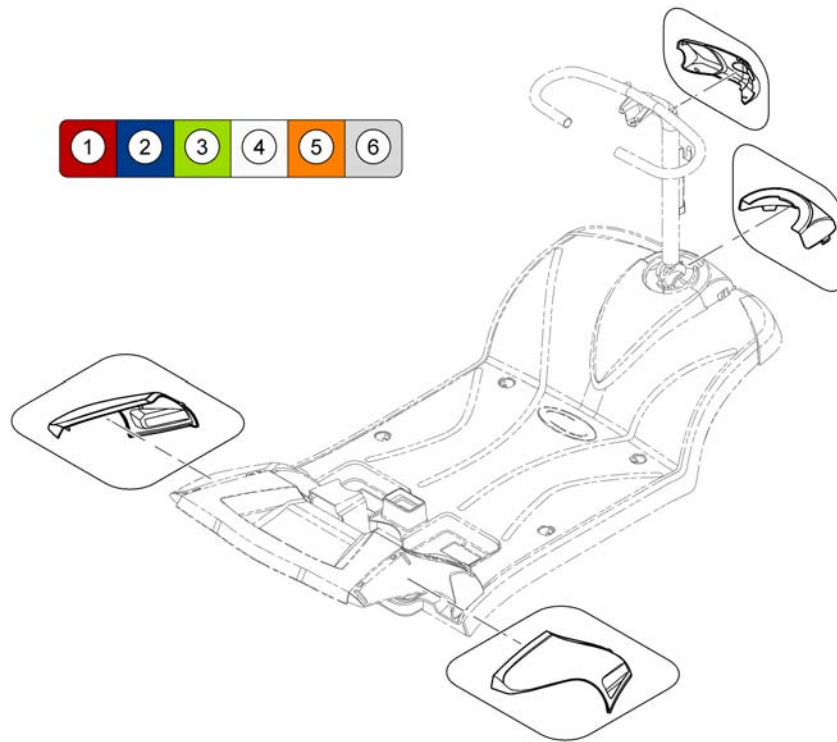

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577169	Color Shrouds Kit 4-Wheel - Jasper Red		1
2	SP1577171	Color Shrouds Kit 4-Wheel - Sapphire Blue		1
3	SP1577170	Color Shrouds Kit 4-Wheel - Jade Green		1
4	SP1577168	Color Shrouds Kit 4-Wheel - Marble White		1
5	-	No longer available.		1
6	SP1604353	Color Shrouds Kit 4-Wheel - Diamond Silver		1

Rear Chassis


1577579 180

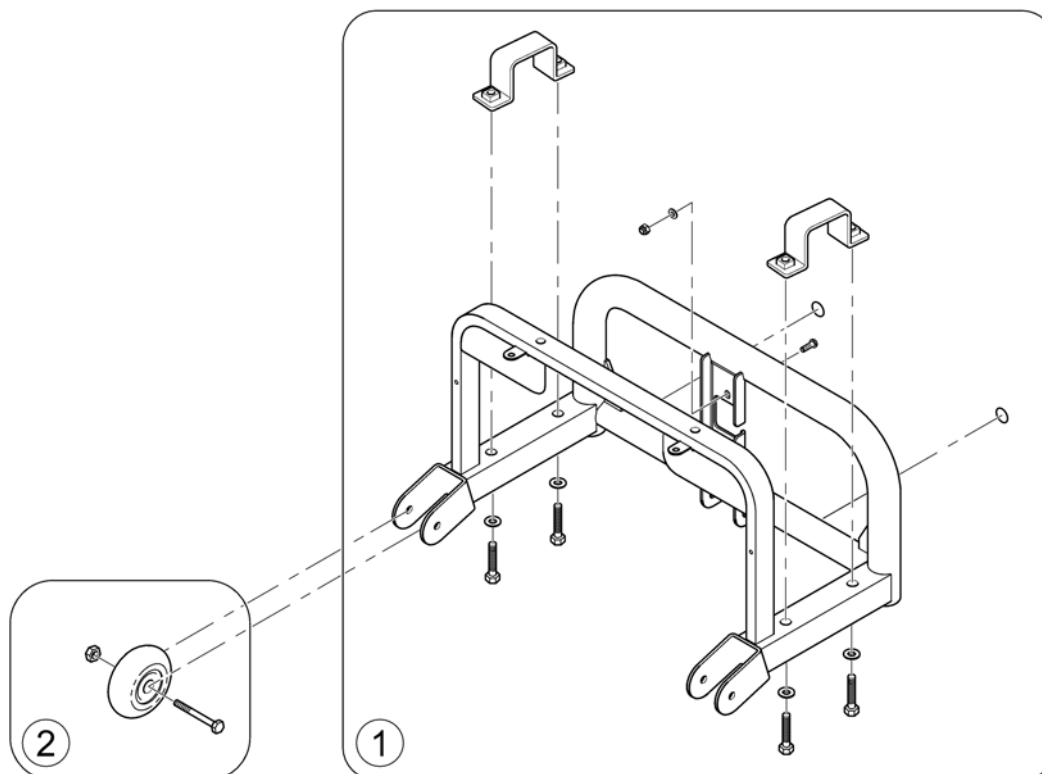

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577434	Rear Frame, cpl.		1
2	SP1577450	Anti Tipper wheel (48mm), cpl. <i>Only in combination with drive wheels 200x50mm!</i>		2
2	SP1645146	Anti Tipper wheel (55mm), cpl. <i>Only in combination with drive wheels 210x65mm!</i>		2


Rear Shroud

1577560.iss

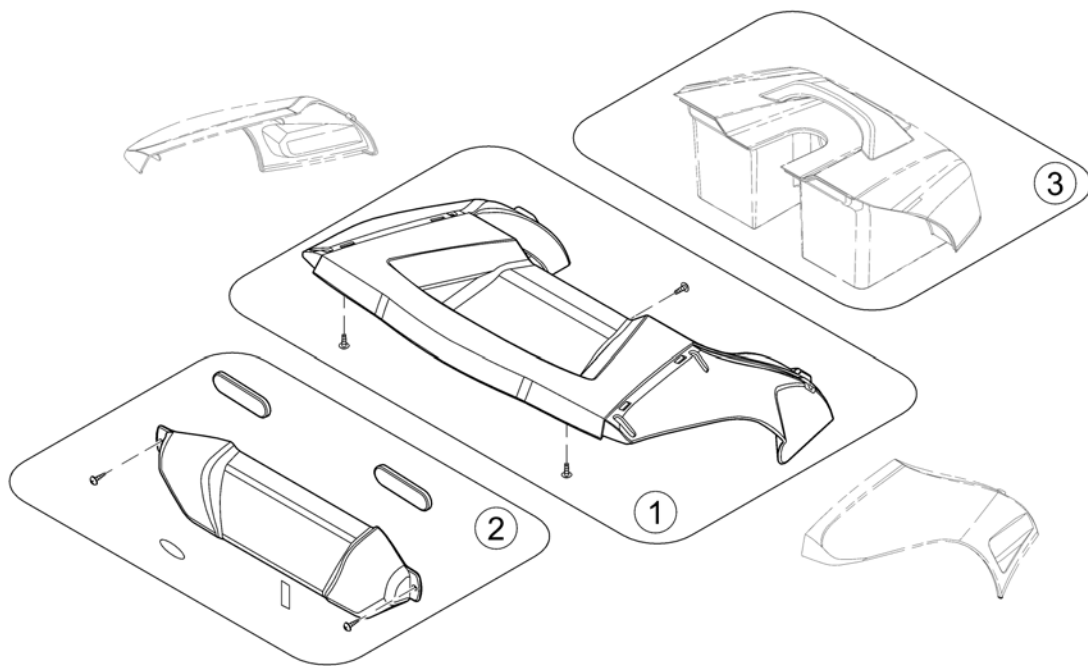

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577436	Rear Shroud		1
-	SP1585421	Rear Reflector Kit		1
2	SP1577435	Shroud Motor Protection		1
3	-	Batteries		0

Batteries

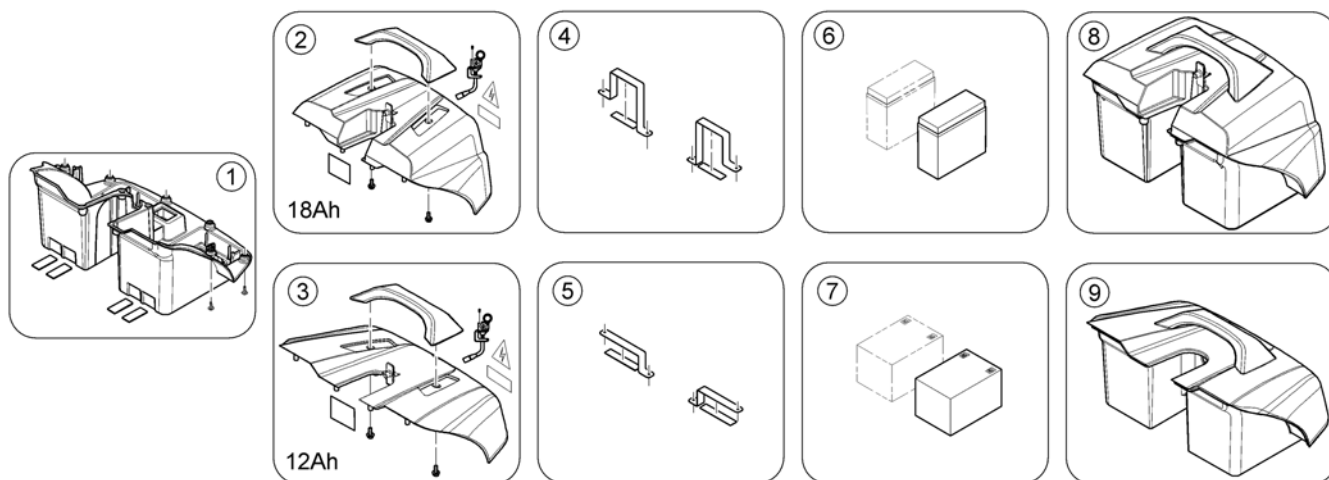


1577614.ISO

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577440	Battery Box		1
2	SP1577439	Battery Lid 18Ah, cpl.		1
3	SP1577438	Battery Lid 12Ah, cpl.		1
4	SP1577516	Mounting Kit 18Ah		1
5	SP1577518	Mounting Kit 12Ah		1
-		Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries)		0
6	SP1577442	Battery, 12V/18Ah, AGM		1
7	SP1506493	Battery, 12V/12Ah, AGM		1
8	SP1581758	Battery Box 18Ah Assy, cpl. with batteries		1
9	SP1581757	Battery Box 12Ah Assy, cpl. with batteries		1


Steering & Tiller Shrouds 3-Wheel

1605743.kso

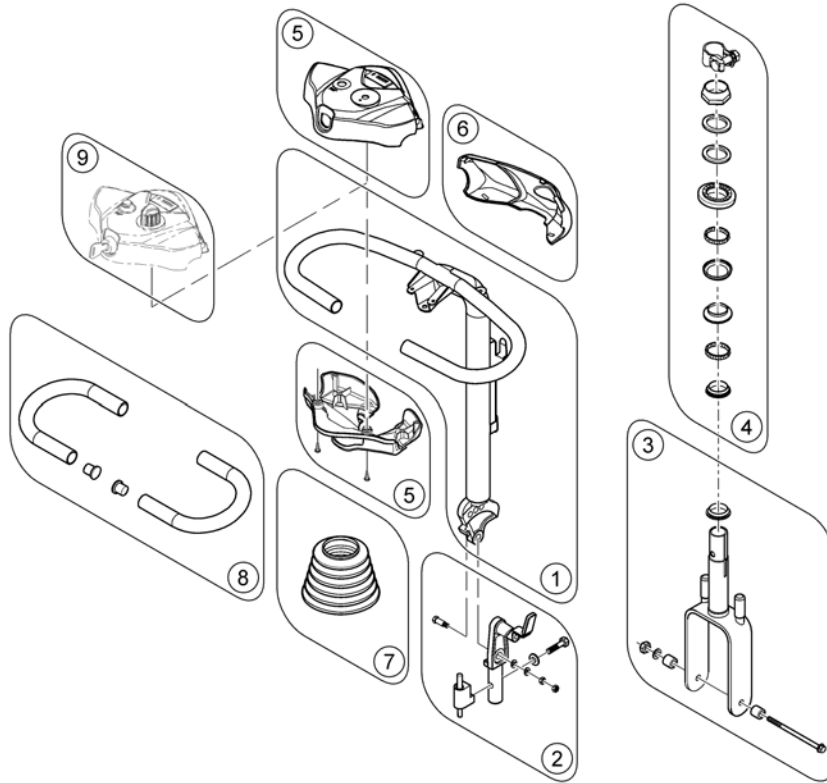


Pos.	Part number	Description	Valid from → until	Qty
-	SP1585426	Reflector Kit Steering Bar		1
1	SP1577432	Steering Bar		1
2	SP1577433	Steering Pivot		1
3	SP1577512	Steering Fork 3-Wheel		1
4	SP1577417	Steering Bearing Kit		1
5	SP1577427	Tiller Shrouds Kit		1
6	-	Colour Shrouds 3-Wheel		1
7	SP1505401	Cover, Steering		1
8	SP1577429	Handle Bar Grip		1
9	-	Electronics & Controller		1

1577612.00

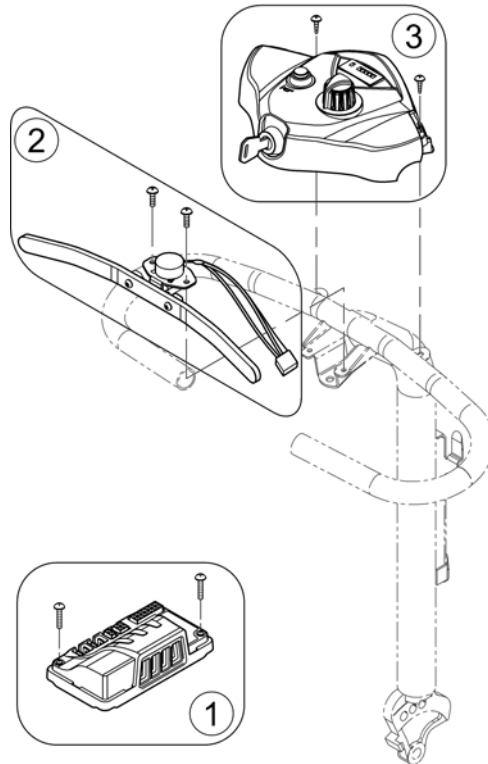

Pos.	Part number	Description	Valid from → until	Qty
1	1506390	Powermodule, Rhino, 50A, DR50-A01		1
2	SP1577428	Wig Wag (incl. Poti)		1
3	SP1577426	Tiller Electronic, cpl.		1
-	SP1530212	Keys (Set of 2 pcs)		1


Steering & Tiller Shrouds 4-Wheel

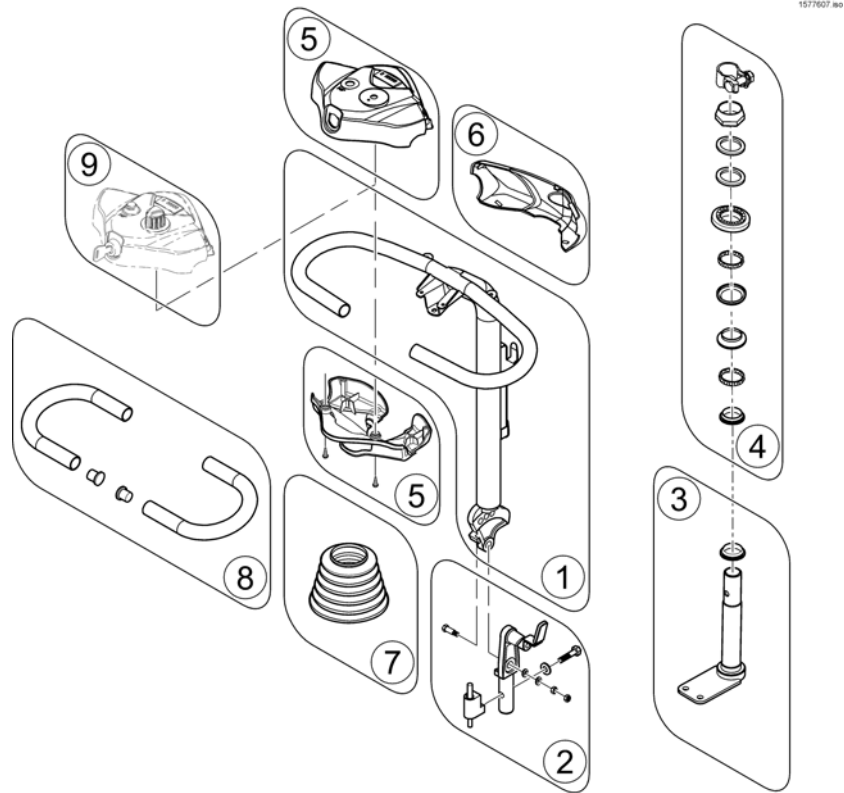

Pos.	Part number	Description	Valid from → until	Qty
-	SP1585426	Reflector Kit Steering Bar		1
1	SP1577432	Steering Bar		1
2	SP1577433	Steering Pivot		1
3	SP1577418	Steering Tube		1
4	SP1577417	Steering Bearing Kit		1
5	SP1577427	Tiller Shrouds Kit		1
6	-	Colour Shrouds 4-Wheel		1
7	SP1505401	Cover, Steering		1
8	SP1577429	Handle Bar Grip		1
9	-	Electronics & Controller		1

Seat & Armpads


1577008e

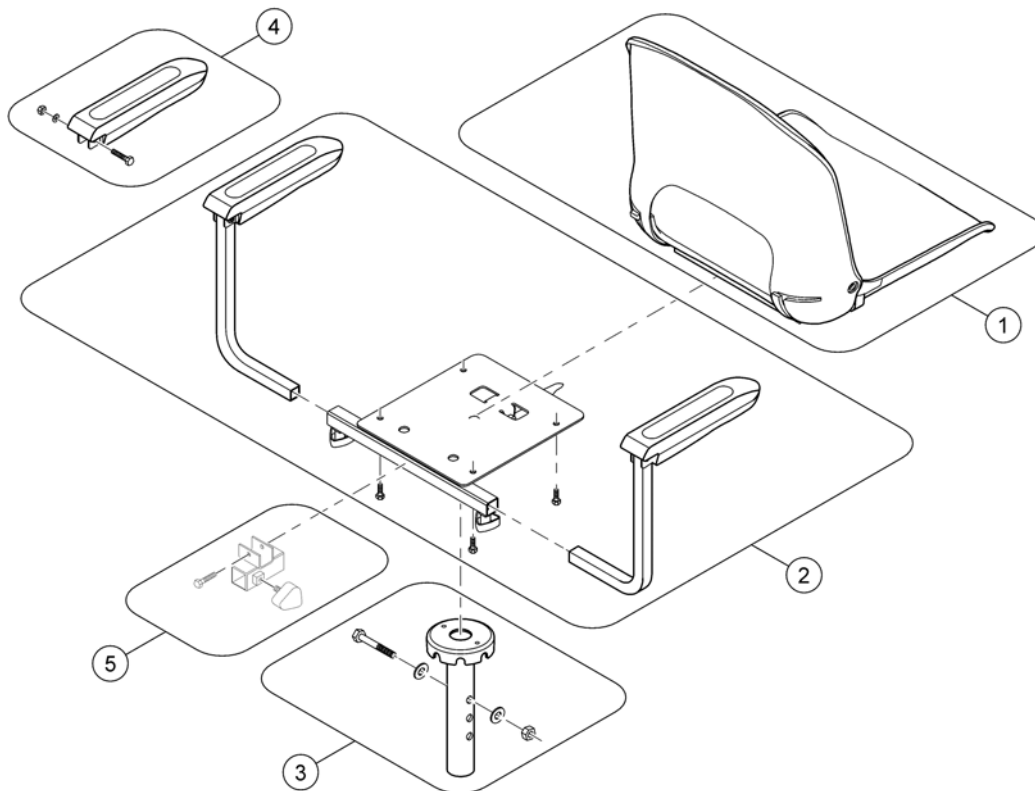

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577444	Seat		1
2	SP1577445	Seat Support		1
3	SP1577446	Seat Post		1
4	SP1577443	Arm Pad LH/RH		1
5	-	Options & Accessories		1

Options & Accessories


1577615.no

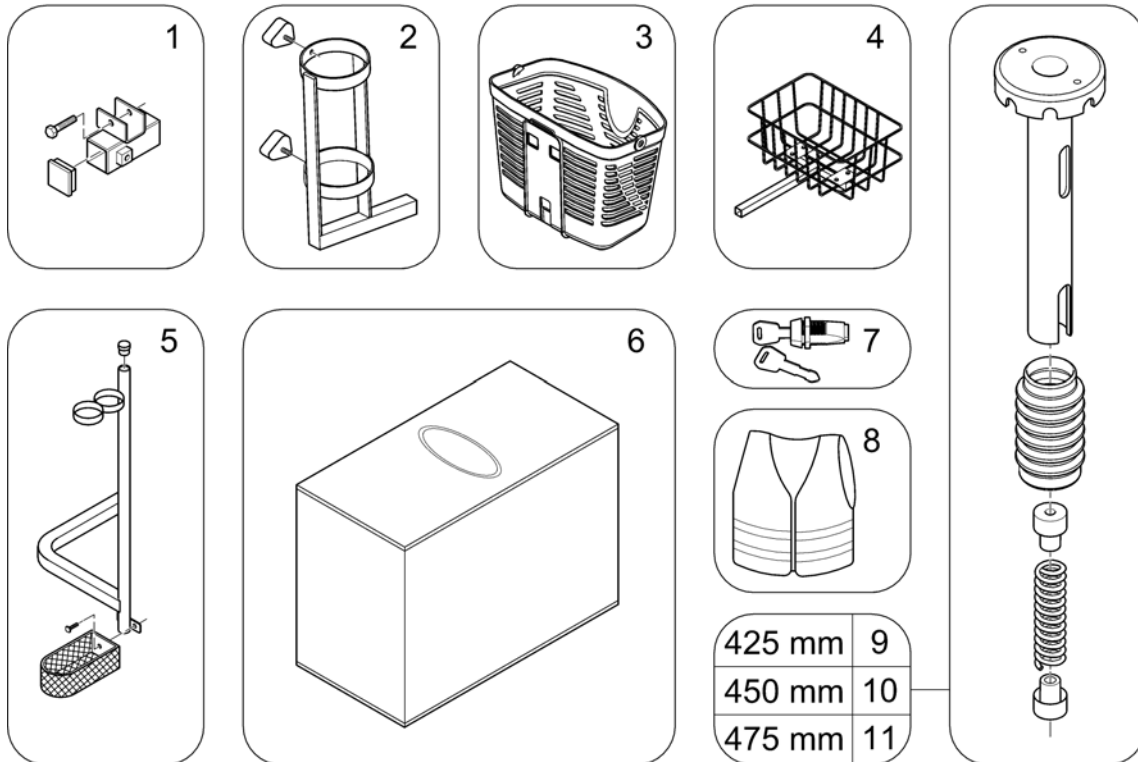

Pos.	Part number	Description	Valid from → until	Qty
1	SP1507388	Accessory Bracket, cpl. <i>Needed in conjunction with positions 2, 4 & 5 / Benötigt in Verbindung mit Pos. 2, 4 & 5</i>		1
2	SP1572971	Oxygen Bottle Holder, cpl. <i>If retrofitting oxygen bottle holder, also order SP1507388</i>		1
3	SP1577430	Front Basket		1
4	SP1604197	Rear Basket (Colibri) <i>If retrofitting rear basket, also order SP1507388</i>		1
5	SP1528274	Crutchholder, cpl. <i>If retrofitting crutchholder, also order SP1507388</i>		1
6	SP1509434	Storage Cover		1
7	SP1543153	Personal Key Cylinder + 2 Keys		1
8	SP1521592	Safety vest		1
9	SP1577212	Seat Suspension low, 425mm		1
10	SP1577213	Seat Suspension mid, 450mm		1
11	SP1577214	Seat Suspension high, 475mm		1


Front Wheels 3-Wheel

1605744 iso

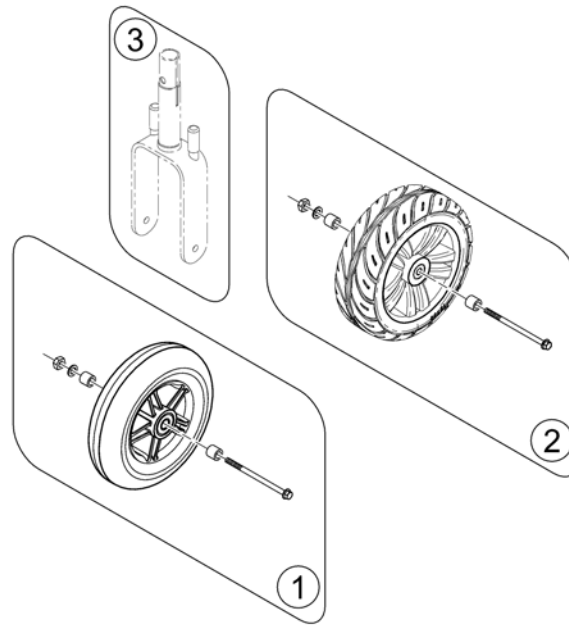

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577513	Front Wheel 200x55mm 3-Wheel		1
2	SP1577453	Front Wheel 210x65mm 3-Wheel		1
3	-	Steering & Tiller Shrouds 3-Wheel		1


1577609 iso

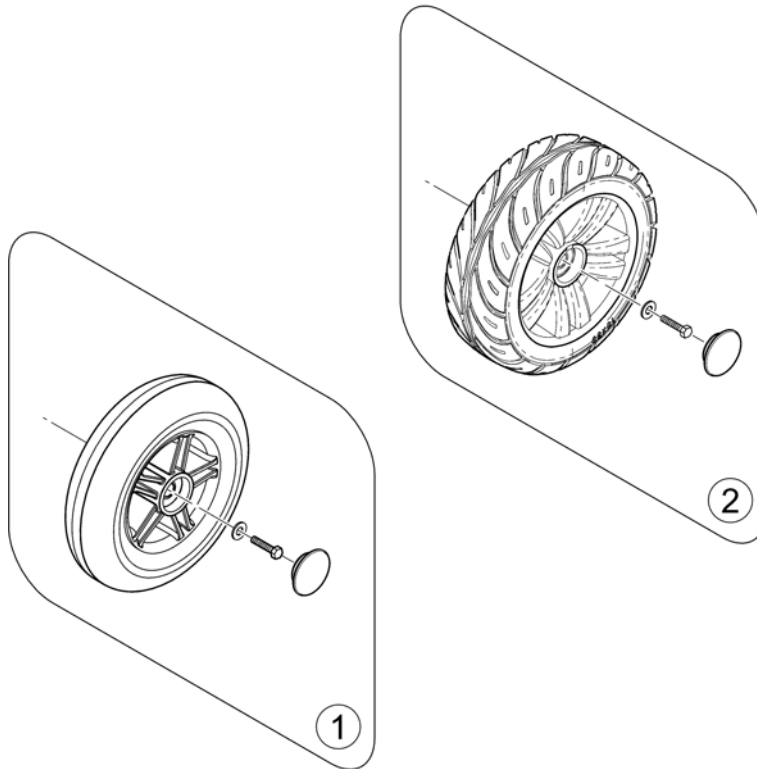


Pos.	Part number	Description	Valid from → until	Qty
1	SP1577425	Front Wheel 200x50mm, cpl.		1
2	SP1577452	Front Wheel 210x65mm, cpl.		1

Rear Wheels

1577610 iso

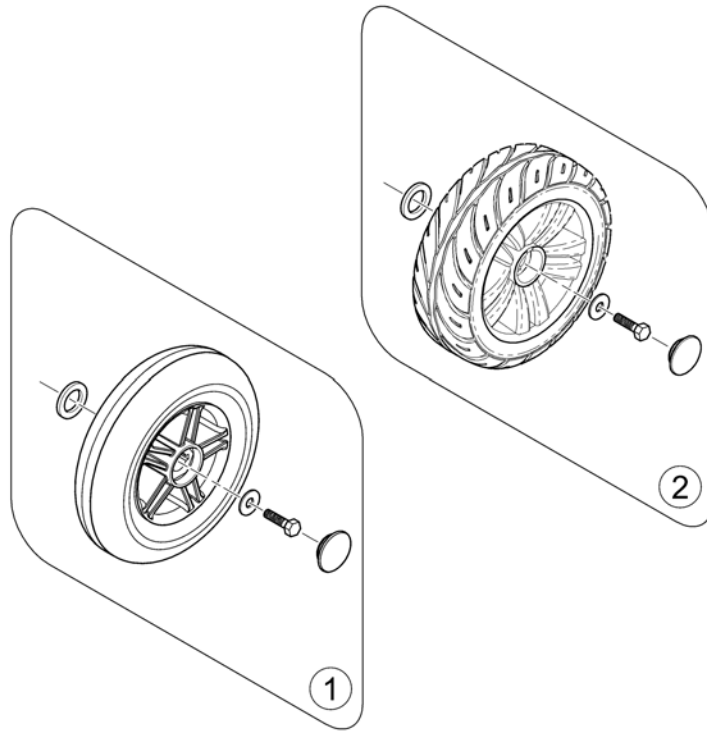


Pos.	Part number	Description	Valid from → until	Qty
1	SP1577448	Rear Wheel 200x50mm, cpl.		1
2	SP1577451	Rear Wheel 210x65mm, cpl.		1

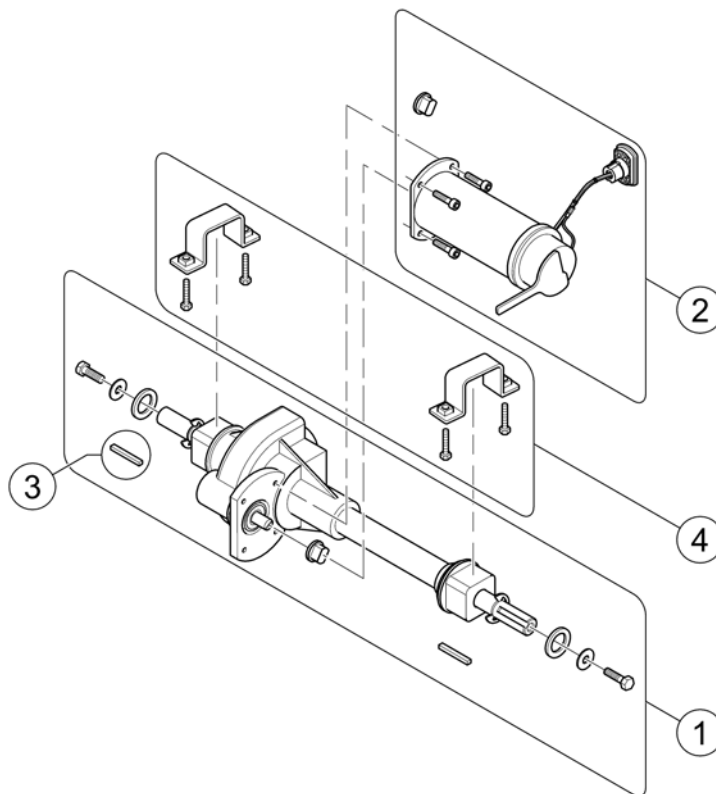

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577447	Transaxle, cpl.		1
2	SP1577449	Motor incl. Mounting Hardware		1
-	SP1588469	Motor Brake - Colibri		1
3	SP1577454	Key-Kit (10 pcs)		1
4	SP1567811	Transaxle bracket		1


Cables

1577613.060

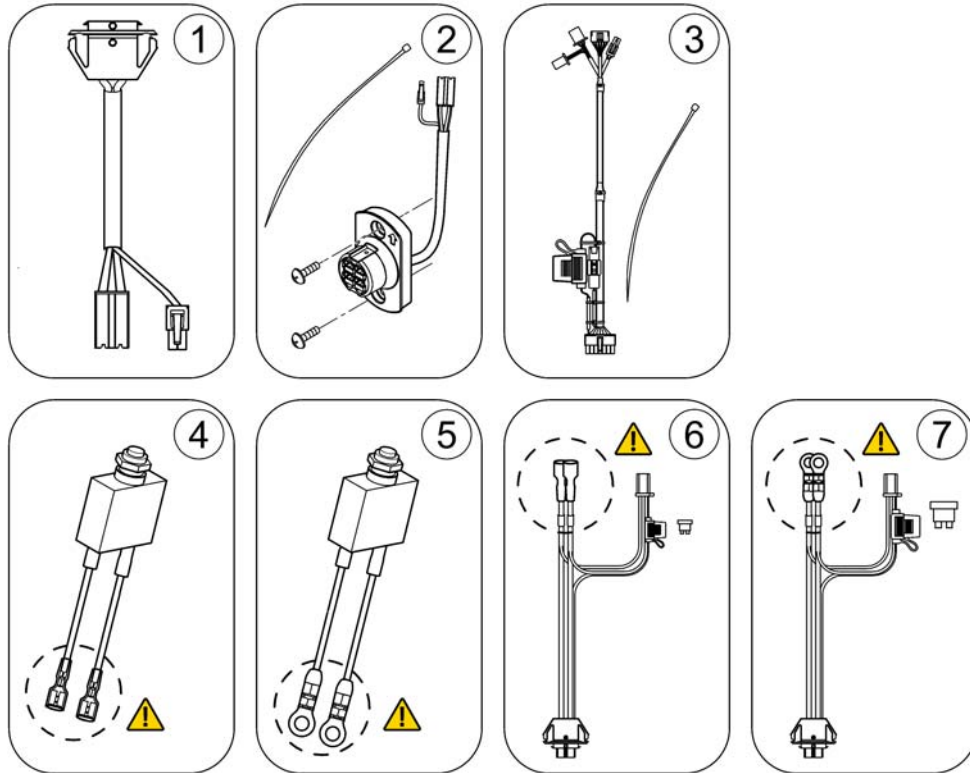

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577421	Battery Connector		1
2	SP1577420	Motor / Battery Connector		1
3	SP1577431	Wire Harness		1
4	SP1577521	Circuit Breaker Cable 12Ah		1
5	SP1577522	Circuit Breaker Cable 18Ah		1
6	SP1577519	Battery Cable 12Ah		1
7	SP1577520	Battery Cable 18Ah		1

Chargers

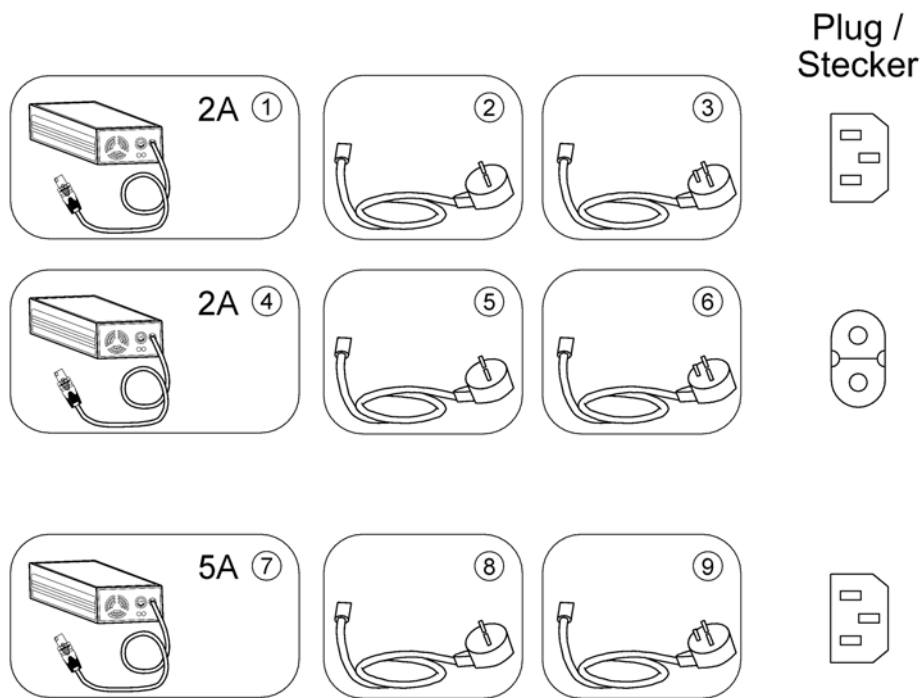

1645170.iso

Pos.	Part number	Description	Valid from → until	Qty
1	-	No longer available - please refer to Replacement Info for which part is to be used instead. <i>SP1577437 is obsolete - Please use SP1645070</i>	→ 2017-11-09	1
2	P21079	Power Cord EU (IEC 60210-C13 / CEE 7/7 Type F 'Schuko')	→ 2017-11-09	1
3	P21080	Power Cord UK (IEC 60320-C13 / BS 1363 Type G)	→ 2017-11-09	1
4	SP1645070	Charger 2A	2017-11-10 →	1
5	SP1645034	Power Cord EU (IEC 60320-C7 / CEE 7/16 Type C)	2017-11-10 →	1
6	SP1645035	Power Cord UK (IEC 60320-C7 / BS 1363 Type G)	2017-11-10 →	1
7	SP1578923	Charger 5A		1
8	P21079	Power Cord EU (IEC 60210-C13 / CEE 7/7 Type F 'Schuko')		1
9	P21080	Power Cord UK (IEC 60320-C13 / BS 1363 Type G)		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånå
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk